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## **Instructions for use**

Samson transport lifting device  
DRVF02

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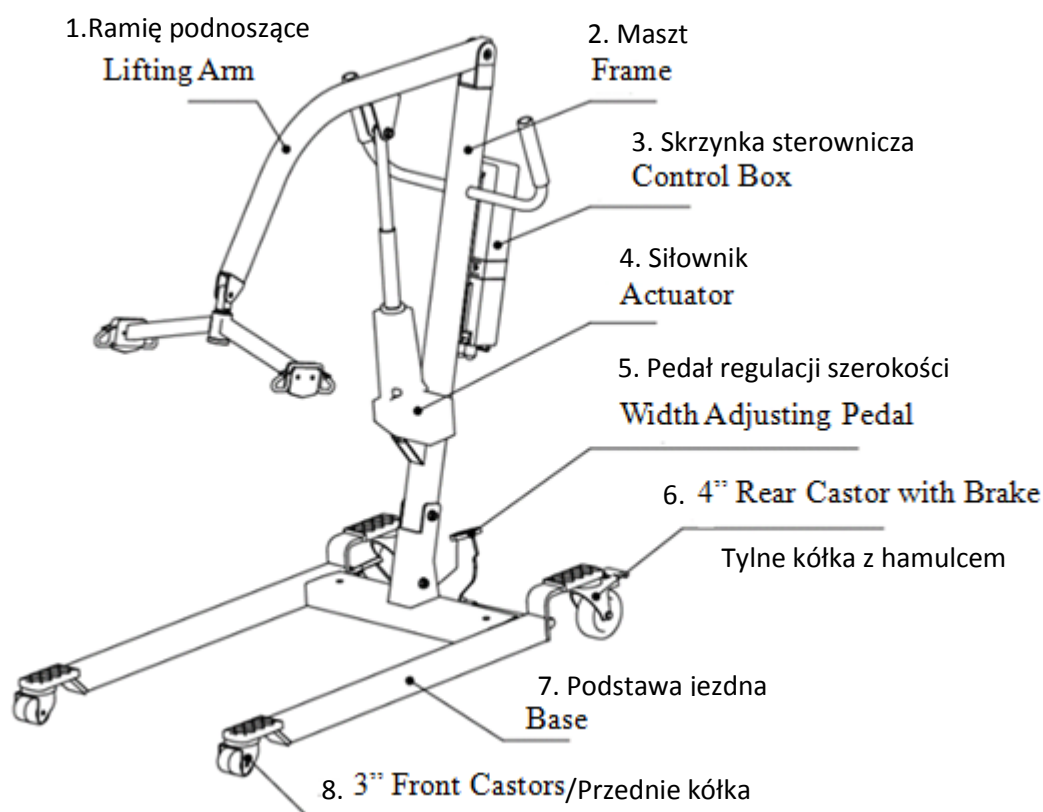
PLEASE READ CAREFULLY BEFORE USING  
WHOLE OF INSTRUCTIONS

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Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual was made in order to provide the necessary information on the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to distinguish that to date we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual.

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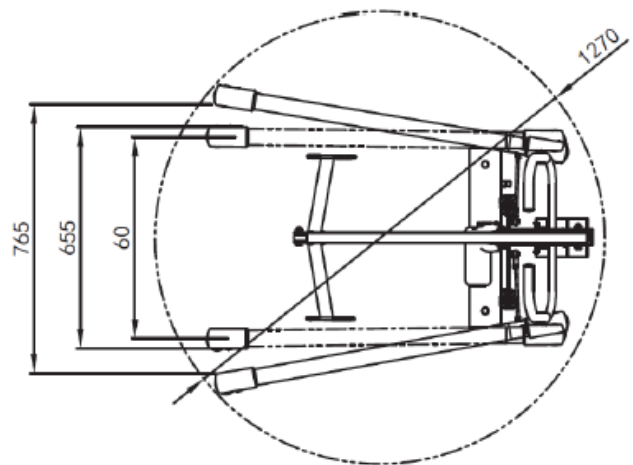
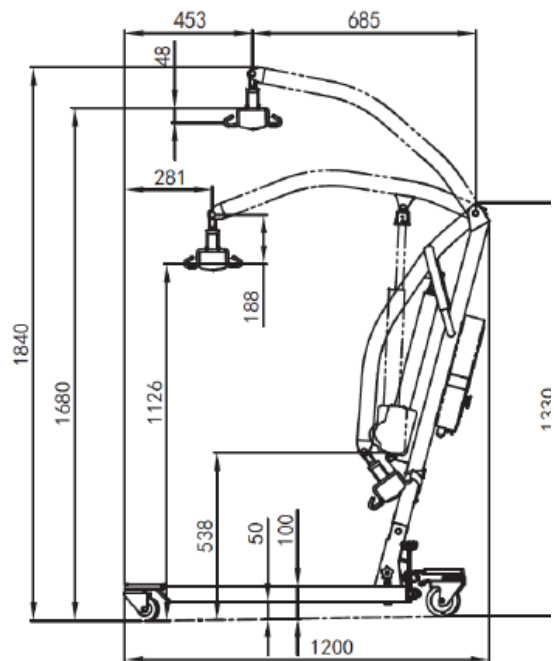
1. Lifting arm
2. Frame
3. Control box
4. Actuator
5. Width adjusting pedal
6. 4" rear castor with brake
7. Base
8. 3" front castor

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Elements that should be inside the packaging:

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Samson transport lifting device is designed to carry only one patient at a time. For safety during lifting and transport, patient should be under supervision of trained caregiver.

Each time in order to maneuverer should use handle. Thanks to the solutions implemented from the device can be used patients affected:

- lower limbs paresis
- paraplegic including lower limbs
- paralysis

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- amputation
- circulatory system insufficiency
- Reduction in joint mobility and weakening of muscle strength.

Samson transport lifting device can also be used in the care of elderly persons.

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Samson transport lifting device is not designed to carrying patient at longer distances. Should choose slings for each patient recommended by doctor. During transport the patient by using Samson should not move it by carpet edge, doorstep or any other uneven surface. In case side effects occur should discontinue patient transport.

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- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above.
- Ensure that all components have no visible damage. If any doubts, please contact the manufacturer.
- Do not use the lifting device until all parts are properly connected.

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#### INSTALLATION PROCEDURE

##### Step 1:


- Release the locknut and bolt and then raise up the frame and align the holes. Fix bolt as shown in drawing and have nut properly attached (make sure no movement on the frame).

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
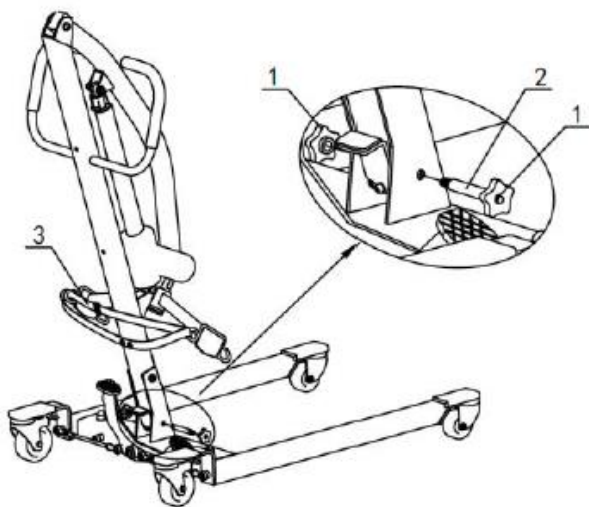
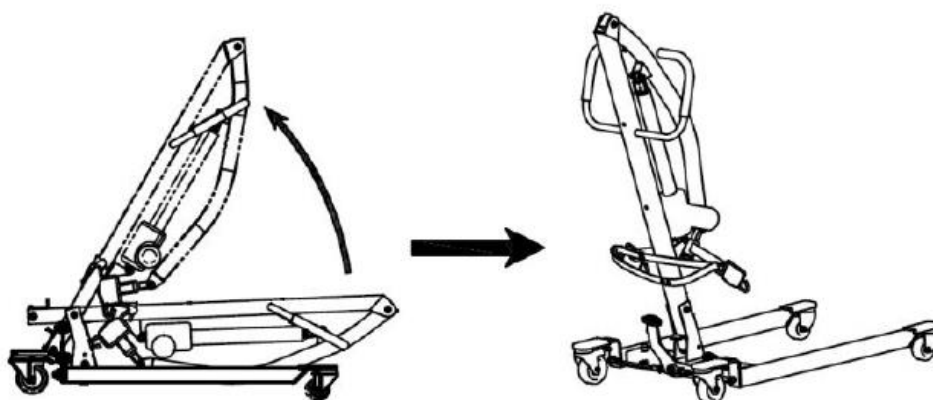
**Step 3:****Installation of control box**

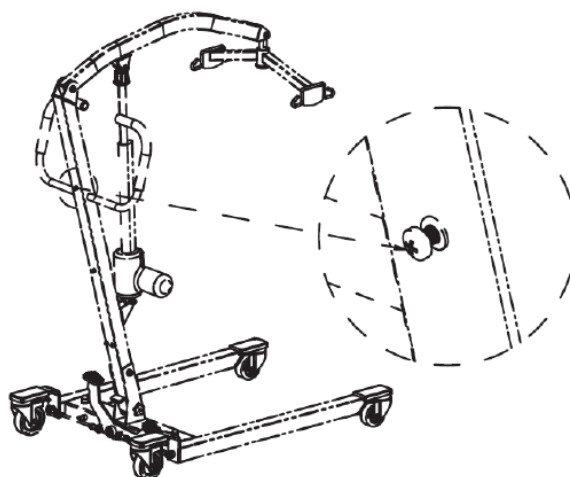- The frame has two screw holes for fixing the control box with a battery (Illustration 3).

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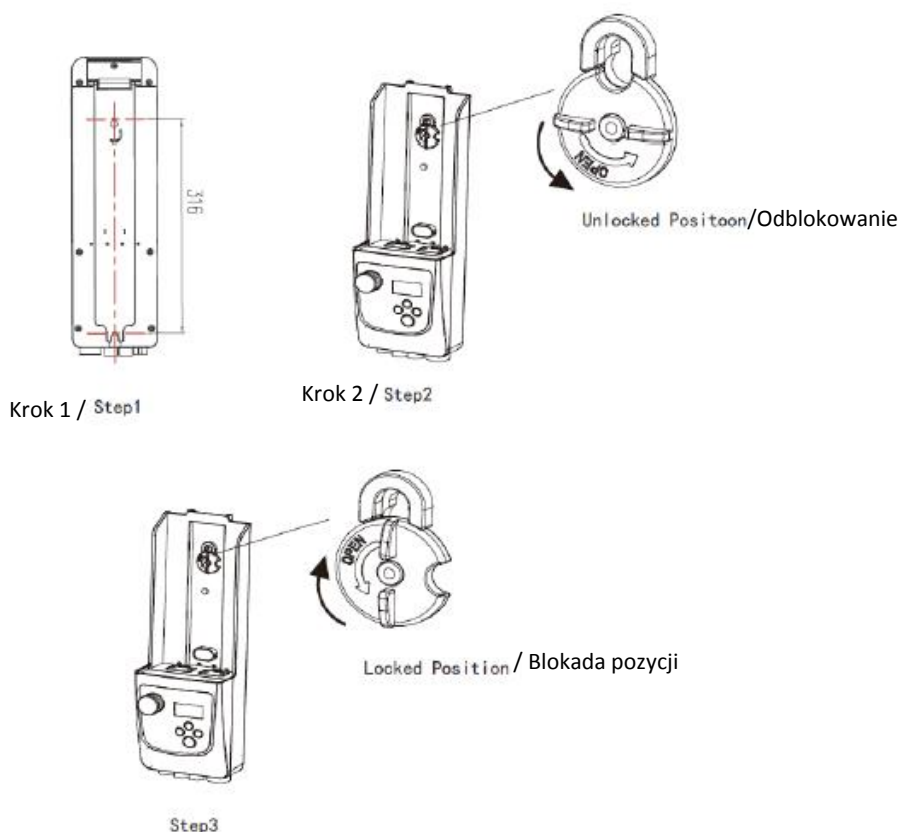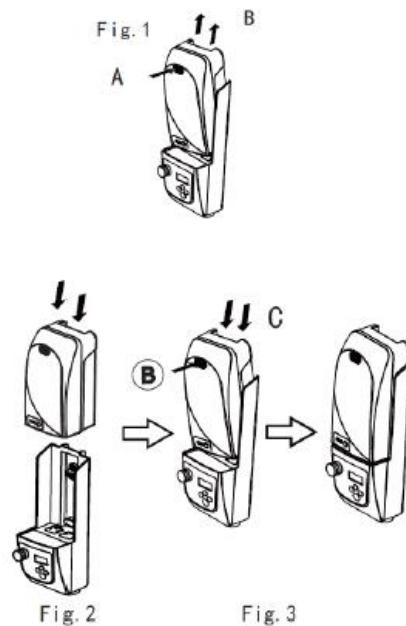
#### Step 4:

##### Remove and install the battery box.

Removal: Press down the switch A and remove battery along direction of B.

Installation: Place the battery in place and press the switch along direction of C. Ensure the battery won't be separated without pressing the switch.

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#### REMOTE CONTROL

- Lifting and lowering the jib is simple and it is performed by remote control.
- After using or adjusting the lifter arm height, the remote control should be placed in a safe and easily accessible place on the handle.

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- To raise or lower the jib, press the button with the corresponding symbol.
- If the remote control does not work properly, press the red "Emergency stop" button

### EMERGENCY STOP BUTTON

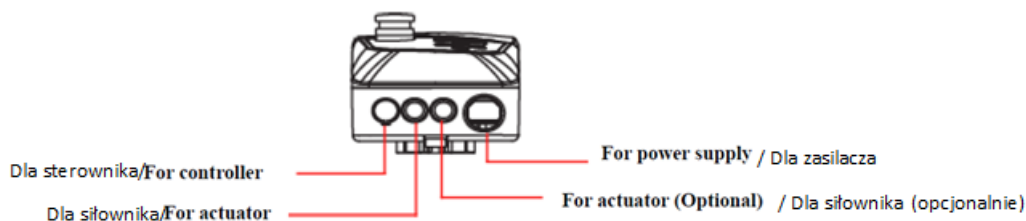
- After pressing the emergency button, the power supply from the battery to the working motor is interrupted, which immediately interrupts the process of lifting or lowering the patient.
- The emergency stop button is unlocked by turning it to the right. The emergency switch pops into the zero position.

### MANUAL EMERGENCY DESCENT AND ASCENT

- In the event of overload or failure of the electrical installation (eg battery failure), mechanical emergency lowering allows manual withdrawal of the loaded actuator.
- To lower the rod you should:
  - Move the protection to the left,
  - Lower the lifter arm,
  - After lowering, fold the securing back to the right again

### CONNECTION OF CONTROL DEVICE

There are 4 ports on the control device, 1 for controller, 2 for actuator and 1 for power supply.

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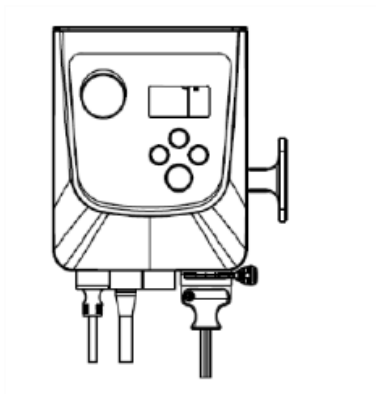
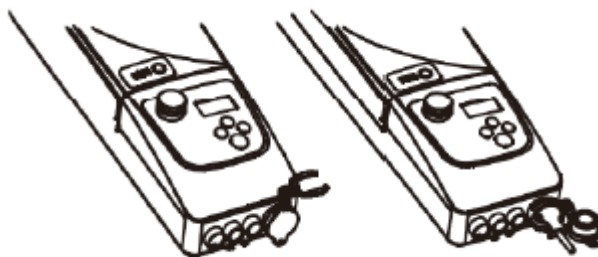
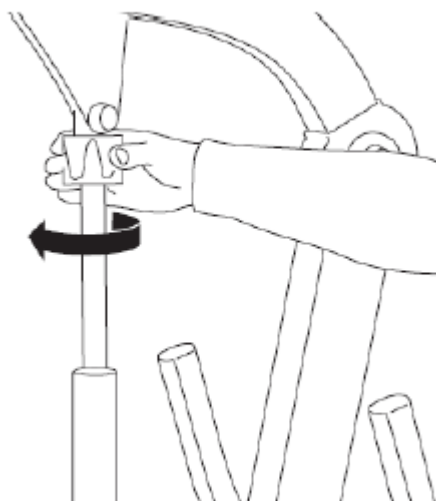
#### **MANUAL EMERGENCY DESCENT AND ASCENT**

Rotate the red sleeve along the direction of arrow to get manual emergency descent, the reverse to manually ascend.

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
## LOCK AND UNLOCK OF REAR WHEEL

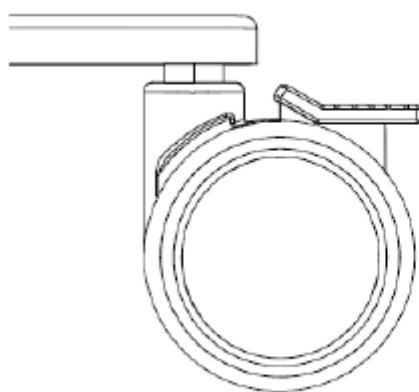Rear wheels can be locked for no movement. Operate with your foot to lock and unlock the rear wheels.

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**Attention:** It is unnecessary to lock the rear wheels for most of a ascent and descent movement. We suggest only to lock the wheels when patient lift moves and may cause personal injury.

**Warning:** To lock the rear wheels of patient lift increase the risk of tipping during the ascent

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and descent movement.

- a) Before using the lifting device refer to this manual and keep it for future reference.
- b) The lifting device is designed to use indoors only on hard and even surfaces.
- c) Using the lifting device on an unstable surface (covered with ice, wet, greasy, sandy) is unacceptable- it causes risk of falling,
- d) You can only use the lifting device on surfaces where all four wheels are touching the ground.
- e) Do not leave the lifting device with the user close to open sources of fire.
- f) Be aware that the surface temperature of the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of lifting device. It may cause burns or frostbite parts of the body.
- g) In order to transfer lifting device, grab the frame. Never transfer the lifting device by grabbing movable elements or control unit.
- h) The lifting device is designed for people up to 180 kg.
- i) Lifting device must be easy to fold and to unfold. In case of problems with folding, you should re-read the instruction or contact the manufacturer.
- j) Brakes cannot slow down the lifting device during ride, You should use them only to prevent the unintended movements.
- k) Oil, grease, water or other chemicals, may cause that brakes may have reduced effectiveness.
- l) It is extremely important to keep all lifting devices components clean in order to maintain the product parameters.
- m) Storing the lifting device in areas with very high humidity (baths, laundry, etc.) or in the open air, also the lack of hygiene causes rapid loss of

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operating parameters and aesthetic of the product.

- n) It is unacceptable to use the lifting device after discovering any visible or detectable damages or when the wear of some parts may cause an accident. All the repairs need to be performed by a skilled personnel and during the warranty period only by the manufacturer's service.
- o) During transport (e.g. In the trunk), do not burden lifting device elements, it may weaken or damage its structure.
- p) Keep out of the reach of children under five years of age.
- q) Should not insert fingers between the lifting device components - risk of injury or entrapment.
- r) Keep extreme caution while unfolding / folding the lifting device. Lack of attention may result in finger entrapment.
- s) Always maneuver the lifting device with the handle provided.
- t) Do not use a sling that is not recommended for the lift.
- u) Only trained and qualified caregivers should transfer a person.
- v) Lifting device is designed for transferring patients only. Do not use the lifting device for any other purpose.
- w) Before each patient lifting , make sure that base legs were blocked in maximum wide position.
- x) While moving the patient, should bypass obstacles at safe distance, which might cause the lifting device to tip over .
- y) Before each lifting device usage, make sure that slings are in good condition. Usage slings showing signs of damage is forbidden.
- z) Before using lifting device, should charge the battery for about 6 – 8 hours. Make sure, the red warning lights is off.

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### **Lifting the patient:**

- First, set the ride base in the maximum enlarged position. To do this, press the pedal of the head adjusting the width of the transport lifting device.
- For easy and comfortable lifting of the patient should be prepared a transporter lifting device, a sling with a leg separator, a sling with head stabilization and a wheelchair.
- The patient should lie in the middle of the bed, then turn he/she to the side, in the opposite direction to the caregiver.
- The sling should be placed under the patient.
- Examples of slings are shown in the Illustration 12 and 13.
- Turn the patient towards the caregiver and place he/she in the middle of the sling so that the back loops are in the upper back of the patient and the thigh loops be placed within the patient;s thighs.
- Hanging loops should be hung on a four-point cradle.
- Loops have different colors to make it easier to put on slings at the same height.

*Ilustracja / Illustration 12. Nosidło z separatorem nóg/ Sling with a leg separator*

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The patient can be placed on a sling in a sitting position, e.g. when being transported from a wheelchair or a bed as follows:

- Place the sling around the back of the patient.
- Donning is easier if the patient bends slightly forward.
- Guide the sling carefully at the patient up on the height of the tailbone.
- Move the thigh belts forward at patient's thighs until they are on both sides at the same height.
- Be careful not to bend the material.
- Wear the seat belts under the patient's slightly parted legs.
- Donning is easier if the patient slightly raises thighs.

In order for the patient to sit comfortably on a wheelchair, it should be planted as deep as possible.

- During the process, lowering the patient, lightly press the patient's knee so that his/her back is closest to the backrest of the wheelchair.

### **Foldable and portable frame**

- Low down the actuator and then fix the lifting arm completely

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- Remove the bolt before folding the frame
- You should release the support plate before the frame stands up

Fig. 1 /rys. 1

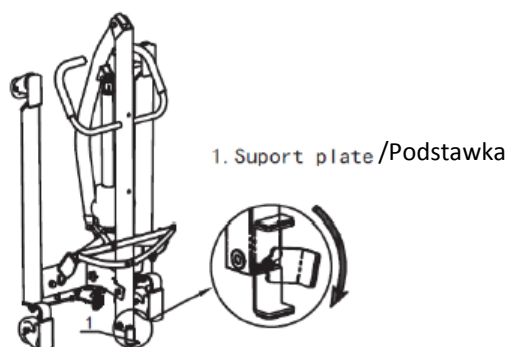
Fig. 3 /rys. 3

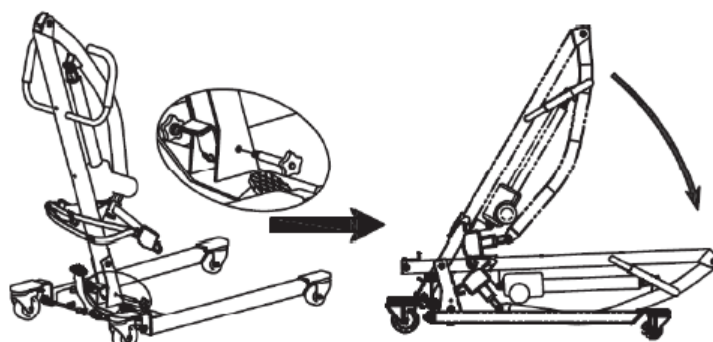
Fig. 2 /rys. 2

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Notice: Please remove the control box before folding the frame.

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### Taking off the sling:

- Remove the hook loop from the four-point cradle.
- Thigh belts should be carefully pulled under the thighs from the side to the down.
- Remove the sling carefully from the patient's back area, moving them towards the legs.
- Removing the belt will be easier if the patient tilts a little forward.

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### Battery charging

1. When the battery has only 1 bar left, controller box shall keep beeping 4 times and remind you to charge immediately. Actuator shall not work without charging (battery level as shown in the picture.

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## 2. Battery Charging Procedure

When battery beeps, switch off the emergency stop button and plug in the controller box. Once battery is fully charged, the state of battery level shows 4 bars (100%) and LED on the control box no longer flashes.

Notes:

1. Do switch off the emergency stop button and stop using the patient lift when charging.
2. Remove the control box and keep it in a shade once patient lift shall be not used for a long time. Charge the battery once every month.
3. Do not charge the battery in a humid atmosphere.
4. In the event of that patient lift is not in daily use, it is strongly recommended that to switch off the emergency stop button so that to cut off the currency and keep the battery level.

## Care and maintenance

Routine inspection listed below shall be performed everyday for the sake of safety.

1. Check if there is any damage or wear by visual inspection.
2. Make sure screws and nuts on lifting arm are secured.
3. Check if hook on lifting arm works properly.
4. Controller, actuator and base width adjustment verification.
5. Emergency descend function verification (both manually and electrically)

If needed, to clean the patient lift with warm water or alcohol. Make sure there is no dust and hair on the

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wheels.

Attention: to clean the patient lift with chemicals with Phenol and chlorine is prohibited, which may cause damage on plastic powder or plastic material.

Transportation and Storage:

Press the emergency stop button for transportation or not using the patient life for a certain of time. Temperature for storage of patient life shall be higher than 0°C and relative moisture ≤60%.

Inspection:

Make a full inspection at least once a year. Pay close attention on those worn part such as connection and screw bolt.

## **SLINGS**

Clean the seats with a damp cloth. Difficult impurities must be removed with a commercially available mild detergent. Stains can be removed with a cloth or a soft brush.

Wash off with a wet cloth (with clean water) and allow it to dry.

### **Advice:**

Do not use:

- solvents,
- toilet cleaners
- sharp brushes and hard objects,
- caustic cleaners.

## **Washing instructions**

Advice:

Before washing, follow the instructions on the label of the belt.

The belts can be washed in a washing machine at 30 °C.

Advice:

- Do not use chlorine bleaching agents.
- Do not dry in clothes dryers.
- Do not iron.

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Description	Trouble Shooting
Patient lift can not work (ascend and descend)	<ol style="list-style-type: none"> <li>1. Check if the emergency stop button is pressed down.</li> <li>2. Verify the controller and cables from actuator is properly connected.</li> <li>3. Check the battery level from display screen on the control box.</li> <li>4. Verify if there is any problem or damage on battery contact panel.</li> <li>5. Ask for after-sale service if patient lift still can not work.</li> </ol>
Charger can not charge the battery	<ol style="list-style-type: none"> <li>1. Check if the charger fully plugs into socket.</li> <li>2. Verify if there is any problem or damage on battery contact panel.</li> <li>3. Ask for after-sale service if patient lift still can not work.</li> </ol>
Patient lift stops when ascending	<ol style="list-style-type: none"> <li>1. Check if the emergency stop button is pressed down.</li> <li>2. Employ the emergency descend device.</li> <li>3. Ask for after-sale service if patient lift still can not work.</li> </ol>

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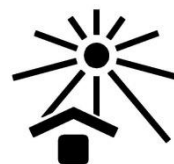
#### DISPOSAL OF WASTE ELECTRICAL AND ELECTRONIC EQUIPMENT

This symbol on the products or accompanying documents means that used electrical or electronic products should not be disposed of with the normal household waste. For proper disposal, refurbishment or recycling, please take these products to the collection points for this type of waste. Alternatively, in some EU countries or other European countries, you may return your product to your local retailer when purchasing a similar new product. By disposing of this product correctly, you will help to conserve valuable natural resources and support the prevention of potential negative effects on the environment and human health, which could result from incorrect disposal. For further information, please contact your local municipality or collection point waste collection points. Penalties may be imposed in accordance with local regulations if this product is disposed of in an inappropriate manner. For parties within the European Union, If you wish to discard electrical or electronic equipment, please obtain the necessary information from your retailer or supplier. Elimination in countries outside the European Union. This symbol is valid in the European Union. If you wish to decommission this appliance, please obtain the correct decommissioning information from your local authorities or from your retailer.

#### 10. Dane teleadresowe \ Contact details

mdh Sp. z o.o.  
ul. Maratońska 104,  
94-007, Łódź, Polska  
tel. +48 42 674 83 84  
fax. +48 42 636 52 21  
www.mdh.pl www.viteacare.com