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Anti-bedsore alternating pressure bubble mattress VCM202

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**PLEASE READ WHOLE INSTRUCTION
CAREFULLY BEFORE USE.**

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Introduction

Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasise that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual.

Product description

Alternating pressure bubble anti-bedsore mattress equipped with end flaps which allows to place the mattress on the bed. Maximum weight capacity: 120 kg

Indications

Prevention and support of treatment of stage I and II bedsores. The mattress is inflated by a system of pipes. The cells are alternately inflated which ensures changing position of body support points. Patient's body rests on the same, invariable height regardless of the phase of pump functioning.

Contraindications

In case of adverse skin changes you should immediately consult a physician or medical personnel.

Use

- Place the mattress on a standard bed so that the air intake would be located on the side of patient's legs. Put the end flaps under the bed mattress.
- Check if the switch is in the "off" position.

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- Connect the pipes to the pump – insert them into the air outlet located on both sides of pump and on the mattress.
- Place a pump on hard surface or hang it on hinges. The place where the pump will stand must be clean and dry. Dust, hair or other impurity may block the filter.
- Please, check if the pipes are not folded or pressed.
- Switch on the pump.
- Set the pressure regulator on maximum. It will shorten the time of pumping which usually takes 20-30 min.
- After inflating the mattress, you should choose the pressure according to the doctor's recommendation.
- Place a patient on the mattress.

In order to receive exact instructions concerning an appropriate use and optimal pressure, please, consult your doctor. The mattress and pump create effect of "constant movement" (changing position of body support points), provide constant muscle massage and improve blood circulation. Thus, the mattress is an effective tool to prevent bedsores. But the position of the patient on the mattress needs to be changed despite of using it.

Switching off

- Turn the switch into the "OFF" position.
- Disconnect the pipes from the air outlet in order to prevent air from escaping.

Safety of use WARNIGNG!

- Before using the mattress, refer to this manual and keep it for future reference.
- Permitted user weight 120 kg.

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- t) Always consult your doctor before using the mattress.
- u) Using the isopropyl alcohol or dissolvent may damage the pump.
- v) Please, make sure that the pump is completely dry before setting on.
- w) The mattress can be folded while storing. Before folding, check if it is completely dry which protects it against mold. The mattress should be stored in a dry place.
- x) Do not put the mattress close to the fire.
- y) Keep the mattress away from sharp objects.
- z) Don't put the mattress directly on springs or frame with sharp edges.
- aa) Pay attention to skin changes. If necessary, consult your doctor or nurse.
- bb) Tubes can not be bent.
- cc) Never use the mattress deflated.
- dd) While moving bed, unplug the mattress.

Everyday use and maintenance

Mattress functioning

For the mattress is inflated by a system of pipes. The cells are alternately inflated which means the patient's body is support alternately by various support points. While one part of the body is supported by inflated cells, the other one is completely free from pressure.

Phases of pump functioning:

1 phase: Inflating all the cells

2 phase: Deflating the first section of cells

3 phase: Reinflating all the cells

4 phase: Deflating the second section of cells

The phases occurs in cycles every 10 - 12 minutes

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PROBLEM SOLVING

- Do not try to repair the pump, do not open it as it can lead to electric shock!
- The knob is in "ON" position but the pump does not work.
 - Please, check if the cable is plugged to electric supply and to the pump. If not, you should plug it.
 - Please, check if there is a voltage in the electric circuit. If there is not, find the cause of such situation.
 - Please, check if the cable is not damaged. If so, please contact the manufacturer.
- The pump works too slowly or the pressure is too low.
 - Please, check if the pressure regulator is set on an appropriate pressure. If not, please, regulate it.
 - Please, check if the pipes are correctly connected to the air outlet. If not, press them in such a way which prevents the air from escaping.
 - Please, check if the pipes are not folded or pressed. If so, please, unbend them.
 - Please, check if the cells are not damaged. If so, please, repair it using the reparation kit.
 - Please, check if the pump works correctly – if you can hear some strange sounds like hissing etc., please contact the manufacturer

CARE AND MAINTENANCE

- Disconnect the pump from electricity. Clean all the surfaces with a moist cloth. Before switching on it has to be dry.
- The mattress should be cleaned with water with soap. After cleaning, dry it carefully. The mattress mustn't be washed in high temperature, dried in a washing machine, bleached, chemically washed, ironed or pressed.
- The mattress may be disinfected using a product Incidin Plus, in accordance with the instructions for use of the preparation.

You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,
- Do not clean the exerciser with pressure or steam aggregates!

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Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Once a month	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.

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Repair

- In case of reparation, please contact the producer.

Customer service

- If you have any questions or need help, contact the producer.

Technical parametres

Pump:

Power supply: 220/240V 50Hz

Air output: 5 liter/min.

Pressure range: 4 – 14 kPa

Cycle time: 10 - 12 min

Power consumption: 7 W

Fuse: 1 A

Size: 28cm x 10cm x 10cm

Weight: 1,1 kg

Mattress:

Size after inflation : 200 cm x 90 cm x 6,5 cm

Weight: 2 kg

material: PVC (EN-71)

PVC thickness: 0,30 mm

colour: nude

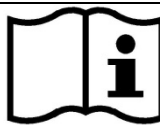
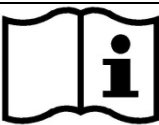
Permitted user weight  120 kg

Series no.: on the packing

Durability: 3 years from production date.

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The product is compliant with standards on electrical and electronic equipment. It should not be disposed with other waste from households. The user is responsible for providing the equipment at the end of its service life to the appropriate collection points, under penalty of sanctions provided by the law applicable in the matter of waste disposal. For further information on the collection schemes available can be obtained from the local waste disposal center.

10. Dane teleadresowe \ Contact details

mdh sp. z o.o.
ul. Maratońska 104,
94-007, Łódź, Polska
tel. +48 42 674 83 84
fax. +48 42 636 52 21
www.mdh.pl www.viteacare.com