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ANTI-BEDSORE

alternating pressure mattress

(tubular structure, including compressor)

Numer katalogowy / Catalogue number: VCM502

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Instructions for use

Wydanie/edition: EJ 09.2024

przejdź do

go to

VITEA
CARE

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English version on next page

Users with visual impairments can find the PDF files together with further information on our website:

<www.viteacare.com>.

☞ Contact your specialist dealer when required.

Alternatively users with visual impairments can have the documentation read out by a helper.

Table of contents

- Introduction 19**
- Product description 20**
 - Indications..... 20
 - Contraindications 20
- Safety of use 20**
 - Security check..... 22
 - Electromagnetic interference – EMI..... 22
- Use 23**
 - Operating principle..... 23
- Troubleshooting..... 24**
- Maintenance and repair 24**
 - Modifications..... 25
- Care and maintenance 25**
 - Maintenance activities..... 25
 - Cleaning..... 25
 - Disinfection..... 25
 - Reuse (multiple product usage) 26
 - Product life cycle..... 26
 - Disposal of the medical device 26
- Technical specifications 26**
- Symbols..... 27**
- Contact details..... 28**

Introduction

Thank you for choosing our product. We are deeply convinced that will meet your expectations. Please read this document before first use. This instructions is provided to give you necessary information on the safe use of the device. Proper use of the recommendations will significantly increase the durability and aesthetics of the product. We would like to point out that we are constantly trying to improve the quality of our products, which may result in minor changes that are not included in the guide. In case of any doubt or damage, please contact your distributor. Make sure that the document “warranty card” is completed, attached and handed over to the distributor, as this will enable you to provide the best service.

ATTENTION

Always observe basic precautions according to the information provided below regarding the safe use of the product. Carefully read the entire instructions before use. Please keep the instructions for future reference.

WARNING!

Improper use can lead to death, serious injury, unsafe conditions or product malfunction.

ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

SUGGESTIONS!

Follow the instructions below to keep your product in good working order.

If the product is used by another person, you must provide them with copies of these instructions for use.

These instructions for use relate to the design and specifications of the product at the time of publication. Due to design changes, some illustrations and photos in the manual may not correspond to the product you have purchased. We reserve the right to make design changes.

The manufacturer disclaims all liability for personal injury or property damage that may result from improper or unsafe use of this product.

Product description

Tubular variable pressure mattress with 17 cells placed in two lines, equipped with a cover. Made of PCV which enable fast and easy cleaning. Manual regulation of pressure in the mattress gives the possibility of its individual adjustment to the patient's weight.

Indications

Prevention and support of treatment of stage I and II bedsores. Due to its tubular structure, the mattress ensures an appropriate arrangement of body weight and correct ventilation (oxygenation and moisture diversion). The air insulation provides an optimal temperature and comfort of use. All the above-mentioned factors play crucial role for bedsores prophylaxis and treatment. The variable pressure mattresses do not eliminate all causes of bedsores but due to changing position of body support points, they reduce the risk of bedsores formation and development.

Contraindications

In case of undesirable skin changes, you should immediately consult your doctor or medical personnel.

Safety of use

WARNING!

Every "serious incident" that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State where the user resides.

WARNING!

Do not modify the product in any way; this will void the warranty.

WARNING!

Exceeding the maximum user weight will void the warranty. The manufacturer is not responsible for injury and damage if the maximum user weight is exceeded.

**WARNING!**

Please read these instructions for use before using the product and keep them for future reference.

- a. Permitted user weight 140 kg.
- b. Please remember that on the mattress should stay only the patient (other people on the mattress are not allowed).
- c. You are not allowed to do the physiotherapy treatment or rehabilitation on inflated mattress.
- d. The mattress should be protected against contact with body care cosmetics. It may cause permanent damage of the mattress.
- e. You must not use the mattress close to inflammable materials. It can lead to explosion!
- f. Always check if the pump is plugged into appropriate socket (220-230 V/50 Hz).
- g. The plug can not be removed from the socket when the pump is on.
- h. Always make sure the pump is in a dry place, remember that it cannot get wet.
- i. Do not touch the pump when it is wet. Always unplug the pump from electricity when it is wet.
- j. Never leave working pump unattended.
- k. Pay special attention if the pump is used in the presence of children or people with disabilities.
- l. Pump can be used according to its purpose only.
- m. Do not use it with other, unauthorized devices.
- n. Never use the pump with damaged or moist cable / plug. You should submit such a pump to a service point.
- o. The device should not be placed in a way that makes it difficult to disconnect it from the electrical outlet.
- p. The cable should be kept away from the hot surfaces.
- q. Never leave the pump in a place where the cable or tubes can be dangerous for people who are not familiar with the device.
- r. Always pull the plug not the cable while disconnecting.
- s. Never block the pump filter. The filter must be free from any impurities such as hair, fur etc.
- t. Always consult your doctor before using the mattress.
- u. Using the isopropyl alcohol or dissolvent may damage the pump.
- v. Please, make sure that the pump is completely dry before setting on.
- w. The mattress can be folded while storing. Before folding, check if it is completely dry which protects it against mold. The mattress should be stored in a dry place.
- x. Do not put the mattress close to the fire.
- y. Keep the mattress away from sharp objects.
- z. Don't put the mattress directly on springs or frame with sharp edges.
- aa. Pay attention to skin changes. If necessary, consult your doctor or medical person.
- ab. Tubes can not be bent.
- ac. Never use the mattress deflated.
- ad. When moving the bed, remove the plug from the outlet.

ae. Be aware that the surface temperature of the parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts. It may cause burns or frostbite parts of the body.

Maximum user weight 140 kg.

WARNING!

To avoid the risk of electric shock, the device must only be connected to a protective earthed mains supply.

Security check

ATTENTION!

For exact instructions on proper use and optimal pressure, consult your doctor. The set creates a “continuous motion” effect, which provides pressure changes to different areas of the body, and also provides continuous muscle massage and improves blood flow. This makes the mattress an excellent tool for preventing bedsores. However, it does not completely exempt from making changes in the patient’s position.

Electromagnetic interference – EMI

Interference:

Devices such as cell phones, computers, radio devices and other sources of strong electromagnetic fields can cause interference with the pump.

Recommendations and safety of use:

Avoid placing the pump in the immediate vicinity of devices that emit electromagnetic fields. In case of abnormal operation of the pump (e.g., changes in operating sound, intermittent operation), immediately disconnect the device from the power supply and contact an authorized dealer or manufacturer’s service center.

Use

- Place the mattress on a standard bed so that the air intake would be located on the side of patient's legs. Put the end flaps under the bed mattress.
- Check if the switch is in the „off“ position.
- Connect the pump to electricity.
- Connect the pipes to the pump – insert them into the air outlet located on both sides of pump and on the mattress.
- Place a pump on hard surface or hang it on hinges. The place where the pump will stand must be clean and dry. Dust, hair or other impurity may block the filter.
- Please, check if the pipes are not folded or pressed.
- Set the knob into “ON” position.
- Set the pressure regulator on maximum. It will shorten the time of pumping which usually takes 15-20 min.
- After inflating the mattress, you should choose the pressure according to the doctor's recommendation.
- Cover the mattress with a sheet, do not stretch.
- Place a patient on the mattress.

End of use:

- Turn off the pump - turn the knob to the (OFF) position.
- Disconnect the tubes from the air outlet to allow the remaining air in the mattress, to escape.

Operating principle

Air is pumped into the mattress via pipes. The chambers of the mattress are alternately inflated with air by the air supply. The patient's body is supported alternately at different points, i.e., part of the body is supported by the currently inflated chambers, and the rest of the body, which is above the pumped-out chambers, is completely free of pressure.

Phases of pump functioning:


Phase 1: filling all cells of the mattress with air.

Phase 2: pumping the air out of the first section of cells.

Phase 3: re-filling of all cells.

Phase 4: deflating the second section of cells.

The phases occurs in cycles every 10 - 12 minutes.

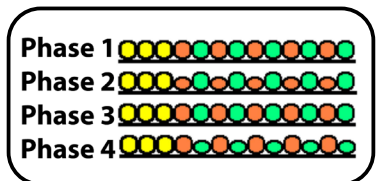Troubleshooting

WARNING!

Do not try to repair the pump, do not open it, as it can lead to electric shock!

Type of fault	Solution
The pump does not work even if the power button is in the "ON" position.	Is the cable connected to the mains and also to the pump? If not, it should be connected.
	Is there voltage in the electrical network? If not, determine the reason for the voltage failure.
	Is the cable not damaged? If so, return to the manufacturer.
The pump works too slowly or the pressure is too low.	Please, check if the pressure regulator is set on an appropriate pressure. If not, please, regulate it.
	Check if the pipes are correctly connected to the air outlet. If not, press them in such a way which prevents the air from escaping.
	Check if the pipes are not folded or pressed. If so, please, unbend them.
	Check if the cells are not damaged. If so, contact the distributor.
	Check if the pump works correctly – if you can hear some strange sounds like hissing etc., please contact the distributor or manufacturer service.

Maintenance and repair

Inspection should be performed regularly, and should be preceded by a complete cleaning of the product (so as to reveal any hidden damage). If repairs are needed or if you have any questions about the product, contact your distributor.

Frequency	Object of control	Comments
Before first use	Checking the technical condition and all the elements within the packaging.	Check that no components have no visible mechanical damage.
Once a month	Cleanliness, general condition.	In case of resulting mechanical damage, contact the distributor immediately.

Modifications

Do not modify, remove, disable or add any parts, components or features of the mattress and pump included in the device.

Care and maintenance

ATTENTION!

Insufficient or abandoned care and maintenance lead to a limitation of the manufacturer's liability.

Maintenance activities

- Disconnect the pump from the mains. Wipe all accessible surfaces with a damp cloth. The pump must dry before turning it on.
- Clean the mattress with soapy water. After washing, it should be thoroughly dried.
- The mattress must not be boiled, tumble dried, bleached, dry cleaned, ironed, or squeezed.
- The mattress can be disinfected using, for example, "Incidin Plus", according to the instructions for use of the product.

Cleaning

To ensure long and efficient operation, the product should be subject to basic maintenance activities and inspections carried out by the service centre. The inspection should be carried out once a year. In case of any doubts or damage, contact a service centre that has all the required qualifications to solve the issue.

WARNING!

Follow the safety instructions provided by the manufacturer of the cleaning and disinfecting products. Failure to observe this requirement may cause skin irritation or early damage to the device.

Do not use:

- solvents,
- toilet cleaners,
- agents with chlorine content,
- sharp brushes and hard objects,
- corrosive cleaning agents.

Do not clean the product with pressure or steam units!

Disinfection

If the product is used by many persons (e.g., at a nursing home), use commercial disinfectants. Before disinfecting the device, clean the outer surface. Only use proven and approved disinfectants.

To obtain information about proven and approved disinfectants and disinfection methods, contact the local sanitary competent authority.

**WARNING!**

Disinfectants may damage the outer surface, reducing the long-term functionality of the components. Always observe the instructions concerning the use of the products specified by the manufacturer.

Reuse (multiple product usage)

The product is suitable for reuse. The number of times it can be reused depends on how often and in which way the product is used. Before reuse, thoroughly clean and disinfect the product and check its safety features. If necessary, take repair action.

Product life cycle

The expected lifetime of a medical device such as an anti-Bedsore mattress with pump depends on the frequency of use, the environment in which it is used, and its care. By using replacement parts and undergoing regular maintenance, the life of the device can be extended.

Disposal of the medical device

The mattress or pump must be disposed of in accordance with applicable local and national regulations. For information on proper disposal of packaging, metal or plastic components, electronics, batteries, neoprene, silicone and polyurethane materials, contact your local waste disposal company or authorized supplier.

Technical specifications

Pump	
Power supply	220/230V 50/60Hz
Air output	4,5 litres/min.
Pressure range	40 mmHg – 100 mmHg
Cycle time	10 - 12 min.
Power consumption	7 W
Fuse:	1,5 A
Dimensions	240 x 120 x 115 mm
Weight	1,3 kg

Mattress	
Dimensions when inflated	1900 x 850 x 110 mm
Weight	3,4 kg
Number and type of cells	17 cylindrical cells
Material	PCV (EN-71)
PCV thickness:	0,32 mm
Colour	Blue
Composition	PCV

Description of the operating environment	  
Storage and operating temperature	From +5°C to +40°C
Humidity for storage and use	From 30% to 70 %

Symbols

The symbols used below on the product are used to identify warnings, prohibitions and mandatory actions. Read them all and understand their meaning.

	Medical device
	Date of manufacture
	Manufacturer
	Distributor
	Conformité Européene (compliance with the European Community)
	Catalog number (to re-order)
	Serial number
	Lot code
	Unique Device Identifier
	Refer to the instructions for use
	Follow the instructions for use
	Warning
	Protect from humidity

	Keep away from sunlight
	Permitted temperature
	For indoor
	Maximum user weight
	The product is designed to be transported by car
	<p>SEPARATE COLLECTION FOR WASTE OF ELECTRICAL AND ELECTRONIC EQUIPMENT.</p> <p>Do not dispose of battery in municipal waste. The symbol indicates separate collection for battery is required. Such products should be handed over to collection sites for this type of waste for proper disposal, renovation or recycling. Alternatively, in some countries of the European Union or other European countries you may return your products to your local retailer when purchasing a similar new product. By correct disposal of this product, you will help to conserve valuable natural resources and help prevent potential negative effects on the environment and human health, which could be caused by incorrect disposal of waste. Further information is available from your local authority or waste collection point. Penalties may apply in accordance with local regulations in the event of improper disposal of this product. For entities in the European Union countries. If you want to dispose of an electrical or electronic device, ask your dealer or supplier for the necessary information.</p> <p>Liquidation in countries outside the European Union: This symbol is valid in the European Union. If you want to dispose of this device, obtain the necessary information on proper disposal at local authorities or from your dealer.</p>

Contact details

Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

www.mdh.pl

www.viteacare.com

We reserve the right to make technical changes due to technical progress.