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TITANUM

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Wydanie EJ 06.2022

INSTRUCTIONS FOR USE

**TITANUM ALUMINIUM WHEELCHAIR
VCWK9AT**

TITANUM STAB

**SPECIAL WHEELCHAIR STABILIZING BACK AND HEAD
VCWK7AT**

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PLEASE READ CAREFULLY BEFORE USING
WHOLE OF INSTRUCTIONS

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Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual was made in order to provide the necessary information on the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to distinguish that to date we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual.

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1. Armrest
2. Seat
3. Parking brake lever
4. Footrest lever

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5. Footrest frame
6. Front wheel
7. Safety belt for feet
8. Footrest platform
9. Higher backrest (*depending on the equipment*)
10. Handle holder
11. Backrest
12. Rear wheels
13. Metal handrim
14. Quick connect handle
15. Front wheel fork

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Elements that should be inside the packaging:

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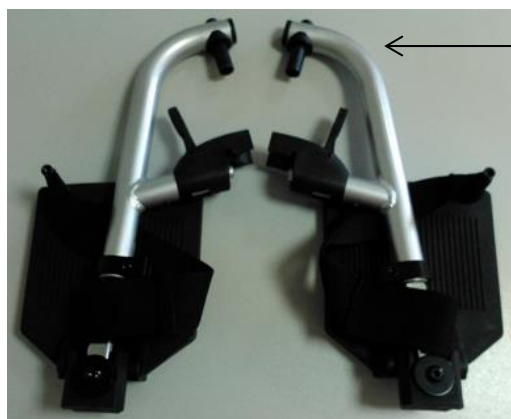
The wheelchair is used in order to commute by people with disabilities e.g. after the spinal cord damages, cerebral palsy or various lower limb injuries, neuromuscular diseases, post-traumatic cranial cerebral or for geriatric patients.

It is also intended for people who want to actively and comfortably participate in everyday life.

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Contraindications to a sitting position. Conditions that require stabilization of the upper spine and head. Patients with mental disorders should not be left unattended in the stroller.

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- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above.
- Ensure that all components have no visible damage. If any doubts, please contact the manufacturer.
- Do not use the wheelchair until all parts are properly connected.

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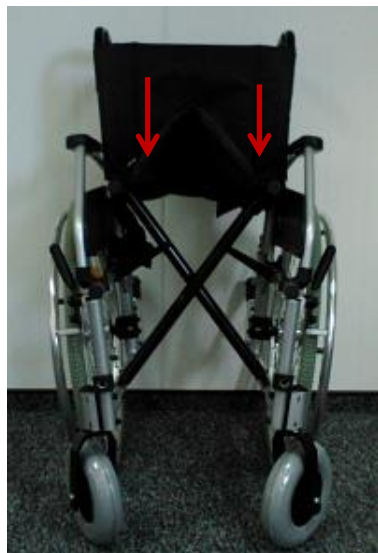
UNFOLDING THE WHEELCHAIR.

- The wheelchair must be placed with rear wheels in front to You
- Unfold the handle holder towards the outside (Illustration 7.).
- Then place wheelchair with front wheels in front of you
- Press seat frames, until complete unfolding (Illustration 8., 9.).

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- To install footrests, hang fixing bracket footrest on the hanger of the wheelchair frame (Illustration 10., 11.) Then rotate it into wheelchair front until it locks ("click" sound).
- Make sure that the footrests are blocked (Illustration 11.).
- Unfold the footrests platform.

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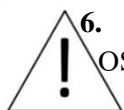
transporting on stairs, just like on stretcher or by pulling or lowering the patient. The construction of the wheelchair and its joining elements are not designed for such a large overload if it is used in this way. This can cause weakness or damage of the entire structure the wheelchair and possible accidents!

- i) The wheelchair is designed to use indoors and outdoors only on hard and even surfaces.
- j) Using the wheelchair on an unstable surface (covered with ice, wet, greasy, sandy) is unacceptable- it causes risk of falling,
- k) You can only use the wheelchair on surfaces where all four wheels are touching the ground.
- l) To overcome architectural barriers (steep or long climbs, thresholds or curbs, etc.) help of caretaker is needed.
- m) If the user does not have sufficient experience in the wheelchair use, should ask caretaker for help with overcoming obstacles.
- n) Do not leave the wheelchair with the user close to open sources of fire.
- o) Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of wheelchair. It may cause burns or frostbite parts of the body.
- p) In order to transfer wheelchair, grab the frame. Never transfer the wheelchair by grabbing the armrests or footrest.
- q) Never use the wheelchair as a seat for the transportation of people in cars, buses or any motor vehicles.
- r) The wheelchair is designed for people up to 125 kg.
- s) Wheelchair must be easy to fold and to unfold.

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In case of problems with folding, you should re-read the instruction or contact the manufacturer.

- t) Keep caution while moving on sloping surfaces. The maximum allowable slope angle up and down is 5%.
- u) Brakes cannot be slow down the wheelchair during ride, You should use them only to prevent the unintended movements.
- v) Both brakes should be properly adjusted. Incorrect setting one of the brakes, results with blocking only one rear wheel. On large slopes, it can cause difficulty to control the rotation the wheelchair around the wheel stopped.
- w) Regularly check the tire pressure. With too low tire pressure than recommended, the brake performance can be significantly reduced!
- x) While reaching objects, avoid excessive leaning out the wheelchair because it can cause a fall (to the side, front, back).
- y) In order to avoid accidental rolling off the wheelchair, always put on the brakes when the stroller is stopped, when patient get in, get off, while changing the position.
- z) Do not leave the patient in the wheelchair (even with turned on both brakes) on the descents, ramps, landings, etc.
- aa) It is necessary to check after each wheel mount if wheel axle connector lock works properly. While mounting should hear "click" of the latch. Pull strongly the wheel to check if it is installed correctly.
- bb) Using the parking brake while driving may result with an accident with all its consequences.
- cc) After each wheelchair adjustment, it is always necessary to check the correct operation of the brake.
- dd) Oil, grease, water or other chemicals, may cause that brakes may have reduced effectiveness.

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ee) Strength of the breaking intensity may be reduced after long exploitation of the wheelchair. You are allowed to adjust breaks yourself. In order to do that you must loosen the screws locking the position of the break, move the mechanism of the break towards the wheel so that the wheel will not rotate when break will be on again (try the adjustment with the user sitting on the wheelchair). After final adjustment secure back the mechanism with the screws.

- ff) It is extremely important to keep all wheelchairs components clean in order to maintain the product parameters.
- gg) Storing the wheelchair in areas with very high humidity (baths, laundry, etc.) or in the open air, also the lack of hygiene causes rapid loss of operating parameters and aesthetic of the product.
- hh) It is unacceptable to use the wheelchair after discovering any visible or detectable damages or when the wear of some parts may cause an accident. All the repairs need to be performed by a skilled personnel and during the warranty period only by the manufacturer's service.
- ii) In case of wheelchair fitted with pumped wheels You should not exceed the maximum permissible inflation pressure of the wheels, the value is indicated on the tire wheels.
- jj) During transport (e.g. In the trunk), do not burden stroller, it may weaken or damage its structure.
- kk) Keep out of the reach of children under five years of age.
- ll) Should not insert fingers between the wheelchair components - risk of injury or entrapment.
- mm) Keep extreme caution while unfolding / folding the wheelchair. Lack of attention may result in finger entrapment:

- when folding the wheelchair
- in the area moving parts.
- in the area of brakes.

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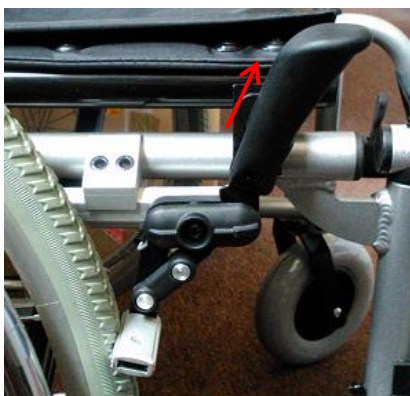
SITTING ON THE WHEELCHAIR:

- Place the wheelchair in a position that will make it easier to transport a patient (by the bed, chair, close to lifting distance)
- Block both brakes of wheelchair, fold or remove the footrests.
- Pull the brake lever up and fold it (Illustration 22., 23., 24.).
- Lift the front of the armrest up pressing the lever of the armrest (Illustration 25.).
- The armrest can be dismantled. To do this, after raising the front part of the armrest, turn the armrest block in to the middle part of the wheelchair (Illustration 27.).
- Place the patient in the wheelchair.
- Mount the armrest, proceeding in the reverse order when dismantling.
- Unfold the brake lever.
- Install and open the footrests and put on it patient's feet.
- Place the patient on the wheelchair so that the patient is in upright sitting position (knees should be bent at an angle of $\sim 90^\circ$).
- For patient comfort, can change the height of the armrest.
- To do this, press the armrest button and pull armrest upwards in order to the armrests be located above or press the armrest button and slightly push the armrest downwards in order to the armrest be located down.

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PUSHING THE WHEELCHAIR BY CARETAKER

- Before use, make sure that no part of the patient's body or his / hers clothing is not exposed to hook with the moving parts of the wheelchair.
- If patient can not sit on one's own, wheelchair should be equipped with stabilizing belts (belts, which are in standard equipment to protect the patient from sliding off the seat).
- Wheelchair should be pushed with both hands.

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MOVING ON SLOPES

- While moving on slopes, use caution and reduce your speed.

- Do not move on too high slopes. The maximum allowable slope angle up and down is 5%.

- Consider the possibility of assistance from the caretaker. If the person on the wheelchair did not have enough force to control the wheelchair, the brakes must be on.
- To prevent tipping over while moving along the sloping surface, You should be slightly lean forward to move your centre of gravity forward.

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GOING DOWN OF THE THRESHOLD OR CURBS

➤ **Ahead without the help of a caretaker**

- Experienced wheelchair user can drive on his own from the threshold or curbs.
- With a low curb, you can drive by moving forward. Make sure that the footrests platform does not touch the ground.
- Please keep balance on the rear wheels to reduce the pressure on the front wheels.
- Then cross the curb.

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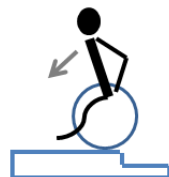
➤ **Ahead with the help of a caretaker**

- Higher curbs can be overcome with caretaker help.
- Caretaker should slightly tilt the wheelchair back.
- Cross the curb moving on the rear wheels.
- Then the wheelchair must be paced on all four wheels.

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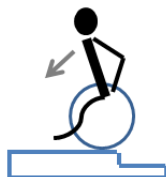
➤ **Backwards without the help of a caregiver**

- Experienced wheelchair user can cross on his own the threshold or curbs.
- Turn the wheelchair in such a way that both rear wheels were turned toward the curb.
- Lean forward to move the centre of gravity in forward.
- Come closer with the wheelchair to the edge of the curb.
- Then, by using the handrim in a controlled manner, user must slide the wheelchair from the curb.

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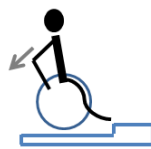
➤ Ahead with the help of a caregiver

- Experienced wheelchair user can drive on his own from the threshold or curbs.
- Caregiver should slightly tilt the wheelchair back.
- Wheelchair user should keep balance on the rear wheels to reduce pressure on the front wheels.
- Go with front wheels on the curb and then put the rear wheels over the curb.

➤ Higher curbs can be crossed going backwards with the help of a caregiver

- Turn the wheelchair in such a way that both rear wheels are turned toward the curb.
- Wheelchair user should keep balance on the rear wheels to reduce the pressure on the front wheels.
- Caregiver should pull the wheelchair over a curb.
- Wheelchair user should return to its normal position on the wheelchair.

➤ Ahead without the help of a caregiver.

- Experienced wheelchair user can go down on his own from the threshold or curb.
- Place the front wheels to the edge of threshold or curb.

- It is important to make sure that the footrests do not touch the threshold or curb.
- Now, can go with the front wheels on the curb.
- Then, may enter the rear wheels on to the curb.

TRANSPORT IN THE CAR

- In order to make the wheelchair easy to transport, should fold footrest platform and remove it.

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- Then pull up the seat upholstery, it will fold a wheelchair.
- The wheelchair is equipped with system of quick release rear wheels.
- To install or remove the rear wheels, press the system of quick release rear wheels button (Illustration 29), pull or push a wheel.
- The wheelchair have to moved by holding the frame elements only. Never hold the wheelchair by the armrests or footrests.

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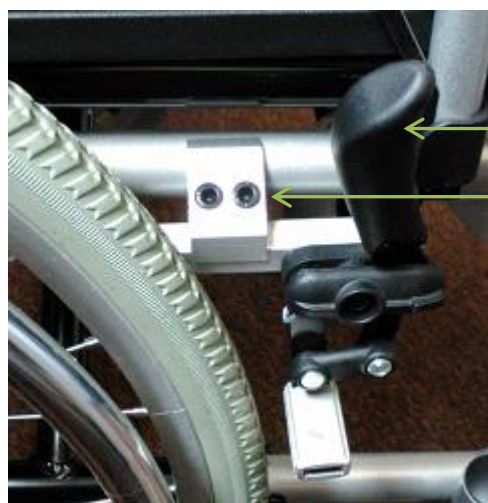
- Illustration 30. shows the brake mechanism.
- To activate the brake, pull the brake lever towards the front wheel to the point where the brake is tightly touching to the rear wheel
- Make sure that the brake lever does not automatically switch off when trying to move the wheelchair.
- To disable the brake, pull the brake lever toward the rear wheel.
- To reposition the brake mechanism, loosen two screws by using Allen wrench 5 mm (Illustration 30.).
- Move the entire mechanism in the direction of the rear wheel and tighten the screws.
- When you turn on the brake, the wheel should not rotate (test the correctness of adjustment carried out in a wheelchair with a patient).

FOOTREST HEIGHT ADJUSTMENT

- Footrest could be adjustable in length.
- To set the desired length of the footrest, should loosen the footrest height adjustment screw by using allen wrench 5 mm (Illustration 31.).
- Depending on the needs push in or pull out the footrest platform located in the frame footrest.
- After setting the proper length of the footrest, tighten the adjustment screw.

FOOTREST ANGLE ADJUSTMENT

- To set desired footrest platform position should loosen the footrest plate adjustment screw by using allen wrench 5 mm (Illustration 31.).
- Set the desired position of the footrest platforms.
- After choosing the footrest platform correct position, tighten the footrest adjustment screw.

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ANTI-FLIP WHEELS

- The wheelchair is equipped with anti flip wheels, which play an important role when a person sitting in a wheelchair has a problem with the stabilization of the body.
- Before using the wheelchair, set the anti-flip wheels.
- To do this, press the anti-flip wheel lock located in the lower part of the anti-flip wheel handle
- Turn anti-flip wheel towards the ground (Illustration 32., 33.).
- This procedure must be performed on both anti-flip wheels.

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WHEELCHAIR HEIGHT

- Prior to change height of the wheelchair, remove the rear wheel.
- Holding the nut with wrench 10mm, unscrew two screws by using allen wrench 5mm located from the inter side hub fixing (Illustration 34.).
- Then select appropriate location of the hub fixing and by holding the nuts tighten the previously removed screws.
- In order to change front wheel position, hold the nut with wrench 13mm, unscrew the front wheel screw by using 5mm allen wrench (Illustration 35.).
- Then select front wheel appropriate location and by holding the nut tighten the previously removed screw.

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SHIFTING CENTER OF GRAVITY

- In order to shift center of gravity hold the 10mm nut with wrench, then unscrew two screws located from the inner side hub fixing by using 5mm allen wrench (Illustration 34.).
- Then turn the hub fixing as shown in Illustration 36.
- Hold the nuts and tighten the previously

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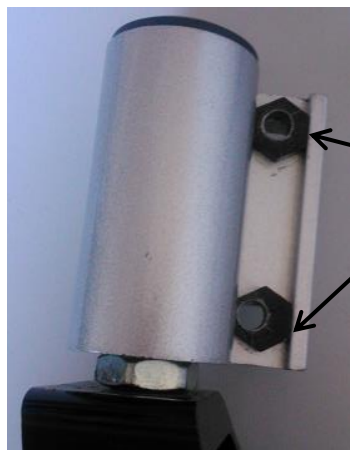
POMMEL ADJUSTMENT OF THE WHEEL FORK

- In order to adjust fork pommel should holding the nut with wrench 10mm, unscrew two mounting screw of fork pommel by using wrench 10 mm mounting (Illustration 35.).
- Select the appropriate arrangement of shims (Illustration 37.).
- Hold the nuts and tighten the previously removed screws.

All of the following adjustment should be made by qualified personnel:


- Pommel adjustment of front wheel fork
- Shift center of gravity
- Wheelchair height

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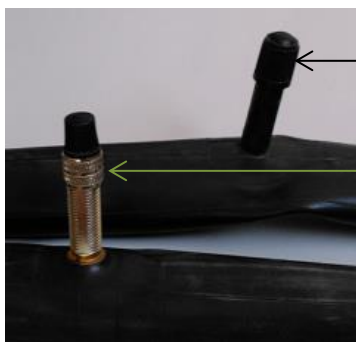
Zawór Schradera
/ Schrader valve

Zawór Presta /
Prest vale

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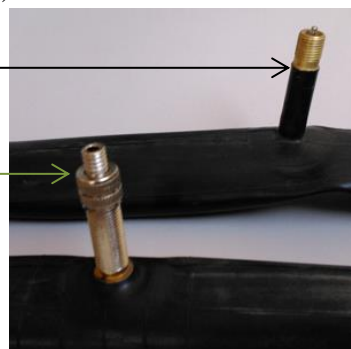PUMPING TYRE (It applies to stroller fitted with wheels pumped)

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- Before pumping the tires, identify the type of valve which is located on the tire. This may be one of the two valves shown in Illustration 38 and 39.
- Presta valve also has an additional small top, which must be loosened before pumping.
- After identifying the valve, prepare the pump designed to valve type.
- Most of the new pumps are equipped to service both Prest valve and Schrader. Many of them let to reverse the tip. Smaller hole is Prest valve, Schrader valve greater. If the pump does not support Prest, you will need to buy yourself the "adapter Prest", which is available at any bike store.
- At this point, if the wheel is provided with a valve Prest, it will be necessary to unscrew the top located on the vent. It is a small top, which can be unscrewed with fingers. When doing this, the top should move upwards. Then valve is considered open.

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CHANGE OF TYRES (It applies to stroller fitted with wheels pumped)

Before beginning the change of tyres it is necessary to get acquainted with the following notes.

First, it is necessary to release air from an inner tube and then by pressing a tyre in the direction of the centre of a rim to loosen a tyre-rim contact. Using a special spoon it is necessary to pull a tyre border out of the rim what will give the possibility to shift the whole wall of the tyre outside. After this operation pulling the inner tube out is very natural.

Prior to the installation of a new inner tube attention should be devoted to ensuring that:

- a bottom band is not damaged and has not foreign bodies,
- external walls of the rim are smooth, without foreign insertions and other damages.

The bottom band and rim should be precisely cleaned if need be.

To install the tyres it is necessary to put a hole of the bottom band over a valve, and then put together to a seating in the rim. Keeping such a position, put the bottom band on the whole perimeter of the rim, paying attention to ensuring that the band is not twisted or damaged and it covers spoke tighteners thoroughly.

Pulling the tyre wall away from the rim, insert the valve to the seating in the rim and slightly pump the inner tube that it achieves a round shape and then to put the inner tube in the rim.

To be sure, it is necessary to check if the inner tube is not twisted (in case of deviation or deformation, pump air additionally). Manually put the tyre wall onto the rim, starting from an opposite side of the valve and ending by it.

It is necessary to check on both sides of the wheel if the inner tube is not pressed by the tyre and if the tyre regularly adheres to the rim. The valve should be slightly pressed inside and then repressed outside that it sets well in the seating.

To check that the tyre installation is correct it is necessary to pump such air that it is possible to press a thumb to the bottom of the rim. Then, check if the tyre adheres evenly to the rim on both sides and if it is centrally arranged. If such defects occur it is necessary to release air from the wheel and correct the setting of the tyres. When an opinion is positive it is necessary to pump the wheel to the maximum pressure value and put a cap onto the valve end.

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PRESSURE IN WHEELS AND BRAKING POWER

- Pressure in wheels should be select to the user weight and then with assistance caregiver check the operation of the parking brakes (wheelchair with user facing toward the fall, can not roll on a plane with a slope of 15%.)
- After all the adjustments, be sure to thoroughly check all connections.

COMMENTS

- Using improper tools or executing incorrect operation can cause serial damages of the rim or tyres.
- In case of plastic rims the bottom band is not used.
- Pay attention to ensuring that during the installation no foreign bodies get into space between a rim and bottom band what can damage a detail or injure a hand.
- Observe proper tire inflation level with the information provided by the manufacturer on the sides of the tire.
- Use a pump with a pressure gauge or manual pump cycling.

EVERYDAY USE AND MAINTANCE

- Clean the upholstery and frame with a moist cloth, without the addition of detergents and other similar chemicals.
- Systematically remove the dirt from the movable elements of the wheelchair (screws, resistance mechanisms).
- Keep clean all wheelchair elements for to save functional parameters of the product.
- Storing in rooms of very high humidity (baths, laundries etc.) or in the open air, in the combination with the lack of hygiene will cause the fast loss of functional and aesthetic parameters.
- Do not expose the wheelchair on to extreme weather conditions (rain, snow very strong sun), because certain parts can change colour.

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You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the exerciser with pressure or steam aggregates!

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TECHNICAL REVIEW

The review should be carried out regularly, it should be preceded by thorough cleaning the wheelchair (so as to reveal any hidden damage):

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Frequency	Object of control.	Comments.
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Before each use	Control of wheels, brakes.	The brakes can worn out, check the condition of the brakes before each use. You should also check the condition of wheels and clean them from dust and sand.
Every two weeks	Screw and nut control regarding their correct fixing.	Make sure if all the screws and top are fixed tightly.
Every two months	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.
Once a month	Wheel axles.	Wheel axles must be conserved, seting between the axle and the hub a few drops of lubricant preservative. In addition, make test each wheel. Raise the wheelchair so that the tested wheel was located above the ground, the wheel should rotate freely but without backlash on axle.
Once a month	State connections, threaded, soldered, the system of quick release rear wheels.	Pay attention to the cracks on frame material in connections threaded, soldered, the system of quick release rear wheels.
Once a month	Condition of the tires.	Make sure that the tire tread is not worn and that the tires have no visible signs of damage.

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Repair

- In case of reparation, please contact the producer.

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Customer service

- If you have any questions or need help, contact the producer.

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