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**English version** on next page

# Table of contents

<b>Introduction from mdh Sp. z o.o.....</b>	<b>27</b>
<b>Product description.....</b>	<b>28</b>
<b>Indications.....</b>	<b>29</b>
<b>Contraindications .....</b>	<b>29</b>
<b>Usage .....</b>	<b>29</b>
<b>Usage safety .....</b>	<b>29</b>
Electric circuit diagram .....	31
Electrical specifications.....	31
<b>Electric wheelchair function controlled by seated user .....</b>	<b>32</b>
<b>Attendant controlled electric wheelchair function .....</b>	<b>34</b>
<b>Wheelchair position calibration .....</b>	<b>37</b>
<b>Unfolding the wheelchair and charging the battery .....</b>	<b>38</b>
Preparing the wheelchair for travel.....	38
Operation and adjustment of the wheelchair accessories .....	39
Handbrake support.....	39
Battery charging.....	40
<b>Error codes .....</b>	<b>40</b>
<b>Technical parameters .....</b>	<b>42</b>
<b>Maintenance and service activities.....</b>	<b>43</b>
Disinfection .....	43
Storage.....	44
Technical review .....	44
Repair .....	44
<b>Symbols .....</b>	<b>45</b>
<b>Contact details .....</b>	<b>47</b>

## Introduction from mdh Sp. z o.o.

The wheelchair was constructed in accordance with the latest technologies. Its design allows maximum adaptation to the user's needs. It combines a solid and durable construction, modern electric controls and safety systems. Before using the wheelchair for the first time, read this instructions for use carefully and thoroughly. If anything in the document seems unclear to you, or if you have any doubts or questions regarding the use of the wheelchair, please contact your dealer. For safety and to keep your wheelchair in good working order, have it inspected regularly. In the event of any doubts or damage, please contact a service centre that is qualified to deal with your problem. Make sure to fill in the enclosed warranty form and submit it to your dealer, as this will ensure the best possible service.

# ATTENTION

Always observe the basic safety precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Keep this manual for future reference.

### WARNING!

Improper use can lead to death, serious injury, unsafe conditions or product malfunction.

### ATTENTION!

Improper use may lead to personal injury and/or damage to the wheelchair.

### SUGGESTION!

Follow the instructions below to keep your wheelchair in good working condition.

## Product description

Nr	Opis elementu
1	Rear control module
2	Rear brake lever
3	Adjustable backrest
4	Locking height adjustment of the guidance handles
5	Height adjustment button for the guidance handles
6	Folding armrest button
7	Front control panel
8	Safety belt
9	Seat
10	Folding footrests
11	Travel bag (load capacity up to 5 kg)
12	Rear wheel with motor - 8" (200x50 mm)
13	Front wheel PU - 8" (200x30 mm)
14	Adjustment backrest belt lock knob

## Indications

The product is intended for disabled people with motor conditions to increase their mobility, when:

- limited or no muscle strength
- limited range of movement
- limited trunk and body stability
- other neurological disorders requiring support

## Contraindications

Contraindications to upright standing. Injuries of upper limbs which make it impossible supporting yourself in a wheelchair.

## Usage

The product is packaged in the assembled position. After removal from the package, please check that it contains all the parts as listed above. Make sure that all parts are free from visible mechanical damage.

If in doubt, contact the manufacturer. Do not use the wheelchair until all components are installed and the battery is charged. The estimated service life is 2 years.

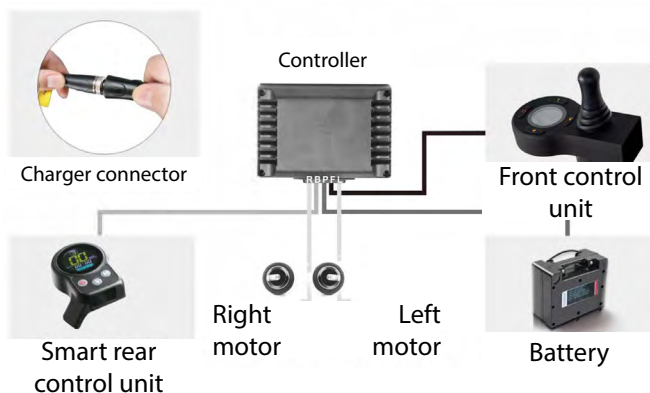
## Usage safety

- a. Before using the wheelchair, read this manual and keep it for future reference.
- b. Use of the wheelchair on unstable and irregular ground such as wet, icy, greasy, heavily dusty ground is not allowed due to the risk of falling.
- c. The wheelchair is intended for use on level surfaces with maximal inclination 6°.
- d. The wheelchair can drive down the slope with a maximum angle of 6°.
- e. It is forbidden to lean outside the cart when carrying people. There is a risk of accident.
- f. Before each use, perform a visual inspection of all structural components of the product including the stability of the product and its support frame.
- g. Do not use the wheelchair if you find any visible or perceptible damage (such as a crack in the frame) as this may result in an accident.
- h. The wheelchair is designed for users with a maximum weight of 100 kg.
- i. The wheelchair is used only for one user to support or sit on it. It is not allowed to stand on the wheelchair or stand with the whole body weight on the footrests.
- j. Do not leave the wheelchair in the direct contact with an open flame source.

- k. Please note that the temperature of the components and frame parts may increase when exposed to the sun or other heat source, while at low temperatures there is a danger of very high cooling of the metal parts of the wheelchair. This can cause burns or frostbite on parts of the body.
- l. All repairs must be performed by qualified personnel of the manufacturer.
- m. Hold the wheelchair with both hands simultaneously in the direction of movement when using the support function.
- n. Check that the locking device for folding and unfolding the wheelchair is working correctly.
- o. It is forbidden by manufacturer to modify the wheelchair by yourself, especially to fit additional wheels. This could lead to a fall and serious injuries. The manufacturer is not responsible for such actions.
- p. There is a possibility of jamming of fingers in the folding area of the wheelchair frame, footrests and armrests.
- q. The wheelchair may only be used on surfaces where all four wheels touch the ground.
- r. Keep out of reach of children under five years of age.
- s. The wheelchair is not intended for the transport of persons in motor vehicles.
- t. The wheelchair should be transported in a folded position. When transporting (e.g. in the boot), do not put any weight on the wheelchair, as this may affect its structure..
- u. The product is not intended for use by a person with a visual, reading or cognitive disability.
- v. Rollator does not have the option of fitting an additional emergency stop device. Rollator has electric and manual brakes.
- w. It is forbidden to move the wheelchair together with the person sitting in it. Move the wheelchair by grasping the frame structure. Never move the wheelchair by the moving parts or by the control module.
- x. Special attention should be paid when braking on a slope as the braking distance increases.

*The product is intended for use by people with disabilities and movement disorders. If you are not sure whether your health condition allows you to use the product, consult a healthcare professional.*

## Electric circuit diagram

## Electrical specifications

Motors: DC 24 V - 2 x 250 W

Batteries: type Li-Ion, capacity 6 Ah

Charger: AC 110-240 V, 50-60 Hz. maximum output current: 2 A.

Rear control unit: maximum output current 40A. Operating current 2-3 A.

**WARNING!**  
The wheelchair may interfere with equipment that emits electromagnetic fields.

**WARNING!**  
No modification of this equipment is allowed.  
- Protect from water jets  
- Do not put in water  
- Protect from dust

**WARNING!**  
No modification of this equipment is allowed.

## The product has the following functions:

- Electric wheelchair controlled by the seated user.
- Power wheelchair steered by a rear operator.
- Rollator with electric drive assist and emergency brake.
- Rollator with constant-speed electric drive.
- Manual rollator with possibility of transporting a seated person.

## Electric wheelchair function controlled by seated user

### Functions description of the front control unit

- Joystick: position **1** in the picture above, it can control the direction and speed of the wheelchair.
- Power switch **4**, when pressed and held for a longer time, it informs with a sound signal and activates the power supply. To turn off the power, press and hold the power switch for 3 seconds, which will be confirmed by a sound.
- Speed reduction and limitation - button **6**: pressing the button reduces the speed.

- Speed increase button **2**: When pressed, the maximum speed limit is increased.
- Sound signal: after pressing and holding the button **7** we activate "horn". The signal is only active while the button is held down. LCD display:
- Icon **5** shows the battery level. The range on one battery charge is 20 km under the following conditions of use: - load 100 kg - ambient temperature 25 ° C - continuous driving on a flat road at an average speed of 6 km / h. The traveled range is influenced by: everyday use of the wheelchair, during which we often turn, stop and resume driving or drive up hills for long distances will affect the actual range of the wheelchair. The range may be affected by external electromagnetic fields.
- Icon **3** shows the current gear (there are 5 speeds to choose from). For your safety, it is recommended to drive at a speed appropriate to your skills and environmental conditions.

## Electric wheelchair use - sitting user

*In order to keep the patient safe while sitting on and getting out of the wheelchair, it is recommended to use the assistance of a caregiver to release the handbrakes. Refer to the section "Handbrake support".*

1. Once in the seat, press the power switch **4**. During the procedure starting, it is forbidden to manipulate the joystick as the system can run irregularly or won't start.  
The LCD display shows the current gear number and remaining power  (as a percentage) until the battery is discharged. The system sets up a safe take-off strategy.
2. The speed of the wheelchair is regulated by buttons **2** and **6** on the controller. There are 5 levels of speed setting. It is better to use low speed when taking off. Do not change the speed level while driving. We recommend that novice users set a low speed to avoid accidents due to inadequate operating skills.
3. Push the joystick **1** slowly away from you to drive the wheelchair forward. Moving the joystick to the left enables the wheelchair to turn to the left. Likewise moving the joystick to the right turns the wheelchair to the right and pulling the joystick towards you will cause the wheelchair to reverse. After a simple understanding of these functions, the wheelchair can be driven freely.
4. If you need to use an acoustic signal, press the button on the controller marked with a horn symbol **7**.

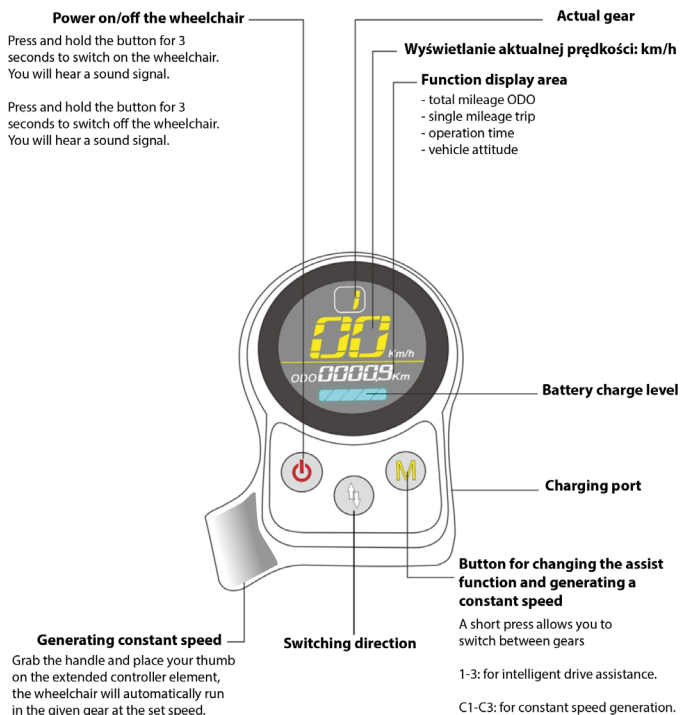
5. The battery charge status is shown on the front controller display as a percentage of energy consumption . It is recommended to regularly charge the battery and carefully monitor its condition so as to take into account the remaining distance that can be covered by the electric wheelchair in relation to the planned movement.
6. To turn off the wheelchair, press the power switch for a long time and observe the display of the power off prompt .

**WARNING:** Improper and / or inappropriate changes to the driving style may have a negative impact on the safety of the electric wheelchair and its user. Danger of an accident!

## Attendant controlled electric wheelchair function

### Description of the functions of the rear control module

- Switching the wheelchair on and off. A long press of the button  turns power on the wheelchair. This is confirmed by an acoustic signal. To turn off the wheelchair, press the  button for a longer time. An acoustic signal will confirm the wheelchair shutdown stage.
- Gear shifting: Short press of the button  shifts gears in the range 01-03    for assistance and C1-C3    for moving at a constant speed. Change of driving direction: short press  switches the direction in which the cart is moving.  Caution: Constant speed still works.
- Grip Detection: Grasp the handle within 10 cm of the controller. The icon will appear  during this time, handle detection is active. If the distance is greater than 10 cm, the wheelchair will be locked and the following message will appear on the display:  .

The functions of the rear control module have two options:

1. Intelligent Automatic drive assistance.
2. Generating constant speed drive.

## Intelligent automatic drive assistance - speed limit

Press the button  several times until the assist mode appears in the range 01-03    . Hold the handle and push the wheelchair. At this point, the automatic drive assist function will be powered on. When pushing, it will assist in forward movement, and when pulled, it will assist moving backwards. A higher gear indicates a higher speed beyond which, the restriction will be triggered. Going down hills will automatically generate a resistance. To make it easier to go up a hill, you can activate constant speed mode.

## Maximum speed limit

When the wheelchair, in assist mode, exceeds the maximum set speed, the system automatically intervenes to reduce it. When the speed drops to half of the set value, the intervention is removed.

## Constant speed drive

- grab the handle, the icon will appear on the display ,
- select a gear by pressing the button  from three levels   
- place your thumb on the constant speed drive generating area, the icon will appear  the wheelchair will automatically start moving at the speed of the selected gear.
- Shortly press the button  to change the direction of travel, the icon  on the controller flashes continuously, indicating you are driving backwards.

**WARNING:** The automatic emergency brake does not operate while generating a constant speed drive.

**WARNING:** When changing the direction of travel, keep a safe distance from the surrounding obstacles (buildings, walls, doors, etc.). Accident risk!

## Rollator with electric drive support and emergency brake.

This feature is for the user of the rear control module and uses the power assist shown above with the ability to rotate the backrest strap for comfortable and safe operation.

## Electrically driven rollator with constant speed function.

This is a user operation mode of the rear control module described above with the backrest belt rotated for comfort and safety. It is also possible to use a transport bag under the seat with a load capacity of up to 5 kg.

## Manual rollator function with the possibility of transporting a sitting person.

After losing the battery energy, the wheelchair can still function as a rollator with hand brakes. It is possible to transport a seated person weighing up to 100 kg.

**WARNING:** When used as a manual rollator without power applied, be careful as the electric brake will not work. Danger of an accident!

## Wheelchair position calibration

*If the error code E-024 (wheelchair position out of calibration) is present after starting, perform the following steps on the rear controller:*

### Method 1:

- When the wheelchair is placed horizontally, the steering controller must be attached in the correct installation position and must not be moved.
- Shortly press the power button .
- To go to the settings section, press the  and  buttons simultaneously.
- In the setting interface, short press the button  to switch the parameter to F4.
- Shortly press the button  which is confirmed by the flickering of the settings screen.
- Press the button  to select the parameter 1. Confirmation will be signaled by an acoustic message.
- The wheelchair begins to calibrate automatically. You should wait until the process is finished, which will be visible when the interface returns to its initial settings.

### Method 2:

- When the wheelchair is placed horizontally, the steering controller must be attached in the correct installation position and must not be moved.
- With the power off, press the button  and hold for 10 seconds until the display shows F4 and is confirmed with an audible message.
- The wheelchair begins to calibrate automatically. You should wait until the process is finished, which will be visible when the interface returns to its initial settings.

# Unfolding the wheelchair and charging the battery

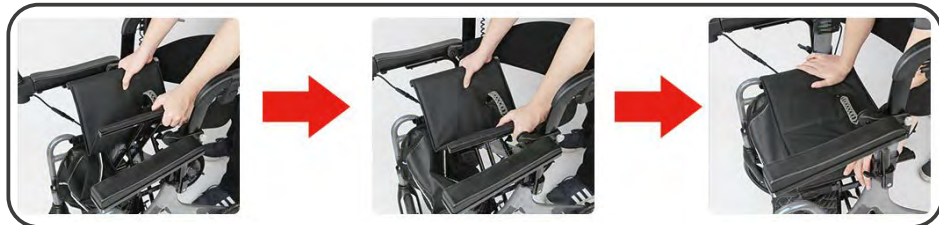
## Preparing the wheelchair for travel

1. Open and remove the wheelchair and all other items contained in the product packaging.
2. Place the wheelchair on an even and level surface.
3. Pull the handle with lock to unlock the wheelchair for unfolding.

handle with lock for folding

4. Press on the edges of the seat to slide the sides apart and unfold the wheelchair.

### WARNING!

Be especially careful when unfolding and folding the wheelchair, lack of attention may cause body parts to be jammed by moving and folding parts!

5. Press the fold button and tilt the armrest upward.

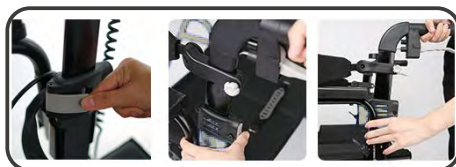
6. To fold the wheelchair, fold the platforms under the feet, then grasping the upper part of the footrest, raise and turn 90° on the outside of the wheelchair as shown below. Then, pull the lock handle in the seat and fold the wheelchair until you hear the lock "click", which locks the wheelchair from sliding the seat apart.

## Operation and adjustment of the wheelchair accessories


Release the lock and then press the button to adjust the height of the handles as shown in the picture.

The front control module can be adjusted by unscrewing the knob under the armrest.

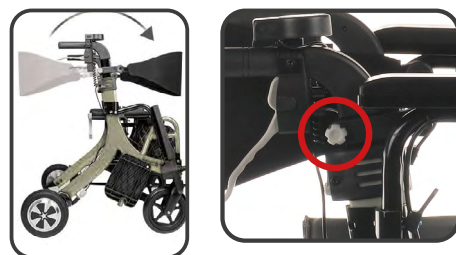The backrest belt can be adjusted by turning or adjusting the height thanks to the knobs fastening to the frame.

## Handbrake support

Parallel movement without tightening the handbrake lever.

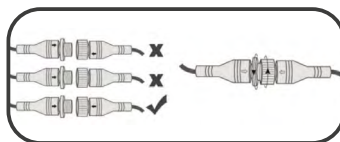
Braked movement by tightening the handbrake lever.

To lock the fixed wheelchair, pull the lever downwards.

## Battery charging

Proper connection of battery cables to the control module.

The charging port is located under the front control module.

The charger does not come with the wheelchair and is an external charger. To charge the battery, connect the charger to the charging socket on the front control module.

**WARNING:** When charging, ensure that there is adequate ventilation and heat removal through the battery and charger.

**WARNING:** To avoid the risk of electric shock, the device must only be connected to a supply mains with protective earth.

**WARNING:** Any "serious incident" related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

## Error codes


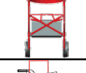
When a flashing symbol  appears, it means an error code. Corresponding number displayed in the rear control module display represents the malfunction.

Error codes	Cause of failure	Solution
E0002	Left engine malfunction	Contact the manufacturer's service.
E0004	Left engine malfunction	Contact the manufacturer's service.
E0008	Left motor current detection error	Contact the manufacturer's service to replace the controller.
E0010	Left motor current detection error	Contact the manufacturer's service to replace the controller.

Error codes	Cause of failure	Solution
E0020	Battery power detection error	Contact the manufacturer's service to replace the controller.
E0040	15V voltage error	Contact the manufacturer's service to replace the controller.
E0800	Communication error with the front control module	Contact the manufacturer's service to replace the controller.
E-021	Communication error with the front control module	Contact the manufacturer's service to replace the controller.
EE	Fall alarm	Automatic fault cancellation after releasing the braking resistance.
E-023	Wheelchair position sensor malfunction	Contact the manufacturer's service to replace the controller.
E-024	Wheelchair position out of calibration	Calibration should be performed in accordance with the instructions for use in the point: "Calibrating the position of the wheelchair".
E-025	Grip detection sensor malfunction	Contact the manufacturer's service to replace the controller.
E-026	Grab detection sensor not calibrated	Contact the manufacturer's service to replace the controller.
E-027	The touch panel is malfunctioning	Contact the manufacturer's service to replace the controller.
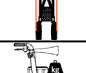
## Technical parameters

Dimensions of the unfolded wheelchair (length x width x height)	-	1005 x 642 x (813-900) mm
Dimensions of the folded wheelchair (length x width x height)	-	717 x 334 x 813 mm
Wheel axis distance		520 mm
Overall width		642 mm
Total height of the wheelchair		813-900 mm
Seat width		425 mm
Effective seat depth		400 mm
Seat height from the ground		525 mm
The width of the folded wheelchair		334 mm
Maximum load of the bag under the seat		5 kg
Maximum user weight		100 kg
Clearance of the footrest step	-	115 mm - 175 mm
Distance of the footrest step from the seat	-	390, 420, 450 mm
Power / voltage of electric motors	-	2 x 250 W / DC 24V
Battery type / capacity	-	Li-Ion / 5 Ah
Driving range	-	up to 10 Km
Maximum Speed - Front / Rear	-	6 Km/h / 2.5 Km/h
Battery charging time	-	4-5 hours
Weight of the wheelchair	-	18,8 kg
Type of frame	-	Aluminum
Cross bracket	-	Aluminum, black

Upholstery	-	Polyurethane + foam
Front wheels	2 pcs	8 inch (PU)
Fork	-	Plastic
The rear wheels	2 pcs	8 inch with motors
Handles	-	PVC, black

## Maintenance and service activities

*Before use, make sure that all elements do not have any visible mechanical damage. If in doubt, contact the manufacturer.*

1. Check and, if necessary, tighten bolts on the front wheels, drive wheels, brakes and other components. If any problems persist after adjusting the tire pressure, inform the manufacturer's service department.
  2. The battery should be charged at least once a week or when discharged if it occurs earlier.
  3. In the case of a damaged tire, contact the manufacturer's service center. Do not use the wheelchair with a damaged tire.
  4. Do not expose the wheelchair to intense moisture.
  5. In order to maintain the operational parameters of the product, it is extremely important to keep all its elements clean and to adapt to the method of its storage.
- Clean the wheelchair frames with a damp cloth.
  - Remove contamination on a regular basis.
  - In order to maintain the operational parameters of the product, it is extremely important to keep all its elements clean and to adapt to the method of its storage.
  - Storage in rooms with very high humidity (baths, laundries, etc.), or in the open air, combined with the lack of hygiene, will result in a rapid loss of functional and aesthetic parameters.
  - Do not expose the wheelchair to extreme weather conditions (rain, snow, very strong sun), as some elements may change its color.
  - The tires are made of a rubber compound, which can leave traces that are difficult to remove on some types of floor (e.g. plastic, wooden floors, parquet, carpets, lining). The manufacturer is not liable for damages caused by wear of the tires or as a result of the chemical processes involved in them.
  - Use only tires provided by the manufacturer - 8 inch PU.

## Disinfection

If the product is used by several people (e.g. in a nursing home), commercially available disinfectants should be used. Before disinfecting, clean the upholstery and handles. Only tested and approved disinfectants may be used for disinfection.

Information on recognized and approved disinfectants and disinfection methods is available from your national health authority.

**WARNING!**

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.

**Not allowed to use:**

- Solvents
- Toilet cleaners
- Sharp brushes and hard objects
- Agents containing chlorine
- Harsh cleaners

*Do not clean the wheelchair with pressure or steam generators!*

**Storage**

Store in a well-ventilated room with an ambient temperature ranging from -10°C to +45°C and humidity not exceeding 93%. Protect against gases, flammable and explosive substances. The wheelchair can be transported by car, plane, train or ship. When the wheelchair is not used for a long time, the condition of the battery should be checked and charged regularly (at least once a month) in order not to damage the cells.

**Technical review**

The inspection should be performed regularly and preceded by a thorough inspection cleaning the wheelchair (so as to reveal any hidden damage).

During the technical inspection:

Frequency	Subject of control	Remarks
Before the first assembly	Inspection of the technical condition of all elements included in the package.	Check that no elements have any visible mechanical damage.
Every two weeks	Fastening bolts, nuts and making sure they are correct mountings.	Make sure that all bolts and nuts are tightened securely.
Every two months	Cleanliness, general condition.	In the event of mechanical damage, please contact the manufacturer immediately.

**Repair**

Contact your distributor if repairs are required.

## Symbols

	Medical device
	Date of manufacture
	Manufacturer
	Distributor
	Conformité Européene (EU compliance)
	Catalog number (to be reordered)
	Serial number
	Batch code
	Unique Device Identifier
	Consult instructions for use
	Warning
	Type B applied part
	Protect against moisture
<b>IPX3</b>	Protection against splashing water

	Possibility of jamming fingers
	Permitted temperature
	For indoor use
	Maximum user weight
	Maximum safe inclination
	The product is designed to be transported by car
	The product is not intended to be used as a transport seat in a motor vehicle
	Do not stand on the footrest
	<p><b>Separate collection for waste of electrical and electronic equipment</b></p> <p>Do not dispose of battery in municipal waste. The symbol indicates separate collection for battery is required. Such products should be handed over to collection sites for this type of waste for proper disposal, renovation or recycling. Alternatively, in some countries of the European Union or other European countries you may return your products to your local retailer when purchasing a similar new product. By correct disposal of this product, you will help to conserve valuable natural resources and help prevent potential negative effects on the environment and human health, which could be caused by incorrect disposal of waste. Further information is available from your local authority or waste collection point. Penalties may apply in accordance with local regulations in the event of improper disposal of this product. For entities in the European Union countries. If you want to dispose of an electrical or electronic device, ask your dealer or supplier for the necessary information.</p> <p><b>Liquidation in countries outside the European Union:</b> This symbol is valid in the European Union. If you want to dispose of this device, obtain the necessary information on proper disposal at local authorities or from your dealer.</p>

## Contact details

### **Manufacturer**

mdh Sp. z o. o.

ul. Maratońska 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

[www.mdh.pl](http://www.mdh.pl)

[www.viteacare.com](http://www.viteacare.com)

We reserve the right to make technical changes due to technical progress.

[viteacare.com](http://viteacare.com)

mdh Sp. z o.o.  
ul. Maratońska 104, 94-007 Łódź, Polska  
tel. +48 42 674 83 84, fax. +48 42 636 52 21  
[www.mdh.pl](http://www.mdh.pl)  
[www.viteacare.com](http://www.viteacare.com)