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English version on next page

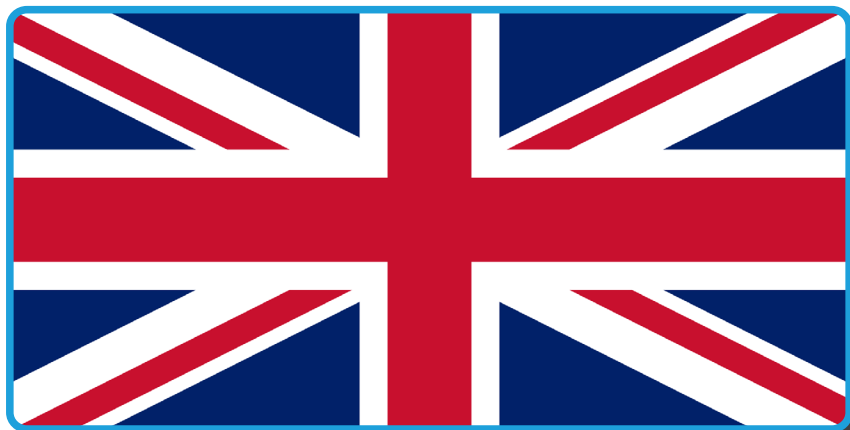
Table of contents

Introduction from mdh Sp. z o.o.....	43
Indications.....	44
Contraindications	44
Safety instructions	44
General safety rules	46
Modifications	46
Security check.....	47
Capacity	47
Transport the wheelchair.....	47
Driving outdoors and precautions in bad weather conditions	48
Physical limitations	48
Information on electromagnetic interference - EMI	49
Practical information concerning the usage	50
Before usage.....	50
Wheelchair operating training.....	50
Moving on rough terrain.....	51
Moving to an upright position.....	51
System of stabilisation straps.....	52
Preparing the wheelchair	52
Transporting the wheelchair	52
Before the first use of the wheelchair.....	53
Hand brake.....	53
Fuse.....	53
Clutch.....	54
Getting off the wheelchair	54
Wheelchair characteristics.....	55
Specifications	55
Wheelchair description	56
Control panel - operation	59
Adjustment and fitting of wheelchair components.....	60
Headrest adjustment.....	60
Folding the seat.....	61

Adjustable lower limb support.....	61
Safety belt (for seat width 420 mm).....	62
Harness belt (for seat width 460 mm)	62
Adjustable knee protector	63
Lifting of the foot platform and armrests	65
Battery and charging	65
Battery charging safety	66
Charging process.....	66
Problems related to the charging process.....	67
Batteries	68
Error codes and troubleshooting	70
Care and maintenance	71
Disinfection.....	71
Disposal of electric wheelchair	75
Symbols	75
Contact details	78

Introduction from mdh Sp. z o.o.

Hero Stand Up is a wheelchair manufactured according to the latest technology. Construction of this wheelchair enables to ensure its maximal adjustment to the requirements of the user. The wheelchair combines solid and durable construction, latest solutions as far as electronic control is concerned, and appropriate safety systems. In order to follow safety principles and to maintain the capability of being fully operational, remember to regularly control your wheelchair in the service center (once a year). In case of any doubts or any damages, please contact your service center, as this is the entity having all competences required to solve your problem. Make sure that you have filled the attached warranty form and that you have submitted it to the distributor, as this will enable us to ensure the best possible service.

ATTENTION

Always follow the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Keep this manual for future reference.

WARNING!

Improper use can lead to death, serious injury, unsafe conditions or product malfunction.

ATTENTION!

Improper use may lead to personal injury and/or damage to the wheelchair.

SUGGESTION!

Follow the instructions below to keep your wheelchair in good working condition.

If the electric wheelchair is used by another person, you must provide them with copies of these instructions for use.

These instructions for use relate to the design and specifications of the product at the time of publication. Due to design changes, some illustrations and photos in the manual may not correspond to the wheelchair you have purchased. We reserve the right to make design changes.

The manufacturer disclaims all liability for personal injury or property damage that may result from improper or unsafe use of this product.

SUGGESTION!

Batteries will be disconnected during transport in order to ensure their long lifetime. Remember to connect the red cord with input marked with „+“ symbol before using the wheelchair for the first time.

Indications

The product is intended for disabled people with musculoskeletal conditions to increase their mobility.

Contraindications

Contraindications to sitting position.

Safety instructions

- Read and follow the information in the instructions for use. It is forbidden to use this product and the available optional equipment without first carefully reading and understanding this manual. If you do not understand the warnings, cautions or instructions, contact your supplier, otherwise personal injury or accident may result. Never attempt to use the electric wheelchair beyond its limitations as described in this manual.
- Never deactivate the control panel when driving, as this will make the wheelchair stop suddenly, and this can cause an injury.
- Consult your doctor and ask if medicines you are taking don't pose an influence on your ability to operate mechanical vehicles.
- To operate an electric wheelchair safely, it should be ridden on flat surfaces where all four wheels have sufficient contact with the ground.
- If the battery level indicator drops to red, recharge the wheelchair as soon as possible. Completely discharging the batteries may damage them or shorten their life.
- Keep metal objects away from the battery connections. Risk of electric shock.

- When cornering, always reduce speed and keep the center of gravity stable. Always reduce speed when negotiating sharp bends.
- Ensure that there are no obstacles or objects behind the wheelchair when reversing. For safety, the reversing speed should be limited to the minimum speed.
- It is prohibited for children to play near or operate the electric wheelchair.
- Please note that the temperature of the components and frame parts may increase when exposed to the sun or other heat source, while at low temperatures there is a danger of very high cooling of the metal parts of the wheelchair. This can cause burns or frostbite on parts of the body.
- Store in clean and dry conditions.
- It is forbidden to drive at night without proper lighting.
- Do not remove the anti-tip wheels or transport wheels.
- Keep hands and feet away from moving parts when driving. Take care that loose clothing does not get caught in the drive wheels.
- Do not connect the battery charger to an extension cord.
- Disassembly of the controller, motor or charger by anyone other than an authorized service is prohibited and will invalidate any applicable warranty conditions.
- It is prohibited to drive the wheelchair on public roads and motorways.
- It is forbidden to use an electric wheelchair under the influence of alcohol, medication or any other substance affecting the ability to drive.
- Be careful when driving in busy areas and shopping centers.
- Under no circumstances should the wheelchair be used as a seat when transported by motor vehicles.
- Do not attempt to lift the wheelchair by gripping anything other than the frame (for example, the seat, armrest or chassis components).
- In the upright position, move only indoors, for short distances. When the seat is raised, the maximum speed will automatically be reduced by half.
- It is forbidden to use a cell phone or other personal communication devices (e.g. CB-radio) while driving.
- Do not use the wheelchair outside when it's raining or snowing, when the temperature is too high (it exceeds 30°C) or too low (below 0°C).
- Do not drive the wheelchair on quaysides, on river banks of lake shores, which have not been secured.
- The wheelchair is not adjusted to be used off-road (dirt roads).
- Avoid sudden stops, turns and making U-turns, as they can lead to fall and injury.
- When operating the wheelchair, keep your forearms on arm-rests, and keep your feet on foot-rests.
- It is forbidden to stand up when operating the wheelchair. Do not lean out to the sides, do not lift any objects from the ground.
- Do not approach any elevations (steps, curbs) with height exceeding 45 mm.
- The wheelchair is not adjusted to move on stairs. Always use wheelchair ramps or lifts.
- The wheelchair is not adjusted to overcome any cracks in the ground that are wider than 100 mm.

- When in vertical position, move only inside the room, on short distances. When the seat is lifted, the maximal speed will automatically be reduced by half.

WARNING!

Any "serious incident" related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

General safety rules

Read the instructions for use carefully with understanding before using the electric wheelchair.

Users may encounter difficulties with maneuvering when passing through narrow doorways, ascending and descending ramps, driving around corners and navigating uneven terrain. Reduce speed, take your time and maneuver the electric wheelchair carefully.

WARNING!

Do not sit in the vehicle when is moving.

Modifications

Do not modify, remove, disable or add any parts, components or features to your electric wheelchair. The manufacturer offers a wide range of electric wheelchair accessories to meet your needs.

WARNING!

Do not modify the electric wheelchair in any way; this will void the warranty.

Security check

ATTENTION - Check the following before use:

- Electrical connections and wiring harnesses - protect and remove corrosion.
- Batteries - fully charged and not corroded.
- Brakes - check that the electromagnetic brake is functioning.
- Upright mechanism - Ensure that the wheelchair is fully extended.
- Condition of the tires- make sure they are not damaged or excessively worn - if there is a problem, contact your dealer.

Capacity

Hero stand up electric wheelchair is designed and built to transport one person with a maximum weight of 100 kg. The wheelchair is not designed to carry goods or objects other than those previously described.

Warning!

Exceeding the maximum user weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

Transport the wheelchair

When transporting the electric wheelchair by vehicle, store it safely in the back of a van, truck or in the boot of a car. Moving parts should be removed or properly secured for transport.

Elements that can be used to attach the wheelchair during transport.

Symbol warning about the risk of jamming the hand. It is essential to make sure that, when changing the position, no elements of garment or of the body can be jammed by mobile elements of the wheelchair.

Symbol indicating that the battery is connected.

Brake release lever

the brake is activated

the brake is deactivated

Brake release lever.

Driving outdoors and precautions in bad weather conditions

Although the electric wheelchair is designed to be used indoors to help with daily living activities, the manufacturer has designed it to perform perfectly outdoors on level surfaces.

ATTENTION - When working outdoors, avoid uneven terrain, soft surfaces, tall grass, loose gravel, loose sand, fresh and salt water, stream banks, lakes and oceans. If you are unsure about a surface, avoid it.

Exposure to adverse weather conditions should always be avoided. If you find yourself in bad weather conditions while using your electric wheelchair, immediately go to the nearest shelter. Dry the electric wheelchair completely before use, before charging or before storing.

Physical limitations

Use awareness, caution, prudence and common sense when operating the wheelchair. Always be aware of your own limitations and rules of use when operating an electric wheelchair.

WARNING!

The product is intended for use by people with disabilities and movement disorders. If you are not sure whether your health condition allows you to use the product, consult a healthcare professional.

WARNING!

It is forbidden to use the wheelchair while under the influence of alcohol or psychotropic drugs. Never smoke while sitting in the wheelchair. Damaged or worn upholstery must be replaced as it increases the risk of fire.

ATTENTION!

When operating an electric wheelchair, please be aware of precautions, warnings and safety issues if you are taking medications prescribed by a doctor or available without a prescription.

WARNING!

Drive the electric wheelchair forward down or up the slope, never reverse. Irregular movements or making forward and backward movements may increase the probability of tipping over. Changes in slope can reduce stability. Under no circumstances should the electric wheelchair be moved up a slope greater than the maximum stability angle of this electric wheelchair.

Information on electromagnetic interference - EMI

Studies of electromagnetic interference (EMI) have shown that it adversely affects the operation and control of electrically powered mobile devices.

EMI can come from a variety of sources, such as cell phones, two-way radios, radio stations, TV stations, amateur radio (HAM) transmitters, wireless computer links, microwave signals, paging transmitters and transmitters used by emergency vehicles. EMI waves can cause unintended movement of the wheelchair or damage to the controller. Every electrically powered mobile device has a certain immunity to electromagnetic interference. The higher the immunity level, the greater the protection. The intensity of interference can be measured in volts per meter (V/m). Current technology allows for an immunity level of 20 V/m, which provides protection against the most common sources of electromagnetic interference. The wheelchair, without any modifications, has an immunity level of 20 V/m. As you get closer to the source, electromagnetic interference becomes more intense. Special attention should be paid to electromagnetic interference from hand-held radio devices. It is possible to unintentionally bring high levels of EMI close to the wheelchair control system, which can affect movement and braking. To prevent possible interference with the wheelchair control system, follow the warnings below.

WARNING!

Do not use hand-held radio transmitters such as a CB-radio or cell phone near a powered-up wheelchair.

WARNING!

Pay attention and stay away from nearby transmitters, such as radio or TV stations.

Warning!

Drive the electric wheelchair in a straight line up or down the slope, never reverse. Irregular forward and backward movements with changes in slope can increase the risk of the wheelchair rollover.

Practical information concerning the usage

Before usage

- Are the batteries fully charged? See the chapter “Batteries and charging”
- Have the manual drive disconnect clutch lever in the rear position. Never leave the lever in the forward position if you do not intend to push the wheelchair manually.

WARNING!

The following operations may adversely affect the steering and stability of the wheelchair:

- Keeping and walking an animal attached to a leash.
- Carrying passengers (including animals).
- Hanging an object on the controller.
- Towing or pushing by another vehicle.

Wheelchair operating training

1. It is necessary to master well the use of the wheelchair and the execution of all maneuvers to move safely. First, master the forward movement. Make sure that the minimum speed is set.
2. Once you have mastered moving in a straight forward direction, you should try to drive by circling the wheelchair in an S shape.
3. After that, you can start practicing driving backwards. Regardless of the programmed forward speed, the wheelchair will always move slower backward.

Moving on rough terrain

Moving on an undulating terrain creates more risk. It is important to follow the safety principles stated below. Failure to comply with these principles may lead to serious injuries.

- It is not recommended to move the wheelchair on slopes with inclination exceeding 10°. It is essential to remember that wet surface will additionally reduce the grip of the wheelchair and will impair the functioning of brakes when ascending and descending a slope.
- When in vertical position, it is not allowed to ascend on a slope with inclination greater than 3°. The speed is automatically reduced, according to the inclination angle and the selected position. In extreme conditions, the system shall make it impossible to move any further.
- It is not recommended to move backwards when descending slopes.
- It is not recommended to move on uneven, dirt roads (sand, gravel, grass).
- When descending a slope, it is recommended to move with minimum speed.
- Do not get off the wheelchair and do not get on a standing wheelchair on an inclined surface (always place the wheelchair in an even, flat surface first).
- When conquering a slope, always ascend following a straight line, according to the inclination of the slope.

Moving to an upright position

- Consult your physician and ask whether an upright standing position is safe in your case.
- When you are moving to an upright position for the first time, the whole process should be supervised by a qualified medical personnel, since many various impairments can occur in such situations.
- Use the verticalization function only when the wheelchair is in the drive mode (the electric power supply is on) - only when the braking system is activated.
- Move to an upright position only when the wheelchair stands on a stable, even and dry surface, otherwise moving to an upright position may result in fall and injury.
- It is not recommended to move in an upright position outside and on long distances. Moving in an upright position is allowed only in small, enclosed spaces (kitchen, office, etc.)

WARNING!

Pay special attention and check that no parts of garments or parts of the body can be jammed when lowering the seat (when moving to a sitting position). Be particularly careful.

Verticalization allows:

- Prevent muscle spasms in lower limbs.
- Regulate functioning of the respiratory, circulatory and excretory system, as well as gastrointestinal tract.

- Regulate muscle tension (prevent spasticity).
- Reduce the risk of osteoporosis (minimise calcium extraction from bones).
- Reduce the risk of decubitus ulcers.
- Increase the feeling of independence when performing activities of daily living (positive psychological aspect).

System of stabilisation straps

WARNING!

Remember to properly fasten knee, chest and hip stabilisation straps before moving to the upright standing position.

Stabilisation straps are used to stabilise the body and limbs when moving forwards and when moving to an upright position. They do not stand as protection in case of possible collision. Before initialising the verticalization process, it is crucial to check if all straps are properly tightened.


Preparing the wheelchair

The wheelchair is delivered in a cardboard package, and its delicate parts are additionally secured against damage during transport. Unpack the wheelchair very carefully, to avoid any damage.

Transporting the wheelchair

The wheelchair should be transported only by means adapted for this purpose. During transport, it should be secured with straps fastened at the places marked with a blue sticker (in the figure opposite, the hitching places are marked with wheels). The wheels should be locked. It is forbidden to stay in a wheelchair during transport, it risks falling and serious injuries. The wheelchair must not be transported by means that are not suitable for this purpose - they do not have a special fastening system, as this risks damaging the wheelchair. Always Ensure that the wheelchair is properly secured in the means of transport, otherwise, it can cause injury to the people moving on the means of transport as well as serious damage to the means of transport itself.

When driving up the ramp to the transporting means, a back belay should be provided to protect the wheelchair from tipping over.

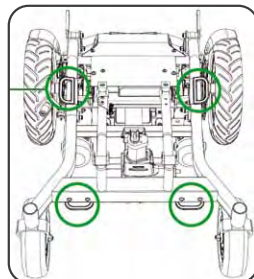All additional elements of the wheelchair should be removed from the wheelchair for the duration of transport or they should be securely attached to the wheelchair, making sure that these elements do not cause injury or damage during sudden braking. In order to reduce the size of the wheelchair during transport, you can do two following actions:

- lift the footrest
- fold the backrest as shown on the illustration

Before the first use of the wheelchair

- Check if batteries are fully charged.
- Check if the drive mode is turned on (clutch lever).
- Always plan your route ensuring that you will omit dangerous areas (boggy, difficult areas) and obstacles, which you won't be able to overcome in a wheelchair.
- Check that the seat is adjusted properly.
- Check if the speed is adjusted to your skills and capacity.
- Check if the horn is functioning properly.

ATTENTION!

- When sitting on the wheelchair do not lean your body weight on arm-rests, as this may lead to personal injuries and the wheelchair may fall.
- Do not stand on foot-rests as the wheelchair may fall (except for the upright position).
- Remember to check whether the wheelchair is deactivated, since inexperienced touching the control shifter when changing seats may make the wheelchair move, and this can lead to fall and personal injury.
- During transfer try to stay as close to the wheelchair as possible, to avoid slip and fall.

Hand brake

When you stop the wheelchair, especially on an uneven ground, remember to pull the hand brake, to increase safety. Pull the hand brake whenever you are sitting on the wheelchair or you are getting off the wheelchair onto another type of seat. Remember that improper pressure in tyres, worn tyre tread or wet tyres can reduce the effectiveness of the hand brake.

Fuse

When the wheelchair overheats, press the fuse to break the circuit and protect it from damage. The location of the fuse is shown in the figure.

Warning!

Do not touch the motor because it risks burns.

Clutch

The wheelchair has a clutch lever enabling to disconnect the electric drive and push the wheelchair manually.

- If you want the electric drive to be **activated**, push the clutch lever forward of the wheelchair.
- If you want the electric drive to be **deactivated**, pull the clutch lever.

WARNING!

Do not deactivate the electric drive, if there is no one who could push your wheelchair.

Do not deactivate the electric drive when the wheelchair is on an inclined surface, as it could roll down and this could lead to serious injuries.

Remember that when the electric drive is deactivated, the electromagnetic braking system is also deactivated.

Always stand behind the wheelchair when you are deactivating the electric drive. Do not deactivate the electric drive when sitting on the wheelchair. When you place your wheelchair in the desired location, remember to activate the electric drive again.

Getting off the wheelchair

WARNING!

Never attempt to get on or off the wheelchair without first removing the key from the power switch. This prevents the product, from being moved by accidentally switching the drive control lever.

1. Stop the wheelchair completely.
2. Turn off the power o control panel
3. Carefully and safely get off the seat and take place next to the wheelchair.

Wheelchair characteristics


The wheelchair consists of the following parts:

- Frame
- Seat
- Backrest
- Knee stabilizer
- Belt or vest to stabilize chest and hips
- Control panel
- Headrest
- Batteries
- Battery charger
- Set of wheels
- Footrest
- Armrests
- Tools set

Specifications

Parameters	HERO STAND UP	
Front wheels	2.80/2.50-4 - pneumatic tire 9"(228,6 mm) 50 PSI(3.5 BAR)	
Rear wheels	3.00-8 - pneumatic tire 14"(355,6 mm) 36 PSI(2.5 BAR)	
Length	1135 mm	
Width	650 mm	620 mm
Height	1025 mm / 1205~1475 mm (with headrest)	
Seat width	460 mm	420 mm
Seat depth	500-550 mm	475-525 mm
Seat height from ground	540 mm	
Armrest Height	195-255 mm	
Backrest Height	500-575 mm	475-525 mm
Motor	2 x 450 W (24 VDC)	
Brakes	Electromagnetic	
Battery	2 x 50 Ah / 12 V	
Energy range	25 km (on the flat road, 100 kg loading)	
Maximum safe inclination	10° (17,63%)	
Maximum speed forward	9,2 km	
Maximum user weight	100 kg	
Wheelchair weight without batteries	78 kg	
Batteries weight	31 kg (2 x 15,5 kg)	

Wheelchair description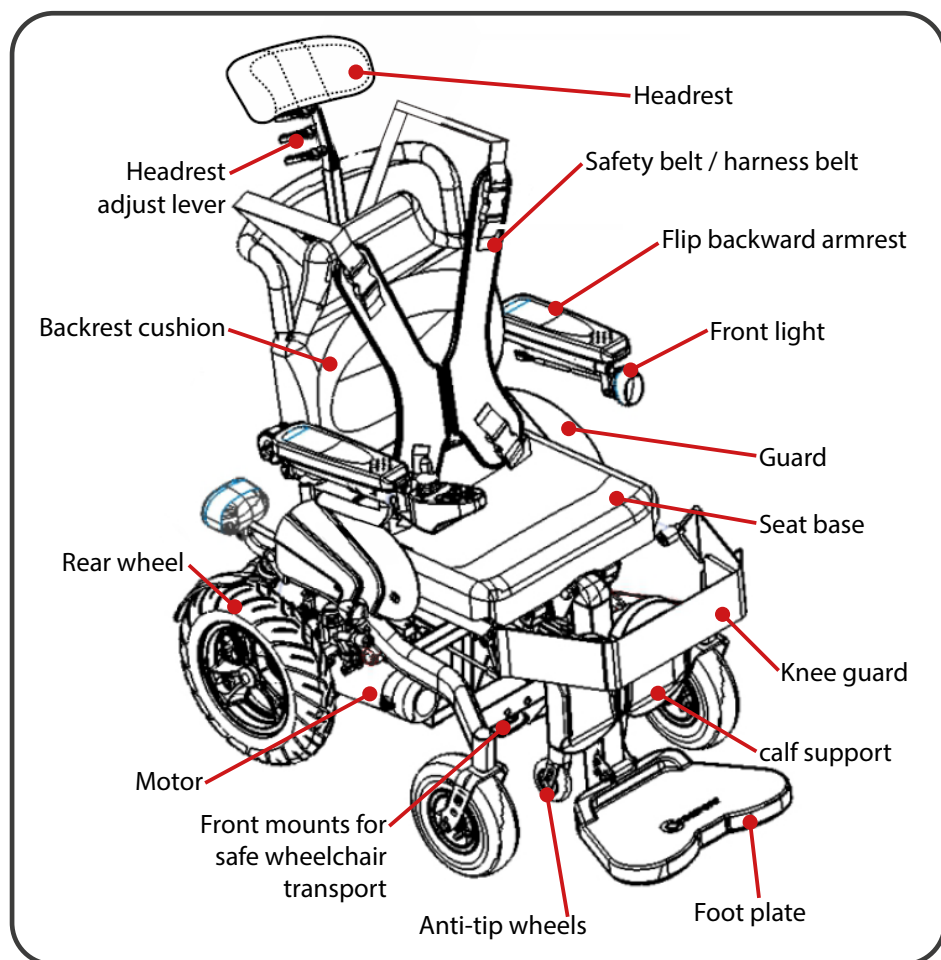


ATTENTION!

Depending on the size of the seat width, the following equipment is available:

For seat width 420 mm - Safety belt




For seat width 460 mm - Harness belt

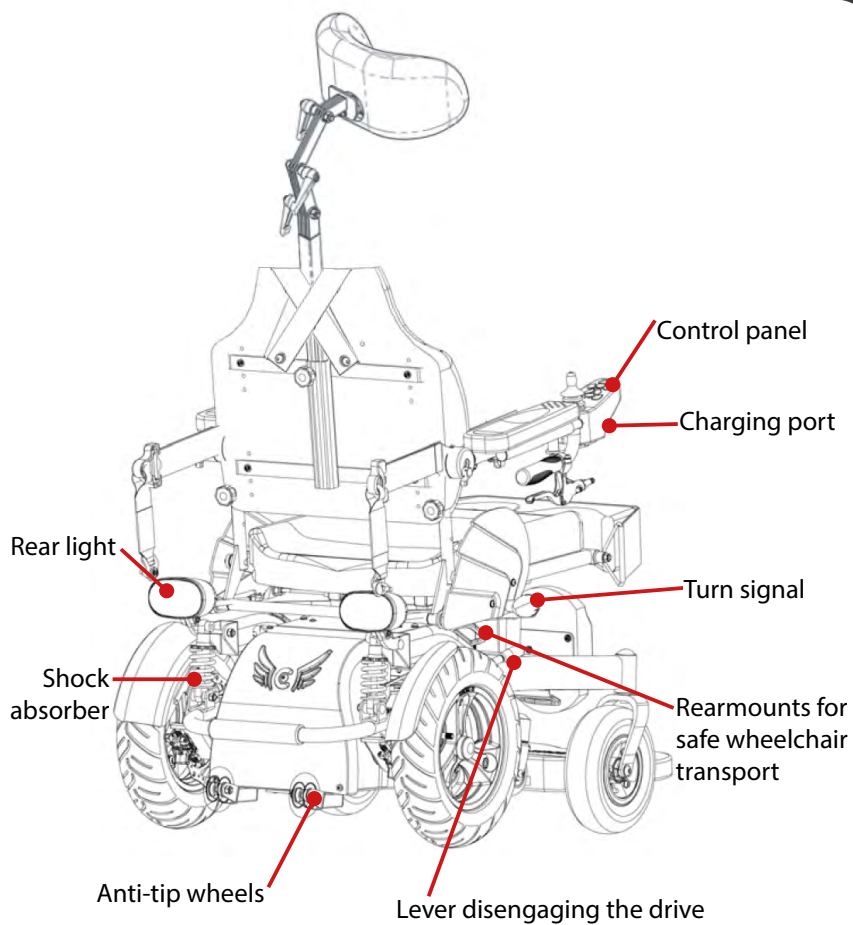
WARNING!

The user should be aware that the surface of the remote control may get heat up when it is exposed to sunlight for an extended period of time.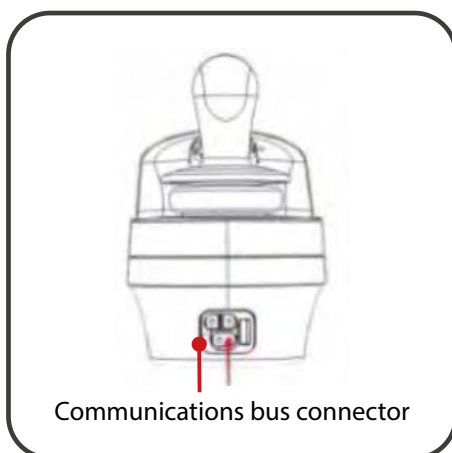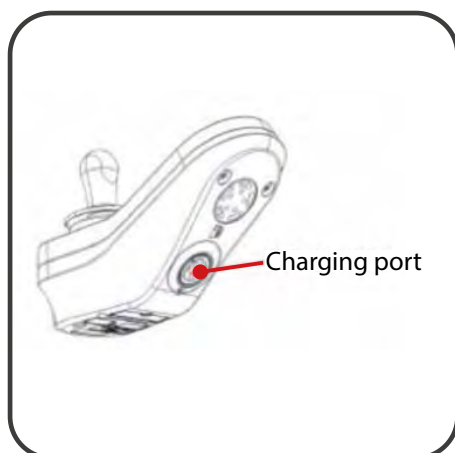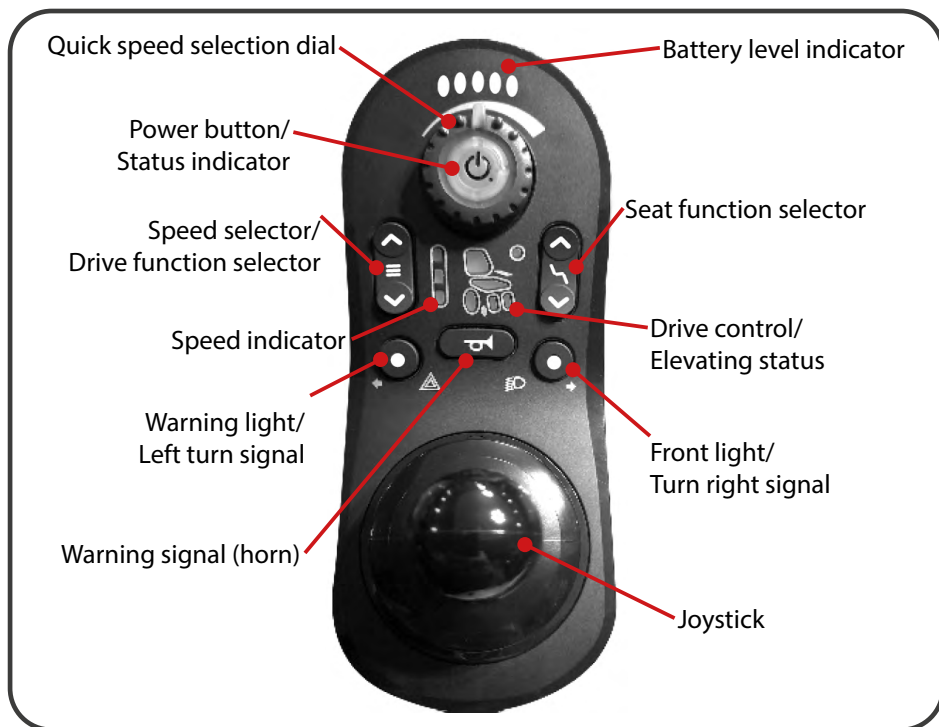

Control panel - operation

Adjustment of wheelchair function operation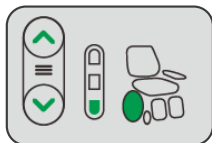

The drive function can be selected using the select button speed, which is located on the control panel below the left side of the power button.


Seat function selector

The seat function can be selected using the seat function switch, which is located on the controller below the right side of the power button.

Quick speed selection dial and speed selector button allow you to adjust the speed of the wheelchair.

Quick speed selection dial

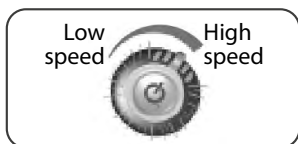To set the lowest speed, turn the dial as far to the left as possible.

To set the highest speed, turn the dial as far to the right as possible.

Speed selector

A speed selector allows the user to limit the wheelchair's maximum speed. Turn the speed selector knob as far to the right as possible to set the highest speed and follow these steps:

Minimum speed  is set at 40% of maximum speed

Medium speed  is set at 70% of maximum speed


Maximum speed  is set at 95% of maximum speed

Indicator lamps and displayed information

To turn on the emergency light, press the button once. To turn off the emergency light, press the left or right light button once.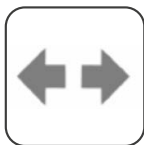

Turn signal lights:

To turn on left (right) signal lights, please press left (or right) lighting button for a while(3 seconds) and release. Left (or right) side LED will light up.

To turn off signal lights, please press left or right lighting button once.

Front lights:


To turn on the headlight, press the button on the controller once. To turn off the headlight, please press once, the left or right front light or turn signal button.

Adjustment and fitting of wheelchair components

Headrest adjustment

Fitting the position of headrest.

Three levers are present to adjust the headrest.

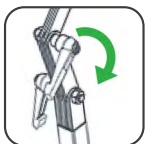- Turn the lever counterclockwise to loosen the joint.
- The user or caregiver can adjust the headrest to the desired position.
- Turn the lever clockwise to lock in position.

1. It is necessary to hold the lever.
2. Press the screw located on the top of the lever.
3. Pull out the lever and slowly rotate it to the desired position.

SUGGESTION!

If there are obstacles between the levers, each lever can be pulled out separately and rotated to the desired position.

ATTENTION!

To avoid damaging the lever, do not pull it out too hard.

Height adjustment headrest

Turn the back button of headrest anticlockwise.
Adjust the height and turn the button clockwise for locking.

ATTENTION!

Hold the headrest and then pull upward.

WARNING!

Possibility of jammed fingers.

Folding the seat


To fold the seat, loosen the two screws located on either side of the backrest and, grasping the handle on top of the backrest, push it forward.

WARNING!

When folding, possibility of getting your hands jammed.

Adjustable lower limb support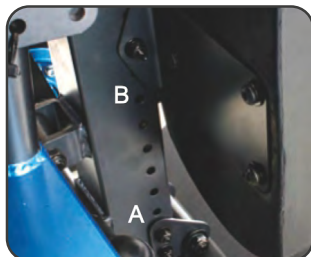

In order to adjust the position of the footplate and to ensure comfortable position of the feet, it is essential to unscrew screws indicates with the letter **A** on the illustration, choose the appropriate height to place the footplate (by choosing proper holes on the fixing arm) and tighten the screws.

In order to adjust the position of calf support, unscrew the screw indicated with the letter **B** on the illustration, choose the appropriate height to place the calf support (by choosing proper holes on the fixing arm) and tighten the screw.

Safety belt (for seat width 420 mm)

1. Move the seat belt.
2. Sit in a comfortable position and adjust the belt.
3. Fasten the seat belt making sure it is locked properly.


SUGGESTION!

The user and attendant can safely adjust the seat belt


Harness belt (for seat width 460 mm)

1. Put on the harness belt and fasten the buckles on both sides of the upper part of the shoulders.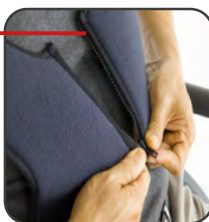

2. Fasten the express zipper.

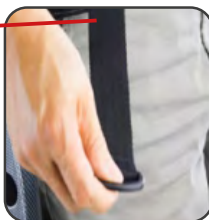3. After putting on the harness belt, fasten the lower buckles on both sides.

ATTENTION!

When adjusting and fitting the harness belt, the help of another person is recommended.

Adjustable knee protector


Adjustment is done with a Velcro strap. It can be fastened tighter or looser depending on your needs.

ATTENTION!


It is important to adjust all parameters in a manner ensuring proper and stable position, both when sitting and when standing upright.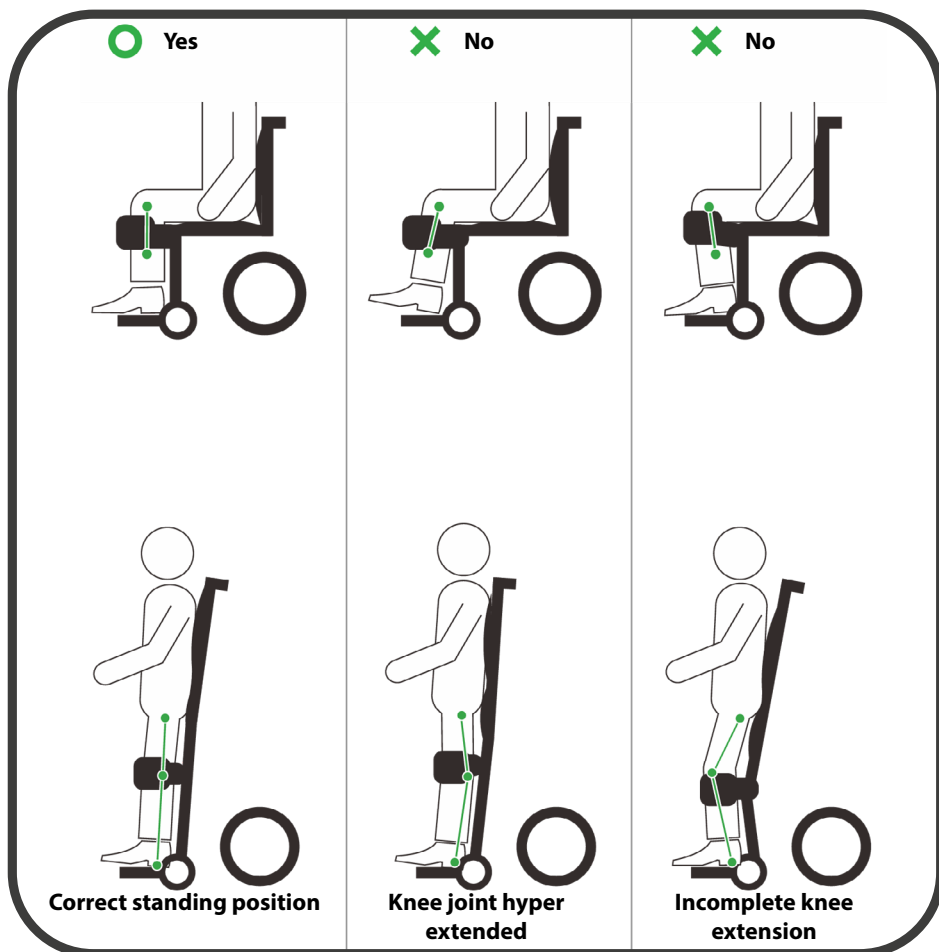

The screw tightening the knee stabiliser should be located on the level of the middle point of the knee joint, to ensure its position in 90° flexion. Proper position of the knee stabiliser will ensure proper standing position (hip, knee, ankle joint in one line) and by the same, proper load bearing of the whole limb.

SUGGESTION!
Properly adjusted pillow on the seat and the backrest is of considerable significance as far as proper standing position is concerned, and it is shown below.

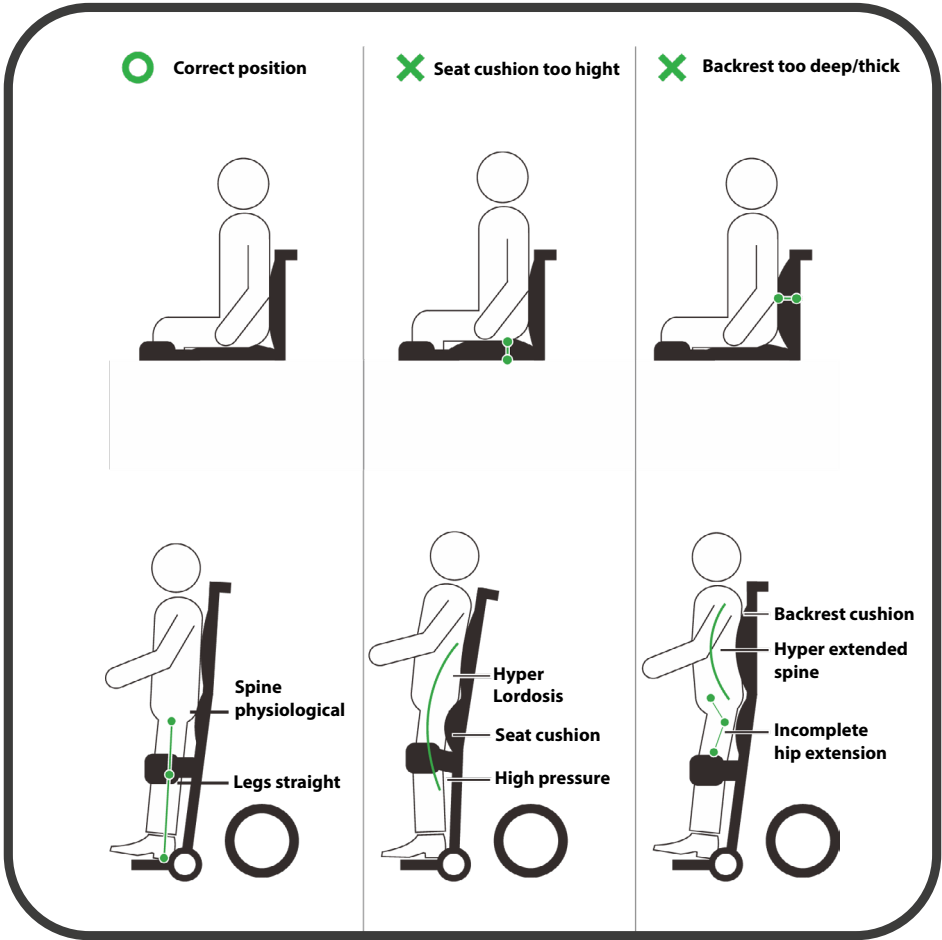Lifting of the foot platform and armrests

If necessary, it is possible to manually fold the footrest by lifting it up (as shown on the illustration below).

In order to facilitate transfer, it is possible to lift armrests.


WARNING!

Always remember to fasten and adjust stabilizing straps. After adjusting all settings of the wheelchair and after sitting in the wheelchair in a comfortable position, fasten your chest stabilizing belt. Make sure that this belt is properly adjusted and fastened.

Battery and charging


Charge the batteries with the battery charger supplied with the wheelchair.

Charger
front view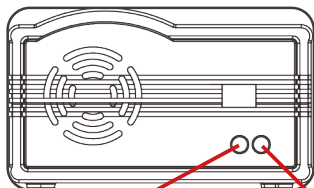

Indicator LED

Power LED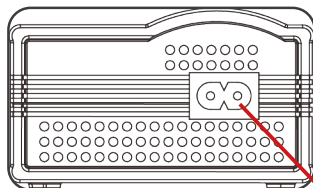

Back of
the charger

Power input socket

Battery charging safety

- Use a standard 3-prong grounded plug and grounded socket.
- Do not use an extension cord or adapters to avoid the risk of fire, electric shock and injury.
- Do not use the same socket for other activities when charging the battery.
- Use the default charger provided for this wheelchair to avoid damage to the wheelchair.
- Do not leave the charger plugged in, if the batteries are not charging.
- Keep the charger and batteries far from water and humidity, do not handle the charger with wet hands.
- Charge the wheelchair in properly aired room.
- Do not cover the ventilator of the charger, to ensure proper heat exchange.
- Leave the device in stand-by mode until the battery is fully charged. Do not sit in the wheelchair when the battery is charging.
- Do not break the battery charging cycle until the battery charge indicator shows that the battery is fully charged..
- Do not put the charger directly on the battery.
- Check whether all parameters of the charger correspond with the parameters of batteries and power supply voltage. Connecting a battery with different parameters may lead to battery damage and to serious injuries of the user.
- When charging the batteries, pay special attention not to expose the wheelchair to high temperatures and smoke, to avoid ignition and explosion.
- In order to avoid electric shock, do not touch the charger or batteries with wet hands or other objects, and do not expose these elements to humidity.
- Check whether the charger switch is set to "O" position before connecting the charger to the mains and before charging the batteries.
- Do not expose the charger to rain, water and humidity.
- The charging process should take place in ambient temperature , ranging between 0°C and 40°C. Charging batteries in other conditions may be unsuccessful or may lead to damage of the batteries.
- If the charging indicator is not illuminated, disconnect cords for 10 minutes and after this period of time try reconnecting them. If this does not help, contact the service department.
- When the charge indicator remains yellow after charging for more than 12 hours, contact the service department.

Charging process

1. Park your wheelchair as close to the grounded socket as possible.
2. Turn off the wheelchair.
3. Make sure that the electric power supply of the wheelchair is turned on (the clutch is in proper position - drive mode).

4. Connect the charger to the charging socket.
5. Connect the charger to grounded socket, remember to check whether the socket has proper voltage - 115V/230V.
6. Turn on the charger (red - the battery is low, yellow - the battery is half charged, green - the battery is fully charged).
In case when LEDs flash in an irregular manner, turn the charger off. Check the connection of the charger and reconnect the charger. In case when the charging procedure will proceed without complications, diodes marking the battery charge level will light up (as described above).
7. The optimal charging time takes 5-6 hours (in winter it lasts slightly longer).
8. When charging is finished, disconnect the charger from the socket, and then take the plug out of the charging socket.

Problems related to the charging process

1. Lack of voltage indication after connecting the charger.
 - Check the connection of power cords.
 - Check the connection socket of the charger.
 - Check the functioning of the ventilator.
2. Red and yellow LEDs on the battery charge indicator are lightened up, but the ventilator does not turn on, despite the fact that the power is on.
 - Check whether the charger is properly connected to the batteries.
3. Red LED on the battery charge indicator lights up permanently and the green LED flashes, lack of charging signal despite the fact that the power is on.
 - Check the antipolarising fuse.
 - Check the connection with batteries.
 - Check the polarisation of batteries.
 - Check whether the batteries are properly located.


Batteries

1. The wheelchair is equipped with batteries with long lifespan. Batteries are sealed and they do not require maintenance (the so-called maintenance-free batteries).
2. The following situations can cause increased energy consumption and hence decrease the range of the wheelchair: ascending hills, overload, winter.
3. Observing the following principles will allow to prolong the lifespan of batteries:
 - Avoid complete discharge of batteries. Charge the battery after everyday use.
 - During first charging, disconnect the plug after 40 hours, when the battery is fully charged.
 - If you are not using the wheelchair on regular basis, to maintain durability of batteries, charge them every three days, observing a full charging cycle.
4. Do not take the cover out from maintenance-free batteries. It is not allowed to add water to the battery.
5. Leak of acid from the battery may cause burns or loss of vision. In case when the acid from the battery comes into touch with the skin, eyes, garments or any other object, rinse it thoroughly with running water and immediately report to undergo medical consultation.
6. Do not use other batteries than the original batteries attached to the wheelchair, as this may result in product damage.
7. Make sure that during transport, both the wheelchair and batteries are properly secured.
8. Batteries are subject to wearing and that is why the range of the wheelchair will decrease. It is essential to exchange the batteries when the range of the wheelchair decreased by half, despite complete charging of the battery.
9. It is essential to change both batteries at the same time, in order to ensure their long-lasting and effective functioning.

Batteries - storage

- In case when the wheelchair will remain unused for a longer period of time, take the batteries out and store them in a dry and cold place.
- Avoid complete failure of the battery.
- If you are using the wheelchair every day, charge batteries every day.
- If you are using the wheelchair once a week, charge batteries once a week.
- It is not allowed to partially charge batteries (f. ex. connect batteries for a shorter time than a full charging cycle), as this may damage the battery or limit its durability.
- Only fully charged batteries can be stored.
- If you are not using your wheelchair on regular basis, in order to maintain durability of batteries, charge them at least once every two weeks, observing a full charging process.
- Always replace the batteries with the ones delivered with your wheelchair.
- In case when the battery is damaged or has to be changed, report to your distributor. Do not throw the battery into the recycle bin, instead, handle it to the distributor.
- In order to prevent short circuit and battery damage never touch battery contacts with metal objects.

Batteries - electrical circuit diagram

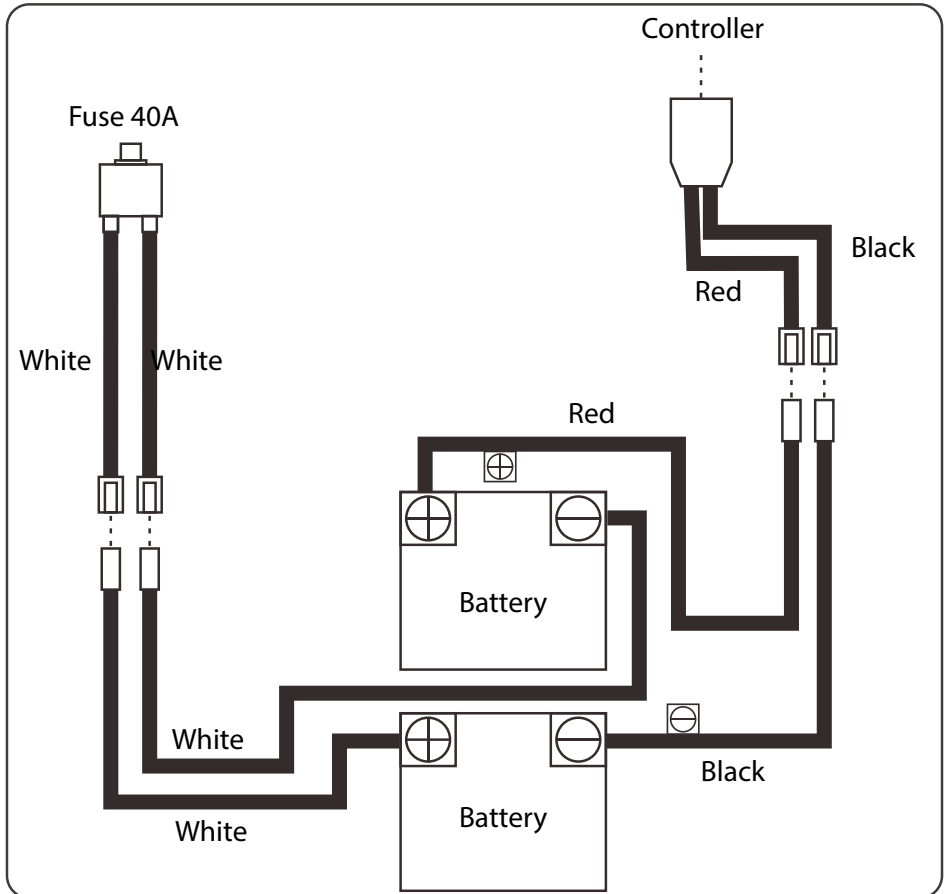1. Faulty connection of the battery may cause battery damage.
2. In case when the battery is exposed to low temperatures (it is frozen), do not try to recharge the battery..
3. In case when the battery freezes, it is essential to leave it for several days in room temperature in order to warm the battery and then charge it.

SUGGESTION!

In case when you have any doubts concerning how to change the battery, contact the service department or the manufacturer.

Error codes and troubleshooting

ATTENTION!

If there is an error with the system when it is powered up, then the status indicator will flash red; the number of flashes will indicate the type of error.

Status
indicator

The table below describes the error indication, and a few possible actions that can be taken to rectify the problem. The actions listed are not in any particular order and are suggestions only; the intention is that one of the suggestions may help you clear the problem. If in doubt, consult your authorized supplier.

Number of flashes	Error description	Possible action
1	Remote / joystick error	Check cables and connectors. Replace Remote module.
2	Network or configuration error	Check cables and connectors. Check Bluetooth pairing. Reconfigure the system. Recharge the battery. Check charger. Replace modules. Contact supplier.
3	Left motor error	Check cables and connectors. Replace Power Module. Check and/or replace left motor.
4	Right motor error	Check cables and connectors. Replace Power Module. Check and/or replace right motor
5	Left park brake error	Check cables and connectors. Check left park brake is released
6	Right park brake error	Check cables and connectors. Check right park brake is released.
7	Module error (other than Remote module)	Check cables and connectors. Check modules. Replace LiNX Access Key. Replace Power Module. Recharge battery.

**WARNING!**

If the wheelchair stalled, reverse away or remove obstacles, or if the wheelchair was moved while turned off, cycle the power.

Care and maintenance

Insufficient or abandoned care and maintenance of an electric wheelchair lead to a limitation of the manufacturer's liability.

In order to ensure long-lasting and efficient activity of the wheelchair, it is essential to perform fundamental maintenance activities and reviews conducted by the service department. Reviews should be performed once a year. In case of any doubts or damages, contact the service department, as the service centre has all competencies required to solve your problem.

**WARNING!**

Follow the safety instructions provided by the manufacturer of cleaning and disinfecting products. Not complying may result in skin irritation or early damage to the upholstery or electric scooter.

Disinfection

If the product is used by several people (e.g. in a nursing home), commercially available disinfectants should be used. Before disinfecting, clean the upholstery and handles. Only tested and approved disinfectants may be used for disinfection. Information on recognized and approved disinfectants and disinfection methods is available from your national health authority.

**WARNING!**

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.

Maintenance plan

When	What	Attention
Before driving	Generally check the correct operation.	Perform the inspection alone or with a support person.
	Magnetic brake control. Set the driving mode switch lever on both sides to driving mode.	Perform the inspection by yourself or with a support person. If the wheelchair can be pushed, immediately contact the service center for brake repair. - Risk of accident!
In especially before driving after dusk	Lighting. Lighting system and reflector, check for proper operation.	Perform the inspection alone or with a support person.
Every 2 weeks (depending on the length of the traveled distances)	Checking tire pressure. Tire pressure: 2,5 bar = 36 psi	Perform the inspection alone or with a support person. Use a manometer.
	Set screw. Check the tightness of the screws and nuts.	Perform the inspection alone or with a person assistance. Tighten loose set screws firmly. If necessary, contact service.
Every 6-8 weeks (depending on the length of the traveled distances)	Bolt fastening. Check nuts and bolts for proper fastening.	Perform these activities by yourself or with the help of another person. Tighten loose wheel nut bolts firmly and re-tighten after 10 hours or 50 km. If necessary, contact service.

Every 2 months (depending on the length of the traveled distances)	Tire profile control Minimum tread depth of tread = 1 mm	Perform the inspection by yourself or with the help of another person. In the case of a worn profile or damaged tires, have the repair done by a specialized service.
Every 6 months (depending on the length of the traveled distances)	Check - Cleanliness - Overall condition	Perform the inspection by yourself or with the help of another person.
Manufacturer's recommendations: Every 12 months (depending on frequency of use)	Inspection work - Vehicle - Loader	Performs specialized service.

Yearly inspection

The inspection should take place at a service center. The purpose of the inspection is to confirm that the wheelchair works correctly, all its systems (electric, electronic, driving) are fully operational. It also helps prevent possible complications resulting from the failure of any of the aforementioned systems.

The review should include:

1. Checking the tires.
2. Checking the batteries and their connections.
3. Checking the operation of the control panel (electronic system).
4. Checking the frame.
5. Checking the condition of the upholstery.

Usage Notes

1. Protect the control panel from strong shocks and impacts.
2. Protect the cart from prolonged exposure to high and low temperatures and moisture.
3. Keep the control panel and the steering handle clean.
4. Protect the wheelchair from direct contact with water.
5. Take care to keep the frame and upholstery clean. If dirty, wipe them with a soft, dry cloth. Do not clean with any abrasive materials. Do not apply any cleaning agents directly to the control panel, battery or electrical parts.
6. Oil the wheel axles once every three months.
7. Use only the batteries supplied with the wheelchair. If it becomes necessary to replace them, report it to the distributor.
8. Charge the batteries regularly. Always make sure the charger is properly connected to the outlet. Do not disconnect the charger by pulling the cord.
9. Check the operation of the control handle when the control panel is off. Check if the control handle is damaged, whether it returns to the neutral position. Watch the rubber that the control handle is covered with, make sure it is not damaged. If any damage is noticed, do not repair it yourself, but contact a service center.
10. Check all cables. If you notice any damage (abrasion, tear), contact the service center.
11. Make sure the control panel is fixed properly.
12. Store the wheelchair in a dry place. Long exposure to moisture can lead to corrosion of frame components and damage to the electronic system.

**WARNING!**

When you find any problem with the motor, charger, control panel or damage to the frame of the forklift, contact the service center immediately. If there are any problems and malfunctions with the forklift, contact the service center to help determine the next procedure.

Disposal of electric wheelchair

The electric wheelchair must be disposed of in accordance with applicable local and national regulations. For information on the proper disposal of packaging, metal frame components, plastic components, electronics, batteries, neoprene, silicone and polyurethane materials, please contact your local waste disposal company or authorized dealer.

Symbols

The symbols used below on the electric wheelchair are used to identify warnings, mandatory actions and prohibited actions. Be sure to read everything and understand their meaning.

	Medical device
	Date of manufacture
	Manufacturer
	Distributor
	Conformité Européene (compliance with the European Community)
	Catalog number (to re-order)
	Serial number

	Lot code
	Unique Device Identifier
	Follow the instructions for use
	Refer to the instructions for use
	Warning
	Protect from humidity
	Possibility of jamming fingers
	Permitted temperature
	For indoor/outdoor use
	Maximum user weight
	Maximum safe inclination
	The product is designed to be transported by car
	The product is not intended to be used as a seat for transport in a motor vehicle
	Indicates attachment points on the scooter

DISPOSAL OF WASTE ELECTRICAL AND ELECTRONIC EQUIPMENT

This symbol on the products or accompanying documents means that used electrical or electronic products should not be disposed of with the normal household waste. For proper disposal, refurbishment or recycling, please take these products to the collection points for this type of waste.

Alternatively, in some EU countries or other European countries, you may return your product to your local retailer when purchasing a similar new product. By disposing of this product correctly, you will help to conserve valuable natural resources and support the prevention of potential negative effects on the environment and human health, which could result from incorrect disposal. For further information, please contact your local municipality or collection point waste collection points. Penalties may be imposed in accordance with local regulations if this product is disposed of in an inappropriate manner. For parties within the European Union, If you wish to discard electrical or electronic equipment, please obtain the necessary information from your retailer or supplier.

Elimination in countries outside the European Union. This symbol is valid in the European Union. If you wish to decommission this appliance, please obtain the correct decommissioning information from your local authorities or from your retailer.

Contact details

Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Polska

tel. +48 42 674 83 84, fax. +48 42 636 52 21

www.mdh.pl

www.viteacare.com

We reserve the right to make technical changes due to technical progress.