# VITEA<sup>®</sup> CARE

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**Umbrella**  
**CHILDREN**  
**REHABILITATIVE**  
**STROLLER**

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### ATTENTION

Always follow the basic precautions according to the information included below and regarding safety measures when using the device. Read the entire instructions carefully before use. Keep this user's manual for future reference.

Umbrella rehabilitative stroller is available in **three sizes**, further known as Umbrella 1, Umbrella 2 & Umbrella 3. For detailed information please refer to the Technical Parameters section.

# 1. INTRODUCTION

Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual was made in order to provide the necessary information on the safe use of the device. The correct application of the recommendations will contribute significantly to extended durability and visual design of the product. We would like to highlight that to date we are constantly improving the quality of our products, which may result in minor changes that are not included in this manual.

## 2. PRODUCT DESCRIPTION

**Fig. 1.** Construction scheme of Umbrella rehabilitative stroller.

1. Handle, 2. Canopy handle, 3. The side lock, 4. Stroller frame, 5. Footrest length adjustment knob, 6. Dual rear wheels (or single in Umbrella 3), 7. The front swivel wheel with possibility of straight ahead ride lock, 8. Canopy, 9. Head stabilizer, 10. Backrest upholstery (with side pelottes and lumbar roller support), 11. Front guard, 12. Abduction wedge, 13. Calf stabilizing belt, 14. Basket for small items, 15. The footrest.

## EQUIPMENT OF THE STROLLER'S PACKAGE

**Fig 2. Stroller**

**Fig 3. Canopy**

**Fig 4. Head stabilizer, front guard (2 versions, black - additional option), abduction wedge, set of side pelottes with lumbar roller support**

**Fig 4. Wheels**

**Fig 5.** Basket for small items, umbrella (additional option), table (additional option), cup

**Fig 6.** Sleeping bag

**Fig 7.** Raincover

### 3. INDICATIONS

The stroller is designed for children with disabilities and / or dysfunctions movement activities as a result of damage of the CNS Central Nervous System. It can be used in children with CP Cerebral Palsy, myelomeningocele, muscle disorders, dysfunctions of the lower limbs.

The maximum weights are **30 kg** (Umbrella 1) **45 kg** (Umbrella 2) & **75 kg** (Umbrella 3).

### 4. CONTRAINDICATIONS

Contradictions against sitting position.

### 5. USAGE

- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above.
- Ensure that all components have no visible damage. If any doubts, please contact the manufacturer.
- Do not use the stroller until all parts are properly connected.

# UNFOLDING THE STROLLER

**Fig 8.** Unfolding the stroller

- Unfolding the stroller should begin with pulling the stroller handles in a direction which is shown in Figure 8. In order to make it easier, you can hold with hand the seat of the stroller.
- When the stroller is partially unfolded, press foot on the stroller blockade (Figure 9.) to the point where stabilizing elements will stop in a horizontal position (Figure 10).

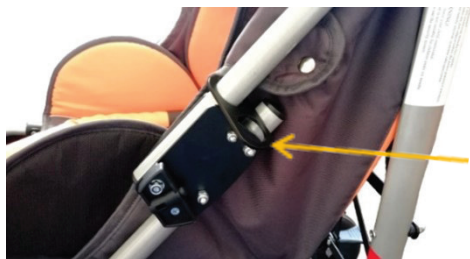
**Fig 9.** Rear lock of the stroller


**Fig 10.** Correct position of rear lock

- 
- To prevent unplanned movement of the stroller, lock the possibility of rotation the two rear wheels.
  - To do this, press the foot brake lever, located at the rear wheel.
  - It is necessary to make sure that the brake has been launched on both wheels.
  - Stroller frame is equipped with the side lock. To lock the side frame, the side lock eyelet should be put on the free end of the frame (Figures 11., 12.).

**Fig 11.** Stroller frame with lock

Side lock  
eyelet

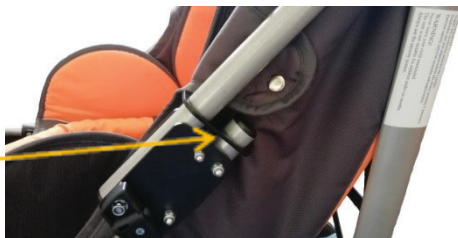**Fig 12.** Locked eyelet on the stroller


**Fig 13.** Assembled and locked  
rehabilitative stroller

- To install the front guard, put the plastic ends into the guides on the stroller frame in the direction from top to bottom (Figures 14., 15.).

**Fig 14.** Front guard lock button (open)

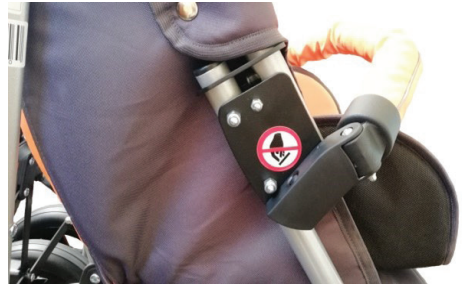**Fig 15.** Front guard lock button (closed)

- By using the snaps attach the upholstery belt to the front guard. But first you need to fasten abduction wedge to the upholstery belt.
- In the same way table can be attached to the stroller (Figure 16).
- Abduction wedge is connected to velcro which should be wrapped around the upholstery waist (Figure 16.).

**Fig 16.** Abduction wedge, table and upholstery waist assembled

- Then install the backrest. Unfasten the velcro fasteners of head stabilizer at the rear part of the upholstery, select the appropriate location of the head stabilizer. Drag velcro headrest through the holes in the top of the backrest upholstery, then fasten velcro.
- To install the canopy put the plastic ends into the guides on the stroller frame in the direction from top to bottom. (Figures 17., 18.)
- The footrest should be lowered on the footrest frame and push it until total blockade.

**Fig 17.** Canopy lock button (open)

**Fig 18.** Canopy lock (closed)

- The back part of the canopy should be located on the outer side of the backrest. Velcro strips on both sides of the canopy should be fastened on the wheelchair frame under the handles.

**Fig 19.** Canopy assembled to the stroller

**Fig 20.** Basket assembled to the stroller

- Basket for small items must be installed between the seat and the cross frame. (Figure 20.) The basket is equipped with velcro to be wrapped around four the furthest located stroller frame.

## FOLDING THE STROLLER

- To fold the stroller, unclip the footrest from one side. Unfasten the upholstery belt. Slide rail in the direction from the bottom to the top by pressing at the same time two front guard lock buttons located at both ends of the rail.
- In order to remove the canopy, press the canopy lock button and slide in the direction from the bottom to the top.
- Unlock two side lock at both side.
- Unlock the rear lock of the stroller by lifting it upwards.
- Now you are ready to fold the stroller by directing the handle to the front wheels and by directing the side frames to each other to the point where the stroller will be looking like as shown in Figure 7.
- The last stage is fastening a safety belt.

**WARNING! Failure to follow these guidelines can result in damage to the wheelchair.**

## 6. SAFETY OF USE

- a) Before using the stroller, refer to this manual and keep it for future reference.
- b) Your child's safety may be compromised case of non-compliance of rules contained in this manual.
- c) Never leave a child unattended. Always watch the child when he/she is in the stroller. Caretakers are responsible for the safety of the child.
- d) The seat is not suitable for children under 6 months.
- e) Always use the seat belts to avoid serious injuries caused by the child falling out or slipping out of the stroller.
- f) It is important to fasten seat belt properly and its adjustment to the user.
- g) The seat and belts do not replace proper supervision of caretaker.
- h) While adjusting the parameters of the stroller, the child should not be in a stroller.
- i) All repairs must be carried out by qualified personnel. And during the warranty period only by the manufacturer's service.
- j) The stroller is not suitable for running or ice-skating and roller skating.
- k) The stroller is designed for transporting only one child.
- l) It is unacceptable to use the stroller, if you found visible or palpable damage, or worn out parts and stroller parts indicates a potential risk of an accident (e.g. Consumption wheels, upholstery, cracks, etc.).
- m) Always check before use if the stroller is properly unfolded. Ensure that all locks blocked before use.
- n) To avoid injury to make sure that the child is far away when the product is being folded and unfolded.
- o) When moving the stroller through doors and other passages, make sure that on both sides there is enough space to avoid injury of hands or damage the stroller.
- p) Do not allow your child to play with this product.
- q) Do not allow your child to stand on the stroller or footrest. The stroller might fall over and hurt the child.
- r) When in motion, always hold the handle of the stroller.
- s) Be especially careful while using the wheelchair near to steps and stairs.
- t) The stroller is not intended for use on stairs and escalators.
- u) Do not to move the stroller if a child is sitting in it.
- v) Do not to move the stroller by holding the front guard. If there is a need to move your stroller, you should hold the fixed elements of the frame.
- w) In order to avoid accidental rolling off the stroller, always put on the brakes and turn on the lock of driving direction when the stroller is stopped, when patient get in, get off, changing the position.


- x) Do not let the child stand on the footrest. Never leave the stroller unattended on sloping ground, even if the brakes are on.
- y) The stroller is designed for children up to 30 kg (Umbrella 1), 45 kg (Umbrella 2) and 75 (Umbrella 3).
- z) Stroller should always be easy to fold out. In case of problems with unfolding. You should re-read the user's manual or contact the manufacturer.
- aa) Keep extreme caution while passing an obstacle. Extreme use of the stroller could cause damage or destroy it.
- bb) Never use the stroller as a seat for the transportation of people in cars, buses or any motor vehicles.
- cc) During transport (e.g. In the trunk), do not burden stroller, it may weaken or damage its structure.
- dd) Do not leave the stroller with the user close to open sources of fire
- ee) Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of stroller. It may cause burns or frostbite parts of the body.
- ff) Keep caution while moving on sloping surfaces. The maximum allowable slope angle up and down is 5%.
- gg) With the stroller you can only use on surfaces where all four wheels touching the ground are to use all of four wheels.
- hh) Stroller is designed for use indoors and outdoors only on paved and even surfaces.
- ii) Use of the product i.e. The icy, wet, greasy, strongly gritty surfaces is unacceptable - it creates a risk of falling.
- jj) It is extremely important to keep all strollers' components clean in order to maintain the product parameters.
- kk) Do not add additional loads (bags, etc.) on the stroller handles.
- ll) Storing the stroller in areas with very high humidity (baths, laundry, etc.) or in the open air, also the lack of hygiene causes rapid loss of operating parameters and aesthetic of the product.
- mm) Do not leave the child in the stroller for longer, because persons with neurological affections who stay in unchanged positions may experience problems with skin nourishment and develop bedsores.
- nn) When the footrests will not be removed before folding the stroller, it can cause damage.
- oo) During use, handing of the product and during its assembly, regulating mechanisms, it might happen trapping and / or squeeze user's or caretaker's body part in holes / gaps between the elements.
- pp) Perform these steps with extreme caution. After adjustment, stabilize the position by carefully tightening the nuts / bolts.

## 7. EVERYDAY USE AND MAINTENANCE

### BRAKE AND TWISTING MOTION LOCK

- The rear wheels are equipped with brake system.
- To prevent unplanned movement of the stroller (Umbrella 1, Umbrella 2), switch on brakes on two rear wheels. (Figure 21A.).
- To do this, press the foot brake, located at the both rear wheels.
- It is necessary to make sure that the brake lever has been launched on both wheels.
- To unlock the brake, lift up the brake lever.
- To prevent unplanned movement of the stroller (Umbrella 3), push on horizontal brakes on two rear wheels. (Figure 21B.).
- It is necessary to make sure that the brake lever has been launched on both wheels.
- To unlock the brake, push the horizontal brake levers, toward the centre of stroller.

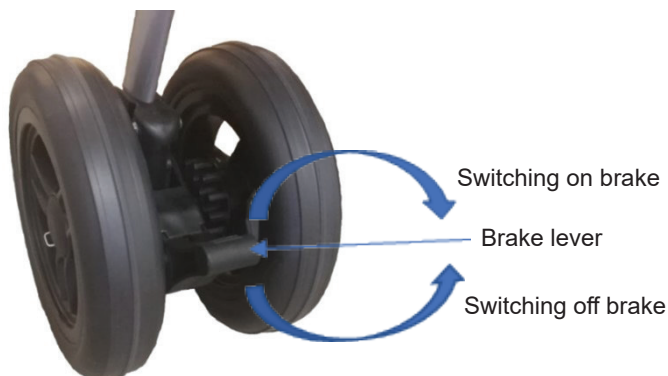**Fig 21A.** Rear brake mechanism (Umbrella 1 & 2)

**Fig 21B.** Rear brake lever mechanism - unlocked & locked (Umbrella 3)

- It is also possible to lock driving direction on the front wheel - drive only straight ahead.
- To lock of driving direction of the front wheel must be set in direction of travel ahead.
- On the two front wheels turn the lock pin of driving direction from the position shown in Figure 22. to the position shown in Figure 23. to the point where the pin is placed in a lock.
- Then check if the wheels do not twist.
- At this point the turning movement of the front wheels is blocked.

**Fig 22.** Lock pin of driving direction (off)

**Fig 23.** Lock pin of driving direction (on)

- To allow again turning movement of the front wheels pull the lock pin of driving direction on both wheels and turn it by 90 degrees from the position shown in Figure 23. to the position shown in Figure 22.

## LENGTH OF THE FOOTREST ADJUSTMENT

- The footrest has a five stage length adjustment. Every 25 mm.
- To change the length of the footrest, unscrew the knob of length adjustment, remove the black plate securing the screw, and then unscrew the screw (Figure 24.)
- Select required length of the footrest, by using the holes in the footrest frame.
- Next, insert the screws into the holes, screw the footrests length adjustment knobs and secure them with the black plates.
- It is important that each side of the screws were placed in holes at the same height.

**Fig 24.** Footrest length adjustment knob

## ADJUSTMENT OF THE HEADREST SETTING

- The stroller is equipped with a headrest that can be placed in two positions.
- To change the position of the head stabilizer, unfasten the velcro fasteners of head stabilizer at the rear part of the upholstery, select the appropriate location of the head stabilizer. Drag velcro headrest through the holes in the top of the backrest upholstery, then fasten velcro.

## ADJUSTMENT OF THE BACKREST TILT ANGLE


- To change angle of the BACKREST TILT, adjust the belts of backrest adjustment. (Figure 25.) To do this, press the button on the buckle and slightly tilt it to the rear (Figure 25). The backrest will start to fall. When you set a desired angle, release the button.
- If you want to tilt the backrest forward, press the button and pull the strap below at the same time.

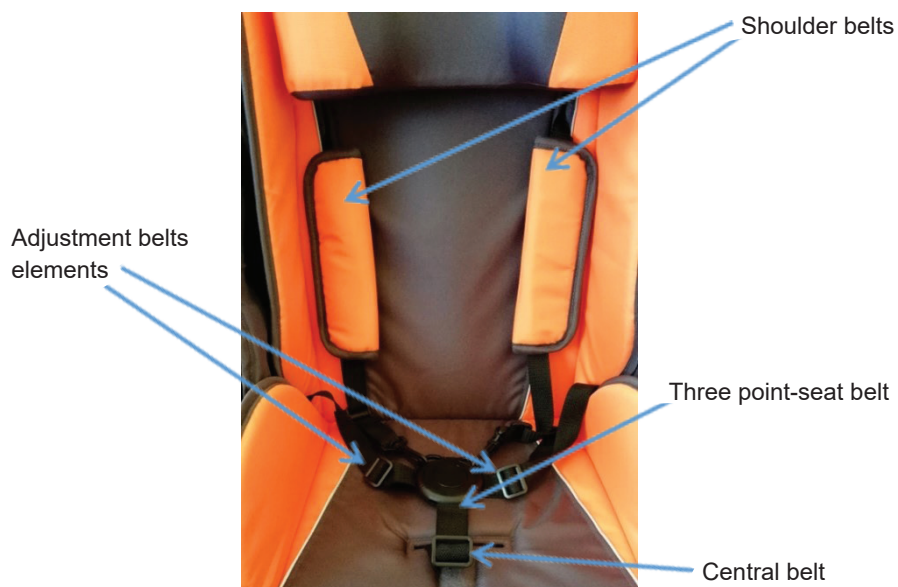**Fig 25.** Belts of backrest adjustment

## SECURITY SYSTEM

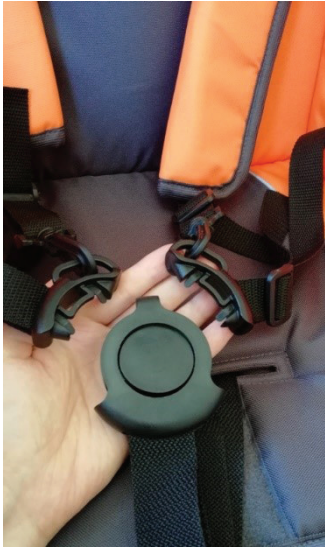
**Fig 26.** Security system

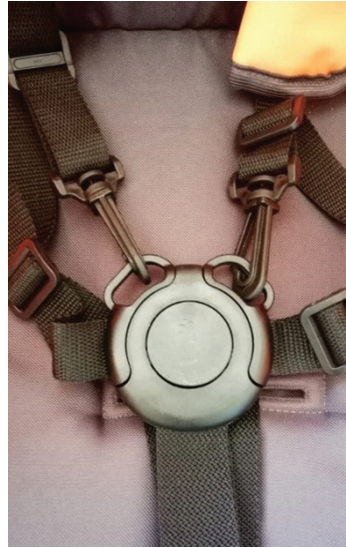
**Fig 26B.** Security system - head stabilizer, side pelottes with lumbar roller support

- The stroller is equipped with a stabilizing strap that can be placed in two different positions with length adjustment, abduction wedge head stabilizer and set of side pelottes with lumbar roller (Figures 26A. & 26B.)
- Settings of all stabilizing elements should be discussed with your doctor or physiotherapist.
- To change belt position, they should be removed from the current position at the top of the backrest upholstery and drag through the hole in the upper part of the backrest upholstery.
- To ensure safety, after placing a child in the stroller, caretaker should immediately buckle them into stabilizing belts.

**Fig 27.** Seatbelt (open)

**Fig 26.** Seatbelt (locked)

- Security system include five-point seat belts with central buckle. Shoulder belts should run along the child body from the shoulders to the waist. Both buckle elements should be placed in the central buckle. To do this, press the button on the buckle and place the elements (Figures 27., 28.)
- While placing elements of buckle into central buckle there must be a click. Only if click is heard, belts are properly fastened.
- To properly protect your child, adjust the length of the belt using the adjustment belt elements.
- To unfasten five-point belts, press the button on the central part of seat belt five-point.

## ASSEMBLING AND DISASSEMBLING OF THE REAR WHEELS

- To remove the rear wheel, press of quick connect handle towards the axis of the wheel, then pull wheel in the line of wheel axis.
- To install the wheel, press quick connect handle and slip the wheel in axle.
- Releasing the handle will lock the wheels on the axle.
- To mount or dismount the inner wheel, the stroller must be unfolded.
- Make sure the wheels after assembling are properly locked.
- For stroller Umbrella 3, use a standard ring wrench to unscrew the wheels.


**Fig 29.** Quick connect handle

## SETTING THE SLEEPING BAG

- Sleeping bag is equipped with a press stud, which should be connected to the press stud located at the bottom part of the canopy. The bottom part of the sleeping bag should hitch onto footrest.

**Fig 30.** Sleeping bag setting

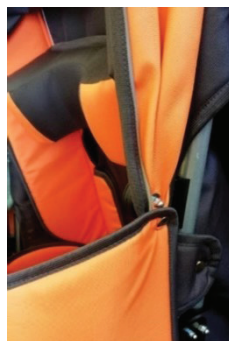**Fig 31.** Sleeping bag (bottom part of the canopy)

## CUP ASSEMBLY

- To mount the cup to the stroller frame, attach the back of the cup to the handle in the frame as shown in Figure 31A., 31B.

Fig 31A. Cup assembly

Fig 31B. Cup assembled correctly

## MOVING ON SLOPES

- While moving on slopes, use caution and reduce your speed.
- Do not move on too high slopes. The maximum allowable slope angle up and down is 5%

## ENTRANCE TO THE TRESHOLD OF CURBS

- To enter stroller to the threshold or curb, it must be set so that the front wheels were set on straight ahead threshold or curb, and near to it.
- Then place a foot on the lock back of stroller, press lightly and push the handle down (Figure 32.)
- The front wheels of stroller will lift slightly upwards, what allows to overcome the obstacles. Once the front wheels are on the obstacle, raise the rear wheels by tilting the stroller forward.


Lock back of stroller


Fig 32. Lock back mechanism position

## PUMPING TYRES

(It applies to stroller fitted with wheels pumped)

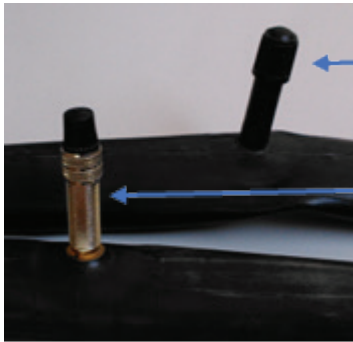**Fig 33.** Valves with cover

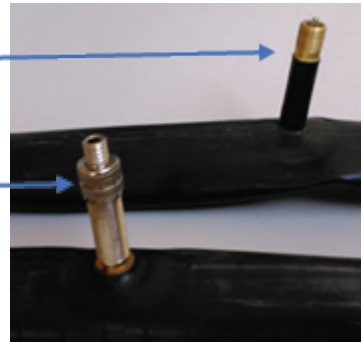
**Fig 34.** Valves w/o cover

- Before pumping the tires, identify the type of valve which is located on the tire. This may be one of the two valves shown in Illustration 33 and 34.
- Presta valve also has an additional small top, which must be loosened before pumping.
- After identifying the valve, prepare the pump designed to valve type
- Most of the new pumps are equipped to service both Presta valve and Schrader. Many of them let to reverse the tip. Smaller hole is Presta valve, Schrader valve greater. If the pump does not support Presta, you will need to buy for yourself the „adapter Presta”, which is available at any bike store.
- At this point, if the wheel is provided with a valve Presta, it will be necessary to unscrew the top located on the vent. It is a small top, which can be unscrewed with fingers. When doing this, the top should move upwards. Then valve is considered open.
- Then remove the plastic top from the valve and set it aside in a safe place. It is important to not lose it, because the valves will get dirt, leading to difficulties in pumping and rapid air loss.
- Place the nozzle of the pump to the valve and check if it stick properly. If it is needed, push the pump to the valve to make sure that the air does not escape to the sides.
- If the pump has a lever with clamp at the nozzle, tighten it. If the nozzle of the pump and valve have a threaded tip, screw them together. As a result, air will be streamed directly to the inner tube and the nozzle cannot detach from the valve.
- At the beginning of pumping the pressure in tyres should be checked at some points. Check tire pressure with your thumbs at the end of the tread. If it is possible, make only a millimetre indentation. It means that the tire has enough air. If necessary, continue pumping.
- After pumping, remove or unscrew the nozzle from the valve outlet while compressing it's ends by thumb. It is recommended to do this quick enough in order to not lose too much air.
- For Presta valve, tighten previously loosened additional small top.
- Screw the plastic top on the valve.

## ADJUSTING THE FOOTREST POSITIONS

(It applies to stroller fitted with adjustable footrest)

- In the upper part of the FOOTREST frame there is footrest adjustment button (Figure 35.).
- When the button is pressed, you can adjust the position of the footrests.
- It should in the same time press two buttons on either side of the footrest.
- After setting correct position, release the buttons.

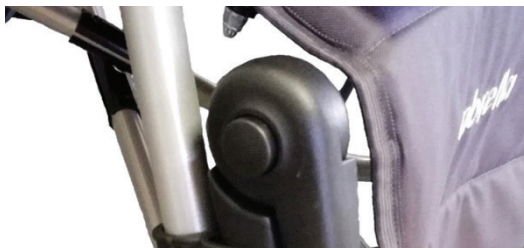**Fig 35.** Footrest adjustment mechanism

**Fig 36.** Footrest adjustment mechanism

**Fig 37.** Footrest adjustment mechanism


# HANDLES ADJUSTMENT

To turn the handles, pull out the safety clip (Figures 38., 39.) and then turn the handle.

**Fig 38.** Safety clip locked

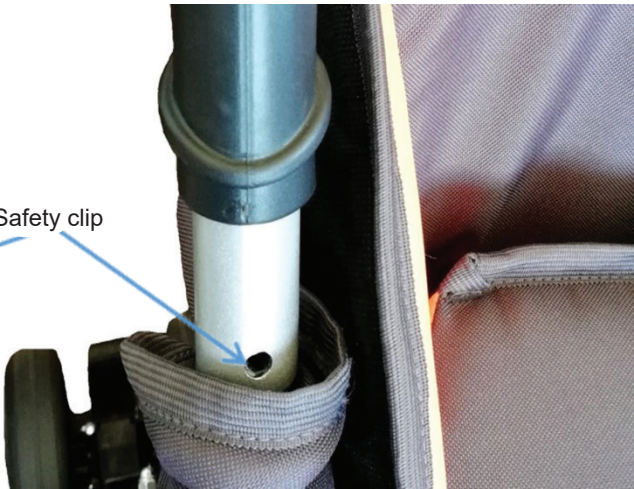**Fig 38.** Safety clip unlocked

- After the adjustment (correction), place the retaining clip in its original position (Figure 40.).

**Fig 40.** Safety clip locked after adjustment

# MAINTENANCE AND CONSERVATION

- Clean the plastic parts and the frame with damp cloth without the addition of detergents or other similar ones.
- Systematically remove the dirt from the movable elements of the stroller (screws, resistance mechanisms).
- Regularly remove dirt from the upholstery with a damp cloth.
- Keep clean all strollers' elements for to save functional parameters of the product.
- Storing in rooms of very high humidity (baths, laundries etc.) or in the open air, in the combination with the lack of hygiene will cause the fast loss of functional and aesthetic parameters. Do not expose the wheelchair on to extreme weather conditions (rain, snow very strong sun), because certain parts can change color.
- The stroller has coverage (coverage is part of the upholstery having the closest contact with the child, does not have stiffeners), which can be washed by hand at a 30°.

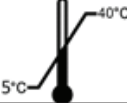
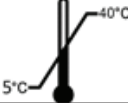
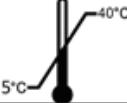
**You must not use:**

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,
- clean the stroller with pressure or steam aggregates.

Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Before each use	Control of wheels, brakes, the lock of driving direction	The brakes can wear out, check the condition of the brakes and the lock of driving direction before each use. You should also check the condition of wheels and clean them from dust and sand.
Every two weeks	Screw and nut control regarding their correct fixing.	Make sure if all the screws and top are fixed tightly.
Every two months	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.
Once a month	Wheel axles	Wheel axles must be conserved, oil between the axle and the hub a few drops of lubricant.


# 8. TECHNICAL PARAMETERS

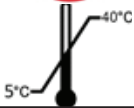Parameter	Umbrella 1	Umbrella 2	Umbrella 3
Permitted user weight	30 kg	45 kg	75 kg
Maximum load of basket for small items	1 kg	1 kg	1 kg
Maximum stroller width	543 mm	580 mm	623 mm
Maximum stroller length	875-888 mm	930-1120 mm	935-1215 mm
Maximum stroller height	1130 mm	1110/1165 mm	1248 mm
Dimensions of the stroller folded with wheels	1230x410x435 mm	1190x430x400 mm	1255x435x570 mm
Seat width	180/290 mm	350 mm	240/360 mm
Seat depth	270 mm	300 mm	400 mm
Backrest height	510 mm	530 mm	720 mm
Front wheels diameter	184 mm	177 mm	178 mm
Rear wheels diameter	233 mm	226 mm	253 mm
Weight of the stroller	13,9 kg	14, 85 kg	19,55 kg
Height of handles above the ground	1080 mm	1060 mm	1060/1200 mm
Footrest version	footrest with 5-step height adjustment	footrest with 6-step height adjustment	footrest with 7-step height adjustment
Type of wheels	Full, rear double, front single	Full, rear double, front single	Pumpred rear single, front single
Material	Aluminum, steel, plastic, rubber	Aluminum, steel, plastic, rubber	Aluminum, steel, plastic, rubber
Maximum safe slope			
Description of the environment in which the product will be used			

temperature of storage and use			
The humidity for the storage and use	30% - 70%	30% - 70%	30% - 70%
Terms of washing upholstery			
Serial number	Given on the product	Given on the product	Given on the product

## 9. SYMBOLS

	Confirmation of product compliance with EU standards
	Production date
	Manufacturer
	See the instructions for use
	WARNING
	Catalogue number
	Serial number
	Protect from moisture
	Protect from sunlight
	Permissible user weight

	Indoor use only
	Possibility of fingers jamming
	Temperature of storage and use
	Maximum safe slope
	The product is intended to transport by car
	The product is not designed for use as a seat to transport in a motor vehicle
	The product is intended to the transport by plane
	It is recommended to do a short and careful hand wash at 30 ° C, do not use bleach, do not iron, do not dry clean, do not use stain removers containing solvent, not use tumble dryer
	Medical device

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