UMBRELLA

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
English version on next page

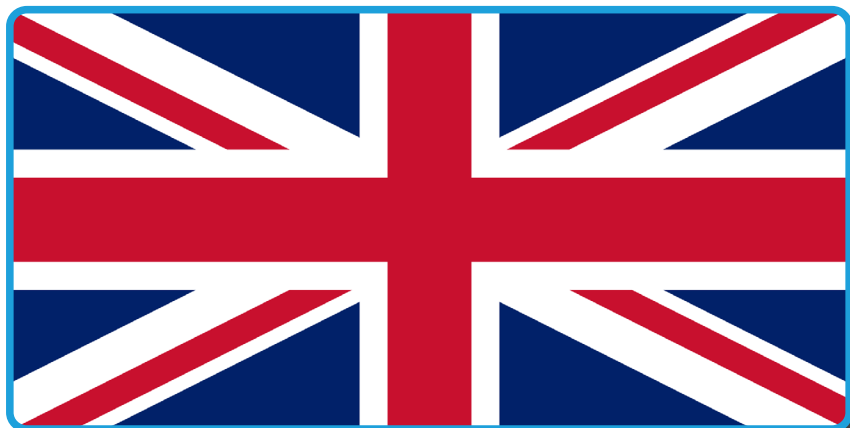Visually impaired people can find PDF files and other information about our products on our website:

<www.viteacare.com>.

👉 Please contact a specialist dealer if necessary.

A visually impaired user can also ask an accompanying person to read the instructions.

Table of contents

Introduction	33
Product characteristics	34
Indications / intended use	34
Contraindications.....	34
Safety of use.....	34
Patient safety	35
Product description.....	37
Preparing the stroller for use.....	40
Assembly and adjustment	40
Unfolding the stroller	40
Footrest length adjustment	43
Footrest position adjustment	44
Headrest adjustment	44
Backrest angle adjustment	44
Guide handle adjustment	45
Installation and removal of rear wheels.....	45
Use	46
Safety systems.....	46
Inflating tyres.....	48
Putting on the sleeping bag	49
Assembly of the cup.....	49
Moving on sloping surfaces	49
Driving up and down thresholds or kerbs	50
Folding the stroller	50
Technical parameters and accessories	50
Reporting technical defects and repairing the device	52
Care and maintenance	52
Cleaning	52
Storage.....	52
Disinfection	53
Product lifetime	53
Product disposal	53
Symbols	54
Contact	56

Introduction

Thank you for choosing our product. We strongly believe that it will meet your expectations. Please read this document before first use. These instructions are provided to give the necessary information on safe use of the device. Proper adherence to the recommendations will significantly contribute to prolonging the lifetime and aesthetics of the device. Ensure that the “warranty card” document is completed, attached and handed in to the distributor, as this will ensure the best service.

ATTENTION!

When using the device, please always observe the basic precautions according to the below information regarding safety. Read the entire instructions carefully before use. Please retain these instructions for use for future reference.

WARNING!

Improper use can lead to death, serious injury, dangerous conditions or malfunction of the product.

ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

SUGGESTIONS!

Follow the instructions below to keep the product in good working condition.

If the product is used by another person, provide them with a copy of these instructions for use.

Instructions for use refer to the design and specifications of the product at the time of publication. Due to design changes, some of the figures and pictures in the instructions for use may not correspond to the product purchased. We reserve the right to make design changes.

The manufacturer disclaims any liability for personal injury or property damage that may result from improper or unsafe use of this product.

Product characteristics

ATTENTION!

Children Rehabilitative Stroller Umbrella is available in three sizes, referred to as Umbrella 1, Umbrella 2 and Umbrella 3. Please read the Technical parameters section for details.

Indications / intended use

The stroller is designed for children with disabilities and / or dysfunctions movement activities as a result of damage of the CNS Central Nervous System. It can be used in children with CP Cerebral Palsy, myelomeningocele, muscle disorders, dysfunctions of the lower limbs.

Contraindications

Contradictions against sitting position.

Safety of use

WARNING!

Each "serious incident" related to the product shall be reported to the manufacturer and the competent authority of the Member State in which the user resides.

WARNING!

The product must not be modified in any way; this will void the warranty.

WARNING!

Exceeding the maximum user weight will void the warranty. The manufacturer is not responsible for injuries or damage caused by failure to comply with the maximum user weight.

WARNING!

Please read these instructions for use before using the product and keep for future reference. Ensure that all information, recommendations and warnings in these instructions for use are fully understood.

ATTENTION!

The product is intended for disabled persons with movement disorders. If you are not sure if health permits to use the product, consult a health professional.

Patient safety

- a. The child's safety may be compromised if the rules in this manual are not followed.
- b. Never leave your child unattended. Always watch the child when he/she is in the stroller. Caretakers are responsible for the safety of the child.
- c. The seat is not suitable for children under 6 months.
- d. Always use the seat belts to avoid serious injuries caused by the child falling out or slipping out of the stroller.
- e. It is important to fasten seat belt properly and its adjustment to the user. The seat belts do not replace proper supervision of caretaker.
- f. When adjusting the various parameters of the stroller, the child should not remain in the stroller.
- g. All repairs must be performed by qualified personnel and, during the warranty period, by a technician designated by the manufacturer..
- h. The stroller is not suitable for running or ice-skating and roller skating. The stroller is only suitable for transporting one child.
- i. It is unacceptable to use the stroller, if you found visible or palpable damage, or worn out parts and stroller parts indicates a potential risk of an accident (e.g. Consumption wheels, upholstery, cracks, etc.).
- j. Always check before use if the stroller is properly unfolded. Ensure that all locks blocked before use.
- k. To avoid injuries, make sure the child is kept away from the stroller when the product is unfolded and folded.
- l. When moving the stroller through doors and other passageways, make sure there is enough space on both sides to avoid injuring your hands or damaging the stroller.
- m. Do not allow your child to play with this product.
- n. It is forbidden for the child to stand on the stroller and footrest. The stroller may tip over and injure the child. Always hold the stroller by the steering handles when driving.
- o. Be especially careful while using the stroller near to steps and stairs.
- p. The stroller is not intended for use on stairs and escalators..
- q. Do not to move the stroller if a child is sitting in it.
- r. Do not to move the stroller by holding the front guard. If there is a need to move your stroller, you should hold the fixed elements of the frame.
- s. In order to avoid accidental rolling off the stroller, always put on the brakes and turn on the lock of driving direction when the stroller is stopped, when patient get in, get off, changing the position. Never leave the stroller unattended on an incline, even if the brakes are locked.
- t. The stroller is designed for children up to 30 kg (Umbrella 1), 45 kg (Umbrella 2) and 75 kg (Umbrella 3).
- u. Stroller should always be easy to fold out. In case of problems with unfolding. You should re-read the instructions for use or contact the manufacturer.
- v. Keep extreme caution while passing an obstacle. Extreme use of the stroller could cause damage or destroy it.

- w. Special care should be taken when driving over obstacles.
- x. Never use the stroller as a seat for the transportation of people in cars, buses or any motor vehicles.
- y. Do not leave the stroller with the user close to open sources of fire.
- z. Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of stroller. It may cause burns or frostbite parts of the body.
- aa. Keep caution while moving on sloping surfaces. The maximum allowable slope angle up and down is 5%. (2,86°).
- ab. The stroller may only be used on surfaces where all four wheels touch the ground and there is sufficient contact to drive the wheels properly.
- ac. Stroller for indoor and outdoor use only on paved and even surfaces.
- ad. Use of the product i.e. The icy, wet, greasy, strongly gritty surfaces is unacceptable - it creates a risk of falling.
- ae. It is extremely important to keep all strollers' components clean in order to maintain the product parameters.
- af. Do not add additional loads (bags, etc.) on the stroller handles.
- ag. Storing the stroller in areas with very high humidity (baths, laundry, etc.) or in the open air, also the lack of hygiene causes rapid loss of operating parameters and aesthetic of the product.
- ah. Accessories not approved by the manufacturer, not shown in this manual, must not be used.
- ai. Do not leave the child in the stroller for longer, because persons with neurological affections who stay in unchanged positions may experience problems with skin nourishment and develop bedsores.
- aj. When the footrests will not be removed before folding the stroller, it can cause damage.
- ak. During use, handing of the product and during its assembly, regulating mechanisms, it might happen trapping and / or squeeze user's or caretaker's body part in holes / gaps between the elements. pp) Perform these steps with extreme caution. After adjustment, stabilize the position by carefully tightening the nuts / bolts.

Product description

Illustration 1. List of items - stroller front.

Illustration 2. List of items - stroller back.

Illustration 3. List of components - bottom of stroller.

Illustration 4. Contents of the package (excluding stroller frame).

Number	List of elements
1	Guide handle
2	Canopy
3 / 26	Head stabilizer
4	Backrest upholstery (with side pelottes and lumbar roller support)
5 / 27	Front guard
6 / 28	Abduction wedge
7	Calf stabilisation belt
8	Footrest
9	Front wheel rotation lock
10	Canopy holder
11	Side lock
12	Footrest angle adjustment button
13	Dual rear wheels (or single in Umbrella 3)
14	Rear wheel quick-release button
15	Footrest length adjustment knob
16	Window cover in the canopy
17	Cup holder connector
18	Backrest tilt adjustment lock
19	Rear locking handle to prevent folding of the stroller
20 / 33	Basket for small items
21	Brake levers on the rear wheels
22	Lower locking handle to prevent folding of the stroller
23	Butterfly screws for abduction wedge connection
24	Sleeping bag
25	Side pelottes and lumbar roller support
29	Table with handle (optional accessory)
30	Rain cover
31	Umbrella (optional accessory)
32	Cup holder

Preparing the stroller for use

The stroller is delivered in cardboard packaging and fragile parts are additionally protected against damage during transport. Unpack the product carefully to avoid any damage.

Assembly and adjustment

Unfolding the stroller

Illustration 5. Unfolding the stroller.

- When unfolding the stroller, start by pulling the stroller handles in the direction shown in illustration 5. For ease of use, you can hold the seat of the stroller with one hand.
- When the stroller is partially unfolded, press with your foot on the rear lock (Illustration 6.) until the stabilising elements of the stroller reach a horizontal position.
- To stabilise the frame of the stroller, spread the bottom lock located under the seat (illustration 7.). To do this, grasp the handle and move it towards the footrest until the cross mechanism of the stroller is fully locked.

Illustration 6. Unfolding the stroller - back.

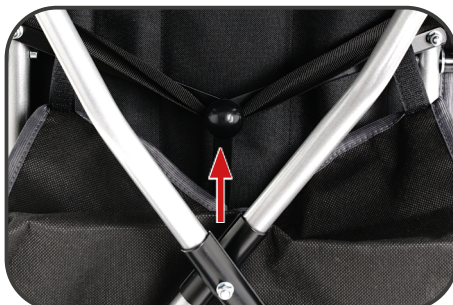
Illustration 7. Unfolding the stroller - bottom.

- The frame of the stroller has been fitted with a side lock. To lock the frame, slide the locking eyelet onto the profile end. (Illustration 8.).

Illustration 8. Frame folding lock.

- Lower the footrest onto the footrest frame and push it down until it locks completely (Illustration 9.).

Illustration 9. Footrest attachment.

- To attach the railing, slide the railing attachment into the guide located on the frame of the stroller in the direction shown (Illustration 10.).

Illustration 10. Front guard installation.

- To fit the abduction wedge, insert the screws into the holes located in the centre of the seat and then fasten from below with thumb screws (Illustration 11.).

Illustration 11. Installation of an abduction wedge.

- The headrest should then be fitted. To do this, unfasten the Velcro of the head stabiliser, select the desired setting and then pull the Velcro through the holes located in the upper part of the backrest upholstery and fasten it.

Illustration 12. Headrest assembly.

- To attach the canopy, slide the canopy attachment from top to bottom into the guide on the frame of the stroller (Illustration 13.).
- The back of the canopy should be on the outside of the backrest. The Velcro on either side of the canopy should be fastened to the frame of the stroller under the handles.

Illustration 13. Canopy assembly.

- To remove the canopy, press the red locking button (Illustration 14.) and then slide upwards from the guide.

Illustration 14. Removing the canopy.

- The small items basket should be fixed between the seat and the cross frame (Illustration 15.)
- The basket is equipped with Velcro to be wrapped around the four outermost frames.

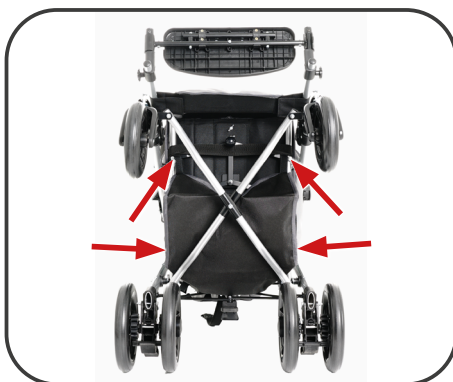
Illustration 15. Assembly of a basket for small items.

Footrest length adjustment

- The footrest has a five-stage length adjustment. The adjustment stroke is 25 mm.
- To change the length of the footrest, unscrew the knob, remove the black plate securing the bolt and then unscrew the bolt (Illustration 16.).
- Choose the right length for the footrest, which is made possible by holes in the footrest frame.
- Then insert the bolts into the holes, screw in the previously removed footrest length adjustment knobs and secure with black plates.
- It is important that the screws are in the holes at the same height on each side.

Illustration 16. Assembly of a basket for small items.

Footrest position adjustment

- At the top of the footrest frame there is a button to adjust the position of the footrest.
- By holding the button down you can adjust the position of the footrest (Illustration 17.).
- Press the two buttons located on both footrest frames simultaneously. Once the correct position has been set, release the buttons.

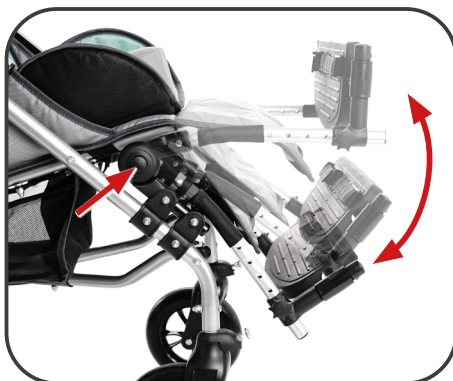
Illustration 17. Footrest position adjustment.

Headrest adjustment

- The stroller is equipped with a head stabiliser that can be placed in two positions.
- To change the position of the head stabiliser, unfasten the head stabiliser velcro (Illustration. 18.), located at the back of the upholstery, select the desired head stabiliser position, pull the head restraint velcro through the holes located at the top of the backrest upholstery and then fasten the velcro.

Illustration 18. Headrest adjustment straps.

Backrest angle adjustment

- To change the backrest angle, adjust the backrest adjustment straps accordingly.
- To do this, press the button on the buckle and tilt back slightly (Illustration 19.) supporting the backrest.
- Assure by supporting the backrest of the wheelchair with your hand, which will start to drop. When you have set the desired angle, let go of the button.
- If you want to raise the position forward, press the button and at the same time pull on the strap shown, supporting the backrest.

Illustration 19. Backrest angle adjustment straps.

Guide handle adjustment

- To turn the handles to guide the stroller, pull out the safety clamp (Illustration 20.) and then turn the handle.
- After the correction has been made, place the safety clamp in its original position.

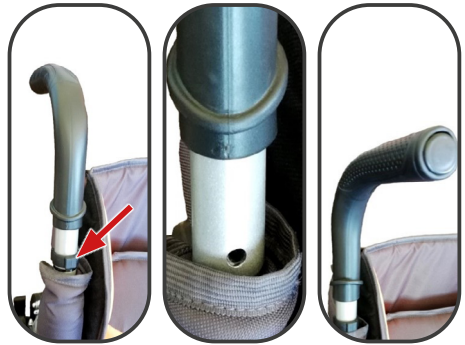
Illustration 20. Handle adjustment.

Installation and removal of rear wheels

- To remove the rear wheels, press the quick-release handle towards the wheel axle and pull it in line with the wheel axle.
- To fit the wheel, press the quick release handle and slide the wheel onto the axle (illustration 21.).
- Releasing the handle will lock the wheel on the axle.
- To fit or remove the inner wheels, the stroller must be unfolded.
- Ensure that the wheels are properly locked after fitting by pulling the wheel outwards against the respective side of the mounting stud.
- For the product labeled Umbrella 3, the wheel appears on one side of the axle.

Illustration 21. Wheel assembly.

Use

Safety systems

- To prevent unplanned movement of the stroller, lock the rotation of both rear wheels.
- To do this, press down on the brake lever located at the rear wheel with your foot..
- It is necessary to ensure that the brake lever is applied to both wheels.

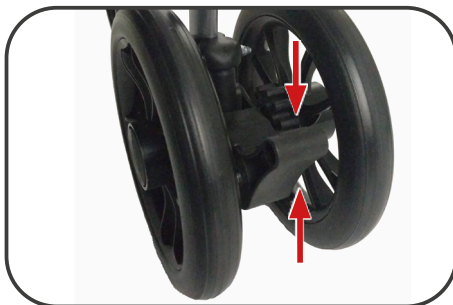
Illustration 22. Rear wheel lock (brake)
(Umbrella 1 i 2).

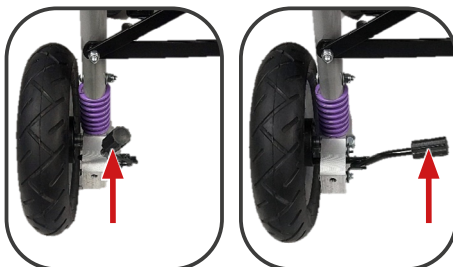
Illustration 23. Rear wheel lock (brake)
(Umbrella 3).

- It is also possible to lock the torsion movement of the front wheels - straight ahead only.
- To lock the swivel movement of the wheels, both wheels must be set to the straight-ahead position.
- On the two front wheels, turn the torsion bar locking pin to the position shown in illustration 24.
- Then check that the wheels do not turn.

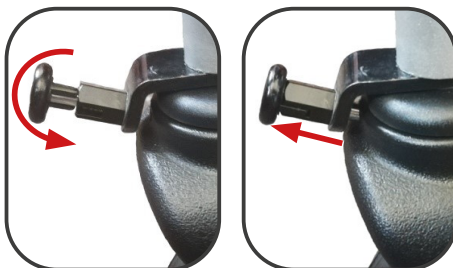
Illustration 24. Front wheel lock.

- The stroller is equipped with stabiliser belts, which can be placed in two different positions and are adjustable in length, as well as with side supports and a lumbar support roll bar (Illustration 25 and 26).
- The settings of all stabilising elements should be consulted with a doctor or physiotherapist.
- To change the position of the straps, remove them from their current position in the upper part of the backrest upholstery and pull them through the corresponding hole in the upper part of the backrest upholstery.
- To ensure safety, once the child is in the stroller, the child should be secured immediately with the stabilising straps.
- The shoulder straps should run the length of the child's torso from shoulder to waist.
- Both parts of the buckle should be placed in the central fastener (Illustration 25.).
- When inserting the buckle elements into the central fastening there must be a characteristic click, only then are the belts properly fastened.
- To properly secure your child, adjust the length of the straps using the belt tensions.
- To release the straps, press the button on the central fastening (buckles).

Illustration 25. Seat belts.

Illustration 26. Side panels and lumbar roller.

Inflating tyres

Refers to the product equipped with inflatable wheels.

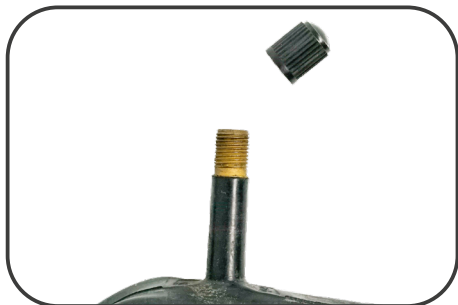
Illustration 27. „Schrader“ valve
(automotive)

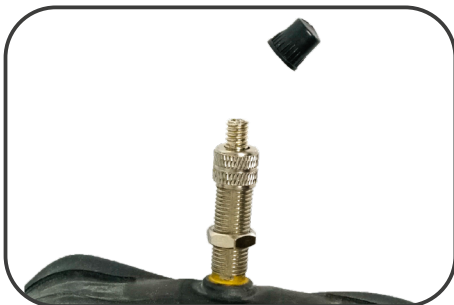
Illustration 28. „Woods“ valve
(bicycle, Dunlop)

- Before inflating tires, identify the type of valve that is located at the tire. It can be one of the two valves shown in illustration 14. and 15.
- Once you have identified the valve, prepare the pump designed for the type of valve installed on the product.
- Most new inflators are equipped with both car or bicycle type valve support. Many of them allow you to reverse the tip. The smaller hole facing outward will be for “Woods”(bicycle), the larger for “Schrader”(car). If the pump does not support a bicycle valve, you will need to equip yourself with the appropriate adapter, which is available at any bicycle store.
- Unscrew and remove the plastic cap from the valve and put it in a safe place. It is important not to lose it, as the valve will become dirty, leading to difficulty in pumping and faster loss of air.
- Apply the pump nozzle to the valve and check that it adheres properly. If necessary, it is advisable to press the pump against the valve so that air does not escape to the sides.
- If the pump has a lever with a clamp at the nozzle, clamp it. If the pump nozzle and the valve have a threaded end, they should be screwed into each other. This will ensure that the air will fall straight into the inner tube, and the nozzle will not detach from the valve.
- When proceeding to pumping, periodically check how much pressure is in the tires. The best way to do this is by pressing the tire with your thumbs at the end of the tread. If it is possible to make only a millimeter dent, it means that there is enough air in the tire. If necessary, continue pumping.
- When you have finished pumping, remove or unscrew the nozzle from the valve by pressing its outlet with your thumb. It is important to do this, smoothly enough so that not too much air escapes.
- Screw the plastic nut onto the valve.

ATTENTION!

Observe the correct degree of inflation of the wheels according to the sizes specified by the manufacturer on the tire pages.

Use a pump with a pressure gauge or a bicycle hand pump.

Putting on the sleeping bag

- The sleeping bag is equipped with a snap element, which must be connected to the snap at the bottom of the canopy.
- The lower part of the sleeping bag should be hooked onto the footrest.

Illustration 29. Sleeping bag fastening.

Assembly of the cup

- To attach the cup to the frame of the stroller, hook the back of the cup onto a bracket in the frame, as shown in illustration 30.

Illustration 30. Cup holder.

Moving on sloping surfaces

- When driving on sloping surfaces, take extra care and reduce speed.
- Do not move on excessive inclines. The maximum permissible upward and downward slope of the ground is 5% (2,86°).

Driving up and down thresholds or kerbs

- To get the stroller onto the threshold or kerb, position yourself so that the front wheels are facing the threshold or kerb and right next to it.
- Then place your foot on the rear lock of the stroller, press lightly and push the handles of the stroller downwards (Illustration 31.) - the front wheels of the stroller will lift up slightly, allowing you to overcome the obstacle.
- Lift the rear wheels as soon as the front wheels hit an obstacle and tilt the stroller forwards.

Illustration 31. Rear frame lock.

Folding the stroller

- To fold the stroller, unfasten the footrest on one side.
- Unbuckle the upholstery strap.
- Slide down the canopy from the bottom to the top by pressing both canopy lock buttons on both ends of the canopy at the same time.
- Remove the canopy by pressing the red canopy lock buttons and slide down in a downward to upward direction.
- Unlock both side locks.
- Unlock the rear locking mechanism of the stroller by lifting it upwards and the lower one by pointing it towards the guide handles.
- You can now proceed to fold the stroller by pointing the stroller handles towards the front wheels and steering the side frames towards each other until the stroller looks like on illustration 5.
- The final step is to fasten the safety belt (orange colour).

Technical parameters and accessories

Parameter	Umbrella 1	Umbrella 2 (till 12.2024)	Umbrella 2 (from 12.2024)	Umbrella 3
Maximum user weight	30 kg	45 kg	45 kg	75 kg
Maximum load for small shopping basket	1 kg	1 kg	1 kg	1 kg
Maximum stroller width	510 mm	590 mm	590	650 mm
Maximum stroller length	920-1050 mm	1000 mm	1150-1200 mm	1050-1340 mm
Maximum stroller height	1120 mm	1160 mm	1160 mm	1260 mm

Parameter	Umbrella 1	Umbrella 2 (till 12.2024)	Umbrella 2 (from 12.2024)	Umbrella 3
Dimensions of stroller folded with wheels (L x W x H)	1220x360x440 mm	1220x440x440 mm	1220x530x420 mm	1260x500x500 mm
Seat width	300 mm	350 mm	360 mm	400 mm
Seat depth	250 mm	300-340 mm	300-340 mm	400-450 mm
Backrest height	530 mm	540 mm	620 mm	750 mm
Diameter of front wheels	AIR-190mm PU-180 mm (+/-5mm)	AIR-190mm PU-180 mm (+/-5mm)	AIR-190mm PU-180 mm (+/-5mm)	AIR-190mm PU-200 mm (+/-5mm)
Diameter of rear wheels	AIR-240mm PU-230 mm	AIR-240mm PU-230 mm	AIR-240mm PU-230 mm	AIR-245mm PU-245 mm (+/-5mm)
Weight of the stroller	12,6 [kg] *	~15 [kg]	16 kg	18 [kg] *
Maximum user height	120 cm	140 cm	150 cm	170 cm
Footrest	footrest with 5-stage height adjustment (in 25 mm steps), with foot and shin straps from 15 cm to 27.5 cm	footrest with 6-position height adjustment (in 25 mm steps), with foot and shin straps from 21.5 cm to 34 cm	footrest with 6-stage height adjustment (in 25mm increments), with foot and shin straps from 19.5cm to 31cm	footrest with 7-stage height adjustment (in 25mm increments), with foot and shin straps from 27.5cm to 42.5cm
Wheels type	Full, rear double, front single	Full, rear double, front single	Full, rear double, front single	Inflatable rear single, front single
Materials	Aluminium, steel, plastic, rubber			
Maximum safe inclination				
Description of the environment in which the product will be used				
Storage and usage temperatures	5°C - 40°C			
Humidity for storage and use	30% - 70%			
Standard accessories	Canopy, shopping basket, cup holder, rain cover, sleeping bag for legs			
Optional accessories	Umbrella, table, railing upholstered in eco-leather			

*weights stroller without canopy, front guard, wedge, cover, on PU wheels (solid).

Reporting technical defects and repairing the device

If a defect or damage to the product is discovered during use or maintenance, it must be reported immediately to the distributor or manufacturer.

WARNING!

Under no circumstances should an unqualified person repair or alter the product or its design.

The only person allowed to make any repairs or adjustments is an employee authorised by the manufacturer.

Care and maintenance

Cleaning

- Paint surfaces should be cleaned with a cloth wet with water. The use of mild cleaning agents for medical devices is permitted..
- Upholstery should be cleaned and maintained with mild detergents. Avoid rubbing too hard so as not to damage the fabric.
- Upholstery should be washed at a temperature of up to 30°C in a bath containing mild chemicals used for washing personal linen or multi-coloured fabrics.

Do not use:

- bleaches,
- toilet cleaners,
- agents with a chlorine content,
- sharp brushes and hard objects,
- corrosive cleaning agents.

Do not clean the product with pressure or steam aggregates!

	Hand wash in 30°C
	Do not chlorinate or bleach
	Do not iron
	Do not dry-clean
	Do not mechanically dry

Storage

The product should be stored in a dry, ventilated room with a relative humidity of no more than 80%. Storage in rooms with very high humidity (baths, laundries, etc.), or in the open air, combined with a lack of hygiene, will result in a rapid loss of performance and aesthetics. Do not expose the stroller to extreme weather conditions (rain, snow very strong sun), as certain parts may change colour.

Disinfection

If the product is used by a large number of people (e.g. in a nursing home), commercially available disinfectants should be used. Before disinfection, the top surface must be cleaned. Only tested and approved disinfection agents may be used for disinfection. Information on recognised and approved disinfectants and disinfection methods can be obtained from the national health authority.

WARNING!

When using disinfectants, external surfaces can be damaged, which can limit the long term functionality of components. In doing so, the manufacturer's application instructions must be followed.

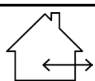
Product lifetime

The expected life of a medical device is 5 years, as long as it is used as intended and all maintenance and servicing is performed. The lifespan of a medical device also depends on the frequency of use, the environment in which it is used and its care. By using spare parts, the life of the product can be extended. Spare parts are generally still available for five years after the model is discontinued. This expected lifetime of a medical device does not imply an additional guarantee.

Product disposal

The product must be disposed of in accordance with applicable local and national regulations. For information on the proper disposal of packaging, metal components, plastic parts, neoprene, silicone and polyurethane materials, contact your local waste disposal company or authorised distributor.

Symbols

	Medical device
	Production date (printed on the product)
	Manufacturer
	Distributor
	Authorised representative in the EU
	Conformité Européene (conformity with EU requirements)
	Catalogue number (to re-order)
	Serial number
	LOT number
	Unique Device Identifier
	Consult instructions for use
	Warning
	Possibility of jamming fingers
	Permitted temperature
	Protect from moisture
	For indoor/outdoor use

	Keep away from sunlight
	The product is designed to be transported by car
	The product is not intended to be used as a seat for transport in a motor vehicle
	Maximum user weight

Contact

Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Poland

Phone No. +48 42 674 83 84, Fax +48 42 636 52 21

www.mdh.pl

www.viteacare.com

We reserve the right to make technical changes due to technical progress.