## Kule łokciowe

---

## Elbow crutches

---

Numer katalogowy/Catalog number: VCBP0042

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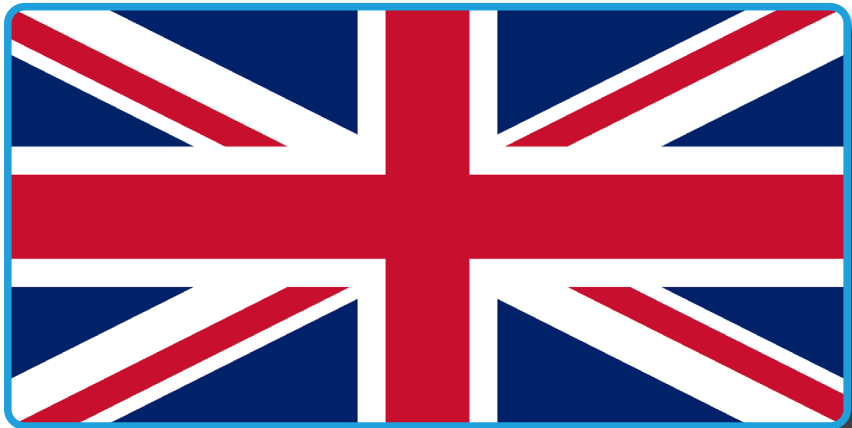
## Introduction

The elbow crutches are made of high quality, round cross section aluminium profiles. They are equipped with the half-elliptical forearm suspender that guarantees perfect elbow stability and the handle (hand grip), designed to fit anatomical shape of a hand and covered with soft in touch, pleasant material that prevents corns creation and with non-skid attachment element. Both the suspender and the handle are made of PU plastics, of especially selected hardness. The adjustments of crutch length and handle height can be performed by steps 25 mm, by means of friction-latch type regulator, stabilized with additional fastener.

## ATTENTION

Always follow the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Keep this manual for future reference.

### WARNING!

Improper use can lead to death, serious injury, unsafe conditions or product malfunction.

### ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

### SUGGESTIONS!

Follow the instructions below to keep your product in good working order.

If the product is used by another person, you must provide them with copies of these instructions for use.

These instructions for use relate to the design and specifications of the product at the time of publication. Due to design changes, some illustrations and photos in the manual may not correspond to the product you have purchased. We reserve the right to make design changes.

The manufacturer disclaims all liability for personal injury or property damage that may result from improper or unsafe use of this product.

## Indications

For persons who need assistance while walking, in order to suspend and move their body weight in case of injuries or orthopaedic disorders, such as fractures, strokes or paresis of lower limb, or while learning to walk.

## Contraindications

Coordination disorders, balance disorders, injuries or traumas of upper limbs that results in unavailability to use crutches.

## Product description

- Universal,
- Made of aluminum
- Suspenders and handle are made of PU plastic
- Height regulation every 25 mm
- Permitted user weight 140 kg
- Rubber made foot attachment: elastic, grey color (does not stain floors or parquets),
- Cat's eye mounted in the handle

## Usage

Crutches are shipped in its minimum length (retracted). After removing from package, crutches need to be set to the desired length (height). It is recommended that the height of the crutch is set so that the handle levels a greater trochanter of the thigh bone.

### WARNING!

Any "serious incident" related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

### ATTENTION!

The product is intended for use by people with disabilities and movement disorders. If you are not sure whether your health condition allows you to use the product, consult a healthcare professional.

## Safety of use

### WARNING!

Do not modify the product in any way; this will void the warranty.

### WARNING!

Exceeding the maximum user weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

1. Crutches are intended for use indoors and outdoors, on various surface types.
2. When using crutches on a surface that does not provide proper support, such as wet, icy, greasy or sandy surfaces, one should be extremely cautious.
3. The crutches are designed to support weights of up to 140 kg.
4. All the construction elements of the crutches are to undergo periodical visual inspection, including:
  - check of friction-latch mechanism of the crutches
  - check of non-skid rubber foot
5. In case of any visible or sensible damages of the crutches (such as cracks), one should not use them, which may result in an injury.
6. It is recommended that all the repairs where done by the qualified and experienced service personnel of the manufacturer.

For the product to maintain its proper characteristics, it is extremely important to maintain all its elements clean at all times and store it properly. Prolonged storing at premises of very high humidity (saunas, laundry rooms, etc.) or outdoors, along with lack of adequate maintenance, will lead to quick loss of the initial utility and aesthetical properties.

**WARNING!**

Maximum weight of user is 140 kg

## Technical parameters

Maximum user weight	140 kg
Usage period	3 lata od daty produkcji
Crutch height	990-1190 mm
Single crutch weight	0,56 kg
Gradual height adjustment	25 mm
Description of the environment in which the product will be used	  
Temperature of storage and use	Od +5°C do +40°C
The humidity for the storage and use	od 30% do 70 %
Materials	Aluminium, PU, guma(PTE)

## Service and repair

The review should be carried out regularly, it should be preceded by thorough cleaning the crutches (so as to reveal any hidden damage). If repairs are needed or if you have any questions about the product, please contact your distributor.

## Care and maintenance

### Cleaning

Aluminium profiles of the crutches are to be cleaned with a damp cloth.

- Hand grip and elbow suspender are to be cleaned with soft detergents,
- All moving elements of the crutches (friction-latch regulator) are to be removed immediately

#### **You must not use:**

- solvents
- toilet detergents
- sharp brushes and hard objects
- detergents that contain chlorine
- corrosive detergents

*Do not clean the crutch with pressure or steam aggregates!*

### Disinfection

If the product is used by several people (e.g. in a nursing home), commercially available disinfectants should be used. Before disinfecting, clean the upholstery and handles. Only tested and approved disinfectants may be used for disinfection.

Information on recognized and approved disinfectants and disinfection methods is available from your national health authority.

#### **WARNING!**

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.

### Product disposal

The product must be disposed of in accordance with applicable local and national regulations. For information on the proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, please contact your local waste disposal company or authorized distributor.

## Symbols

	Medical device
	Date of manufacture (printed on product)
	Manufacturer
	Distributor
	Conformité Européene (compliance with the European Community)
	Catalog number (to re-order)
	Serial number
	Lot code
	Unique Device Identifier
	Refer to the instructions for use.
	Warning
	Possibility of jamming fingers
	Protect from humidity
	Keep away from sunlight

	The product is designed to be transported by car
	Maximum user weight

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tel. +48 42 674 83 84, fax. +48 42 636 52 21

[www.mdh.pl](http://www.mdh.pl) [www.viteacare.com](http://www.viteacare.com)

We reserve the right to make technical changes due to technical progress.

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Erwin Kowsky GmbH & Co. KG