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Laska trójnóg VCBP0024

Wydanie EJ 06.2022

INSTRUCTIONS FOR USE

Quadruped cane VCBP0021

Tripod cane VCBP0024

Edition EJ 06.2022

VITEA[®] CARE

CE

Spis treści

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PLEASE READ CAREFULLY BEFORE
USING WHOLE OF INSTRUCTIONS

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The bottom part of the cane consists of steel and aluminium, covered with chrome construction and equipped with four legs (quadruped cane) and three legs (tripod cane) with non-slip gum strap. An aluminium tube with polyurethane handle builds the walking stick upper part. The handle is slipped into the frame which has 10 holes located every 2.5 cm. The holes enable the walking stick height regulation in a range from 73.5 to 96 cm, every 2.5 cm. A frictional-pawl regulator guarantees a required height.

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1. Universal (for both left and right hand);
2. Made of aluminium and steel;
3. Handle made of pu;
4. Height regulation every 2,5 cm;
5. Max user weight: 100 kg;
6. Gum strap: elastic, non-marking;

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For people who need support during walking, after lower limbs injuries, for balancing and body weight transferring in traumatic and orthopaedic diseases. Also for elderly people as a body balance support during walking.

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Contraindications to the upright position . Balance and motor coordination disorder, upper limb injuries and dysfunctions which preclude correct usage. Patients with coordination disorders, balance and intellectual disabilities should not be left without supervision.

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Product is packed folded. Take out of wrapping. Unfold to required height.

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MAINTENANCE AND SERVICE:

- Clean the cane with a moist cloth;
- Always use gentle detergents which do not damage
- the protective coat of the frame;
- Clean regularly all the mobile elements (pawl blockade mechanism).

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For the product to maintain its proper characteristics, it is extremely important to maintain all its elements clean at all times and store it properly. Prolonged storing at premises of very high humidity (saunas, laundry rooms, etc.) or outdoors, along with lack of adequate maintenance, will lead to quick loss of the initial utility and aesthetical properties.

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You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the cane with pressure or steam aggregates!

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TECHNICAL REVIEW

- The review should be carried out regularly, it should be preceded by thorough cleaning the cane (so as to reveal any hidden damage).

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Repair

- In case of reparation, please contact the producer.

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Customer service

- If you have any questions or need help, contact the producer.

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