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## 3-Wheel rollator

Wersja/Version: BILLY

Typ/Type: VCBK

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**English version** on next page

# Table of contents

<b>Introduction .....</b>	<b>17</b>
Indications.....	18
Contraindications .....	18
<b>Product description.....</b>	<b>18</b>
List of elements in the package.....	19
<b>Usage .....</b>	<b>19</b>
Handles height adjustment .....	20
Folding.....	20
Unfolding.....	21
Brake force adjustment .....	21
Brake system.....	22
Overcoming obstacles .....	22
<b>Safety instructions .....</b>	<b>22</b>
<b>Technical parameters .....</b>	<b>24</b>
<b>Service and repair.....</b>	<b>25</b>
<b>Care and maintenance .....</b>	<b>25</b>
Cleaning.....	25
Disinfection.....	25
Product disposal .....	26
Maintenance plan.....	26
<b>Symbols .....</b>	<b>27</b>
<b>Contact details .....</b>	<b>28</b>

## Introduction

Three wheel rollator with footrest, equipped with two fixed rear wheels with brake and locking the wheels and the one front wheels swivel. It lets you easily navigate through any surface, inside and outside the building. Handles anatomical made of polyethylene, aluminium frame. Make sure that you have filled the attached warranty form and that you have submitted it to the distributor, as this will enable us to ensure the best possible service.

# ATTENTION

Always follow the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Keep this manual for future reference.

### WARNING!

Improper use can lead to death, serious injury, unsafe conditions or product malfunction.

### ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

### SUGGESTIONS!

Follow the instructions below to keep your product in good working order.

If the product is used by another person, you must provide them with copies of these instructions for use.

These instructions for use relate to the design and specifications of the product at the time of publication. Due to design changes, some illustrations and photos in the manual may not correspond to the product you have purchased. We reserve the right to make design changes.

The manufacturer disclaims all liability for personal injury or property damage that may result from improper or unsafe use of this product.

## Indications

Rehabilitation supports are an ancillary equipment to verticalization and moving of patients with different types of motor disorders. They can be used as an aid for patients after surgery (lower limbs) who are unable to use crutches, showing no major coordination disorders, patients with mild dysfunction of the lower limbs or weakness. They are used in patients with in patients with impaired balance preventing them from independent vertical position and locomotion, as well as various types of circulatory dysfunction (e.g. intermittent claudication). Supports can also be used as an aid for geriatric patients.

## Contraindications

Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves, impaired coordination and perception.

## Product description

Figure 1

## List of elements in the package

Figure 2 - Tray for small items

Figure 3 - Rollator

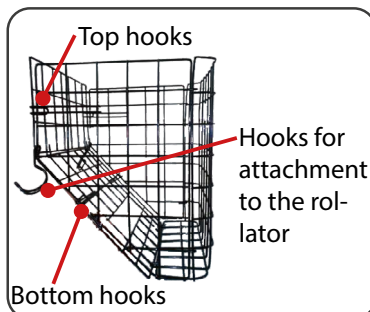
Figure 4 - Basket

Figure 5 - The mounting kit (two screws with knobs, two knobs with two screws and washer to mount handles)

## Usage

- The product is packaged in a form to be assembled.
- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above.
- Ensure that all components have no visible damage. If any doubts, please contact the distributor.
- Do not use the rollator until all parts are properly connected.
- Installation - Please refer to Figure 1.
- Begin to unfold the support by sliding outward the side frames, then lock the horizontal strut hinge. (Figure 6)

Figure 6

Horizontal strut hinge

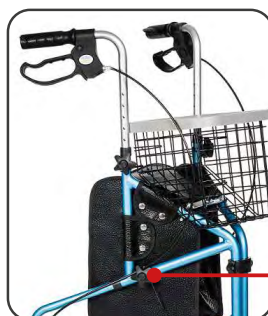
Figure 7

screw with knob

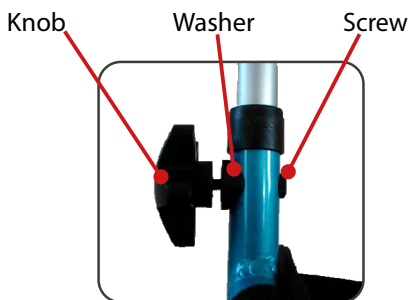
Figure 8

Figure 9

- Install handles in the holes of the side frames (Figure 7).
- From the inside of the side frame, insert the screw, while from the outside, put a washer on the screw and then tighten the knob (Figure 8).
- Then slide out the lower part of the frame to which the wheels are attached, until you hear the characteristic click of the latch.
- In addition, it is necessary to lock the frame.
- Tighten the knob firmly to the rollator frame (Figure 7).

Before connecting the basket to the rollator must be dismantled basket as shown on illustration (Figure 4).

- Hooks for fixing must be attached to the rollator side frames under knobs.
- Then place the tray on the basket so that the cut ends are on the side of the handles.
- To fold the support, proceed in reverse to the unfolding procedure shown.

## Handles height adjustment

- Handles should be located at a height that will provide comfort and a sense of security for the user. It is important to ensure that both hands are positioned at the same height.
- To change the height of the handles unscrew the knob for fixing handles, remove washers, the screws and depending on the needs extend or shorten the frame handles.
- After adjusting the appropriate height put the screw on the inner side of the side frame, while the outer side on the screw put pad, and then tighten the knob (Figure 8).
- Make sure that the handle is properly tightened, so that it prevents movement of the handles.

## Folding

- In order to fold rollator for ease of transport, you must first block rotation of the wheels
- Then it is necessary to remove the tray, basket for small items and emptied a bag for small items.
- Pull up horizontal hinge strut and then submit rollator, directing the handle to each other (Figure 9).

## Unfolding

- To unfold rollator slide outward side frames, then lock the levels hinge of stretcher.
- Before each use, check the operation of the braking system. Squeeze the brake handles to check if the brake pads are right adhere wheels and stop the rollator.
- Press the handle down to activate the locking rotation of the wheels and check if it works properly.

## Brake force adjustment

To adjust the brake, first, loosen a nut located above an adjusting screw of the brake just over the wheel. (Figure 10).

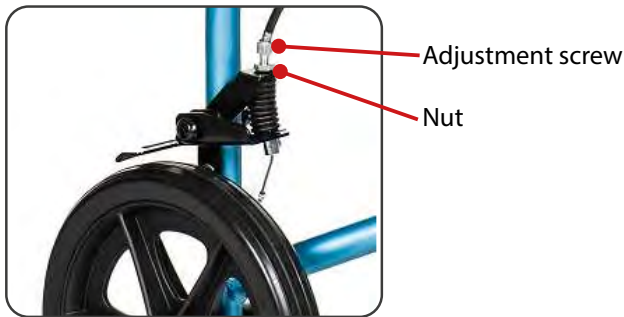
Figure 10

- Then turn the adjusting screw anticlockwise till a proper brake force is achieved.
- When the brake force is adjusted, tighten the nut.
- After adjusting the braking force, it is important to check the operation of the braking system.
- Squeeze the brake handle to check that the brake pads properly adhere and are able to stop rollator. Then pull the handles down to apply the parking brake and check if it works properly.

## Brake system

- The rollator is equipped with multi-functional brake. The brake handle is located under the rollator handle. It can operate as a standard brake as well as parking brake
- To use the standard brake, lift up the brake handle.
- To use a parking brake, push down the brake handle to the point where you can hear the distinctive sound of inclusion brake .
- Use in that same time both, the right and left brake.

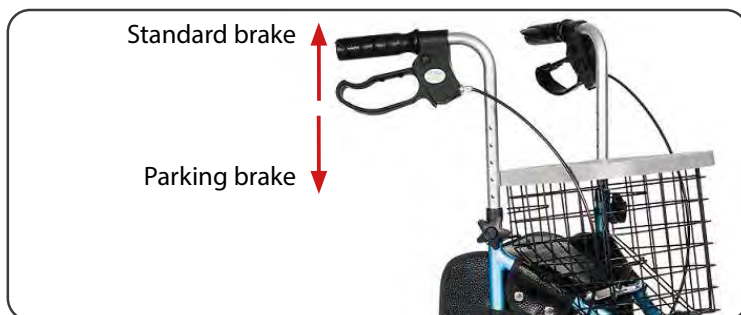
Figure 11

## Overcoming obstacles

- In order to drive the rollator onto the threshold or curb should set the rollator perpendicularly to the edge of the obstacle. Holding the handle, tilt the rollator in rear, raise the front wheels up.
- Then drive the feet to the edge of the obstacle and holding the rollator handle, lift slightly up and enter rollator on the obstacle.

## Safety instructions

### WARNING!

Any “serious incident” related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

### ATTENTION!

The product is intended for use by people with disabilities and movement disorders. If you are not sure whether your health condition allows you to use the product, consult a healthcare professional.

### WARNING!

Do not modify the product in any way; this will void the warranty.

### WARNING!

Exceeding the maximum user weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

**WARNING!**

Before using the rollator, refer to this manual and keep it for future reference.

- At standstill and the use of seat should apply the locking rotation of the wheels both rear wheels.
- The stand will not replace the wheelchair, the seat is used only for temporary resting or transport the patient on a short distance.
- The rollator is used only to provide rest to one user. Do not stand on it.
- The rollator should be used on even surfaces at an incline less than 5% (2,86°)
- With the rollator you can only use on surfaces where all four wheels touching the ground are to use all of four wheels.
- Use of the product on the icy, wet, greasy, strongly gritty surfaces is unacceptable - it creates a risk of falling.
- Before each use of this product:
  - Perform visual inspection of all structural components of the product,
  - Check the operation of the braking system. Squeeze the brake handles to see if the brake pads are fit properly to wheels and are able to stop rollator. Press the handles down to activate the parking brake and check if it works properly.
  - Carefully check the stability and condition of the support frame.
- Do not use the support in the event of any visible or perceptible damage (e.g. cracks in the frame). It may cause an accident.
- Maximum weight that can be moved in the bag - 5kg.
- Do not leave the stroller with the user close to open sources of fire.
- Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source.
- Furthermore, with low temperatures there is a danger of overcooling metal parts of stroller. It may cause burns or frostbite parts of the body.
- All repairs must be carried out by qualified personnel. And during the warranty period only by the manufacturer's service.
- Do not exceed the maximum height of the handles to push, rollator will lose stability.
- Push the rollator with both hands simultaneously in the direction of travel.
- Before using, make sure that all screws are tight.
- Make sure that the lock used to folding and unfolding that product work properly.
- Due to the possibility of jamming the fingers put extreme caution in the area of horizontal hinge strut, fold-out footrest and side frame hinge.
- Do not use the support in the event of any visible or perceptible damage (e.g. cracks in the frame). It may cause an accident.
- Maximum weight that can be moved in the bag - 5kg.
- Do not leave the stroller with the user close to open sources of fire.
- Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source.

- Furthermore, with low temperatures there is a danger of overcooling metal parts of stroller. It may cause burns or frostbite parts of the body.
- Do not exceed the maximum height of the handles to push, rollator will lose stability.
- Push the rollator with both hands simultaneously in the direction of travel.
- Before using, make sure that all screws are tight.
- Make sure that the lock used to folding and unfolding that product work properly.
- Due to the possibility of jamming the fingers put extreme caution in the area of horizontal hinge strut and side frame hinge.

#### WARNING!

Maximum user weight 120 kg

## Technical parameters

Maximum user weight	120 kg
Maximum bag, basket or tray load	5 kg
Rollator hight	800-930 mm
Hight regulation	8
Hight regulation scope	25 mm
Rollator width	680 mm
Rollator length	600 mm
Width when folded	270 mm
Rollator weight	5,3 kg
Minimum turning radius	64°
Wheels diameter	190 mm
Wheels type	Full
Maximum safe inclination	5% (2,86°)
Materials	Aluminium, polyethylene, rubber
Description of the environment in which the product will be used	  
Temperature of storage and use	Od +5°C do +40°C
The humidity for the storage and use	od 30% do 70 %

## Service and repair

The review should be carried out regularly, it should be preceded by thorough cleaning the product (so as to reveal any hidden damage). If repairs are needed or if you have any questions about the product, please contact your distributor.

## Care and maintenance

### Cleaning

- Clean the frame with a damp cloth, clean the seat with damp cloth with mild detergent.
- Remove all soiling from the movable components of the support (wheels, clamping screws, locks).
- In order to maintain the rollator parameters, it is very important to keep all elements clean and stored properly. Storing the rollator in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic of the rollator.
- Do not expose the rollator on to extreme weather conditions (rain, snow very strong sun), because certain parts can change color.

#### **It is not allowed to use:**

- solvents
- toilet detergents
- sharp brushes and hard objects
- detergents that contain chlorine
- corrosive detergents

*Do not clean the rollator with pressure or steam aggregates!*

### Disinfection

If the product is used by several people (e.g. in a nursing home), commercially available disinfectants should be used. Before disinfecting, clean the upholstery and handles. Only tested and approved disinfectants may be used for disinfection.

Information on recognized and approved disinfectants and disinfection methods is available from your national health authority.

#### **WARNING!**

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.


## Product disposal

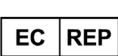The product must be disposed of in accordance with applicable local and national regulations. For information on the proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, please contact your local waste disposal company or authorized distributor.

## Maintenance plan

Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging	Check if any elements have any visible mechanical damage
Before each use	Wheel control	You should also check the condition of wheels and clean them from dust and sand.
Every two weeks	Screw and nut control regarding their correct fixing	Make sure if all the screws and top are fixed tightly
Every two months	Cleanliness and general condition	In case of mechanical damage contact the producer immediately
Once a month	Wheel axles	Wheel axles must be impregnated, between the axle and the hub put a few drops of lubricant

## Symbols

	Medical device
	Date of manufacture (printed on product)
	Manufacturer
	Distributor
	Authorized representative in the European Community
	Conformité Européene (compliance with the European Community)
	Catalog number (to re-order)
	Serial number
	Lot code
	Unique Device Identifier
	Refer to the instructions for use.
	Warning
	Possibility of jamming fingers
	Protect from humidity

	For indoor/outdoor use
	Keep away from sunlight
	Maximum safe inclination
	The product is designed to be transported by car
	The product is not intended to be used as a seat for transport in a motor vehicle
	Maximum user weight

## Contact details

### Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

[www.mdh.pl](http://www.mdh.pl) [www.viteacare.com](http://www.viteacare.com)

We reserve the right to make technical changes due to technical progress.