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Walker with ball

Wersja/Version: BOLA

Typ/Type: VCBP0032

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English version on next page

Table of contents

- Introduction 15**
 - Indications..... 16
 - Contraindications 16
- Product description..... 16**
- Usage 17**
 - Height adjustment..... 17
 - Moving by walker 17
- Safety instructions 18**
- Technical parameters 19**
- Service and repair..... 20**
- Care and maintenance 20**
 - Cleaning..... 20
 - Disinfection..... 20
 - Product disposal 21
 - Maintenance plan..... 21
- Symbols 22**
- Contact details..... 24**

Introduction

Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasise that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual. Please ensure that you complete the attached warranty form and return it to your distributor so that we can provide you with the best possible service.

ATTENTION

Always follow the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Keep this manual for future reference.

WARNING!

Improper use can lead to death, serious injury, unsafe conditions or product malfunction.

ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

SUGGESTIONS!

Follow the instructions below to keep your product in good working order.

If the product is used by another person, you must provide them with copies of these instructions for use.

These instructions for use relate to the design and specifications of the product at the time of publication. Due to design changes, some illustrations and photos in the manual may not correspond to the product you have purchased. We reserve the right to make design changes.

The manufacturer disclaims all liability for personal injury or property damage that may result from improper or unsafe use of this product.

Indications

Walking aid designed for disabled people for those with locomotor disorders. Ideal for rehabilitation process and physiotherapy (verticalisation, lower limb relief).

Contraindications

Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves.

Product description

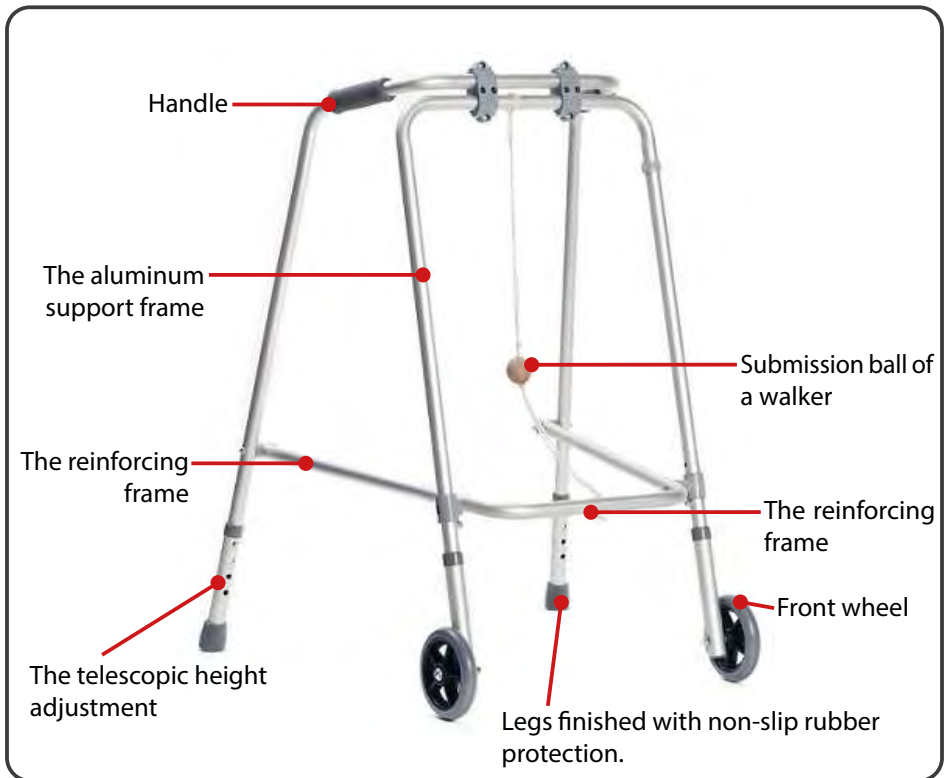
Illustration 1.

Usage

- The product is packaged in a form to be assembled.
- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above (Illustration 1).
- Ensure that all components have no visible damage. If any doubts, please contact the distributor.
- In order to unfold the walker, should move the bar which is placed below the front part of the frame bearer. Push the bar till it reaches horizontal position and leans against blocking element.
- For easy storage and transport, the walker can be folded.
- In order to fold the walker, pull the ball upwards. You can find the ball on the string attached to frame and the bar.

Height adjustment

- Before using a walker, should adjusted to the user's height by telescopically adjustable feet length.
- To do this, press the height adjustment pin of the legs and depending on the needs reducing the height of the legs or extend the leg (Illustration 2)

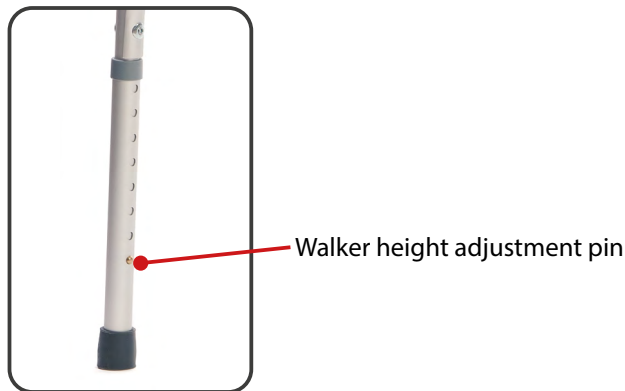
Illustration 2.

Moving by walker

- In the process of using a walker, the person must move walker forward and take a step forward, walk into the frame.

Safety instructions

WARNING!

Any “serious incident” related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

ATTENTION!

The product is intended for use by people with disabilities and movement disorders. If you are not sure whether your health condition allows you to use the product, consult a healthcare professional.

WARNING!

Do not modify the product in any way; this will void the warranty.

WARNING!

Exceeding the maximum user weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

WARNING!

Before using the walker, refer to this manual and keep it for future reference.

- Use of the product on the icy, wet, greasy, strongly gritty surfaces is unacceptable - it creates a risk of falling.
- The walker should be used on even surfaces at an incline less than 5% (2,860).
- Before each use of this product:
 - Perform visual inspection of all structural components of the product,
 - Carefully check the stability and condition of the support frame.
- Do not use the walker in the event of any visible or perceptible damage (e.g. cracks in the frame). It may cause an accident.
- The walker is designed for users with a maximum weight of 110 kg.
- The walker is design to support one person, climbing and standing on the walker is strictly forbidden.
- Do not leave the walker close to open sources of fire.
- Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of walker. It may cause burns or frostbite parts of the body.
- It is unacceptable to use the walker after discovering any visible or detectable damages (e.g. broken frame) as it may lead to accident.
- All the repairs needs to be performed by a skilled personnel.
- Do not exceed the maximum height.

- Keep walker with both hands simultaneously, in the direction of movement.
- Make sure that the lock used to folding and unfolding the walker is working properly.
- Manufacturer prohibits making any walker modifications on their own, especially by assembling additional wheels because product is not equipped with brakes. It might cause a fall or serious injuries. The manufacturer will not take responsibility for such actions.
- There is a risk of jamming fingers in the area for receipt of a walking frame.

WARNING!
Maximum user weight 110 kg.

Technical parameters

Maximum user weight	110 kg
Walker hight	850-920 mm
Walker width	650 mm
Walker weight	2 kg
The scope of control levels	4
Maximum safe inclination	5% (2,86°)
Materials	Aluminum, polyurethane, rubber
Description of the environment in which the product will be used	  
Permitted temperature	From +5°C to +40°C
The humidity for the storage and use	From 30% to 70 %

Service and repair

The review should be carried out regularly, it should be preceded by thorough cleaning the product (so as to reveal any hidden damage). If repairs are needed or if you have any questions about the product, please contact your distributor.

Care and maintenance

Cleaning

- Clean the frame with a damp cloth, clean the seat with damp cloth with mild detergent.
- Remove all soiling from the movable components of the walker (wheels, clamping screws, locks).
- In order to maintain the walker parameters, it is very important to keep all elements clean and stored properly. Storing the walker in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic of the walker.
- Do not expose the walker on to extreme weather conditions (rain, snow very strong sun), because certain parts can change color.

It is not allowed to use:

- solvents
- toilet detergents
- sharp brushes and hard objects
- detergents that contain chlorine
- corrosive detergents

Do not clean the walker with pressure or steam aggregates!

Disinfection

If the product is used by several people (e.g. in a nursing home), commercially available disinfectants should be used. Before disinfecting, clean the upholstery and handles. Only tested and approved disinfectants may be used for disinfection.

Information on recognized and approved disinfectants and disinfection methods is available from your national health authority.

WARNING!

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.


Product disposal

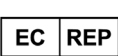The product must be disposed of in accordance with applicable local and national regulations. For information on the proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, please contact your local waste disposal company or authorized distributor.

Maintenance plan

Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging	Check if any elements have any visible mechanical damage
Before each use	Wheel control	Check the condition of wheels and clean them from dust and sand.
Every two weeks	Screw and nut control regarding their correct fixing	Make sure if all the screws and top are fixed tightly
Every two months	Cleanliness and general condition	In case of mechanical damage contact the producer immediately
Once a month	Wheel axles	Wheel axles must be impregnated, between the axle and the hub put a few drops of lubricant

Symbols

	Medical device
	Date of manufacture (printed on product)
	Manufacturer
	Distributor
	Authorized representative in the European Community
	Conformité Européene (compliance with the European Community)
	Catalog number (to re-order)
	Serial number
	Lot code
	Unique Device Identifier
	Refer to the instructions for use.
	Warning
	Possibility of jamming fingers
	Permitted temperature

	Protect from humidity
	For indoor/outdoor use
	Keep away from sunlight
	Maximum safe inclination
	The product is designed to be transported by car
	The product is not intended to be used as a seat for transport in a motor vehicle
	Maximum user weight

Contact details

Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

www.mdh.pl www.viteacare.com

We reserve the right to make technical changes due to technical progress.