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4-Wheel aluminum rollator

Wersja/Version: ROCKY

Typ/Type: VCBK

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Instructions for use

Wydanie/Edition: EJ 02.2023

przejdź do

go to

VITEA
CARE

Spis treści

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English version on next page

Table of contents

Introduction	18
Indications.....	19
Contraindications	19
Product description.....	19
List of elements in the package.....	20
Usage	20
Mounting and dismantling the rollator.....	20
Braking system	22
Adjustment of braking.....	23
Height adjustment of handles	23
Folding.....	24
Elbow crutches attachment.....	24
Overcoming obstacles (thresholds, kerbs)	24
Safety instructions	24
Technical specifications	26
Service and repair.....	27
Care and maintenance	27
Cleaning.....	27
Disinfection.....	27
Maintenance plan.....	28
Product disposal	28
Symbols.....	29
Contact details	32

Introduction

Folding four-wheel aluminum rollator, equipped with two fixed rear wheels with brake and two front swivel wheels. It allows easy movement on any surface, indoors or outdoors.

ATTENTION

Always observe the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Please keep these instructions for future use.

WARNING!

Improper use can lead to death, serious injury, dangerous conditions or product malfunction.

ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

SUGGESTIONS!

Follow the instructions below to keep the product in good working order.

If the device is used by another person, a copy of these instructions for use must be provided.

These instructions for use apply to the design and specification of the product at the time of publication. Due to design changes, some figures and photos in the manual may not correspond to the product purchased. We reserve the right to make design changes.

The manufacturer disclaims any liability for personal injury or property damage that may result from improper or unsafe use of this product.

Indications

Rehabilitation supports are assistive equipment that facilitate the uprighting and movement of patients with different types of motor disorders. They can be used to help patients after surgical procedures (limb operations lower limbs) unable to use crutches, without coordination disorders, patients with mild lower limb dysfunction or weakness. They are used for patients with balance disorders that make it impossible to independent upright standing and locomotion and various types of circulatory dysfunctions (e.g., stoppage chroma). Supports can also be used as a locomotion aid for geriatric patients.

Contraindications

Contraindications to upright standing, upper limb injuries and dysfunctions preventing support, coordination disorders.

Product description

Figure 1.

List of elements in the package

Figure 2. Elements that should be included in the package.

Usage

Mounting and dismounting the rollator

- The product is packaged for assembly.
- When removed from the packaging, check that it contains all the parts as listed above.
- Ensure that all components have no visible mechanical damage. If in doubt, contact your distributor.
- Do not use the support until all parts are properly connected.
- Assembly - please refer to Figure 1.
- Start unfolding the support by pulling the frame apart, then lock the horizontal expansion hinges by pressing down on the bottom of the small item bag.

Figure 3. Unfolding the frame.

Figure 4. Mounting locations for wheels.

Figure 5. Locking screws for seat adjustment.

- Fit the wheels into the lower holes at the side of the frame (Figure 4.).
- Adjust the appropriate height (2 positions available), the same for each of the four wheels.
- Insert a knob with a screw into the mounting hole on the outside of the frame and then tighten it (Figure 5.).
- Ensure that the wheels are securely fastened in the support frame.

Figure 6. Mounting points for guide handles.

Figure 7. Handle/guide bar locking screws and wheel mounting.

- Install handles in the upper holes of the side frames (Figure 6.).
- Adjust the appropriate height of the handles, the same for each handle.
- Insert a screw into the mounting hole, inside of the frame and a knob on the outside, then tighten them (Figure 7.).
- Ensure that the handle is securely fixed in the support frame.

- The handles should be at a height that provides comfort and a sense of security for the user. It is important to make sure that both handles are set at the same height.
- To change the height of the handles, unscrew the handle fixing knobs, remove the washers, remove the screws and slide the handle out or in of the frame as required.
- Once the correct height has been adjusted, tighten the knob.
- Ensure that the knob is properly tightened to prevent the handles from moving.

Figure 8. Rollator backrest.

- The backrest must be fitted in the holders provided for it on the side of the rollator frame (the clips on the backrest must snap into the holes provided in the holders).
- Once the backrest is fitted, pull it to make sure it is properly secured.

Figure 9. Backrest assembly.

WARNING!

Special care must be taken when folding the rollator as there is a risk of jamming fingers or hands.

Braking system

- The support is equipped with a multifunctional brake. The brake handle is located under the handle. It can operate as a standard brake and as a parking brake.
- To use the brake as standard, pull the brake handle upwards.
- To use the parking brake, push the brake handle down until you can hear the characteristic sound of the brake being applied (use the right and left brake at the same time). This will cause the rear wheels to lock.
- To unlock the wheels, pull the brake handle upwards.

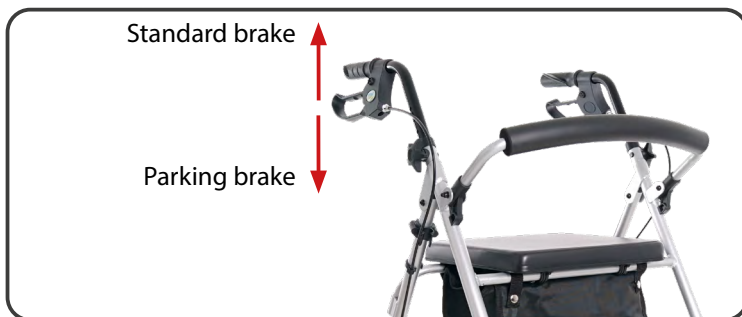
Figure 10. Brake operation.

Adjustment of braking

- To adjust the brake force, first loosen the nut located above the brake adjustment screw just above the wheel (Figure 11.).

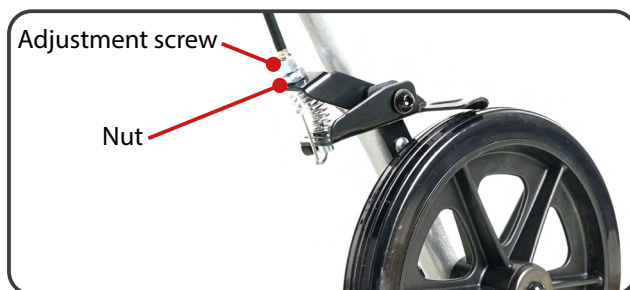
Figure 11. Brake adjustment.

- Then turn the adjustment screw counter-clockwise until the correct brake force is achieved.
- After adjusting the brake force, tighten the nut.
- After adjusting the brake force, it is important to check the function of the brake system.
- Squeeze the brake handle to check that the brake blocks fit correctly to the wheels and are able to stop, then pull the handles down to apply the parking brake and check that it works properly.

Height adjustment of handles

- The handles should be at a height that provides comfort and a sense of security for the user. It is important to make sure that both handles are set at the same height.
- To change the height of the handles, unscrew the handle fixing knobs, remove the washers, remove the screws and slide the handle frames out or in as required.
- Once the correct height has been adjusted, tighten the knob (Figure 7).
- Ensure that the knob is properly tightened to prevent movement of the handles.

Folding

- To fold the rollator for ease of transport, pull the horizontal expansion hinge upwards using a small amount of force.

Elbow crutches attachment

- On the right side of the frame there are two crutches holders: a lower and an upper one.
- Place the crutch in the bottom bracket and then push into the top handle
- Ensure that the crutch/stick is firmly seated in the holder and that it does not fall out of the holder when moving.

Overcoming obstacles (thresholds, kerbs)

- To drive the rollator over a threshold or kerb, bring to the edge of the obstacle, holding the handles, tilt back which will lift the front wheels up.
- After the front wheels have been placed on the obstacle, bring the rear wheels to the edge of the obstacle, then, holding the handles of the rollator, lift up slightly.

Safety instructions

WARNING!

Any 'serious incident' relating to a device must be reported to the manufacturer and to the competent authority of the Member State in which the user is resident.

ATTENTION!

The product is intended for people with disabilities and musculoskeletal conditions. If you are unsure whether your health condition allows you to use the product, consult a healthcare professional.

WARNING!

The product must not be modified in any way; this will void the warranty.

WARNING!

Exceeding the user's maximum weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

WARNING!

Please read these instructions for use before using the rollator and keep them for future reference.

- The rollator is not a substitute for a wheelchair, the seat is only used for temporary rest.
- While using the seat, apply the parking brake for both rear wheels.
- The rollator is not a substitute for a pushchair, the seat is only used for temporary rest.
- The seat is only for resting for one user, standing on it is prohibited.
- The support is designed for use on level surfaces and on surfaces with a slope not exceeding 5% (2,86°).
- The rollator may only be used on surfaces where all wheels touch the ground and the contact is sufficient to drive the wheels adequately.
- The use of the rollator on ground that does not provide secure support, i.e. wet, icy, greasy or very dusty ground, is not permitted due to the risk of falling.
- Before each use of the product:
 - perform a visual inspection of all structural components of the product,
 - check the operation of the braking system. Squeeze the brake handles to check that the brake blocks fit correctly against the wheels and are able to stop the device. Press the handles down to activate the parking brake and check that it is working properly.
 - carefully check the stability and condition of the support frame.
- Do not use the rollator if there is any visible or perceptible damage (e.g. breakage of the frame) as this may cause an accident.
- Do not leave the rollator nearby of an open flame source.
- Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of the rollator. It may cause burns or frostbite parts of the body.
- Any repairs should be entrusted to a qualified manufacturer's service.
- Do not exceed the maximum height level of the push handles as the rollator will lose stability.
- Guide the rollator with both hands simultaneously in the direction of travel.
- Ensure that all screws are tightened securely before use.
- Check that the locking mechanism for folding and unfolding the rollator is working correctly.
- Special care should be taken in the area of the horizontal strut hinge, the fold-out footrest and the side frame hinge due to the possibility of jamming fingers.
- The maximum total weight of small items placed in the bag is 1.5 kg.

WARNING!
Maximum user weight is 120 kg

Technical specifications

Maximum user weight	120 kg
Maximum support height	970 mm
Maximum bag load	1.5 kg
Height of guide handles	800-970 mm
Support width	595 mm
Length of the rollator when unfolded	720 mm
Seat height adjustment	510-560 mm
Seat width	350 mm
Seat length	320 mm
Length of the rollator when folded	300 mm
Weight of the rollator	7.45 kg
Minimum turning radius	80°
Diameter of wheels	190 mm
Type of wheels	Full
Maximum safe slope	5% (2,86°)
Frame material	Aluminum
Description of the environment in which the product will be used	  
Storage and usage temperature	From +5°C to +40°C
Air humidity for storage and use	30% to 70%

Service and repair

The inspection should be carried out regularly and should be preceded by a thorough cleaning of the product (so as to reveal any hidden damage). If repairs are needed or if you have any questions about the product, please contact your distributor.

Care and maintenance

Cleaning

- Clean the support frame with a damp cloth, the seat with mild detergents.
- Remove dirt from moving rollator components (wheels, bolts, clamp locks) on an ongoing basis.
- In order to maintain the performance of the product, it is extremely important to keep all its components clean and stored in appropriate conditions. Storage in rooms with very high humidity (baths, laundries, etc.), or in the open air, combined with a lack of hygiene, will result in a rapid loss of performance and aesthetics.
- Do not expose the rollator to extreme weather conditions (rain, snow very strong sun), as certain components may change colour.

Must not be used:

- solvents
- toilet cleaners
- sharp brushes and hard objects
- chlorine-containing agents
- corrosive cleaning agents

Do not clean the rollator with pressure or steam unit!

Disinfection

If the product is used by a large number of people (e.g. in a nursing home), commercially available disinfectants should be used. Prior to disinfection, the top surface should be cleaned. Only tested and approved disinfectants may be used for disinfection. Information on recognised and approved disinfectants and disinfection methods can be obtained from the national health authority.

WARNING!

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.


Maintenance plan

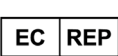Frequency	Subject of the inspection	Comments
Before the first installation	Checking the condition of all components in the package	Check that no components have visible mechanical damage
Before each use	Wheel and brake inspection	Brakes can wear out, check the condition of the brakes before each use. Wheels should be checked for wear and cleaned of dust and sand
Every two weeks	Screws and nuts - making sure they are securely fastened	Ensure that all screws and nuts are tightened securely
Every two months	Cleanliness, general condition	In the event of mechanical damage, contact the manufacturer immediately
Once a month	Wheel axles	Wheel axles must be maintained, by spraying a few drops of preservative-lubricant between the axle and the hub

Product disposal

The product must be disposed of in accordance with current local and national regulations. For proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, contact your local waste disposal company or authorised distributor.

Symbols

	Medical device
	Production date (printed on the product)
	Manufacturer
	Distributor
	Authorised representative in the EU
	Conformité Européene (conformity with EU requirements)
	Catalogue number (to be reordered)
	Serial number
	LOT number
	Unique identification code
	Refer to the instructions for use
	Warning
	Possibility of jammed fingers
	Protect from moisture

	For indoor/outdoor use
	Keep away from sunlight
	Maximum safe inclination
	The product is designed to be transported by car
	The product is not intended to be used as a seat for transport in a motor vehicle
	Maximum user weight

Contact details

Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

www.mdh.pl www.viteacare.com

We reserve the right to make technical changes due to technical progress.