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4-Wheel rollator

Wersja/Version: AMBONA

Typ/Type: VCBK

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English version on next page

Table of contents

Introduction	19
Indications.....	20
Contraindications	20
Product description.....	20
List of elements in the package.....	21
Usage	22
Brake system.....	25
Overcoming obstacles (threshold, curbs).....	25
Stairs.....	25
Safety instructions	25
Technical parameters	27
Service and repair.....	28
Care and maintenance	28
Cleaning.....	28
Disinfection.....	28
Product disposal	29
Maintenance plan.....	29
Symbols.....	30
Contact details	32

Introduction

Four-wheel walker type is a modern, stable, durable and safe product. It was created to support the rehabilitation of the patient, allow the patient to move without help of the others. Make sure that you have filled the attached warranty form and that you have submitted it to the distributor, as this will enable us to ensure the best possible service. Make sure that you have filled the attached warranty form and that you have submitted it to the distributor, as this will enable us to ensure the best possible service.

ATTENTION

Always follow the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Keep this manual for future reference.

WARNING!

Improper use can lead to death, serious injury, unsafe conditions or product malfunction.

ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

SUGGESTIONS!

Follow the instructions below to keep your product in good working order.

If the product is used by another person, you must provide them with copies of these instructions for use.

These instructions for use relate to the design and specifications of the product at the time of publication. Due to design changes, some illustrations and photos in the manual may not correspond to the product you have purchased. We reserve the right to make design changes.

The manufacturer disclaims all liability for personal injury or property damage that may result from improper or unsafe use of this product.

Indications

Rehabilitation supports are an ancillary equipment to facilitate tilting and moving of patients with different types of motor disorders. They can be used as an aid for patients after surgery (lower limbs) who are unable to use crutches, showing no coordination disorders, patients with mild dysfunction of the lower limbs or weakness. They are used in patients with in patients with impaired balance preventing them from independent vertical position and locomotion, as well as various types of circulatory dysfunction (e.g. intermittent claudication). Supports can also be used as an aid for geriatric patients.

Contraindications

Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves, impaired coordination and perception.

Product description

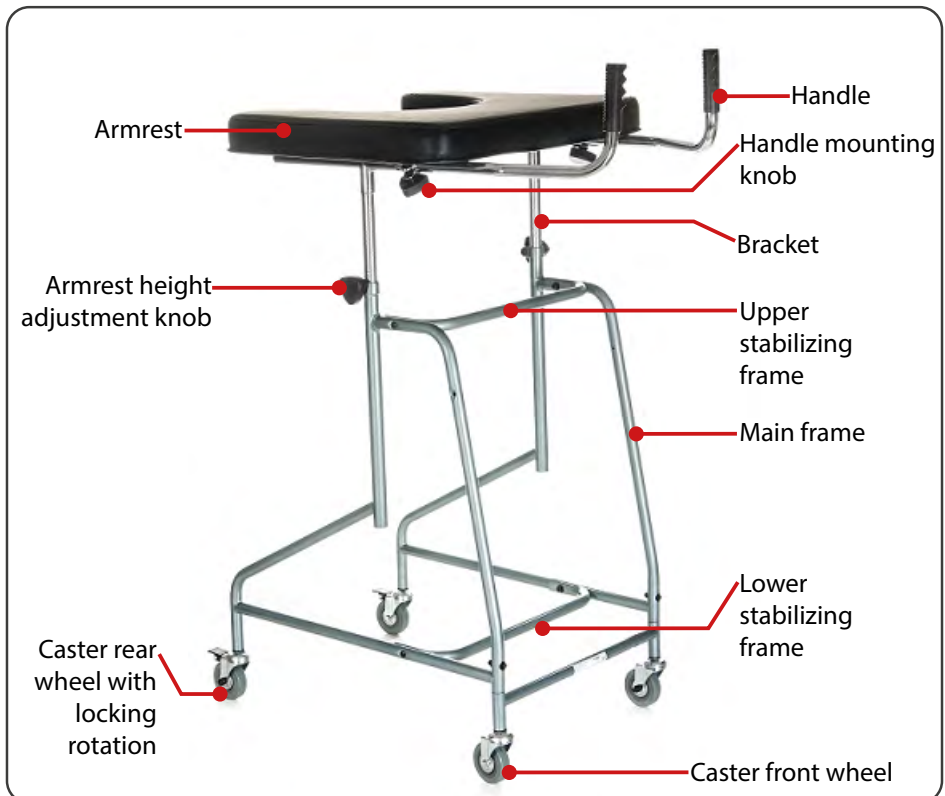
Illustration 1

List of elements in the package

Illustration 2 - Armrest

Illustration 3 - Two brackets

Illustration 4 - Stabilizing upper and lower frame

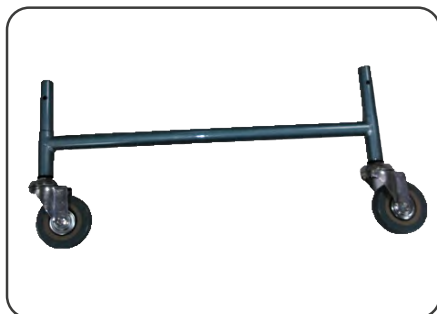
Illustration 5 - Front wheels

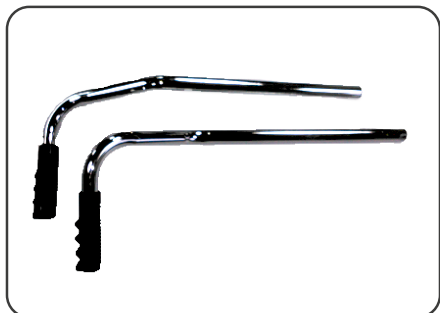
Illustration 6 - Handles

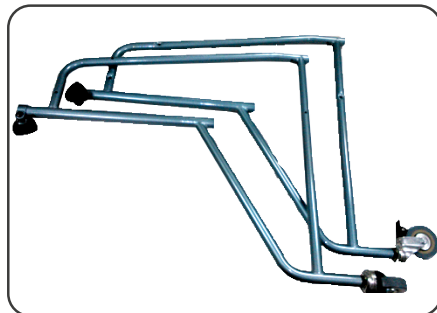
Illustration 7 - Main frames

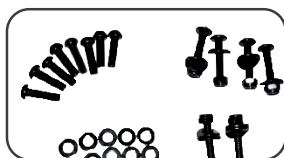
Illustration 8

The mounting kit:

- two screws with two pads to mount front wheels
- four screws with pads to mount main frames
- eight screws with pads to mount armrests

Usage

- The product is packaged in a form to be assembled.
- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above.
- Ensure that all components have no visible damage. If any doubts, please contact the distributor.
- Do not use the rollator until all parts are properly connected.
- Installation - Please refer to Illustration 1.
- Assembling should start by connecting the front wheels with the two main frames.
- In the lower parts of the main frames are holes into which should slide the front wheels. (Illustration 9.)
- The connected elements must be secured with screws (Illustration 10).
- Unfolding rollator should begin unfolded the frame, then lock the levels hinge of stretcher.

Illustration 9

Illustration 10

- The main framework for stabilizing the frame should be fixed by using four screws and washers (Illustration 11, 12).

Illustration 11

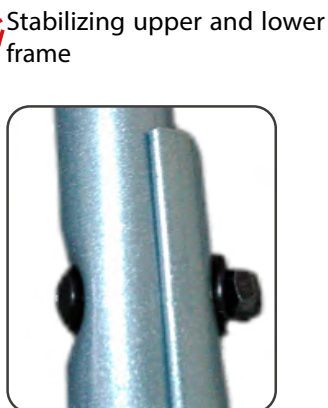
Illustration 12

Differences in the upper frame mounting method

Illustration 13 - VCBK2A

Illustration 14 - VCBK2A_01

Differences in height adjustments of the armrests

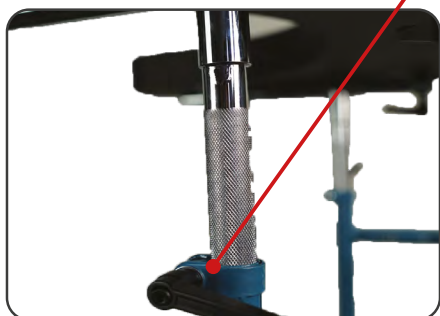
Illustration 15 - VCBK2A

Illustration 16 - VCBK2A_01

- In the lower part of the armrest are located eight holes for mounting screws.
- Bearer must be fixed in such a way that the knobs are positioned to the outer side in front of the support.

Illustration 17

- Bracket with the armrest must be installed in the main frames.
- Depending on user's needs, can set a convenient armrest height.
- Tighten the armrest height adjustment knob.

Illustration 18

Illustration 19

- Insert the handles in bracket and tighten the mounting knobs.

Illustration 20 - VCBK2A

Illustration 21 - VCBK2A_01

Brake system

The walker has a braking system located on the rear wheels. The braking system is the locking device. In order to stop the walker and immobilize it press down the lock located on the rear wheels. To unlock the wheels, pull up the lock.

Overcoming obstacles (threshold, curbs)

In order to drive the rollator onto the threshold or curb should set the rollator perpendicularly to the edge of the obstacle. Holding the handle, tilt the rollator in rear raise the front wheels up.

Then drive the rear wheels to the edge of the obstacle and holding the rollator handle, lift slightly up and enter rollator on the obstacle.

Stairs

When the patient climbs the stairs or down the stairs needs belaying of two people. One person should assist the patient, the other person should carry walker, because the patient after reaching the top will again need support. While moving the equipment, it is recommended to hold it for the frame.

Safety instructions

WARNING!

Any "serious incident" related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

ATTENTION!

The product is intended for use by people with disabilities and movement disorders. If you are not sure whether your health condition allows you to use the product, consult a healthcare professional.

WARNING!

Do not modify the product in any way; this will void the warranty.

WARNING!

Exceeding the maximum user weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

**WARNING!**

Before using the rollator, refer to this manual and keep it for future reference.

- When stopped, apply the brake to both rear wheels.
- Do not stand on rollator.
- Do not sit on the armrest, because there is a danger of falling.
- The rollator should be used on even surfaces at an incline less than 5% (2,86°).
- With the rollator you can only use on surfaces where all four wheels touching the ground are to use all of four wheels.
- Use of the product on the icy, wet, greasy, strongly gritty surfaces is unacceptable - it creates a risk of falling.
- Before each use of this product:
 - Perform visual inspection of all structural components of the product,
 - Check the operation of the braking system. Squeeze the brake handles to see if the brake pads are fit properly to wheels and are able to stop rollator.
 - Carefully check the stability and condition of the support frame.
- Do not use the support in the event of any visible or perceptible damage (e.g. cracks in the frame). It may cause an accident.
- Do not leave the stroller with the user close to open sources of fire.
- Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with temperatures there is a danger of overcooling metal parts of stroller. It may cause burns or frostbite parts of the body.
- All repairs must be carried out by qualified personnel. And during the warranty period only by the manufacturer's service.
- Do not exceed the maximum height of the handles to push, rollator will lose stability.
- Before using, make sure that all screws are tight.
- Due to the possibility of jamming the fingers put extreme caution during assembly rollator.

**WARNING!**

Maximum user weight 125 kg

Technical parameters

Parameters	Symbol (Illustration 22)	VCBK2A	VCBK2A_01
Total width of the rollator base	C	525 mm	530 mm
Inner width of the rollator base	D	455 mm	460 mm
Total length with brake (handles extended) from min. up to max.	A-B	760-970 mm	750-970 mm
Ground clearance at the front frame	E	145 mm	150 mm
Ground clearance at foot bracket height	F	180 mm	185 mm
Total height with handles (height to the top edge of the pillow + 110 mm handles) from min. up to max.	H-G	1080-1480 mm	975-1370 mm
Cushion measurements (w x l x h)	K x L x 50/60mm	590x490x60 mm	580x490x50 mm
Total width of the rollator (at the height of the cushion)	K	590 mm	580 mm
Measurements of the cutouts in the cushion	O	370x290 mm	360x280 mm
Handles	N	110 mm	110 mm
Handles spacing	J	310 mm	290 mm
Rollator weight		11 kg	
Maximum user weight		125 kg	

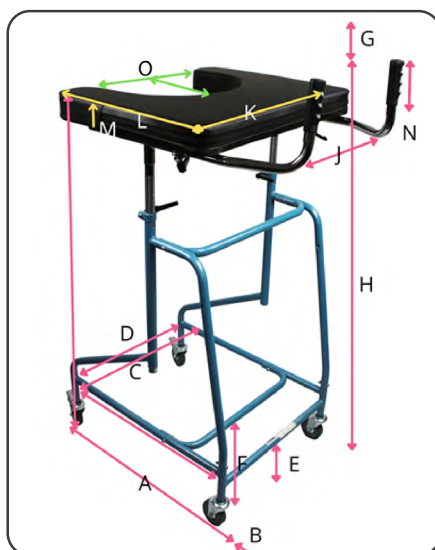
Illustration 22

Service and repair

The review should be carried out regularly, it should be preceded by thorough cleaning the product (so as to reveal any hidden damage). If repairs are needed or if you have any questions about the product, please contact your distributor.

Care and maintenance

Cleaning

- Clean the frame with a damp cloth, clean the seat with damp cloth with mild detergent.
- Remove all soiling from the movable components of the support (wheels, clamping screws, locks).
- In order to maintain the walker's parameters, it is very important to keep all elements clean and stored properly. Storing the walker in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic of the wheelchair.
- Do not expose the wheelchair on to extreme weather conditions (rain, snow very strong sun), because certain parts can change colour.
- If walker has wetted in the rain, it must be dried, use anticorrosive to keep the components in good condition.
- Regularly check the mobility of the rotating elements, and if its need to apply grease.
- Avoid contact tires of the wheels with the acid to prevent the deterioration of the tire rubber quality.

It is not allowed to use:

- solvents
- toilet detergents
- sharp brushes and hard objects
- detergents that contain chlorine
- corrosive detergents

Do not clean the rollator with pressure or steam aggregates!

Disinfection

If the product is used by several people (e.g. in a nursing home), commercially available disinfectants should be used. Before disinfecting, clean the upholstery and handles. Only tested and approved disinfectants may be used for disinfection.

Information on recognized and approved disinfectants and disinfection methods is available from your national health authority.

WARNING!

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.

Product disposal

The product must be disposed of in accordance with applicable local and national regulations. For information on the proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, please contact your local waste disposal company or authorized distributor.

Maintenance plan

Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging	Check if any elements have any visible mechanical damage
Before each use	Wheel control	You should also check the condition of wheels and clean them from dust and sand.
Every two weeks	Screw and nut control regarding their correct fixing	Make sure if all the screws and top are fixed tightly
Every two months	Cleanliness and general condition	In case of mechanical damage contact the producer immediately
Once a month	Wheel axles	Wheel axles must be impregnated, between the axle and the hub put a few drops of lubricant

Symbols

	Medical device
	Date of manufacture (printed on product)
	Manufacturer
	Distributor
	Authorized representative in the European Community
	Conformité Européene (compliance with the European Community)
	Catalog number (to re-order)
	Serial number
	LOT NUMBER
	Unique Device Identifier
	Refer to the instructions for use.
	Warning
	Possibility of jamming fingers
	Permitted temperature

	Protect from humidity
	For indoor/outdoor use
	Keep away from sunlight
	Maximum safe inclination
	The product is designed to be transported by car
	The product is not intended to be used as a seat for transport in a motor vehicle
	Maximum user weight

Contact details

Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

www.mdh.pl www.viteacare.com

We reserve the right to make technical changes due to technical progress.