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3-Wheel Rollator

Wersja/Version: TOM

Typ/Type: VCBK

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English version on next page

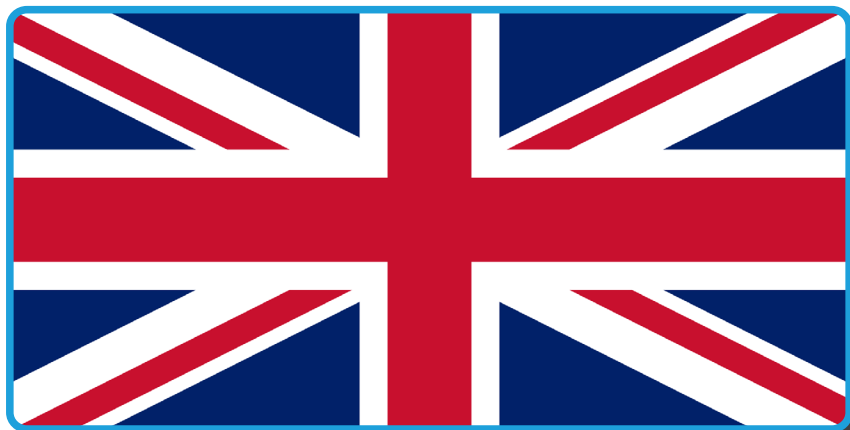
Table of contents

Introduction	18
Indications.....	19
Contraindications	19
Product description.....	19
List of elements in the package.....	20
Usage	20
Mounting and unfolding the rollator.....	20
Braking system	22
Adjustment of braking.....	22
Height adjustment of the handles	23
Folding.....	23
Unfolding.....	23
Overcoming obstacles (thresholds, kerbs)	23
Safety instructions	24
Technical parameters	26
Care and maintenance	27
Cleaning.....	27
Disinfection.....	27
Maintenance plan.....	28
Product disposal	28
Symbols	28
Contact details	30

Introduction

Three wheel steel rollator with footrest, equipped with two fixed rear wheels with brake and locking the wheels and the one front wheels swivel. It lets you easily navigate through any surface, inside and outside the building. Anatomical handles made of polyethylene, steel frame. Make sure that you have filled the attached warranty form and that you have submitted it to the distributor, as this will enable us to ensure the best possible service.

ATTENTION

Always observe the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Please keep these instructions for future use.

WARNING!

Improper use can lead to death, serious injury, dangerous conditions or product malfunction.

ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

SUGGESTIONS!

Follow the instructions below to keep the product in good working order.

If the device is used by another person, a copy of these instructions for use must be provided.

These instructions for use apply to the design and specification of the product at the time of publication. Due to design changes, some figures and photos in the manual may not correspond to the product purchased. We reserve the right to make design changes.

The manufacturer disclaims any liability for personal injury or property damage that may result from improper or unsafe use of this product.

Indications

Rehabilitation supports are an ancillary equipment to facilitate tilting and moving of patients with different types of motor disorders. They can be used as an aid for patients after surgery (lower limbs) who are unable to use crutches, showing no coordination disorders, patients with mild dysfunction of the lower limbs or weakness. They are used in patients with impaired balance preventing them from independent vertical position and locomotion, as well as various types of circulatory dysfunction (e.g. intermittent claudication). Supports can also be used as an aid for geriatric patients.

Contraindications

Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves, impaired coordination and perception.

Product description

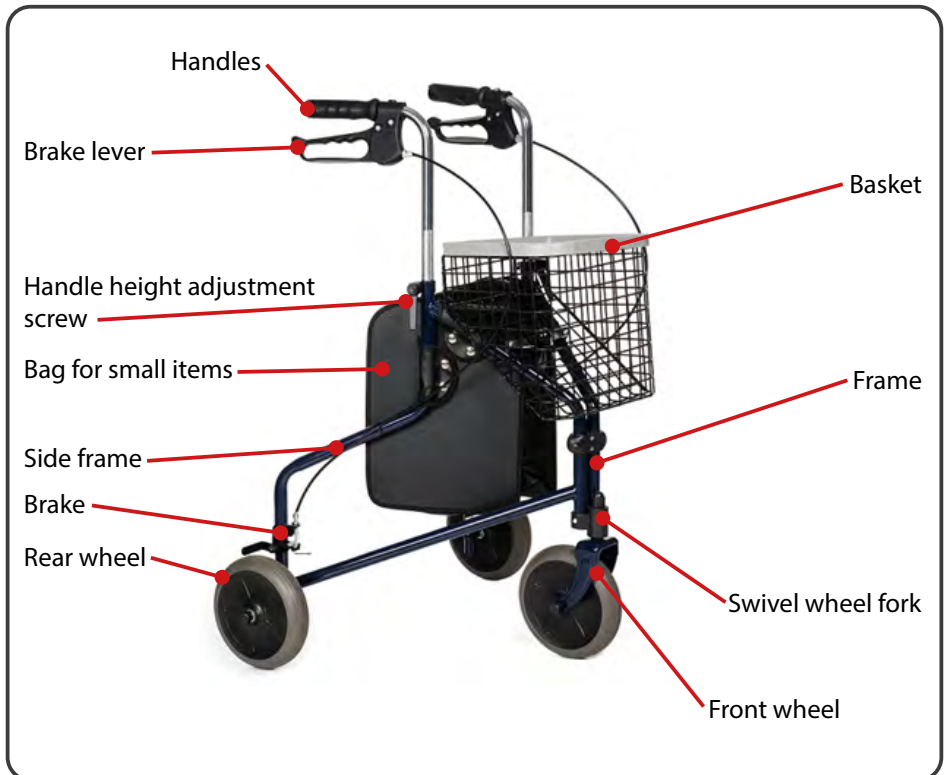
Illustration 1.

List of elements in the package

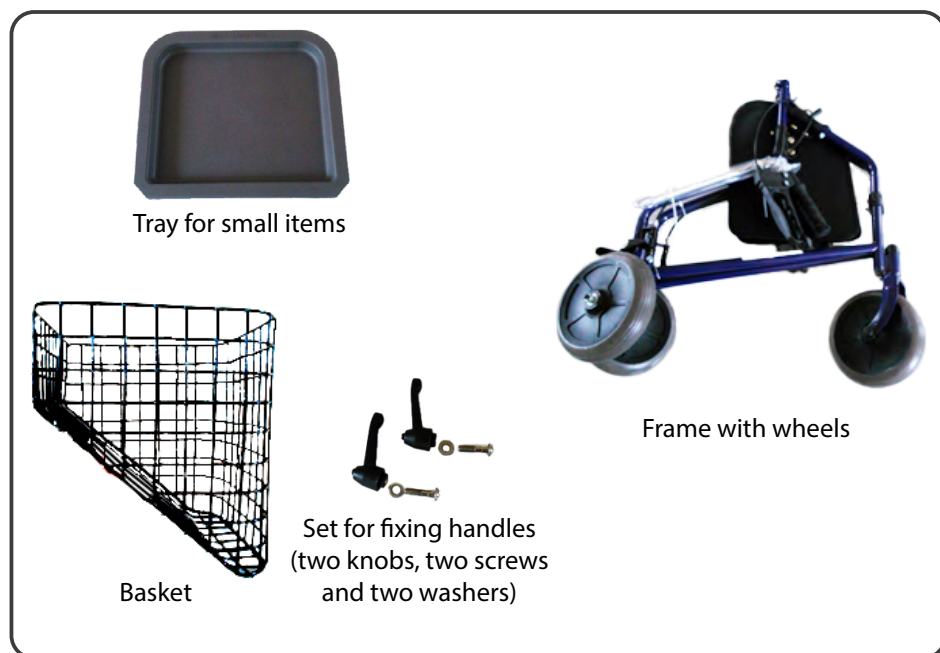
Illustration 2. Items that should be included in the package.

Usage

Mounting and unfolding the rollator

WARNING!

Special care must be taken when folding the rollator as there is a risk of jamming fingers or hands.

- Product packaged for assembly.
- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above (Illustration 2.).
- Ensure that all components have no visible damage. If any doubts, please contact the distributor.
- Do not use the rollator until all parts are properly connected.
- Assembly - please refer to Illustration 1.
- When unfolding the rollator, start by pulling apart in the outward direction of the side frames then lock the horizontal strut hinge. (Illustration 3.)

Illustration 3. Unfolding the frame.

Illustration 4. Handle assembly.

- Install the handles in the holes of the side frames (Illustration 4.).
- From the inside of the side frame, insert a screw, while from the outside, put a washer on the screw and then tighten the knob. (Illustration 5.)

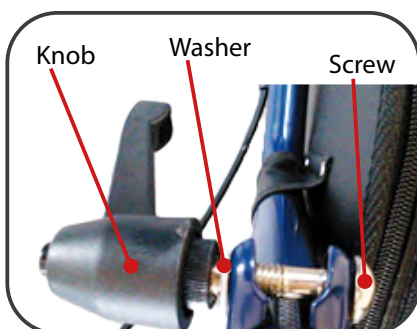
Illustration 5. Handle height adjustment screw.

- A basket for small items should be attached to the basket mounting hooks.
- The basket should be attached, as shown in Illustration 6.
- Then place the tray on the basket so that the cut ends are on the side of the handles
- To fold the support, proceed in reverse to the unfolding diagram shown.

Illustration 6. Mounted basket.

Braking system

- The rollator is equipped with a multi-functional brake. The brake handle is located under the handle. It can operate as a standard brake and as a parking brake.
- To use the brake as standard, pull the brake handle upwards.
- To use the parking brake, push the brake handle down until you can hear the characteristic sound of the brake being applied (use the right and left brake at the same time). This will cause the rear wheels to lock.
- To unlock the wheels, pull the brake handle upwards.

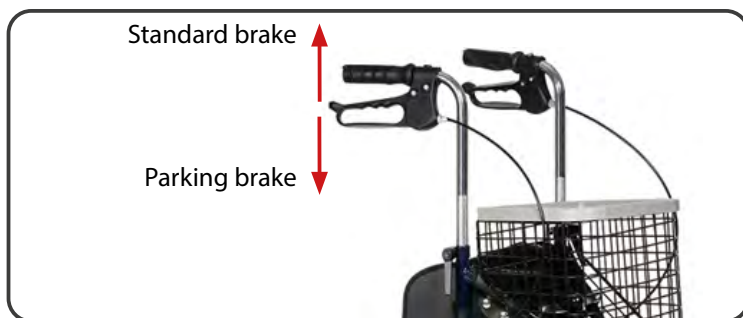
Illustration 7. Brakes operation.

Adjustment of braking

- To adjust the brake force, first loosen the nut located under the brake adjustment screw just above the wheel (Illustration 8.).

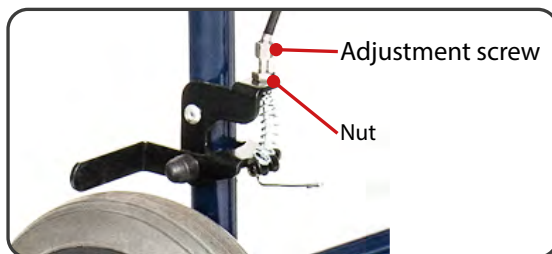
Illustration 8. Brake adjustment.

- Then turn the adjustment screw counter-clockwise until the correct brake force is achieved.
- After adjusting the brake force, tighten the nut.
- After adjusting the brake force, it is important to check the function of the brake system.
- Squeeze the brake handle to check that the brake blocks fit correctly to the wheels and are able to stop, then pull the handles down to apply the parking brake and check that it works properly.

Height adjustment of the handles

- The handles should be at a height that provides comfort and a sense of security for the user. It is important to make sure that both handles are set at the same height.
- To change the height of the handles, unscrew the knobs for fixing the handles (Illustration 5.), and slide the handle frames out or in as needed.
- After adjusting the appropriate height, tighten the knob.
- Make sure that the knob is properly tightened so that it prevents the handles from moving.

Folding

- In order to fold the stand for ease of transport, it is first necessary to block the possibility of rotation of the rear wheels.
- Then it is necessary to remove the basket for small items and empty the bag for small items.
- It is necessary to pull, in an upward direction, the horizontal strut hinge, and then fold the support by pointing the handles towards each other. (Illustration 9.)

Illustration 9. Folding the rollator

Unfolding

- To unfold the support, spread outward the side frames, then lock the horizontal strut hinge.
- Before each use, check the operation of the brake system. Squeeze the brake handles to check that the brake blocks fit snugly against the wheels and are able to stop the stand.
- Press the handles down to apply the handbrake and check that it works properly.

Overcoming obstacles (thresholds, kerbs)

- In order to drive the rollator onto the threshold or curb should set the rollator perpendicularly to the edge of the obstacle.
- Holding the handle, tilt the rollator in rear raise the front wheels up.
- Then drive the rear wheels to the edge of the obstacle and holding the rollator handle, lift slightly up and enter rollator on the obstacle.

Safety instructions

WARNING!

Any 'serious incident' relating to a device must be reported to the manufacturer and to the competent authority of the Member State in which the user is resident.

ATTENTION!

The product is intended for people with disabilities and musculoskeletal conditions. If you are unsure whether your health condition allows you to use the product, consult a healthcare professional.

WARNING!

The product must not be modified in any way; this will void the warranty.

WARNING!

Exceeding the user's maximum weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

WARNING!

Please read these instructions for use before using the rollator and keep them for future reference.

- Before using the seat stand, read these instructions for use and keep them for future reference.
- When stopping and using the seat, apply the parking brake to both rear wheels.
- The rollator is not a substitute for a wheelchair, the seat is only for temporary rest.
- The seat is only for resting for one user, it is forbidden to stand on it.
- The support is designed for use on level surfaces and on surfaces with a slope not exceeding 5% (2,86°).
- The rollator may only be used on surfaces where all three wheels touch the ground and the contact is sufficient to drive the wheels adequately.
- Use of the rollator on a surface that does not provide secure support, i.e. wet, icy, greasy, heavily dusty, is not permitted due to the risk of falling.
- Before each use of the product, you should:
 - perform a visual inspection of all structural components of the product,
 - check the operation of the brake system.Squeeze the brake handles to check that the brake blocks fit snugly against the wheels and are able to stop the stand. Squeeze the handles down to apply the parking brake and check if it works properly.
 - Carefully check the stability and condition of the subframe.
- Do not use the support if there is any visible or perceptible damage (e.g. frame breakage) as this may cause an accident.

- The support is designed for users with a maximum weight of 110 kg.
- The maximum total weight of small items placed in the bag and/or basket is 5kg.
- Do not leave the stand in the immediate vicinity of an open flame source.
- Note that the surface temperature of the textile and frame components may increase after exposure to the sun or other heat source, while in cold temperatures there is a danger of very high cooling of the rollator's metal parts. This can cause burns or frostbite to body parts.
- Any repairs should be entrusted to qualified personnel of the manufacturer.
- Do not exceed the maximum height level of the push handles, the support will lose stability.
- Push the support with both hands simultaneously in the direction of travel.
- Make sure all screws are tightened securely before use.
- Check that the locking mechanism used to fold and unfold the footrest works properly.
- Due to the possibility of jamming fingers, take special attention in the area of the horizontal strut hinge, the folding footrest and the side frame hinge.
- The grip's longitudinal axis (the longer edge of the handle) must be parallel to the direction of movement.

WARNING!

Maximum user weight is 110 kg.

Technical parameters

Maximum user weight	110 kg
Rollator height	830-955 mm
Adjustment	Stepless
Range of adjustment	125 mm
Maximum bag load	5 kg
Outer width of the rollator	690 mm
Rollator length	580 mm
Rollator width when folded	280 mm
Weight	6,6 kg
Minimum turning radius	67°
Wheel diameter	200 mm
Type of wheels	Full
Maximum safe inclination	5% (2,86°)
Materials	Steel, polyethylene, rubber
Description of the environment in which the product will be used	  
Storage and usage temperature	From +5°C to +40°C
Air humidity for storage and use	From 30% to 70 %

Care and maintenance

Cleaning

- Clean the rollator frame with a damp cloth, the seat with mild detergents.
- Remove dirt from moving rollator components (wheels, bolts, clamp locks) on an ongoing basis.
- In order to maintain the performance of the product, it is extremely important to keep all its components clean and store in appropriate conditions. Storage in rooms with very high humidity (baths, laundries, etc.), or in the open air, combined with a lack of hygiene, will result in a rapid loss of performance and aesthetics.
- Do not expose the rollator to extreme weather conditions (rain, snow very strong sun), as certain components may change colour.

Must not be used:

- solvents
- toilet cleaners
- sharp brushes and hard objects
- chlorine-containing agents
- corrosive cleaning agents

Do not clean the rollator with pressure or steam unit!

Disinfection

If the product is used by a large number of people (e.g. in a nursing home), commercially available disinfectants should be used. Prior to disinfection, the top surface should be cleaned. Only tested and approved disinfectants may be used for disinfection. Information on recognised and approved disinfectants and disinfection methods can be obtained from the national health authority.

WARNING!

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.

Maintenance plan

Frequency	Subject of the inspection	Comments
Before the first installation	Checking the condition of all components in the package	Check that no components have visible mechanical damage
Before each use	Wheel and brake inspection	Brakes can wear out, check the condition of the brakes before each use. Wheels should be checked for wear and cleaned of dust and sand
Every two weeks	Screws and nuts - making sure they are securely fastened	Ensure that all screws and nuts are tightened securely
Every two months	Cleanliness, general condition	In the event of mechanical damage, contact the manufacturer immediately
Once a month	Wheel axles	Wheel axles must be maintained, by spraying a few drops of preservative-lubricant between the axle and the hub

Product disposal

The product must be disposed of in accordance with current local and national regulations. For proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, contact your local waste disposal company or authorised distributor.

Symbols

	Medical device
	Production date (printed on the product)
	Manufacturer

	Distributor
	Authorised representative in the EU
	Conformité Européene (conformity with EU requirements)
	Catalogue number (to be reordered)
	Serial number
	LOT number
	Unique identification code
	Refer to the instructions for use
	Warning
	Possibility of jammed fingers
	Permitted temperature
	Protect from moisture
	For indoor/outdoor use
	Keep away from sunlight

	Maximum safe inclination
	The product is designed to be transported by car
	The product is not intended to be used as a seat for transport in a motor vehicle
	Maximum user weight

Contact details

Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

www.mdh.pl www.viteacare.com

We reserve the right to make technical changes due to technical progress.

Notes

mdh Sp. z o.o.
ul. Maratońska 104, 94-007 Łódź, Polska
tel. +48 42 674 83 84, fax. +48 42 636 52 21
www.mdh.pl
www.viteacare.com