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4-Wheel rollator with footrest

Wersja/Version: ROSE

Typ/Type: VCBK

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
Instructions for use

Wydanie/Edition: EJ 05.2023

z koszykiem / with basket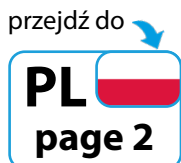

VITEA
CARE

Spis treści

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English version on next page

Table of contents

- Introduction 19**
 - Indications..... 20
 - Contraindications 20
- Product description..... 20**
 - List of elements in the package..... 21
- Usage 21**
 - Mounting and dismounting the rollator..... 21
 - Backrest assembly 23
 - Braking system 24
 - Adjustment of braking..... 24
 - Height adjustment of handles 25
 - Folding..... 25
 - Overcoming obstacles (thresholds, kerbs) 25
- Safety instructions 26**
- Technical specifications 28**
- Service and repair..... 28**
- Care and maintenance 29**
 - Cleaning..... 29
 - Disinfection..... 29
 - Product disposal 29
 - Maintenance plan..... 30
- Symbols..... 30**
- Contact details 32**

Introduction

Four wheel aluminum rollator with footrest, equipped with two fixed rear wheels with brake and locking the wheels and the two front wheels swivel. It lets you easily navigate through any surface, inside and outside the building. Anatomical handles made of polyethylene, aluminum frame. Make sure that you have filled the attached warranty form and that you have submitted it to the distributor, as this will enable us to ensure the best possible service.

ATTENTION

Always observe the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Please keep these instructions for future use.

WARNING!

Improper use can lead to death, serious injury, dangerous conditions or product malfunction.

ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

SUGGESTIONS!

Follow the instructions below to keep the product in good working order.

If the device is used by another person, a copy of these instructions for use must be provided.

These instructions for use apply to the design and specification of the product at the time of publication. Due to design changes, some figures and photos in the manual may not correspond to the product purchased. We reserve the right to make design changes.

The manufacturer disclaims any liability for personal injury or property damage that may result from improper or unsafe use of this product.

Indications

Rehabilitation supports are an ancillary equipment to facilitate tilting and moving of patients with different types of motor disorders. They can be used as an aid for patients after surgery (lower limbs) who are unable to use crutches, showing no coordination disorders, patients with mild dysfunction of the lower limbs or weakness. They are used in patients with impaired balance preventing them from independent vertical position and locomotion, as well as various types of circulatory dysfunction (e.g. intermittent claudication). Supports can also be used as an aid for geriatric patients.

Contraindications

Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves, impaired coordination and perception.

Product description

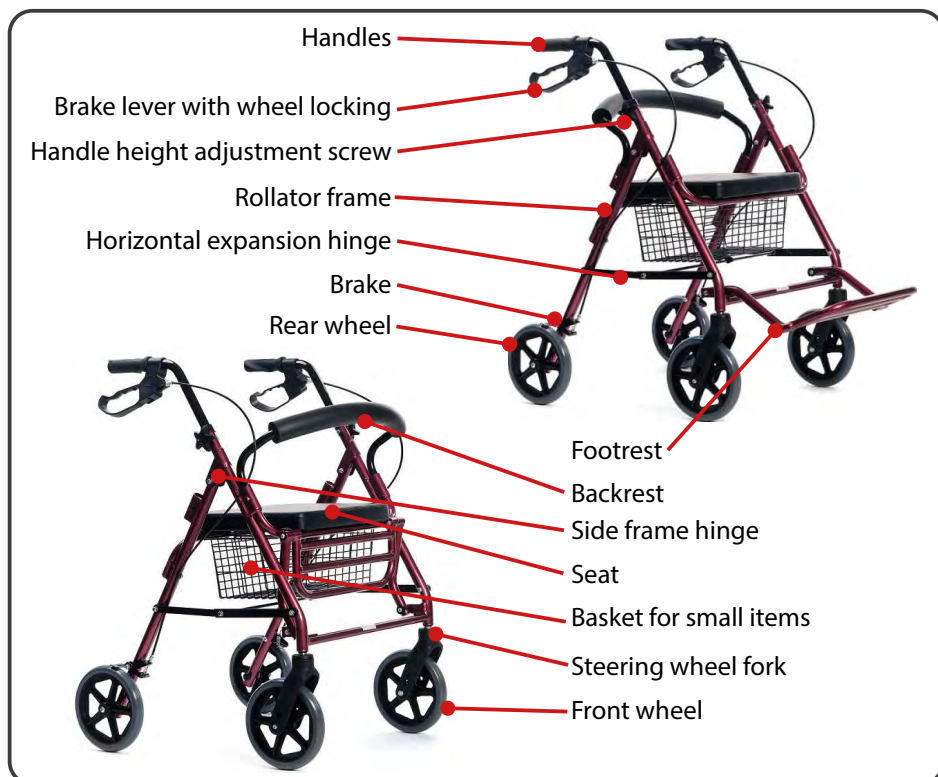
Illustration 1.

List of elements in the package

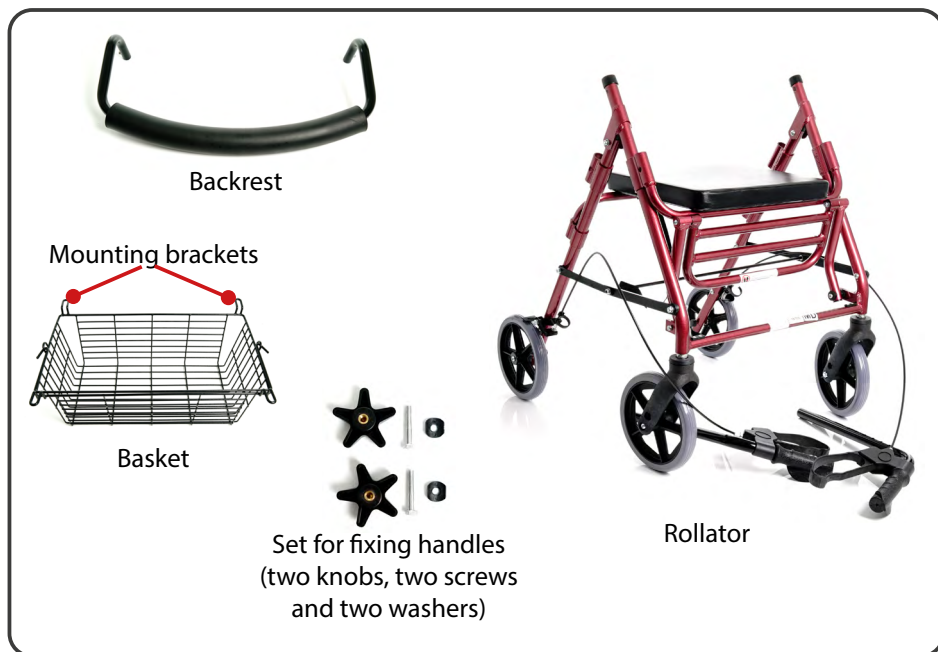
Illustration 2. Items that should be included in the package.

Usage

Mounting and dismounting the rollator

WARNING!

Special care must be taken when folding the rollator as there is a risk of jamming fingers or hands.

- Product packaged for assembly.
 - After unpacking the package, check if it contains all parts in accordance with the list of elements set out above.
 - Ensure that all components have no visible damage. If any doubts, please contact the distributor.
 - Do not use the rollator until all parts are properly connected.
 - Begin unfolding the support by pulling apart the front and rear frame parts, then lock the horizontal strut hinges by pressing on the connecting joint.
 - After removing all foil protection, the footrest and seat can be unfolded.
- (Illustration 4, 5.)

Illustration 3. Unfolding the frame.

Illustration 4. Unfolded frame.

- In the holes of the side frames should be mounted handle (Illustration 6).
- Should adjust appropriate height of the handles, the same high for each handle.
- In the mounting opening located on outer side of the frame, insert knob equipped with a screw and tighten it.
- Make sure that handles are securely fixed in the rollator frame.
- Backrest should be installed in designated for it handle located on the side frame rollator (locking tabs located on the backrest must snap into the holes localized in handle).
- After installing the backrest, pull it to make sure it is properly secured.

Illustration 5. Unfolding the footrest.

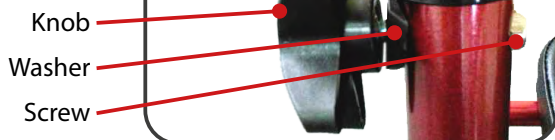
Illustration 6. Handle height adjustment screw.

Backrest assembly

- Depending on the additional function that rollator is to perform, it is necessary to adjust the backrest.
- In a situation where the seat is intended only for temporary resting, when the patient moves independently without a caretaker backrest should be positioned as shown in Illustration 7., on the opposite side of handles.
- In a situation when the seat is designed to make transport over short distances Patient backrest should be placed as shown in Illustration 8., on the side of handles.

Illustration 7.

Illustration 8.

Illustration 9. Locking backrest attachment.

- To remove the backrest of the brackets, simultaneously press the two latches located on the mounting brackets back, and then pull it up (Illustration 9.).
- Attach the backrest in the desired position.
- In addition to transport the patient on a short distance, unfold the footrest. (Illustration 5.)

Braking system

- The rollator is equipped with a multi-functional brake. The brake handle is located under the handle. It can operate as a standard brake and as a parking brake.
- To use the brake as standard, pull the brake handle upwards.
- To use the parking brake, push the brake handle down until you can hear the characteristic sound of the brake being applied (use the right and left brake at the same time). This will cause the rear wheels to lock.
- To unlock the wheels, pull the brake handle upwards.

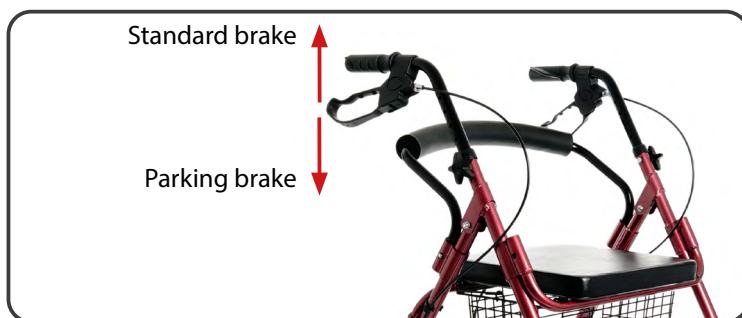
Illustration 10. Brakes operation.

Adjustment of braking

- To adjust the brake force, first loosen the nut located under the brake adjustment screw just above the wheel (Illustration 8.).

Illustration 11. Brake adjustment.

- Then turn the adjustment screw counter-clockwise until the correct brake force is achieved.
- After adjusting the brake force, tighten the nut.
- After adjusting the brake force, it is important to check the function of the brake system.
- Squeeze the brake handle to check that the brake blocks fit correctly to the wheels and are able to stop, then pull the handles down to apply the parking brake and check that it works properly.

Height adjustment of handles

- The handles should be at a height that provides comfort and a sense of security for the user. It is important to make sure that both handles are set at the same height.
- To change the height of the handles, unscrew the handle fixing knobs (Illustration 6.), remove the washers, remove the screws and slide the handle out or in frame as required.
- Once the correct height has been adjusted, tighten the knob.
- Ensure that the knob is properly tightened to prevent movement of the handles.

Folding

- To fold the rollator for ease of transport, pull the horizontal expansion hinge upwards using a small amount of force.

Illustration 12. Folding the rollator

Overcoming obstacles (thresholds, kerbs)

- To drive the rollator over a threshold or kerb, bring to the edge of the obstacle, holding the handles, tilt back which will lift the front wheels up.
- After the front wheels have been placed on the obstacle, bring the rear wheels to the edge of the obstacle, then, holding the handles of the rollator, lift up slightly.

Safety instructions

WARNING!

Any 'serious incident' relating to a device must be reported to the manufacturer and to the competent authority of the Member State in which the user is resident.

ATTENTION!

The product is intended for people with disabilities and musculoskeletal conditions. If you are unsure whether your health condition allows you to use the product, consult a healthcare professional.

WARNING!

The product must not be modified in any way; this will void the warranty.

WARNING!

Exceeding the user's maximum weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

WARNING!

Please read these instructions for use before using the rollator and keep them for future reference.

- Before using the rollator, refer to this manual and keep it for future reference.
- At standstill and the use of seat should apply the locking rotation of the wheels both rear wheels.
- The rollator do not replace the wheelchair, the seat is used only for temporary resting or transport the patient on a short distance.
- The rollator is used only to provide rest to one user. Do not stand on it.
- The rollator should be used on even surfaces at an incline less than 5% (2,86°).
- The rollator can only be used on surfaces where all four wheels touch the ground and the contact is sufficient to drive the wheels properly.
- Use of the product on the icy, wet, greasy, strongly gritty surfaces is unacceptable - it creates a risk of falling.
- Before each use of this product:
 - perform visual inspection of all structural components of the product,
 - check the operation of the braking system. Squeeze the brake handles to see if the brake pads are fit properly to wheels and are able to stop rollator.
 - carefully check the stability and condition of the support frame.
- Press the handles down to activate the parking brake and check if it works properly.
- Do not use the support in the event of any visible or perceptible damage (e.g. cracks in the frame). It may cause an accident.

- The maximum total weight of small items placed in the basket is 1.5 kg.
- Do not leave the rollator with the user close to open sources of fire.
- Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of stroller. It may cause burns or frostbite parts of the body.
- All repairs must be carried out by qualified personnel. And during the warranty period only by the manufacturer's service.
- Do not exceed the maximum height of the handles to push, rollator will lose stability.
- Push the rollator with both hands simultaneously in the direction of travel.
- Before using, make sure that all screws are tight.
- Make sure that the Levels hinge of stretch securing before submitting rollator, work properly.
- Due to the possibility of jamming the fingers put extreme caution in the area of horizontal hinge strut, fold-out footrest and side frame hinge.

WARNING!

Maximum user weight is 120 kg.

Technical specifications

Maximum user weight	120 kg
Maximum rollator height	770-880 mm
Height adjustment steps	6
Adjustment range	25 mm
Maximum bag load	1.5 kg
Rollator outside width	600 mm
Rollator inside width	440 mm
Seat height	535 mm
Seat width x length	350 x 320 mm
Length of the rollator with unfolded footrest	900 mm
Length of the rollator with folded footrest	670 mm
Length of the folded rollator	320 mm
Rollator Weight	8,2 kg
Minimum turning radius	95°
Wheel diameter	200 mm
Wheels type	Full
Maximum safe inclination	5% (2,86°)
Frame material	Aluminum, poliethylen, rubber
Description of the environment in which the product will be used	  
Storage and usage temperature	From +5°C to +40°C
Air humidity for storage and use	From 30% to 70 %

Service and repair

The inspection should be carried out regularly and should be preceded by a thorough cleaning of the product (so as to reveal any hidden damage). If repairs are needed or if you have any questions about the product, please contact your distributor.

Care and maintenance

Cleaning

- Clean the support frame with a damp cloth, the seat with mild detergents.
- Remove dirt from moving rollator components (wheels, bolts, clamp locks) on an ongoing basis.
- In order to maintain the performance of the product, it is extremely important to keep all its components clean and store in appropriate conditions. Storage in rooms with very high humidity (baths, laundries, etc.), or in the open air, combined with a lack of hygiene, will result in a rapid loss of performance and aesthetics.
- Do not expose the rollator to extreme weather conditions (rain, snow very strong sun), as certain components may change colour.

Must not be used:

- solvents
- toilet cleaners
- sharp brushes and hard objects
- chlorine-containing agents
- corrosive cleaning agents

Do not clean the rollator with pressure or steam unit!

Disinfection

If the product is used by a large number of people (e.g. in a nursing home), commercially available disinfectants should be used. Prior to disinfection, the top surface should be cleaned. Only tested and approved disinfectants may be used for disinfection. Information on recognised and approved disinfectants and disinfection methods can be obtained from the national health authority.

WARNING!

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.

Product disposal


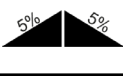
The product must be disposed of in accordance with current local and national regulations. For proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, contact your local waste disposal company or authorised distributor.

Maintenance plan

Frequency	Subject of the inspection	Comments
Before the first installation	Checking the condition of all components in the package	Check that no components have visible mechanical damage
Before each use	Wheel and brake inspection	Brakes can wear out, check the condition of the brakes before each use. Wheels should be checked for wear and cleaned of dust and sand
Every two weeks	Screws and nuts - making sure they are securely fastened	Ensure that all screws and nuts are tightened securely
Every two months	Cleanliness, general condition	In the event of mechanical damage, contact the manufacturer immediately
Once a month	Wheel axles	Wheel axles must be maintained, by spraying a few drops of preservative-lubricant between the axle and the hub

Symbols

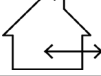	Medical device
	Production date (printed on the product)
	Manufacturer
	Distributor
	Authorised representative in the EU
	Conformité Européene (conformity with EU requirements)
	Catalogue number (to be reordered)

	Serial number
	LOT number
	Unique identification code
	Refer to the instructions for use
	Warning
	Possibility of jammed fingers
	Permitted temperature
	Protect from moisture
	For indoor/outdoor use
	Keep away from sunlight
	Maximum safe inclination
	The product is designed to be transported by car
	The product is not intended to be used as a seat for transport in a motor vehicle
	Maximum user weight

Contact details

Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

www.mdh.pl www.viteacare.com

We reserve the right to make technical changes due to technical progress.