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PRODUCT DESCRIPTION:

The shower stool is a light construction made of warm in contact polyurethane and aluminum. Stool legs of regulated height assure stable support. There are additional flow openings placed on the stool seat which allows a patient to hold the stool for better balance.

INDICATIONS:

Hygiene activities with the use of bath or shower;
Providing a patient with more safe position;
Significant or partial limitation of movement (dysfunctions of motor system, problems with body balance and coordination)
Designed for patients in serious conditions, elderly patients,
Protection against wound infection or dipping,
Ideal for hospitals, homecare.

CONTRAINDICATIONS:

Contraindications to a sitting position, the lack of trunk stabilization. Patients with coordination disorders, balance and intellectual disabilities should not be left without supervision.

STOOL'S EQUIPMENT:

1. Polyurethane seat,
2. Aluminum frame,
3. Telescopic regulation of height,
4. Rubber straps on the top of stool's legs,
5. Openings on the sides of the seat which play the role of handles,
6. Colours: silver, white

USING (REGULATION):

The originally packed product consists of separate parts. In order to fold it, insert the legs into the tubes attached to seat, until the telescopic mechanism latches (with characteristic click sound). Remember to set all legs in the same length. Asymmetrical length setting may cause falling down.

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SAFETY OF USING:

1. The shower stool is designed to be used on flat, straight surfaces.
2. The use of shower stool on the rough surface (of the slope more than 5%) may result in falling down and injury.
3. It is strictly forbidden to stand on the stool.
4. Maximum load of the stool - 100kg.
5. Use the stool in case of observing any visual or detectable damages (such as frame split) because it may result in falling down.

SERVIS AND CONSERVATION:

1. Clean the shower stool frame with a moist cloth.
2. Do not use sharp cleaning tools (risk of damaging the protective cover).
3. Clean with gentle detergent.
4. Regularly clean the shower stool's mobile elements (pawl blockade mechanism).

Keeping all the elements clean is essential for preserving the product working parameters. Keeping the walking stick for a long time in a very moist room or outdoor may result in loss of esthetic and functional parameters of the product.

TECHNICAL PARAMETERS:

Height: 45-56 cm

Width: 51 cm

Weight: 2,1 kg

Materials: Aluminum, polyurethane, rubber.

Serial number/ production date: on the product

Product durability: 5 years from production date

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Medical device

Dane teleadresowe producenta

mdh Sp. z o.o.

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tel. +48 42 674 83 84

fax. +48 42 636 52 21

www.mdh.pl www.viteacare.com