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Instructions for use
Shower seat
VCTP0021
Edition EJ 06.2022

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PLEASE READ WHOLE INSTRUCTION
CAREFULLY BEFORE USE.

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Introduction

Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasise that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual

Product description

The shower seat is a light construction, made of warm in contact polyurethane. The shower seat has legs of regulated height assuring stabile support. It is light, easy to clean, equipped with flow openings and additional openings which allow a patient to hold the seat for better balance.

Indications

Professional supporting equipment, simplifying/enabling the patient to shower.

Contraindications

Contraindications to a sitting position, the lack of trunk stabilization. Patients with coordination disorders, balance and intellectual disabilities should not be left without supervision.

Use

The originally packed product consists of separate parts. In order to fold it, insert the legs into the tubes attached to seat, until the telescopic mechanism latches (with characteristic click sound). Remember to set all legs in the same length. Asymmetrical length setting may cause falling down. Every time after using the seat, clean it with a moist cloth. Do not use chemical detergents.

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Safety of use

WARNING !

- a) Before using the cushion, refer to this manual and keep it for future reference.
- b) The shower seat is designed to be used on flat, straight surfaces without inclination.
- c) It is forbidden to stand on a chair.
- d) The use of shower seat on the rough surface may result in falling down and injury.
- e) It is strictly forbidden to stand on the commode chair.
- f) Maximum load of the commode chair is 100 kg.
- g) When using this product for the first time, assess the risk and suitability of a given person using the product.
- h) Before use, always make sure the chair is stable and the legs are on the same level. Make sure that all sides of the frame are completely locked in a stable position and the toilet cover and seat are firmly attached to the rear frame.
- i) Regularly (about every 3 months) Check the condition of screw connections.
- j) Regularly check the equipment for damage and wear. If the equipment is damaged, stop using it.
- k) Keep away from direct sunlight and heat source.

Everyday use and maintenance

- 1. Clean the shower stool frame with a moist cloth.
- 2. Do not use sharp cleaning tools (risk of damaging the protective cover).

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Keeping all the elements clean is essential for preserving the product working parameters. Keeping the commode chair for a long time in a very moist room or outdoor may result in loss of esthetic and functional parameters of the product.

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Set height: 45 - 56 cm
 Seat width: 51 cm
 Weight: 2,95 kg
 Materials: aluminum, polyurethane, rubber.
 Serial number/ production date: on the product
 Product durability: 5 years from production

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Repair

- In case of reparation, please contact the producer.

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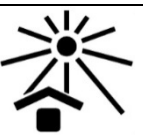
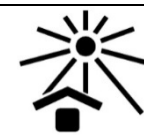
Customer service

- If you have any questions or need help, contact the producer.

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