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VITEA®
CARE

Instructions for use
Recliner Extra
Special wheelchair stabilizing back and head
VCWK702
Edition EJ 08.2023

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PLEASE READ CAREFULLY BEFORE USING
WHOLE OF INSTRUCTIONS

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Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual was made in order to provide the necessary information on the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to distinguish that to date we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual.

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Elements that should be inside the packaging:

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The wheelchair is used in order to commute by people with disabilities e.g. after the spinal cord damages, cerebral palsy or various lower limb injuries, neuromuscular diseases, post-traumatic cranial cerebral or for geriatric patients.

It is also intended for people who want to actively and comfortably participate in everyday life.

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Contraindications to a sitting position

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- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above.
- Ensure that all components have no visible damage. If any doubts, please contact the manufacturer.
- Do not use the wheelchair until all parts are properly connected.

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UNFOLDING THE WHEELCHAIR

- The wheelchair must be placed with rear wheels in front to You.
- Unfold the handle holder towards the outside

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(Illustration 8.).

- Then place wheelchair with front wheels in front of you.
- Press seat frames, until complete unfolding. (Illustration 9., 10.)

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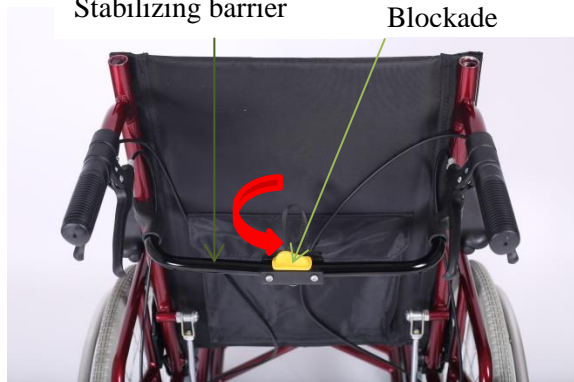
- Then should lower the stabilizing barrier to the horizontal position (Illustration 11.)
- Stabilizing barrier has to be locked, by turning the lock by 45°. (Illustration 12.)
- To unlock the stabilizing barrier, turn lock by 45°.

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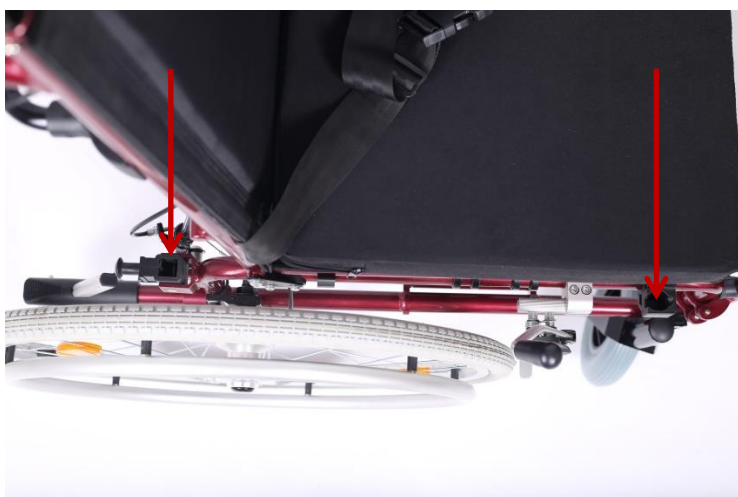
- Armrests should to be mounted in openings at a side edge of the seat. Armrest safety arm has to be faced to wheelchair rear part, and Height adjustment button has to be faced to the outer side of the wheelchair.

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- In order to the armrest be properly attached, pull the armrest block located in wheelchair rear and press the rear edge of the armrest (Illustration. 13).
- Then press the armrest lever located in the armrest front part and press armrest front edge. (Illustration 14).
- Procedure should be performed for both armrests.
- Make sure, that both armrests are assembled properly.
- In order to remove the armrests, press the armrest lever toward the front edge of the armrest, lift up the front edge of the armrest. Then pull the armrest lock and lift the rear edge of the armrest up.

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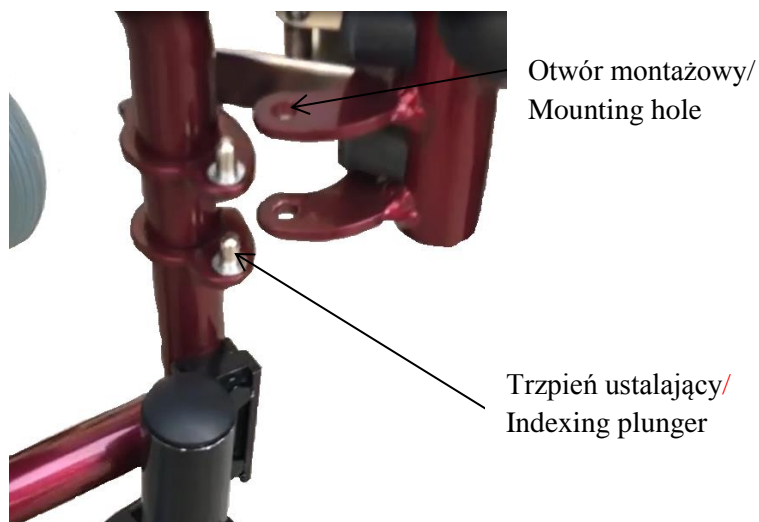
- Then attach the higher backrest. Into the holes located in the rear part of the backrest should insert higher backrest frame. (Illustration 16., 17.)
- Headrest should attach to the higher backrest by using Velcro, on height convenient for the patient.
- Fasten the seat cushion using velcro.
- In order to remove the higher backrest, press the two pins located on the backrest frames and pull up the higher backrest.

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
- To install footrests, place the mounting holes in the indexing plungers (Illustration 18., 19.), then rotate footrests into wheelchair front until it locks (you will hear click).
- Unfold the footrests platform.
- To lower the footrest, release the footrest lever. (Illustration 19.).
- In order to remove the footrest, lower the footrest and tilt the latching mechanism outwards with the whole footrest.

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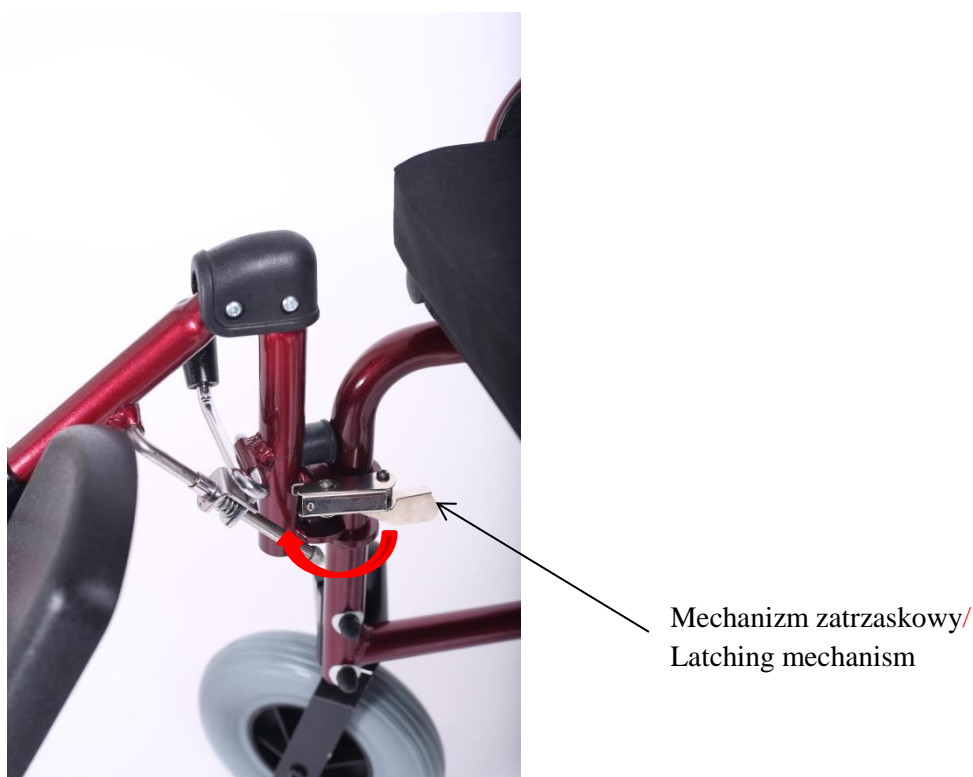- Before using the wheelchair refer to this manual and keep it for future reference.
- While getting on and out of the wheelchair, the brakes always have to be on!
- Before getting on or getting off the wheelchair, do not stand on the footrest, you must fold or completely set aside footrests first.

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p) Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of wheelchair. It may cause burns or frostbite parts of the body.

- q) In order to transfer wheelchair, grab the frame. Never transfer the wheelchair by grabbing the armrests or footrest.
- r) Never use the wheelchair as a seat for the transportation of people in cars, buses or any motor vehicles.
- s) The wheelchair is designed for people up to 120 kg.
- t) Wheelchair must be easy to fold and to unfold. In case of problems with folding, you should re-read the instruction or contact the manufacturer.
- u) Keep caution while moving on sloping surfaces. The maximum allowable slope angle up and down is 5%.
- v) Brakes cannot be slow down the wheelchair during ride, You should use them only to prevent the unintended movements.
- w) Both brakes should be properly adjusted. Incorrect setting one of the brakes, results with blocking only one rear wheel. On large slopes, it can cause difficulty to control the rotation the wheelchair around the wheel stopped.
- x) Regularly check the tire pressure. With too low tire pressure than recommended, the brake performance can be significantly reduced!
- y) While reaching objects, avoid excessive leaning out the wheelchair because it can cause a fall (to the side, front, back).
- z) In order to avoid accidental rolling off the wheelchair, always put on the brakes when the wheelchair is stopped, when patient get in, get

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a skilled personnel and during the warranty period only by the manufacturer's service.

- jj) In case of wheelchair fitted with pumped wheels You should not exceed the maximum permissible inflation pressure of the wheels, the value is indicated on the tire wheels.
- kk) During transport (e.g. In the trunk), do not burden stroller, it may weaken or damage its structure.
- ll) Keep out of the reach of children under five years of age.
- mm) Should not insert fingers between the wheelchair components - risk of injury or entrapment.
- nn) Keep extreme caution while unfolding / folding the wheelchair. Lack of attention may result in finger entrapment:
 - when folding the wheelchair
 - in the area moving parts.
 - in the area of brakes.

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SITTING ON THE WHEELCHAIR:

- Place the wheelchair in a position that will make easier transporting patient (by the bed, chair, close to lifting distance)
- Block both brakes of wheelchair, fold or remove the footrests.
- Depending on the method of moving the patient, remove the correct armrest (it is recommended to leave the second armrest as a handrail for the patient and prevent slipping out of the wheelchair).
- Place the patient the wheelchair.
- Put on the armrest.
- Install and open the footrests and put on it patient's feet.
- Place the patient on wheelchair so that the patient is in upright sitting position (knees should be bent at an angle of ~ 90 °).

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PUSHING THE WHEELCHAIR BY CARETAKER

- Before use, make sure that no part of the patient's body or his / hers clothing is not exposed to hook with the moving parts of the wheelchair.
- If patient can not sit on one's own, wheelchair should be equipped with stabilizing belts (belts, which are in standard equipment to protect the

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patient from sliding off the seat).

- Wheelchair should be pushed with both hands.

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- While moving on slopes, use caution and reduce your speed.

- Do not move on too high slopes. The maximum allowable slope angle up and down is 5%.

- Consider the possibility of assistance from the caretaker. If the person on the wheelchair did not have enough force to control the wheelchair, the brakes must be on.
- To prevent tipping over while moving along the sloping surface, You should be slightly lean forward to move your centre of gravity forward.

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- Ahead without the help of a caretaker

- Experienced wheelchair user can drive on his own from the threshold or curbs.

- With a low curb, you can drive by moving forward. Make sure that the footrests platform does not touch the ground.
- Please keep balance on the rear wheels to reduce the pressure on the front wheels.
- Then cross the curb.

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
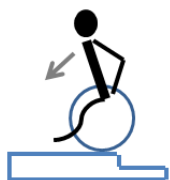
- Ahead with the help of a caretaker

- Higher curbs can be overcome with caretaker help.

- Caretaker should slightly tilt the wheelchair back.
- Cross the curb moving on the rear wheels.
- Then the wheelchair must be paced on all four wheels.

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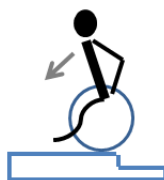- Backwards without the help of a caregiver
 - Experienced wheelchair user can cross on his own the threshold or curbs.
 - Turn the wheelchair in such a way that both rear wheels were turned toward the curb
- Lean forward to move the centre of gravity in forward.
- Come closer with the wheelchair to the edge of the curb.
- Then, by using the handrim in a controlled manner, user must slide the wheelchair from the curb.

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- Ahead with the help of a caregiver
 - Experienced wheelchair user can drive on his own from the threshold or curbs.
 - Caregiver should slightly tilt the wheelchair back.
- Wheelchair user should keep balance on the rear wheels to reduce pressure on the front wheels.
- Go with front wheels on the curb and then put the rear wheels over the curb.

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- Higher curbs can be crossed going backwards with the help of a caregiver
 - Turn the wheelchair in such a way that both rear wheels are turned toward the curb.
 - Wheelchair user should keep balance on the rear wheels to reduce the pressure on the front wheels.
- Caregiver should pull the wheelchair over a curb.
- Wheelchair user should return to its normal position on the wheelchair.

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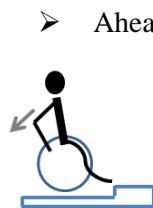
➤ Ahead without the help of a caregiver.

- Experienced wheelchair user can go down on his own from the threshold or curb.

- Place the front wheels to the edge of threshold or curb.

- It is important to make sure that the footrests do not touch the threshold or curb

- Now, can go with the front wheels on the curb.

- Then, may enter the rear wheels on to the curb.

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TRANSPORT IN THE CAR

- In order to make the wheelchair easy to transport, should fold footrests platforms and remove the footrests.
- Then remove the seat cushion and higher backrest.
- It is necessary to unlock the stabilizing barrier and to point the barrier arms up.
- Then pull up the seat frame, it will submit a wheelchair.
- The wheelchair is equipped with system of quick release rear wheels.
- To install or remove the rear wheels, press the system of quick release rear wheels button (Illustration 20) and pull or push a wheel.
- The wheelchair have to be moved by holding the frame elements only. Never hold the wheelchair by the armrests or footrests.

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- Illustration 23 shows the brake mechanism.
- To activate the brake, pull the brake lever towards the front wheel to the point where the brake tightly touches to the rear wheel.
- Make sure that the brake lever does not switch off when you try to move the wheelchair.
- To disable the brake, pull the brake lever toward the rear wheel.

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BRAKE MECHANISM – BRAKING FORCE

- To adjust the position of the brake mechanism to the rear wheel, should allen wrench 5 mm to unscrew the two screws.
- Change position the brake mechanism and tighten the screws.
- Then fasten the rear wheel to the wheelchair and check that the brake mechanism is properly adjusted to the wheel, prevents the rotation of the wheels.

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FOOTREST

- To change the height of the footrest, lift it up. In order to lower it, pull the lever lowering the footrest (Illustration 24) towards the rear wheel.
- Calfrest has three step adjustment.
- To adjust the calfrest height (Illustration 24) just turn it to the outside and depending on the need, lift it up or lower it down and then turn it

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lock to the inside.

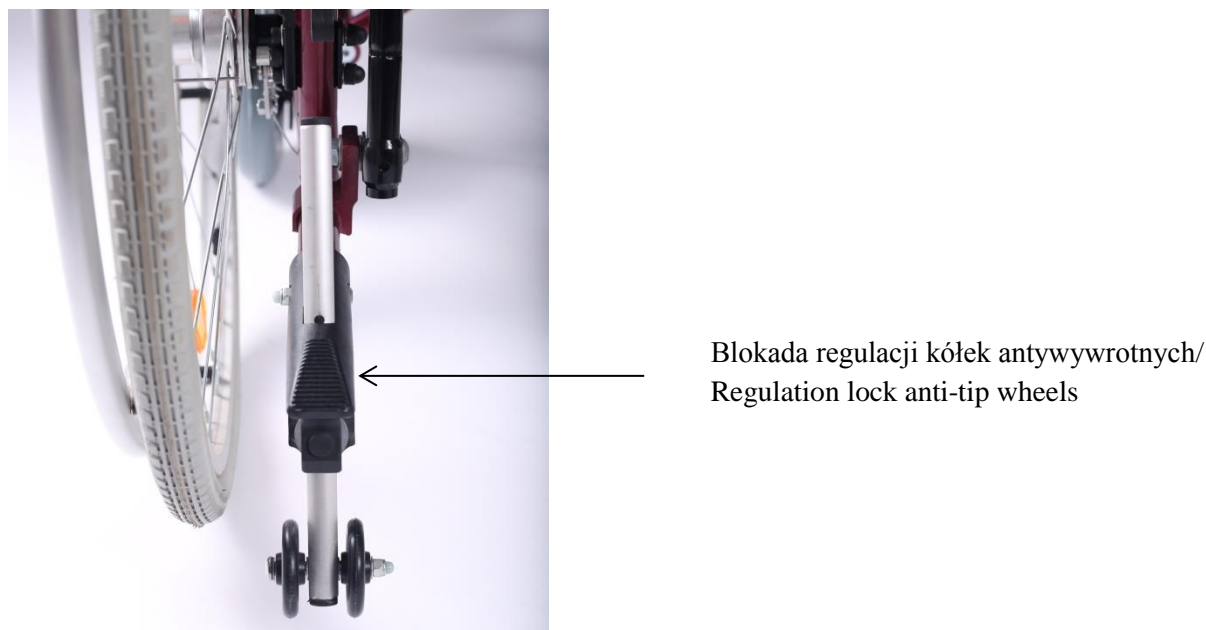
- By loosening the Allen wrench 5 footrest length adjustment screw, you can to change the length of the footrest.
- After determining the appropriate length of the footrests, you should tighten footrest length adjustment screw.
- The length of the footrest is subject to continuous regulation.
- The maximum length of footrest adjustment is marked on the rail foot.

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
- The wheelchair is equipped with anti-tip wheels, which play an important role when a person sitting on a wheelchair has a problem with the stabilization of the body.
- The height of the anti-tip wheels should be individualized to the patient depending on experience of using wheelchair.
- To adjust the height of the anti-tip wheels pull the adjustment lock the anti-tip wheels are (Illustration 25) located in the wheel handle, set the desired height and release the lock.
- Anti tip wheel height adjustment is five-point regulation.

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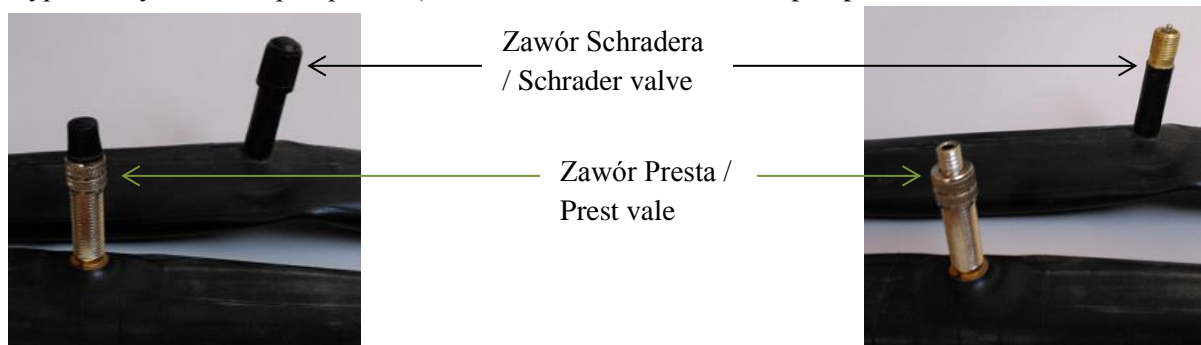PUMPING TYRE (It applies to stroller fitted with wheels pumped)

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- Before pumping the tires, identify the type of valve which is located on the tire. This may be one of the two valves shown in Illustration 26 and 27.
- Presta valve also has an additional small top, which must be loosened before pumping.
- After identifying the valve, prepare the pump designed to valve type.
- Most of the new pumps are equipped to service both Presta valve and Schrader. Many of them let to reverse the tip. Smaller hole is Presta valve, Schrader valve greater. If the pump does not support Presta, you will need to buy yourself the "adapter Presta", which is available at any bike store.
- At this point, if the wheel is provided with a valve Presta, it will be necessary to unscrew the top located on the vent. It is a small top, which can be unscrewed with fingers. When doing this, the top should move upwards. Then valve is considered open.
- Then remove the plastic top from the valve and set it aside in a safe place. It is

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important to not lose it, because the valves will get dirt, leading to difficulties in pumping and rapid air loss.

- Place the nozzle of the pump to the valve and check if it stick properly. If it is needed, push the pump to the valve to make sure that the air does not escape to the sides.
- If the pump has a lever with clamp at the nozzle, tighten it. If the nozzle of the pump and valve have a threaded tip, screw them together. As a result, air will be streamed directly to the inner tube and the nozzle can not be detached from the valve.
- At the begining of pumping the pressure in tyres should be checked at some points. Check tire pressure with your fingers at the end of the tread. If it is possible, make only a millimeter indentation. It means that the tire has enough air. If necessary continue pumping.
- After pumping, remove or unscrew the nozzle from the valve outlet while compressing it's ends by thumb. It is recommended to do this quick enough in order to not loose too much air.
- For Prest valve, tighten previously loosened additional small top.
- Screw the plastic top on the valve.

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CHANGE OF TYRES (It applies to stroller fitted with wheels pumped)

Before beginning the change of tyres it is necessary to get acquainted with the following notes.

First, it is necessary to release air from an inner tube and then by pressing a tyre in the direction of the centre of a rim to loosen a tyre-rim contact. Using a special spoon it is necessary to pull a tyre border out of the rim what will give the possibility to shift the whole wall of the tyre outside. After this operation pulling the inner tube out is very natural.

Prior to the installation of a new inner tube attention should be devoted to ensuring that:

- a bottom band is not damaged and has not foreign bodies,
- external walls of the rim are smooth, without foreign insertions and other damages.

The bottom band and rim should be precisely cleaned if need be.

To install the tyres it is necessary to put a hole of the bottom band over a valve, and then put together to a seating in the rim. Keeping such a position, put the bottom band on the whole perimeter of the rim, paying attention to ensuring that the band is not twisted or damaged and it covers spoke tighteners thoroughly.

Pulling the tyre wall away from the rim, insert the valve to the seating in the rim and slightly pump the inner tube that it achieves a round shape and then to put the inner tube in the rim.

To be sure, it is necessary to check if the inner tube is not twisted (in case of deviation or deformation, pump air additionally). Manually put the tyre wall onto the rim, starting from an opposite side of the valve and ending by it.

It is necessary to check on both sides of the wheel if the inner tube is not pressed by the tyre and if the tyre regularly adheres to the rim. The valve should be slightly pressed inside and then repressed outside that it sets well in the seating.

To check that the tyre installation is correct it is necessary to pump such air that it is possible to press a thumb to the bottom of the rim. Then, check if the tyre adheres evenly to the rim on both sides and if it is centrally arranged. If such defects occur it is necessary to release air from the wheel and correct the setting of the tyres. When an opinion is positive it is necessary to pump the wheel to the maximum pressure value and put a cap onto the valve end.

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PRESSURE IN WHEELS AND BRAKING POWER

- Pressure in wheels should be select to the user weight and then with assistance caregiver check the operation of the parking brakes (wheelchair with user facing toward the fall, can not roll on a plane with a slope of 15%).
- After all the adjustments, be sure to thoroughly check all connections.

COMMENTS

- Using improper tools or executing incorrect operation can cause serial damages of the rim or tyres;
- In case of plastic rims the bottom band is not used;
- Pay attention to ensuring that during the installation no foreign bodies get into space between a rim and bottom band what can damage a detail or injure a hand;
- Observe proper tire inflation level with the information provided by the manufacturer on the sides of the tire.
- Use a pump with a pressure gauge or manual pump cycling.

EVERYDAY USE AND MAINTANCE

- Clean the upholstery and frame with a moist cloth, without the addition of detergents and other similar chemicals.
- Systematically remove the dirt from the movable elements of the wheelchair (screws, resistance mechanisms).
- Keep clean all wheelchair elements for to save functional parameters of the product.
- Storing in rooms of very high humidity (baths, laundries etc.) or in the open air, in the combination with the lack of hygiene will cause the fast loss of functional and aesthetic parameters.
- Do not expose the wheelchair on to extreme weather conditions (rain, snow very strong sun), because certain parts can change colour.

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You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the wheelchair with pressure or steam aggregates!

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TECHNICAL REVIEW

The review should be carried out regularly, it should be preceded by thorough cleaning the wheelchair (so as to reveal any hidden damage).

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Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Before each use	Control of wheels, brakes,	The brakes can worn out, check the condition of the brakes before each use. You should also check the condition of wheels and clean them from dust and sand.
Every two weeks	Screw and nut control regarding their correct fixing.	Make sure if all the screws and top are fixed tightly.
Every two months	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.
Once a month	wheel axles	Wheel axles must be conserved, seting between the axle and the hub a few drops of lubricant preservative. In addition, make test each wheel. Raise the wheelchair so that the tested wheel was located above the ground, the wheel should rotate freely but without backlash on axle.
Once a month	State connections, threaded, soldered, the system of quick release rear wheels	Pay attention to the cracks on frame material in connections threaded, soldered, the system of quick release rear wheels.
Once a month	Condition of the tires	Make sure that the tire tread is not worn and that the tires have no visible signs of damage.

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Repair

- In case of reparation, please contact the producer.

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Customer service

- If you have any questions or need help, contact the producer.

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