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## PRODUCT DESCRIPTION:

The bench is available in two commercial versions:

- Made of lightweight and durable anodized aluminum frame and seat made of plastic
- Entirely made of plastic

The corresponding profile of the benches allows them to firmly anchor on the lateral edges of the tub. Width adjustment allows you to adjust the width of the bench for almost any type of bath.

## INDICATIONS

The bench facilitates patients transfer from and into the bath. It also facilitates every day toilet activities for people with motor limitations. It also provides a safe position for patient with bending limitations in hips e.g. after the hip endoprosthesis.

## CONTRAINDICATIONS

Contraindications to a sitting position, lack of self-stabilization of the trunk, impaired balance.

## USE (REGULATION):

Before installing the bench adjust its width to the size of the tub. Attach the bench on the upper edges of the tub. Make sure that the bench is anchored firmly before the transfer of the patient.

## SAFETY OF USING:

1. The bench can be used only by one person at the same time - a maximum load of 100 kg.
2. Before each use make sure that a bench is properly attached.
3. Do not stand on the bench seat, it threatens with a collapse.
4. At least once every three months should be performed visual inspection of all components of the product:
  - check the function of the telescopic width adjustment mechanism,
  - thoroughly check the stability and appearance of the frame and the seat.
5. Do not use the bench in the case of any visible or detectable defects (for example crack frame or seat), it may cause an accident.
6. Patients with balance disorders and intellectual disabilities should not be left unattended on the bench.

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## USE AND CONSERVATION:

1. Clean the frame of the bench with moist cloth.
2. Clean the seat with a moist cloth and mild detergent.
3. On an ongoing basis remove the contaminants from the adjustment mechanism.

Lack of proper care for the purity of the individual elements can cause rapid loss of functional parameters and aesthetic product.

### Regulation of bench width spacing grip:

- 62 – 70 cm for bench with aluminum frame
- 41 – 63 cm for bench made of plastic

Materials: Aluminum, polyurethane, rubber

Serial number/production date: on the product

Durability of product: 2 years from date of manufacture.

Medical device

### Dane teleadresowe \ Contact details

mdh Sp. z o.o.

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94-007, Łódź, Polska

tel. +48 42 674 83 84

fax. +48 42 636 52 21

www.mdh.pl www.viteacare.com