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Raised toilet seat

Numer katalogowy/Catalog number: VCTP0031

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English version on next page

Table of contents

- Product description..... 11**
 - Indications..... 12
 - Contraindications 12
- Usage 12**
- Assembly 12**
- Specifications..... 14**
- Care and maintenance 14**
 - Cleaning..... 14
 - Disinfection..... 14
- Service and repair..... 15**
 - Product disposal 15
- Symbols 15**
- Contact details 16**

Product description

The raised toilet seats with fixing to a toilet, lightweight, easy to clean, resistant to water, seats are made of contact-warm polyurethane material. Seats have screws clamp to a toilet and rubber pads that ensure stability.

ATTENTION

Always follow the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Keep this manual for future reference.

WARNING!

Improper use can lead to death, serious injury, unsafe conditions or product malfunction.

ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

SUGGESTIONS!

Follow the instructions below to keep your product in good working order.

If the product is used by another person, you must provide them with copies of these instructions for use.

These instructions for use relate to the design and specifications of the product at the time of publication. Due to design changes, some illustrations and photos in the manual may not correspond to the product you have purchased. We reserve the right to make design changes.

The manufacturer disclaims all liability for personal injury or property damage that may result from improper or unsafe use of this product.

Indications

Support equipment, facilitating - enabling handicapped and elderly persons to use the toilet.

Expected benefits

Facilitate handling daily physiological needs of people with reduced mobility, having trouble standing up, sitting down, bending over, and experiencing pain when trying to assume various positions.

Contraindications

No possibility of adopting a sitting position. For safety reasons, people with balance disorders should not remain without supervision when using the equipment.

Usage

The product in its original packaging folded, ready to be mounted on the toilet. In the version with handles, they should be mounted in the raised toilet seat beforehand. The product placed directly on the toilet seat and fasten by tightening the screw in the front of the product. Each time you use the product clean it with a damp cloth, using a mild detergent.

WARNING!

Any "serious incident" related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

ATTENTION!

The product is intended for people with disabilities with musculoskeletal disorders. If you are not sure whether your health condition allows you to use the product consult a health care professional.

Assembly

WARNING!

Do not modify the product in any way; this will void the warranty.

WARNING!

Exceeding the maximum user weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

1. Always be sure that the seat is correctly and securely locked in place before using.
As shown in the illustrations below.

For models VCTP0031 05-K6 i VCTP0031 05-6

For model VCTP0031 05-P6

2. Be sure that the adjustment knob in front of the seat is always tight.
3. This product is designed for use with the most common types of toilets. It will not fit all commode types; be sure that the unit is correctly installed, fits properly and is stable before use.
4. On models with arm rests: The arms are for the purpose of aiding when getting on and off the seat. The seat is contoured to provide safe and comfortable elevated seating. Removable handles are padded with a soft foam that provides a secure nonslip grip for added stability when lowering or rising from the seat.

**WARNING!**

The maximum capacity is 110 kg

It is unacceptable to use a product that has not been securely fastened to the toilet bowl - risk of falling.

Do not attempt to transfer the full weight of the body to the armrests when using the cap, as this may cause damage to the device.

Specifications

Model	VCTP0031 05-6	VCTP0031 05-K6	VCTP0031 05-P6
Maximum user weight	110 kg	110 kg	110 kg
Outside length	390 mm	390 mm	430 mm
Outside width	350 mm	350 mm	410 mm
Height	120 mm	120 mm	110 mm
Colour	white	white	white
Hole dimensions	220x290 mm	220x290 mm	220x240 mm
Height of handles	-	-	100 mm
Composition	polyurethane, rubber		
Durability	5 years from date of sale		

Care and maintenance

Cleaning

- Clean the wheelchair frames with a damp cloth.
- Remove contamination on a regular basis.
- In order to maintain the operational parameters of the product, it is extremely important to keep all its elements clean.
- Do not expose the product and the drive to extreme weather conditions (rain, snow, very strong sun), as some elements may change its color.

Disinfection

If the product is used by several people (e.g. in a nursing home), commercially available disinfectants should be used. Before disinfecting, clean the upholstery and handles. Only tested and approved disinfectants may be used for disinfection.

Information on recognized and approved disinfectants and disinfection methods is available from your national health authority.

**WARNING!**

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.

Service and repair

If repairs are needed or if you have any questions about the product, please contact your distributor.

Product disposal

The product must be disposed of in accordance with applicable local and national regulations. For information on the proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, please contact your local waste disposal company or authorized distributor.

Symbols

	Medical device
	Date of manufacture (printed on product)
	Manufacturer
	Distributor
	Conformité Européene (compliance with the European Community))
	Catalog number (to re-order)
	Serial number
	LOT NUMBER
	Unique Device Identifier

	Refer to the instructions for use
	Warning
	Possibility of jamming fingers
	Permitted temperature
	Keep away from sunlight
	The product is designed to be transported by car
	Maximum patient weight

Contact details

Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Polska

tel. +48 42 674 83 84, fax. +48 42 636 52 21

www.mdh.pl www.viteacare.com

We reserve the right to make technical changes due to technical progress.