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
**English version** on next page

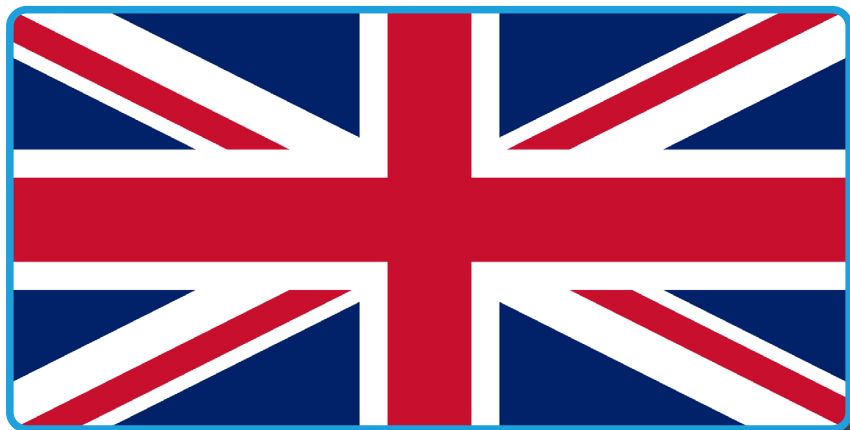**Users with visual impairments can find the PDF files together with further information on our website:**

**<[www.viteacare.com](http://www.viteacare.com)>.**

**☞ Contact your specialist dealer when required.**

**Alternatively users with visual impairments can have the documentation read out by a helper.**

# Table of contents

- Introduction ..... 17**
  - Indications..... 18
  - Contraindications ..... 18
- Product description..... 18**
- Usage ..... 19**
  - Assembly ..... 19
- Safety instructions ..... 20**
- Technical specifications ..... 21**
- Care and maintenance ..... 21**
  - Cleaning..... 21
  - Disinfection..... 22
  - Reuse (multiple product usage) ..... 22
- Service and repair..... 22**
  - Life span..... 22
  - Product disposal ..... 22
- Symbols ..... 23**
- Contact details ..... 24**

## Introduction

Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasise that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual. Please ensure that you complete the attached warranty form and return it to your distributor so that we can provide you with the best possible service.

# ATTENTION

Always observe the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Please keep these instructions for future use.

### WARNING!

Improper use can lead to death, serious injury, dangerous conditions or product malfunction.

### ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

### SUGGESTIONS!

Follow the instructions below to keep the product in good working order.

If the device is used by another person, a copy of these instructions for use must be provided.

These instructions for use apply to the design and specification of the product at the time of publication. Due to design changes, some figures and photos in the manual may not correspond to the product purchased. We reserve the right to make design changes.

The manufacturer disclaims any liability for personal injury or property damage that may result from improper or unsafe use of this product.

## Indications

For patients with significant movement limitations, generally poor physical status, aging persons; using a bath or shower for hygienic practices. For use by patients who cannot themselves perform actions and hygiene procedures. Stool secure the stable position of the patient, protects against the risk of falling. Protection against wound infection or dipping during hygiene procedures. Stool can be used during hospitalization and home care.

## Contraindications

Contraindications to a sitting position, the lack of trunk stabilization. Patients with major coordination disorders, balance and intellectual disabilities should not be left without supervision.

## Product description

The shower stool is a light construction made of aluminum. Stool legs with regulated height assure stable support.

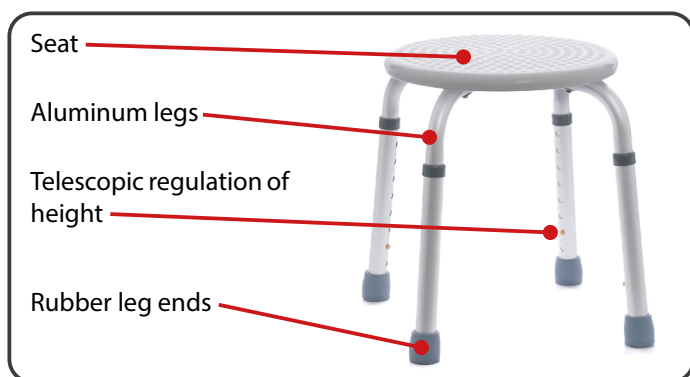
Illustration 1. Stool elements.

## Usage

After unpacking the package, check that it contains all the parts according to the list of components above. Ensure that all parts are free from visible mechanical damage. In case of doubt contact the manufacturer. Do not use the stool until all parts have been properly assembled.

## Assembly

Start assembling the stool by connecting the stool frame by inserting the bolt and washer into the mounting hole located in the centre of the frame (Illustration 2.).

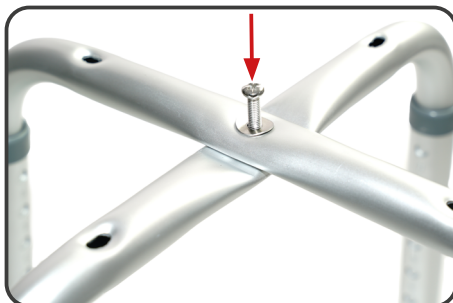
Illustration 2. Assembly of the frame with legs.

- Then, using a 10 mm open-ended key and a cross-head screwdriver (Philips), tighten the screw and make sure that the seat frame is securely connected. Make sure that all the legs are set to the same height and that they are properly attached.

Illustration 3. Assembly of the frame with legs.

- Assemble the seat using the supplied screws by inserting them into the holes in the frame and then screwing them into the mounting points on the seat using a Phillips screwdriver (Illustration 4.).

Illustration 4. Seat assembly.

## Safety instructions

### WARNING!

Any 'serious incident' relating to a device must be reported to the manufacturer and to the competent authority of the Member State in which the user is resident.

### ATTENTION!

The product is intended for people with disabilities and musculoskeletal conditions. If you are unsure whether your health condition allows you to use the product, consult a healthcare professional.

### WARNING!

The product must not be modified in any way; this will void the warranty.

### WARNING!

Exceeding the user's maximum weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

### WARNING!

Before using the product, please read these instructions for use and keep them for future reference.

- a. The stool is designed for use on flat surfaces.
- b. Using the stool on unsupported surfaces, i.e. rough surfaces (with a slope of more than 5%) may lead to falling down and injury.
- c. Standing on the stool is not permitted.
- d. Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of the product. It may cause burns or frostbite parts of the body.
- e. Do not use the stool if there is any visible or visible damage (e.g. cracked frame) as this can cause an accident.

### WARNING!

Maximum user weight is 100 kg

## Technical specifications

Maximum user weight	100 kg
Stool height	360-540 mm
Stool width	400 mm
Seat diameter	320 mm
Stool weight	1,5 kg
Materials	Aluminum, polypropylene, PVC
Description of the environment in which the product will be used	  
Storage and usage temperature	From +5°C to +40°C
Air humidity for storage and use	30% to 70 %
Required tools for assembly	Cross head (Phillips) screwdriver, 10 mm open-ended spanner

## Care and maintenance

### Cleaning

- Clean the stool frame with a damp cloth.
- Do not use harsh cleaning materials (risk of damaging the protective coating).
- Plastic elements should be cleaned with mild detergents.
- In order to maintain the performance of the product, it is extremely important to keep all its components clean.
- Do not expose the product to extreme weather conditions (rain, snow, very strong sun) as certain parts may change colour.
- After use, the product should be thoroughly dried to extend its expected life.

#### **Must not be used:**

- solvents
- toilet cleaners
- sharp brushes and hard objects
- chlorine-containing agents
- corrosive cleaning agents

*Do not clean the product with pressure or steam unit!*

## Disinfection

If the product is used by a large number of people (e.g. in a nursing home), commercially available disinfectants should be used. Prior to disinfection, the top surface should be cleaned. Only tested and approved disinfectants may be used for disinfection. Information on recognised and approved disinfectants and disinfection methods can be obtained from the national health authority.

### WARNING!

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of the product. Follow the instructions provided by manufacturer of the chemical product.

## Reuse (multiple product usage)

The product is suitable for reuse. The number of times it can be reused depends on how often and in which way the product is used. Before reuse, thoroughly clean and disinfect the product and check its safety features. If necessary, take repair action.

## Service and repair

The inspection should be carried out regularly and should be preceded by a thorough cleaning of the product (so as to reveal any hidden damage). If repairs are needed or if you have any questions about the product, please contact your distributor.

## Life span

The expected average life span of a medical device is 4 years, provided it is used as intended and all maintenance and servicing. The lifespan of a medical device also depends on the frequency of use, the environment in which it is used and its care. By using spare parts, the life of the device can be extended. Spare parts are generally available for a further 5 years after a model is discontinued.

The indicated lifespan does not constitute additional guarantee.

## Product disposal

The product must be disposed of in accordance with current local and national regulations. For proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, contact your local waste disposal company or authorised distributor.

## Symbols

	Medical device
	Production date (printed on the product)
	Manufacturer
	Distributor
	Conformité Européene (conformity with EU requirements)
	Catalogue number (to be reordered)
	Serial number
	LOT number
	Unique identification code
	Refer to the instructions for use
	Warning
	Possibility of jammed fingers
	Permitted temperature
	Protect from moisture

	For indoor use
	Keep away from sunlight
	The product is designed to be transported by car
	Maximum user weight

## Contact details

### Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

[www.mdh.pl](http://www.mdh.pl)

[www.viteacare.com](http://www.viteacare.com)

We reserve the right to make technical changes due to technical progress.