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## Rotor exerciser

Modele/Models: Genesis, Genesis Plus  
Numer katalogowy/Catalog number: DRVF06

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
**English version** on next page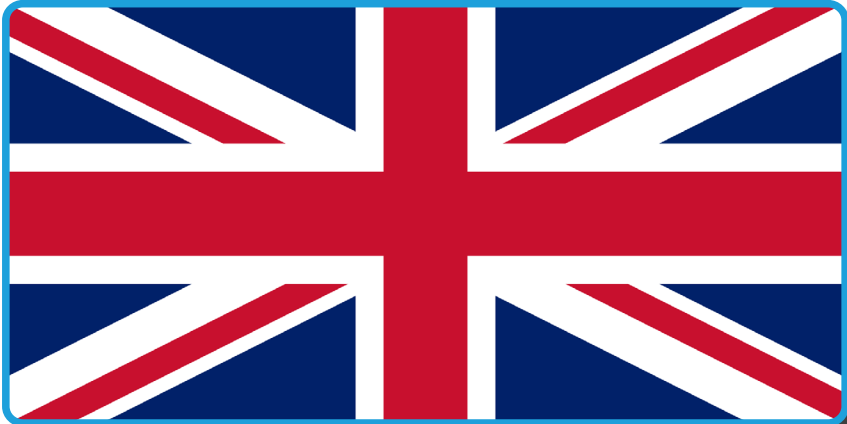

# Table of contents

<b>Introduction from mdh Sp. z o.o.</b> .....	<b>16</b>
Indications.....	17
Contraindications.....	17
<b>Product description</b> .....	<b>17</b>
<b>Usage</b> .....	<b>18</b>
Unfolding.....	18
Assembling.....	18
Lower and upper limbs exercise.....	18
Adjustment.....	18
Calorie counter installation only for models with calorie counter.....	19
Operation of the calorie counter.....	20
Battery replacement .....	21
<b>Safety instructions</b> .....	<b>21</b>
<b>Technical parameters</b> .....	<b>22</b>
<b>Service and repair</b> .....	<b>22</b>
<b>Care and maintenance</b> .....	<b>23</b>
Cleaning .....	23
Disinfection .....	23
Product disposal.....	23
Maintenance plan.....	24
<b>Symbols</b> .....	<b>24</b>
<b>Contact details</b> .....	<b>28</b>

## Introduction from mdh Sp. z o.o.

Thank you for choosing our product. We are strongly believe it will meet your expectations. Please get acquainted with this document before first use. This instruction is delivered to give necessary information about safe use of the device. Obeying the recommendations will influence the durability and aesthetics of the product. We would like to indicate that we keep trying to improve the quality of our products which may cause minor changes that may not be included in the instruction. Please ensure that you complete the attached warranty form and return it to your distributor so that we can provide you with the best possible service.

# ATTENTION

Always follow the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Keep this manual for future reference.

### WARNING!

Improper use can lead to death, serious injury, unsafe conditions or product malfunction.

### ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

### SUGGESTIONS!

Follow the instructions below to keep your product in good working order.

If the product is used by another person, you must provide them with copies of these instructions for use.

These instructions for use relate to the design and specifications of the product at the time of publication. Due to design changes, some illustrations and photos in the manual may not correspond to the product you have purchased. We reserve the right to make design changes.

The manufacturer disclaims all liability for personal injury or property damage that may result from improper or unsafe use of this product.

## Indications

Rehabilitation and recreation, recommended for upper and lower limbs paralysis and orthopaedic diseases. Exercises with use of the exerciser allow to form the muscle power, improve blood circulation within limbs, stimulate nervous system and improve the joint range of motion. Especially recommended for exercises during paralyse of lower limb peripheral nerves and for those who have experienced the cerebral stroke or other neurological dysfunction. The possibility of applying the exerciser at hospitals as well as in household conditions.

## Contraindications

Conditions requiring limited upper and lower limbs motion. Contraindication to physical movements (high blood pressure, cardio-vascular and breathing disorder)

## Product description

Figure 1.

# Usage

## Unfolding

- The product is packaged in a form to be assembled.
- After removing the packaging, make sure the exerciser is not mechanically damaged.
- Then straighten the base frame until the folding lock is latched in the main frame.
- Make sure the legs are properly installed and do not move.
- Knob governing the resistance of the pedal slightly screw into the hole located in the central part of the frame.

## Assembling

- To fold the exerciser, press folding lock and assemble his arms toward the main frame.

### WARNING!

Special care must be taken when folding the rotor exerciser as there is a risk of jamming fingers or hands.

## Lower and upper limbs exercise

- Sitting on a chair, set the exerciser in front of you, with the pedals located within the user's legs.
- Sitting on a chair, set the exerciser in front of you on a table or other piece of furniture with a flat area with that the pedals located within the user's hand.
- Next, insert your feet / hands into pedals stirrups. If you feel discomfort caused by improper distance between the pedals and the chair, it is recommended to set a proper distance.
- It is ready to start doing the exercises.

## Adjustment

- In the central part of the exerciser is a knob which regulate the resistance of the pedals.
- Turning the knob clockwise, the resistance is increased , this means that more force must be applied to move the pedals.
- Turning the knob counterclockwise reduces the resistance, this means that less force must be used to move the pedals.

## Calorie counter installation only for models with calorie counter

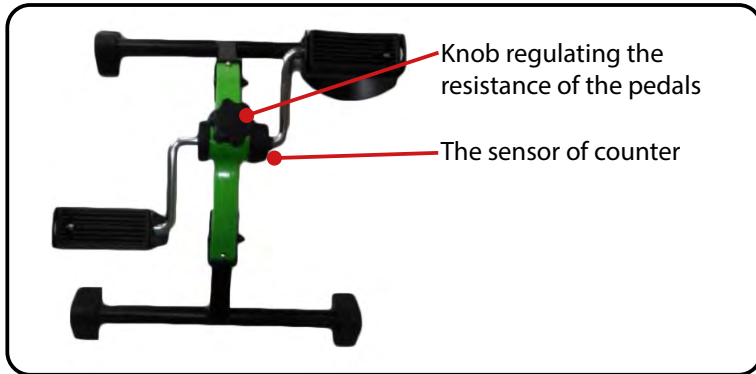
Figure 2.

- Before installing the calorie counter, unscrew the knob regulating the resistance of the pedal.

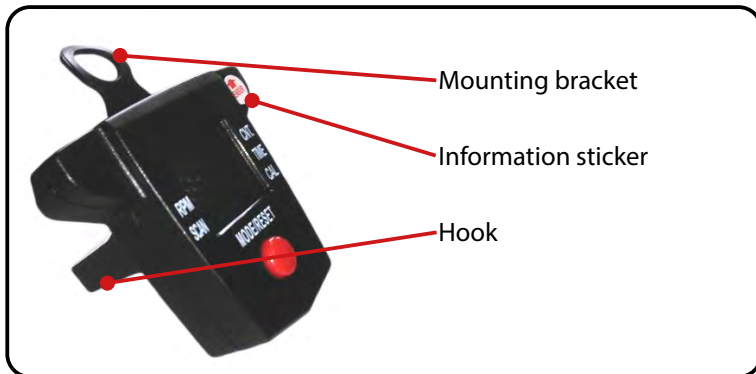
Figure 3.

- The calorie counter should be installed in such a way that information sticker was on the same side like of the sensor of counter.
- Thread must be surrounded by the mounting bracket . Knob regulating the resistance of the pedals must be screwed in the thread. Hooks should cover the main frame of exerciser (Illustration 4.)

Figure 4.

## Operation of the calorie counter

- The calorie counter is activated automatically after a few seconds of starting work of the pedals, or by pressing the red button **MODE/RESET**.
- A few seconds after you stop exercising, a **STOP** sign will appear in the top left corner, disappear when you resume exercising and start counting the parameters of the work done.

Figure 5.

- The calorie counter has different functions: (Figure 5.)
  - > **CNT**: gives the amount of rotations of the pedals (count how many meters on the pedal pass the calorie counter).
  - > **CAL**: gives the total calories amount consumed from the exercise session.
  - > **TIME**: gives the total time of the exercise session.
  - > **RMP**: shows the momentary frequency (number of rotations per minute).
  - > **MODE/RESET**: changes displayed value (**SCAN**, **CAL**, **TIME**, **RPM**), if the **MODE / RESET** button is pressed for more than 4 seconds, it will reset the timer and reset all parameters which were displayed.
  - > **SCAN**: by pressing **MODE / RESET** arrow points to **SCAN** option, exercise should be stopped and the computer will prepare the final results. After the **MODE / RESET** button is pressed again, they will be displayed.
  - > The calorie counter displays the value of the option, near which features an triangle-shaped arrow.
  - > To move to other option, press the **MODE / RESET** button.

## Battery replacement

- The calorie counter is operated by one battery AAA 1,5V.
- When the battery is running low, must be replaced.
- To replace the battery, pull out the battery cover. (Figure 6.)
- Then remove the old battery and insert a new one (Figure 7.)

Figure 6.

Figure 7.

## Safety instructions

### WARNING!

Any “serious incident” related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

### ATTENTION!

The product is intended for use by people with disabilities and movement disorders. If you are not sure whether your health condition allows you to use the product, consult a healthcare professional.

### WARNING!

Do not modify the product in any way; this will void the warranty.

### WARNING!

Exceeding the maximum user weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

### WARNING!

Before using the rotor, refer to this manual and keep it for future reference.

- Before using exerciser for upper and lower limbs , refer to this manual and keep it for future reference.
- Before using the exerciser should consult your doctor.
- Stop exercises and consult your specialist immediately if you feel dizziness, severe muscle/joint pain or chest pain.

- Warm-up and stretch about 5 minutes before beginning your exercises.
- Exercises should begin slowly and stay in controlled manner.
- Keep a minimum safety area clearance of 1 meter of the pedal exerciser. Be sure these area remains clear during use of the exerciser.
- Keep it away from children. The exerciser shall not be used as a toy.
- When it is used by, on or near children, invalids or disabled persons close supervision is necessary.
- The exerciser is designed for using on flat, even surfaces.
- Using the exerciser on a surface which does not provide the certain support, i.e. wet, greasy is unacceptable for the reason of the risk of the body injury.
- Exercises of lower limbs on the exerciser can be done in a sitting position only. Choose a stable chair.
- Select shoes, whose sole do not slip off the exerciser pedals.
- Do not stand on the exerciser pedals.
- Risk of burns – the exerciser can be warm after training, it is caused by the friction of moving parts.
- Do not leave the walker close to open sources of fire.
- Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of walker. It may cause burns or frostbite parts of the body.
- The acceptable pressure on the rotor's pedals amounts to 50 kg (total).
- Do not use the exerciser in case of discovering any visible or perceptible damage (e.g. the crack of the pedal, brackets), because it can cause the accident.
- There is a risk of jamming your fingers or hands in around the folding of base arms.
- All of repairs should be made by qualified manufacturer personnel.

## Technical parameters

Maximum device load	50 kg
Height	380 mm
Width	385 mm
Length	465 mm
Weight	2,4 kg
Power supply in model GENESIS PLUS	Battery type AAA 1,5 V
Materials	Steel, polyethylene, rubber
Description of the environment in which the product will be used	  
Permitted temperature	From +5°C to +40°C
The humidity for the storage and use	From 30% to 70%

## Service and repair

The review should be carried out regularly, it should be preceded by thorough cleaning the product (so as to reveal any hidden damage). If repairs are needed or if you have any questions about the product, please contact your distributor.

## Care and maintenance

### Cleaning

- Clean the frame with a damp cloth, clean the seat with damp cloth with mild detergent.
- Remove all soiling from the movable components (clamping screws, locks).
- In order to maintain the rotor parameters, it is very important to keep all elements clean and stored properly. Storing the rotor in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic of the product.
- Do not expose the rotor on to extreme weather conditions (rain, snow very strong sun), because certain parts can change color.

#### **It is not allowed to use:**

- solvents
- toilet detergents
- sharp brushes and hard objects
- detergents that contain chlorine
- corrosive detergents

*Do not clean the rotor with pressure or steam aggregates!*

### Disinfection

If the product is used by several people (e.g. in a nursing home), commercially available disinfectants should be used. Before disinfecting, clean the upholstery and handles. Only tested and approved disinfectants may be used for disinfection.

Information on recognized and approved disinfectants and disinfection methods is available from your national health authority.

#### **WARNING!**

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.

### Product disposal

The product must be disposed of in accordance with applicable local and national regulations. For information on the proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, please contact your local waste disposal company or authorized distributor.

## Maintenance plan

Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging	Check if any elements have any visible mechanical damage
Every two weeks	Screw and nut control regarding their correct fixing	Make sure if all the screws and top are fixed tightly
Every two months	Cleanliness and general condition	In case of mechanical damage contact the producer immediately

## Symbols

	Medical device
	Date of manufacture (printed on product)
	Manufacturer
	Distributor
	Conformité Européene (compliance with the European Community)
	Catalog number (to re-order)
	Serial number
	Lot code
	Unique Device Identifier
	Refer to the instructions for use

	Follow the instructions for use
	Warning
	Protect from humidity
	Keep away from sunlight
	Possibility of jamming fingers
	Permitted temperature
	For indoor/outdoor use
	Maximum user weight
	<p><b>DISPOSAL OF WASTE ELECTRICAL AND ELECTRONIC EQUIPMENT</b></p> <p>This symbol on the products or accompanying documents means that used electrical or electronic products should not be disposed of with the normal household waste. For proper disposal, refurbishment or recycling, please take these products to the collection points for this type of waste.</p> <p>Alternatively, in some EU countries or other European countries, you may return your product to your local retailer when purchasing a similar new product. By disposing of this product correctly, you will help to conserve valuable natural resources and support the prevention of potential negative effects on the environment and human health, which could result from incorrect disposal. For further information, please contact your local municipality or collection point waste collection points. Penalties may be imposed in accordance with local regulations if this product is disposed of in an inappropriate manner. For parties within the European Union, If you wish to discard electrical or electronic equipment, please obtain the necessary information from your retailer or supplier.</p> <p>Elimination in countries outside the European Union. This symbol is valid in the European Union. If you wish to decommission this appliance, please obtain the correct decommissioning information from your local authorities or from your retailer.</p>

# **Notatki**

## **Notes**

**Notatki**  
**Notes**

## Contact details

### **Manufacturer**

mdh Sp. z o. o.

ul. Maratońska 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

[www.mdh.pl](http://www.mdh.pl)

[viteacare.com](http://viteacare.com)

We reserve the right to make technical changes due to technical progress.