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VITEA®
CARE

USER MANUAL
CARTER Wheeled walker <DWSD>
VCBP0032
Edition MG 01.2016

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PLEASE READ CAREFULLY BEFORE
USING WHOLE OF INSTRUCTIONS

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Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasise that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual.

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1. Handle
2. The aluminum support frame
3. The reinforcing frame
4. The telescopic height adjustment
5. Submission button of a walker
6. The reinforcing frame
7. Legs finished wheels
- 8.

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Walking aid designed for disabled people for those with locomotor disorders.

Ideal for rehabilitation process and physiotherapy (verticalisation, lower limb relief).

Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves.

- The product is packed In folder position.
- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above.
- Ensure that all components have no visible damage. If any doubts, please contact the manufacturer

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- In order to unfold the walker, should leaning its sides maximally outside till the moment of latching the blockade. (Illustration 3.)
- For easy storage and transport, the walker can be folded.
- To do that, should put lock pin moving function on the reinforcing frame.
- Than push the submission button placed on the frame and fold side frames.

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- Release submission button. Walker should be folded to position shown on the Illustration 2.
- Walker can be moved, holding by side frames.

- Before using the walker, refer to this manual and keep it for future reference.
- Use of the product on the icy, wet, greasy, strongly gritty surfaces is unacceptable - it creates a risk of falling.
- The walker should be used on even surfaces at an incline less than 5%.
- Before each use of this product:
 - Perform visual inspection of all structural components of the product,
 - Carefully check the stability and condition of the support frame.
- Do not use the walker in the event of any visible or perceptible damage (e.g. cracks in the frame). It may cause an accident.
- The walker is designed for users with a maximum weight of 110 kg.
- The walker is design to support one person, climbing and standing on the walker is strictly forbidden.
- Do not leave the walker close to open sources of fire.
- Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of walker. It may cause burns or frostbite parts of the body.

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- j) It is unacceptable to use the walker after discovering any visible or detectable damages (e.g. broken frame) as it may lead to accident.
- k) All the repairs needs to be performed by a skilled personnel.
- l) Do not exceed the maximum height.
- m) Keep walker with both hands simultaneously, in the direction of movement.
- n) Make sure that the lock used to folding and unfolding the walker is working properly.
- o) Manufacturer does not recommend making any walker modifications on their own, especially by assembling additional wheels because product is not equipped with brakes. It might cause a fall or serious injuries. The manufacturer will not take responsibility for such actions.
- p) There is a risk of jamming fingers in the area for receipt of a walking frame.

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HEIGHT ADJUSTMENT

- Before using a walker, should adjusted to the user's height by telescopically adjustable feet length.
- To do this, press the height adjustment pin of the legs and depending on the needs reducing the height of the legs or extend the leg (Illustration 4.)

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MOVING WITH WALKER WITH WHEELS

- In the process of using a walker, the person must move walker forward and take a step forward, walk into the frame.

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CARE AND MAINTENANCE

- Clean the walker frame with a damp cloth
- Remove contaminants on a regular basis
- In order to maintain the walker's parameters, it is very important to keep all elements clean and stored properly.
- Storing the walker in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic.

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You must not use:

- solvents,
 - toilet detergents,
 - sharp brushes and hard objects,
 - detergents that contain chlorine,
 - corrosive detergents,
- Do not clean the rollator with pressure or steam aggregates!

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TECHNICAL REVIEW

- The review should be carried out regularly, it should be preceded by thorough cleaning the rollator (so as to reveal any hidden damage).

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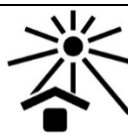
Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Every two weeks	Screw, nut and rivets control regarding their correct fixing.	Make sure if all the screws, top and rivets are fixed tightly.
Every two months	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.

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