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USER MANUAL
FRANK
4-WHEEL STEEL ROLLATOR VCBK221
Edition MG 12.2015

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**PLEASE READ WHOLE INSTRUCTION
CAREFULLY BEFORE USE**

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Four wheel steel rollator with footrest, equipped with two fixed rear wheels with brake and locking the wheels and the two front wheels swivel. It lets you easily navigate through any surface, inside and outside the building. Handles anatomical made of polyethylene, steel frame.

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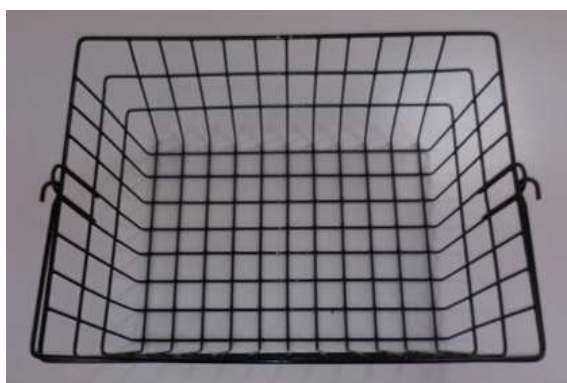
1. Handle holder
2. Brake lever with locking rotation of the wheels.
3. Handles height screw adjustment
4. Lock
5. Side frame
6. Rear wheel
7. Tray
8. Seat
9. Transverse frame
10. Basket for small items
11. Seat
12. Caster front wheels

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Elements that should be inside the packaging:

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Rehabilitation supports are an ancillary equipment to facilitate tilting and moving of patients with different types of motor disorders. They can be used as an aid for patients after surgery (lower limbs) who are unable to use crutches, showing no coordination disorders, patients with mild dysfunction of the lower limbs or weakness. They are used in patients with impaired balance preventing them from independent vertical position and locomotion, as well as various types of circulatory dysfunction (e.g. intermittent claudication). Supports can also be used as an aid for geriatric patients.

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Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves, impaired coordination and perception.

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- Do not use the rollator until all parts are properly connected.
- Unfold the frame, pulling its front and back crosspiece. Set the frame on an even surface.
- Place your hand on the seat and transverse frame, push firmly and block the blockade (it is placed on the side of the frame) until its automatic latch. Please, make sure that the safety lever reached its ultimate position. (Illustraton 8.)

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- Remove the plugs from slots, place the screw on yoke of the frame stand so that the marking on the screw head would be placed inside the slot. Place handle holder into the guide of frame stand and screw in the

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blocking screw. Please, make sure that the screws are screwed in tightly and that the both handle holder are at the same height.

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FOLDING:

- In order to fold rollator for ease of transport, you must first block rotation of the wheels
- Then it is necessary to remove the tray, basket for small items.
- turn the blockade lever upward.
- Draw the transverse frame upwards

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UNFOLDING

- Place your hand on the seat, push firmly and block the blockade (it is placed on the side of the frame) until its automatic latch. Please, make sure that the safety lever reached its ultimate position.
- Before each use check the operation of the braking system. Press the brake holders to control whether the brake pads adhere to the wheel well and are able to stop the rollator.
- Press the holders downward to activate the hand brake and check whether it operates correctly.

BRAKE FORCE ADJUSTMENT

- The breaking force can be adjusted – screw or unscrew the thumb, which is nearby Bowden cable. (Illustration 12.)
- Brake blocks cannot touch the ring of a wheel.

- When the brake force is adjusted, tighten the nut.
- After adjusting the braking force, it is important to check the operation of the braking system.
- Squeeze the brake handle to check if the brake pads properly adhere and are able to stop rollator. Then pull the handles down to apply the parking brake and check if it works properly.

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BRAKE SYSTEM

- The rollator is equipped with multi-functional brake. The brake handle is located under the rollator handle. It can operate as a standard brake as well as parking brake
- To use the standard brake, lift up the brake handle.
- To use a parking brake, push down the brake handle to the point where you can hear the distinctive sound of inclusion brake (You should use in that same time both the right and left brake).
- You should use in that same time both the right and left brake.

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ELBOW CRUTCHES

- On the right side of rollator, there are two elbow crutches holders: upper and lower.
- Place a elbow crutches in the bottom holder, and then push it in the top holder. Make sure that the elbow crutches is firmly seated in the holder, and that it will not fall out when you move.

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OVERCOMING OBSTACLES (THRESHOLD, CURBS)

- In order to drive the rollator onto the threshold or curb should set the rollator perpendicularly to the edge of the obstacle. Holding the handle, tilt the rollator in rear raise the front wheels up.
- Then drive the rear wheels to the edge of the obstacle and holding the rollator handle, lift slightly up and enter rollator on the obstacle.

CARE AND MAINTENANCE

- Clean the frame with a damp cloth, clean the seat with damp cloth with mild detergent.
- Remove all soiling from the movable components of the support (wheels, clamping screws, locks).
- In order to maintain the walker's parameters, it is very important to keep all elements clean and stored properly. Storing the walker in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic of the wheelchair.
- Do not expose the wheelchair on to extreme weather conditions (rain, snow very strong sun), because certain parts can change color.

You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the rollator with pressure or steam aggregates!

TECHNICAL REVIEW

The review should be carried out regularly, it should be preceded by thorough cleaning the rollator (so as to reveal any hidden damage).

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Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Before each use	Control of wheels, brakes,	The brakes can worn out, check the condition of the brakes before each use. You should also check the condition of wheels and clean them from dust and sand.
Every two weeks	Screw and nut control regarding their correct fixing.	Make sure if all the screws and top are fixed tightly.
Every two months	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.
Once a month	wheel axles	Wheel axles must be impregnated, between the axle and the hub put a few drops of lubricant.

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
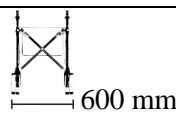
Repair

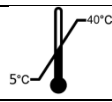- In case of reparation, please contact the producer.

Customer service

- If you have any questions or need help, contact the producer.

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