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PLEASE READ WHOLE INSTRUCTION  
CAREFULLY BEFORE USE

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Four-wheel walker ambona type is a modern, stable, durable and safe product. It was created to support the rehabilitation of the patient, allow the patient to move without help of the others.

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1. Armrest
2. Armrest height adjustment knob.
3. Caster rear wheel with locking rotation of the wheels.
4. Handle holder.
5. Handle holder mounting knob
6. Barrier
7. Upper stabilizing frame
8. Main frame

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- 9. Lower stabilizing frame
- 10. Caster front wheel

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Elements that should be inside the packaging:

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*mocowania podłokietników) / The mounting kit ( two screws with two pads to mount front wheels, four screws with pads to mount mounting frames, eight screws with pads to mount armrests)*

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Rehabilitation supports are an ancillary equipment to facilitate tilting and moving of patients with different types of motor disorders. They can be used as an aid for patients after surgery (lower limbs) who are unable to use crutches, showing no coordination disorders, patients with mild dysfunction of the lower limbs or weakness. They are used in patients with in patients with impaired balance preventing them from independent vertical position and locomotion, as well as various types of circulatory dysfunction (e.g. intermittent claudication). Supports can also be used as an aid for geriatric patients.

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Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves, impaired coordination and perception.

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knobs are positioned to the outer side in front of the support.

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- In bearer insert the handles and tighten the mounting knob handles.

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To unfold the rollator, should follow the opposite to the above manner of conduct

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- Before using the rollator, refer to this manual and keep it for future reference.
- At standstill should apply the locking rotation of the wheels both rear wheels.
- Do not stand on rollator.
- Do not sit on the armrest, because there is a danger of falling.
- The rollator should be used on even surfaces at an incline less than 5%
- With the rollator you can only use on surfaces where all four wheels touching the ground are to use all of four wheels.
- Use of the product on the icy, wet, greasy, strongly gritty surfaces is unacceptable - it creates a risk of falling.
- Before each use of this product:
  - Perform visual inspection of all structural components of the product,
  - Check the operation of the braking

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system. Squeeze the brake handles to see if the brake pads are fit properly to wheels and are able to stop rollator.

- Carefully check the stability and condition of the support frame.
- Do not use the support in the event of any visible or perceptible damage (e.g. cracks in the frame). It may cause an accident.
- The rollator is designed for users with a maximum weight of 100 kg.
- Do not leave the stroller with the user close to open sources of fire.
- Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of stroller. It may cause burns or frostbite parts of the body.
- All repairs must be carried out by qualified personnel. And during the warranty period only by the manufacturer's service.
- Do not exceed the maximum height of the handles to push, rollator will lose stability.
- Before using, make sure that all screws are tight.
- Due to the possibility of jamming the fingers put extreme caution during assembly rollator.

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### BRAKE SYSTEM

- The walker has a braking system located on the rear wheels. The braking system is the

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locking device. In order to stop the walker and immobilize it press down the lock located on the rear wheels. To unlock the wheels, pull up the lock.

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#### OVERCOMING OBSTACLES (THRESHOLD, CURBS)

- In order to drive the rollator onto the threshold or curb should set the rollator perpendicularly to the edge of the obstacle. Holding the handle, tilt the rollator in rear raise the front wheels up.
- Then drive the rear wheels to the edge of the obstacle and holding the rollator handle, lift slightly up and enter rollator on the obstacle.

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#### STAIRS

- When the patient climbs the stairs or down the stairs needs belaying of two people. One person should assist the patient, the other person should carry walker, because the patient after reaching the top will again need support. While moving the equipment, it is recommended to hold it for the frame.

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#### CARE AND MAINTENANCE

- Clean the frame with a damp cloth, clean the seat with damp cloth with mild detergent.
- Remove all soiling from the movable components of the support (wheels, clamping screws, locks).
- In order to maintain the walker's parameters, it is very important to keep all elements clean and stored properly. Storing the walker in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic of the wheelchair.
- Do not expose the wheelchair on to extreme weather conditions (rain, snow very strong

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sun), because certain parts can change colour.

- If walker has wetted in the rain, it must be dried, use anticorrosive to keep the components in good condition.
- Regularly check the mobility of the rotating elements, and if its need to apply grease.
- Avoid contact tires of the wheels with the acid to prevent the deterioration of the tire rubber quality.

You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the rollator with pressure or steam aggregates!

#### TECHNICAL REVIEW

The review should be carried out regularly, it should be preceded by thorough cleaning the rollator (so as to reveal any hidden damage).

New user

The rollator can be passed to a new user after technical inspection performed by qualified medical personnel. The former user should provide complete technical documentation which is crucial for safety of using.

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Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Before each use	Control of wheels, brakes,	The brakes can worn out, check the condition of the brakes before each use. You should also check the condition of wheels and clean them from dust and sand.
Every two weeks	Screw and nut control regarding their correct fixing.	Make sure if all the screws and top are fixed tightly.
Every two months	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.
Once a month	wheel axles	Wheel axles must be impregnated, between the axle and the hub put a few drops of lubricant.

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#### Repair

- In case of reparation, please contact the producer.

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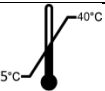
#### Customer service

- If you have any questions or need help, contact the producer.

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