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USER MANUAL
ROSE
4-WHEEL ROLLATOR WITH FOOTREST
VCBK222T
Edition MG 04.2017

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**PLEASE READ WHOLE INSTRUCTION
CAREFULLY BEFORE USE**

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Four wheel aluminium rollator with footrest, equipped with two fixed rear wheels with brake and locking the wheels and the two front wheels swivel. It lets you easily navigate through any surface, inside and outside the building.

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1. Handles.
2. Brake lever with locking rotation of the wheels.
3. Height adjustment screw of handles.
4. Rollator frame.
5. Expansion hinge.
6. Brake.
7. Rear wheel.
8. Additional handles for the seated person.
9. Backrest.
10. Side frame hinge.
11. Seat .
12. Bag for small items.
13. Footrest.
14. Steering wheel fork.
15. Front wheel.

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Elements that should be inside the packaging:

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Rehabilitation supports are an ancillary equipment to facilitate tilting and moving of patients with different types of motor disorders. They can be used as an aid for patients after surgery (lower limbs) who are unable to use crutches, showing no coordination disorders, patients with mild dysfunction of the lower limbs or weakness. They are used in patients with impaired balance preventing them from independent vertical position and locomotion, as well as various types of circulatory dysfunction (e.g. intermittent claudication). Supports can also be used as an aid for geriatric patients.

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Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves, impaired coordination and perception.

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- The product is packed In folder position.
- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above.
- Ensure that all components have no visible damage. If any doubts, please contact the manufacturer.
- Do not use the rollator until all parts are properly connected.

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- properly to wheels and are able to stop rollator. Press the handles down to activate the parking brake and check if it works properly.
- Carefully check the stability and condition of the support frame.
- Do not use the support in the event of any visible or perceptible damage (e.g. cracks in the frame). It may cause an accident.
- The rollator is designed for users with a maximum weight of 120kg.
- Maximum weight that can be moved in the bag- 1,5kg.
- Do not leave the stroller with the user close to open sources of fire.
- Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of stroller. It may cause burns or frostbite parts of the body.
- All repairs must be carried out by qualified personnel. And during the warranty period only by the manufacturer's service.
- Do not exceed the maximum height of the handles to push, rollator will lose stability.
- Push the rollator with both hands simultaneously in the direction of travel.
- Before using, make sure that all screws are tight.
- Make sure that the Levels hinge of stretch securing before submitting rollator, work properly.
- Due to the possibility of jamming the fingers put extreme caution in the area of horizontal hinge strut, fold-out footrest and side frame hinge.

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HEIGHT ADJUSTMENT HANDLES.

- Handles should be located at a height that will provide comfort and a sense of security for the user. It is important to ensure that both hands are positioned at the same height.
- To change the height of the handles unscrew the knob for fixing handles, depending on the needs extend or shorten the frame handles.
- After adjusting the appropriate height, in the mounting opening located on outer side, the side frame should insert knob equipped with a screw and tighten it.
- Make sure that the handle is properly tightened, so that it prevents movement of the handles.

FOLDING:

- In order to fold rollator for ease of transport, you must first block rotation of the rear wheels.
- In order to remove backrest should press two clasp located in backrest mounting opening and at the same time pull the backrest

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upwards.

- In order to remove handle should unscrew the knobs and remove the handles.
- Then it is necessary to fold footrest.
- Pick up the seat, pull upward levels hinge of stretcher, then fold the rollator according to move expansion hinge. (Illustration 12.)
- When moving the rollator, always hold the side frame of rollator.

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UNFOLD:

- To unfold rollator slide outward side frames, then lock the expansion hinge.
- In case of earlier dismantling of the handles and backrest, should install them in accordance with point 5.
- Before each use, check the operation of the braking system. Squeeze the brake handles to check if the brake pads are right adhere wheels and stop the rollator.
- Press the handle down to activate the locking rotation of the wheels and check if it works properly.

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ADJUSTMENT OF BRAKING

- To adjust the brake force at first loosen the

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nut is located under the adjusting screw just above the wheel brake.

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BRAKE SYSTEM

- The rollator is equipped with multi-functional brake. The brake handle is located under the rollator handle. It can operate as a standard brake as well as parking brake
- To use the standard brake, lift up the brake handle.
- To use a parking brake, push down the brake handle to the point where you can hear the distinctive sound of inclusion brake (You should use in that same time both the right and left brake).
- You should use in that same time both the right and left brake.

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OVERCOMING OBSTACLES (THRESHOLD, CURBS)

- In order to drive the rollator onto the threshold or curb should set the rollator perpendicularly to the edge of the obstacle. Holding the handle, tilt the rollator in rear raise the front wheels up.
- Then drive the rear wheels to the edge of the obstacle and holding the rollator handle, lift slightly up and enter rollator on the obstacle.

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CARE AND MAINTENANCE

- Clean the frame with a damp cloth, clean the seat with damp cloth with mild detergent.
- Remove all soiling from the movable components of the support (wheels, clamping screws, locks).
- In order to maintain the walker's parameters, it is very important to keep all elements clean and stored properly. Storing the walker in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic of the wheelchair.

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- Do not expose the wheelchair on to extreme weather conditions (rain, snow very strong sun), because certain parts can change color.

You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the rollator with pressure or steam aggregates!

TECHNICAL REVIEW

The review should be carried out regularly, it should be preceded by thorough cleaning the rollator (so as to reveal any hidden damage).

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Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all	Check if any elements have any visible

	the elements within the packaging.	mechanical damage.
Before each use	Control of wheels, brakes,	The brakes can worn out, check the condition of the brakes before each use. You should also check the condition of wheels and clean them from dust and sand.
Every two weeks	Screw and nut control regarding their correct fixing.	Make sure if all the screws and top are fixed tightly.
Every two months	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.
Once a month	wheel axles	Wheel axles must be impregnated, between the axle and the hub put a few drops of lubricant.

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Repair

- In case of reparation, please contact the producer.

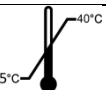
Customer service

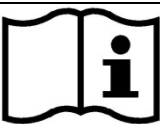
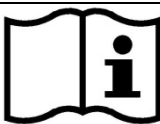
- If you have any questions or need help, contact the producer.

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