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Kule łokciowe

VCBP0042

Wydanie MG 05.2016


USER'S MANUAL

Elbow crutches

VCBP0042

Edition MG 05.2016

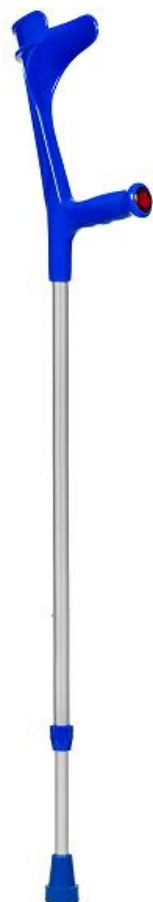VITEA[®]
CARE

CE

Spis treści

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PLEASE READ CAREFULLY BEFORE
USING WHOLE OF INSTRUCTIONS

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The elbow crutches are made of high quality, round cross section aluminium profiles. They are equipped with the half-elliptical forearm suspender that guarantees perfect elbow stability and the handle (hand grip), designed to fit anatomical shape of a hand and covered with soft in touch, pleasant material that prevents corns creation and with non-skid attachment element. Both the suspender and the handle are made of PU plastics, of especially selected hardness. The adjustments of crutch length and handle height can be performed by steps 2.5 cm, by means of friction-latch type regulator, stabilized with additional fastener.

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1. Universal,
2. Made of aluminium,
3. Suspender and handle are made of PU plastics,
4. Height adjustments in step of 2.5 cm,
5. Permitted user weight 140 kg.
6. Rubber made foot attachment: elastic, grey (does not stain floors or parquets),
7. Cat's eye mounted in the handle.

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For persons who need assistance while walking, in order to suspend and move their body weight in case of injuries or orthopaedic disorders, such as fractures, strokes or paresis of lower limb, or while learning to walk

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Coordination disorders, balance disorders, injuries or traumas of upper limbs that results in unavailability to use crutches.

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Crutches are shipped in its minimum length (retracted). After removing from package, crutches need to be set to the desired length (height). It is recommended that the height of the crutch is set so that the handle levels a greater trochanter of the thigh bone.

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For the product to maintain its proper characteristics, it is extremely important to maintain all its elements clean at all times and store it properly. Prolonged storing at premises of very high humidity (saunas, laundry rooms, etc.) or outdoors, along with lack of adequate maintenance, will lead to quick loss of the initial utility and aesthetical properties.

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MAINTENANCE AND SERVICE:

- Aluminium profiles of the crutches are to be cleaned with a damp cloth.
- Hand grip and elbow suspender are to be cleaned with soft detergents,
- All moving elements of the crutches (friction-latch regulator) are to be removed immediately

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You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the crutch with pressure or steam aggregates!

TECHNICAL REVIEW

- The review should be carried out regularly, it should be preceded by thorough cleaning the crutches (so as to reveal any hidden damage).

Repair

- In case of reparation, please contact the producer.

Customer service

- If you have any questions or need help, contact the producer.

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