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Kule pachowe

VCBP0041

Wydanie MG 05.2016

USER'S MANUAL

Armpit crutches

VCBP0041

Edition MG 05.2016

**VITEA<sup>®</sup>**  
**CARE**

**CE**

## Spis treści

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PLEASE READ CAREFULLY BEFORE  
USING WHOLE OF INSTRUCTIONS

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Armpit crutches are product made of oval-shaped aluminum profiles. The crutch is equipped with an armpit, hand grip, frame support element, anti-slip end cap.

Armpits and handles are made of rubber foam material of carefully chosen hardness. The crutches are equipped with frictional-pawl regulator which allows handle and armpit regulation. The height regulation range is 20 cm (every 2.5 cm).

1. Universal,
2. Made of aluminum.
3. Armpits and handles made of rubber foam material
4. Height regulation every 2.5 cm.
5. Maximum weight 100 kg.
6. Gum end cap: elastic, black, (does not stain floors or parquets).

Designed for people who need support during walking, after lower limbs injuries, for better balance, while transferring the body weight in case of traumatic and orthopaedic diseases, after fractures, in case of palsies and paresis.

Coordination disorders, balance disorders, injuries or traumas of upper limbs that results in unavailability to use crutches.

Crutches are shipped in its minimum length (retracted). After removing from package, crutches need to be set to the desired length (height). It is recommended that the height of the crutch is set so that the handle levels a greater trochanter of the thigh bone.

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For the product to maintain its proper characteristics, it is extremely important to maintain all its elements clean at all times and store it properly. Prolonged storing at premises of very high humidity (saunas, laundry rooms, etc.) or outdoors, along with lack of adequate maintenance, will lead to quick loss of the initial utility and aesthetical properties.

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### **MAINTENANCE AND SERVICE:**

- Clean the crutches frames with a moist cloth.
- Clean the armpits and handles with gentle detergent.
- Regularly clean the crutch mobile elements (pawl
- blockade mechanism)

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You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the crutch with pressure or steam aggregates!

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#### TECHNICAL REVIEW

- The review should be carried out regularly, it should be preceded by thorough cleaning the crutches (so as to reveal any hidden damage).

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#### Repair

- In case of reparation, please contact the producer.

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#### Customer service

- If you have any questions or need help, contact the producer.

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