Kule pachowe

Armpit crutches

Numer katalogowy/Catalog number: VCBP0041

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English version on next page

Table of contents

- Introduction 11**
 - Indications..... 12
 - Contraindications 12
- Product description..... 12**
- Usage 12**
- Safety of use..... 12**
- Technical parameters 13**
- Service and repair..... 14**
- Care and maintenance 14**
 - Cleaning..... 14
 - Disinfection..... 14
 - Product disposal 14
- Symbols 15**
- Contact details 16**

Introduction

Armpit crutches are product made of oval- shaped aluminum profiles. The crutch is equipped with an armpit, hand grip, frame support element, anti-slip end cap. Armpits and handles are made of rubber foam material of carefully chosen hardness. The crutches are equipped with frictional-pawl regulator which allows handle and armpit regulation. The height regulation range is 200 mm (every 25 mm).

ATTENTION

Always follow the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Keep this manual for future reference.

WARNING!

Improper use can lead to death, serious injury, unsafe conditions or product malfunction.

ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

SUGGESTIONS!

Follow the instructions below to keep your product in good working order.

If the product is used by another person, you must provide them with copies of these instructions for use.

These instructions for use relate to the design and specifications of the product at the time of publication. Due to design changes, some illustrations and photos in the manual may not correspond to the product you have purchased. We reserve the right to make design changes.

The manufacturer disclaims all liability for personal injury or property damage that may result from improper or unsafe use of this product.

Indications

For persons who need assistance while walking, in order to suspend and move their body weight in case of injuries or orthopaedic disorders, such as fractures, strokes or paresis of lower limb, or while learning to walk.

Contraindications

Coordination disorders, balance disorders, injuries or traumas of upper limbs that results in unavailability to use crutches.

Product description

- Universal,
- Made of aluminum
- Suspenders and handle are made of PU plastic
- Height regulation every 25 mm
- Maximum user weight 100 kg
- Rubber made foot attachment: elastic, grey color (does not stain floors or parquets),
- Cat's eye mounted in the handle

Usage

Crutches are shipped in its minimum length (retracted). After removing from package, crutches need to be set to the desired length (height). It is recommended that the height of the crutch is set so that the handle levels a greater trochanter of the thigh bone.

WARNING!

Any "serious incident" related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

ATTENTION!

The product is intended for use by people with disabilities and movement disorders. If you are not sure whether your health condition allows you to use the product, consult a healthcare professional.

Safety of use

WARNING!

Do not modify the product in any way; this will void the warranty.

WARNING!

Exceeding the maximum user weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

1. Crutches are intended for use indoors and outdoors, on various surface types.
2. When using crutches on a surface that does not provide proper support, such as wet, icy, greasy or sandy surfaces, one should be extremely cautious.
3. The crutches are designed to support weights of up to 100 kg.
4. All the construction elements of the crutches are to undergo periodical visual inspection, including:
 - check of friction-latch mechanism of the crutches
 - check of non-skid rubber foot
5. In case of any visible or sensible damages of the crutches (such as cracks), one should not use them, which may result in an injury.
6. It is recommended that all the repairs where done by the qualified and experienced service personnel of the manufacturer.

For the product to maintain its proper characteristics, it is extremely important to maintain all its elements clean at all times and store it properly. Prolonged storing at premises of very high humidity (saunas, laundry rooms, etc.) or outdoors, along with lack of adequate maintenance, will lead to quick loss of the initial utility and aesthetical properties.

WARNING!
Maximum weight of user is 100 kg

Technical parameters

Maximum user weight	100 kg		
Usage period	3 years from the date of purchase		
Size	S	M	L
Crutch height	930-1130 mm	1130-1330 mm	1330-1530 mm
Weight of two crutches	1,6 kg	1,7 kg	1,8 kg
Step adjustment	25 mm		
Description of the environment in which the product will be used			
Temperature of storage and use	Od +5°C do +40°C		
The humidity for the storage and use	od 30% do 70 %		
Materials	Aluminium, polyurethane foam, rubber		

Service and repair

The review should be carried out regularly, it should be preceded by thorough cleaning the crutches (so as to reveal any hidden damage). If repairs are needed or if you have any questions about the product, please contact your distributor.

Care and maintenance

Cleaning

Aluminium profiles of the crutches are to be cleaned with a damp cloth.

- Hand grip and elbow suspender are to be cleaned with soft detergents,
- All moving elements of the crutches (friction-latch regulator) are to be removed immediately

You must not use:

- solvents
- toilet detergents
- sharp brushes and hard objects
- detergents that contain chlorine
- corrosive detergents

Do not clean the crutch with pressure or steam aggregates!

Disinfection

If the product is used by several people (e.g. in a nursing home), commercially available disinfectants should be used. Before disinfecting, clean the upholstery and handles. Only tested and approved disinfectants may be used for disinfection.

Information on recognized and approved disinfectants and disinfection methods is available from your national health authority.

WARNING!

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.

Product disposal

The product must be disposed of in accordance with applicable local and national regulations. For information on the proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, please contact your local waste disposal company or authorized distributor.

Symbols

	Medical device
	Date of manufacture (printed on product)
	Manufacturer
	Distributor
	Conformité Européene (compliance with the European Community)
	Catalog number (to re-order)
	Serial number
	Lot code
	Unique Device Identifier
	Refer to the instructions for use.
	Warning
	Possibility of jamming fingers
	Protect from humidity
	Keep away from sunlight

	The product is designed to be transported by car
	Maximum user weight

Contact details

Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

www.mdh.pl www.viteacare.com

We reserve the right to make technical changes due to technical progress.