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PLEASE READ WHOLE INSTRUCTION
CAREFULLY BEFORE USE.

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Introduction

Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasise that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual

Product description

Innovative, patented system of pressure relief enables proper pressure distribution which prevents skin abrasions and bedsores.

Breathable, easy to clean cushion cover provides optimal comfort of use.

Anti-slip pad maintains the cushion on place.

Pressure regulation allows to change cushion height and softness.

Application

The cushion prevents skin abrasions caused by sitting for long periods, relieves lumbar spine, provides proper pelvic position which prevents pain related to lumbar spine overloading.

Contraindications

Contraindications to a sitting position, bedsores V° stage

Use

- Take the cushion out of the box and make sure that the valve is in open position, wait 30 seconds and the cushion will inflate automatically, without any additional devices.

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- When fully inflated, the cushion can support max 180 kg.
- Close the valve by turning it counter-clockwise. The white line should reach the white dot.
- In order to regulate softness and height of the cushion, sit on the cushion and turn the valve clockwise.
- Adjust the cushion softness to individual needs to obtain maximum comfort.

Safety of use

WARNING !

- a) Before using the cushion, refer to this manual and keep it for future reference.
- b) Permitted user weight 180 kg.
- c) Avoid using the cushion in extremely high or low temperature, do not expose it to UV rays.
- d) Keep away from fire.
- e) Keep away from sharp objects.
- f) The cushion can be used only with original cushion cover.
- g) Please remember that on the cushion should stay only the patient (other people on the mattress are not allowed).
- h) Pay attention to skin changes. If necessary, consult your doctor or nurse.
- i) The cushion can be used on the flat surfaces only. Using the mattress on the orthopaedic adjustable beds it is not recommended.
- j) Never deflate completely the cushion while sitting on it. After choosing the proper softness of the cushion, close the valve (turn it counter-clockwise) so that the white line would reach the white dot-it will prevent air from escaping.

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Everyday use and maintenance

For hygiene reason anti-bedsore cushion should be used only by one patient.

The cushion cover should be washed up by hand in warm water using mild detergents. Do not chlorinate. Do not whirlpool, wring gently before drying. Dry in horizontal position. Dry in a distance of artificial sources of heat. Do not iron. Close the valve before cleaning the cushion. The cushion should be washed up with the damp cloth using mild detergents. Dry in a distance of artificial sources of heat. Do not iron.

Always put on the cover when the cover and cushion are completely dry.

Technical parameters

Cushion available in the following sizes:

- 38 cm x 40 cm x 8 cm
- 40 cm x 40 cm x 8 cm
- 45 cm x 40 cm x 8 cm

150 kg

COMPOSITION:

Cushion - visco-elastic polyurethane foam, 75 D polyester, waterproof
Cushion cover: 100% poliester

Repair

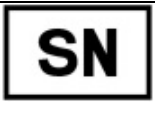
- In case of reparation, please contact the producer.

Customer service

- If you have any questions or need help, contact the producer.

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