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Wydanie 03.16MG/ Edition 03.16MG

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Anti-bedsore alternating pressure mattress of tubular structure VCM502

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PLEASE READ WHOLE INSTRUCTION
CAREFULLY BEFORE USE.

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Introduction

Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasise that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual

Product description

Tubular variable pressure mattress with 17 cells placed in two lines, equipped with a cover. Made of PCV which enable fast and easy cleaning. Smooth manual pressure adjustment adjusted individually to patient's weight.

Application

Prevention and support of bed sore treatment. Due to its tubular structure, the mattress ensures an appropriate arrangement of body weight and correct ventilation (oxygenation and moisture diversion). The air insulation provides an optimal temperature and comfort of use. All the above-mentioned factors play crucial role for bed sore prophylaxis and treatment. The variable pressure mattresses do not eliminate all causes of bedsores but due to changing position of body support points, they reduce the risk of bedsores formation and development.

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Contraindications

In case of adverse skin changes you should immediately consult a physician or medical personnel.

Use

- Place the mattress on a standard bed so that the air intake would be located on the side of patient's legs. Put the end flaps under the bed mattress.
- Check if the switch is in the "off" position.
- Connect the pump to electricity.
- Connect the pipes to the pump – insert them into the air outlet located on both sides of pump and on the mattress.
- Place a pump on hard surface or hang it on hinges. The place where the pump will stand must be clean and dry. Dust, hair or other impurity may block the filter.
- Please, check if the pipes are not folded or pressed.
- Set the knob into “ON” position.
- Set the pressure regulator on maximum. It will shorten the time of pumping which usually takes 15-20 min.
- After inflating the mattress, you should choose the pressure according to the doctor's recommendation.
- Place a patient on the mattress.

In order to receive exact instructions concerning an appropriate use and optimal pressure, please, consult your doctor. The mattress and pump create effect of "constant movement" (changing position of body support points), provide constant muscle massage and improve blood circulation. Thus, the mattress is an effective tool to prevent bedsores. But the position of the patient on the mattress needs to be changed despite of using it.

Switching off

- Set the knob into "OFF" position..
- Disconnect the pipes from the air outlet in order to prevent air from escaping.

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Safety of use

WARNING!

- a) Before using the mattress, refer to this manual and keep it for future reference.
- b) Permitted user weight 140 kg.
- c) Please remember that on the mattress should stay only the patient (other people on the mattress are not allowed).
- d) You are not allowed to do the physiotherapy treatment or rehabilitation on inflated mattress.
- e) The mattress should be protected against contact with body care cosmetics. It may cause permanent damage of the mattress.
- f) You must not use the mattress close to inflammable materials. It can lead to explosion !
- g) Always check if the pump is plugged into appropriate socket (220-230 V/50 Hz).
- h) The plug can not be removed from the socket when the pump is on.
- i) Always make sure the pump is in a dry place, remember that it cannot get wet.
- j) Do not touch the pump when it is wet. Always unplug the pump from electricity when it is wet.
- k) Never leave working pump unattended.
- l) Pay special attention if the pump is used in the presence of children or people with disabilities.
- m) Pump can be used according to its purpose only.
- n) Do not use it with other, unauthorized devices.
- o) Never use the pump with damaged or moist cable / plug. You should submit such a pump to a service point.
- p) The cable should be kept away from the hot surfaces.

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Everyday use and maintenance

The mattress is inflated by a system of pipes. The cells are alternately inflated which means that the patient's body is supported alternately by various support points. While one part of the body is supported by inflated cells, the other one is completely free from pressure.

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Phases of pump functioning:

1 phase: Inflating all the cells

2 phase: Deflating the first section of cells

3 phase: Reinflating all the cells

4 phase: Deflating the second section of cells

The phases occurs in cycles every 10 - 12 minutes

PROBLEM SOLVING

- Do not try to repair the pump, do not open it as it can lead to electric shock!
- The knob is in "ON" position but the pump does not work.
 - Please, check if the cable is plugged to electric supply and to the pump. If not, you should plug it.
 - Please, check if there is a voltage in the electric circuit. If there is not, find the cause of such situation.
 - Please, check if the cable is not damaged. If so, please contact the manufacturer.
- The pump works too slowly or the pressure is too low.
 - Please, check if the pressure regulator is set on an appropriate pressure. If not, please, regulate it.
 - Please, check if the pipes are correctly connected to the air outlet. If not, press them in such a way which prevents the air from escaping.
 - Please, check if the pipes are not folded or pressed. If so, please, unbend them.
 - Please, check if the cells are not damaged. If so, please, repair it using the reparation kit.
 - Please, check if the pump works correctly – if you can hear some strange sounds like hissing etc., please contact the manufacturer

CARE AND MAINTENANCE

Disconnect the pump from electricity. Clean all the surfaces with a moist cloth. Before switching on it has to be dry.

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- The mattress should be cleaned with water with soap. After cleaning, dry it carefully. The mattress mustn't be washed in high temperature, dried in a washing machine, bleached, chemically washed, ironed or pressed.
- The mattress may be disinfected using a product Incidin Plus, in accordance with the instructions for use of the preparation

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You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,
- Do not clean the exerciser with pressure or steam aggregates!

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Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Once a month	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.

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Repair

- In case of reparation, please contact the producer.

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Customer service

- If you have any questions or need help, contact the producer.

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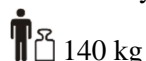
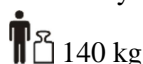
components: PCV

Series no.: on the packing

Durability: 3 years from production date.

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The product is compliant with standards on electrical and electronic equipment. It should not be disposed with other waste from households. The user is responsible for providing the equipment at the end of its service life to the appropriate collection points, under penalty of sanctions provided by the law applicable in the matter of waste disposal. For further information on the collection schemes available can be obtained from the local waste disposal center.

Contact details

 mdh sp. z o.o.
22/24 ks. W. Tymienieckiego Street
90-349 Łódź, Poland
tel. +48 42 674 83 84
fax. +48 42 636 52 21
www.mdh.pl www.viteacare.com