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Introduction

Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasise that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual.

ATTENTION

Always follow the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Keep this manual for future reference.

WARNING!

Improper use can lead to death, serious injury, unsafe conditions or product malfunction.

ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

SUGGESTIONS!

Follow the instructions below to keep your product in good working order.

If the product is used by another person, you must provide them with copies of these instructions for use.

These instructions for use relate to the design and specifications of the product at the time of publication. Due to design changes, some illustrations and photos in the manual may not correspond to the product you have purchased. We reserve the right to make design changes.

The manufacturer disclaims all liability for personal injury or property damage that may result from improper or unsafe use of this product.

Indications

The cushion prevents skin abrasions caused by sitting for long periods, relieves lumbar spine, provides proper pelvic position which prevents pain related to lumbar spine overloading.

Contraindications

Contraindications to a sitting position, bedsores V° stage.

Product description

Innovative, patented system of pressure relief enables proper pressure distribution which prevents skin abrasions and bedsores.

Breathable, easy to clean cushion cover provides optimal comfort of use.

Anti-slip pad maintains the cushion on place.

Pressure regulation allows to change cushion height and softness.

Usage

- Take the cushion out of the box and make sure that the valve is in open position, wait 30 seconds and the cushion will inflate automatically, without any additional devices.
- When fully inflated, the cushion can support max 180 kg.
- Close the valve by turning it counter-clockwise. The white line should reach the white dot.
- In order to regulate softness and height of the cushion, sit on the cushion and turn the valve clockwise.
- Adjust the cushion softness to individual needs to obtain maximum comfort.

WARNING!

Any "serious incident" related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

ATTENTION!

The product is intended for use by people with disabilities and movement disorders. If you are not sure whether your health condition allows you to use the product, consult a healthcare professional.

Safety of use

WARNING!

Do not modify the product in any way; this will void the warranty.

WARNING!

Exceeding the maximum user weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

- Before using the cushion, refer to this manual and keep it for future reference.
- Permitted user weight: 180 kg.
- Avoid using the cushion in extremely high or low temperature, do not expose it to UV rays.
- Keep away from fire.
- Keep away from sharp objects.
- The cushion can be used only with original cushion cover.
- Please remember that on the cushion should stay only the patient (other people on the mattress are not allowed).
- Pay attention to skin changes. If necessary, consult your doctor or nurse.
- The cushion can be used on the flat surfaces only. Using the mattress on the orthopaedic adjustable beds it is not recommended.
- Never deflate completely the cushion while sitting on it. After choosing the proper softness of the cushion, close the valve (turn it counter-clockwise) so that the white line would reach the white dot - it will prevent air from escaping.

WARNING!

Maximum weight of user is 180 kg

Technical parameters

Width	380 mm	400 mm	450 mm
Length	400 mm		
Height	80 mm		
Maximum user weight	180 kg		
Material composition	visco-elastic polyurethane foam, 75 D polyester, waterproof		
Cover composition	100% poliester		

Service and repair

If repairs are needed or if you have any questions about the product, please contact your distributor.

Product disposal

The product must be disposed of in accordance with applicable local and national regulations. For information on the proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, please contact your local waste disposal company or authorized distributor.

Care and maintenance

Cleaning

For hygiene reason anti-bedsore cushion should be used only by one patient. The cushion cover should be washed up by hand in warm water using mild detergents. Do not chlorinate. Do not whirlpool, wring gently before drying. Dry in horizontal position. Dry in a distance of artificial sources of heat. Do not iron. Close the valve before cleaning the cushion. The cushion should be washed up with the damp cloth using mild detergents. Dry in a distance of artificial sources of heat. Do not iron. Always put on the cover when the cover and cushion are completely dry.

Disinfection

Before disinfecting, clean the upholstery and handles. Only tested and approved disinfectants may be used for disinfection.

Information on recognized and approved disinfectants and disinfection methods is available from your national health authority.

WARNING!

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.

Symbols

	Medical device
	Date of manufacture
	Manufacturer
	Distributor
	Conformité Européene (compliance with the European Community)
	Catalog number (to re-order)
	Serial number
	Lot number
	Unique Device Identifier
	Follow the instructions for use
	Refer to the instructions for use.
	Warning
	Possibility of jamming fingers
	Permitted temperature

	Protect from humidity
	Keep away from sunlight
	The product is designed to be transported by car
	Maximum user weight
	Wash by hand - maximum temperature 40°C, do not use bleach, do not iron, do not dry clean, do not tumble dry.

Contact details

Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Polska

tel. +48 42 674 83 84, fax. +48 42 636 52 21

www.mdh.pl

www.viteacare.com

We reserve the right to make technical changes due to technical progress.