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English version on next page

Users with visual impairments can find the PDF files together with further information on our website:

<www.viteacare.com>.

☞ Contact your specialist dealer when required.

Alternatively users with visual impairments can have the documentation read out by a helper.

Table of contents

Introduction	44
Product characteristics	45
Indications	45
Contraindications	45
Safety instructions	46
General safety rules	47
Modifications	47
Security check.....	48
Capacity	48
Transport the scooter	48
Inclination guidance.....	48
Driving outdoors and precautions in bad weather conditions	49
Physical limitations	49
Information on electromagnetic compatibility (EMC)	50
Product information.....	51
Clutch lever for disengaging the drive	52
Battery charge indicator.....	52
Drive control lever	53
Power switch	53
Speed control dial	53
Horn.....	53
Directions lights	53
Operate.....	53
Before you start using	53
Getting on the scooter.....	54
Pre-ride inspection and adjustment.....	54
Operating a scooter	54
Getting off the scooter	55
Power off function after a certain time.....	55
Regulation and adjustment of scooter components	56
Delta steering column and rear view mirrors adjustment.....	56
Steering column angle adjustment.....	56
Seat rotation adjustment.....	57
Armrest spacing adjustment	57
Horizontal seat adjustment.....	57
Seat height adjustment.....	58
Disassembly and assembly the scooter	58
Disassembly	58
Disassembly procedures	58
Disassembly the frame	59

Assembly	59
Basic troubleshooting	60
Batteries and charging	61
First charge.....	61
Connecting the charger	62
Operating temperatures	62
Battery operation and charging	62
Battery charging process	62
Reporting technical defects and repairing the device	64
Care and maintenance	64
Tires.....	64
Wheel Replacement.....	64
External surfaces	64
Cleaning.....	65
Disinfection.....	65
Battery terminal connections.....	65
Electric wiring harnesses	65
Daily checks	65
Weekly checks.....	66
Monthly checks	66
Yearly checks	66
ABS plastic covers.....	66
Axle bearings and motor/transaxle	66
Console, Charger, and Rear Electronics	66
Nylon Lock Nut Replacement.....	66
Storage of the scooter	67
Re-use (reusable product)	67
Product life cycle.....	67
Replacement parts.....	67
Disposal of an electric scooter	68
Warranty.....	68
Technical parameters	69
Product safety symbols.....	71
Symbols.....	71
Contact details	76

Introduction

The CLEVER TRIP scooter has been designed and manufactured to the highest quality standards. Before using this product for the first time, please read these operating instructions. Improper use of the scooter may result in personal injury or traffic accidents. If you still have questions after reading this manual, please contact your specialist dealer. Ensure that the 'Warranty Card' document is completed and enclosed and handed over to the distributor, as this will enable claims to be processed efficiently.

ATTENTION

Always follow the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Keep this manual for future reference.

WARNING!

Improper use can lead to death, serious injury, unsafe conditions or product malfunction.

ATTENTION!

Improper use may lead to personal injury and/or damage to the scooter.

SUGGESTION!

Follow the instructions below to keep your scooter in good working order.

If the electric scooter is used by another person, you must provide them with copies of these instructions for use.

These instructions for use relate to the design and specifications of the product at the time of publication. Due to design changes, some illustrations and photos in the manual may not correspond to the scooter you have purchased. We reserve the right to make design changes. The product shipped in its original packaging requires partial assembly - follow the information in the "Disassembly and assembly the scooter" section. The manufacturer shall not be responsible for damage resulting from improper or unsafe use of the medical device, if such use was against the instructions and recommendations contained in the product documentation. The manufacturer shall ensure that the product has been designed and manufactured in accordance with the applicable safety and quality standards.

Product characteristics

An electrically-powered scooter, intended as a medical device, is a mobility aid for people with disabilities, the elderly or those with reduced mobility.

Construction: The Clever Trip scooter is available in a four-wheel version which provides good stability.

Drive: The vehicle is powered by an electric motor.

Energy range: On a single charge it can be up to several tens of kilometres, depending on battery capacity and conditions of use.

Safety: The vehicle has road lights, electromagnetic brakes and seat belts.

Comfort: The product is equipped with an adjustable seat, adjustable steering column angle, armrests, transport basket and cushioned suspension..

Indications

The product is intended for disabled people with musculoskeletal conditions to increase their mobility.

Contraindications

Contraindications to upright standing. Injury or paresis of the upper limbs preventing the use of a power wheelchair.

Safety instructions

- a. Please note that this is an electric scooter and is intended for outdoor use. Special care should be taken when driving around bends, obstacles and inclines.
- b. Always ensure that the electric scooter is fully assembled before getting on.
- c. Do not fold the scooter until you have dismounted completely, folded the scooter seat back and lowered the armrests to their lowest position.
- d. The folding mechanism of the electric scooter is equipped with an anti-trapping function. This is primarily to prevent damage to the armrests and backrests. Folding will be locked when the user is seated in the scooter. Assistance may be required to unfold the scooter. Folding the scooter while you are in it is strictly prohibited.
- e. Do not operate the folding mechanism while the user is on the seat.
- f. Read and follow the information in the instructions for use. It is forbidden to use this product and the available optional equipment without first carefully reading and understanding this manual. If you do not understand the warnings, cautions or instructions, contact your supplier, otherwise personal injury or accident may result. Never attempt to use the electric scooter beyond its limitations as described in this manual.
- g. Place your feet on the footrest when driving. Do not stand on the footrest.
- h. To operate an electric scooter safely, ride on flat surfaces where all four wheels make sufficient contact with the ground.
- i. If the battery level indicator drops to red, recharge the scooter as soon as possible. Completely discharging the batteries may damage them or shorten their life.
- j. Keep metal objects away from the battery connections. Risk of electric shock.
- k. When cornering, always reduce speed and keep the center of gravity stable. Always reduce speed when negotiating sharp bends.
- l. When reversing your electric scooter, make sure there are no obstacles behind you.
- m. It is prohibited for children to play near or operate the scooter.
- n. Some parts of the electric scooter are susceptible to temperature changes. The controller can only operate at temperatures between -10°C ~ 50°C (-14°F to 122°F).
- o. In extremely cold temperatures, the batteries may freeze and the electric scooter may not operate. In extremely high temperatures, the scooter may run at a lower speed due to the controller's heat protection function, which prevents damage to the motors and other electrical components.
- p. Store in clean and dry conditions.
- q. It is forbidden to drive at night without lighting.
- r. Do not remove the anti-tip wheels.
- s. Keep hands and feet away from moving parts when driving. Take care that loose clothing does not get caught in the drive wheels.
- t. Do not connect the battery charger to an extension cord.
- u. Disassembly of the controller, motor or charger by anyone other than an authorized service is prohibited and will invalidate any applicable warranty conditions.
- v. It is prohibited to drive the scooter on public roads and motorways.
- w. It is forbidden to use an electric scooter under the influence of alcohol, medication

- or any other substance affecting the ability to drive.
- x. Be careful when driving in busy areas and shopping centers.
 - y. Under no circumstances should the scooter be used as a seat when being transported by motor vehicles.
 - z. Do not attempt to lift the scooter by gripping anything other than the frame (for example, the seat, armrest or chassis components).
 - aa. Please note that the temperature of the components and frame parts may increase when exposed to the sun or other heat source, while at low temperatures there is a danger of very high cooling of the metal parts of the wheelchair. This can cause burns or frostbite on parts of the body.

WARNING!

The product is intended for use by people with disabilities and movement disorders. If you are not sure whether your health condition allows you to use the product, consult a healthcare professional.

WARNING!

Any "serious incident" related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

General safety rules

Read the instructions for use carefully with understanding before using the electric scooter.

Users may encounter difficulties with maneuvering when passing through narrow doorways, ascending and descending ramps, driving around corners and navigating uneven terrain. Reduce speed, take your time and maneuver the electric scooter carefully.

Modifications

Do not modify, remove, disable or add any parts, components or features to your electric scooter. The manufacturer offers a wide range of electric scooter accessories to meet your needs.

**WARNING!**

Do not modify the electric scooter in any way - this will void the warranty.

Security check

**ATTENTION!**

Check the following before use:

- Electrical connections and wiring harnesses - protect and remove corrosion.
- Batteries - fully charged and not corroded.
- Brakes - check that the electromagnetic brake is functioning.
- Folding mechanism - make sure the scooter is fully unfolded.
- Condition of tires - make sure they are not damaged or excessively worn - if there is a problem, contact your dealer.

Capacity

Clever Trip scooter is designed and built to transport one person with a maximum weight of 150 kg. The scooter is not designed to carry goods or objects other than those previously described.

**Warning!**

Exceeding the maximum user weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

Transport the scooter

When transporting the electric scooter by vehicle, store it safely in the back of a van, truck or in the boot of a car. Moving parts should be removed or properly secured for transport.

**Warning!**

Do not sit in an electric scooter while it is being transported in a motor vehicle.

Inclination guidance

Remain cautious when going up and down hills. If necessary, reduce your speed when going downhill. If you are moving up a hill try to keep the scooter moving forward. If necessary, stop. Be careful and slowly accelerate your scooter forward. If you are descending, reduce the speed of the electric scooter to the lowest setting and then proceed with caution. If the scooter is descending faster than you anticipated, slowly release the drive control lever until it stops and then push the lever slightly to continue descending safely.

**WARNING!**

Drive the electric scooter forward down or up a slope, never reverse. Irregular movements or making back and forth movements may increase the probability of tipping over. Changes in slope can reduce stability. Under no circumstances should you travel up a slope with a gradient greater than the maximum stability angle of this electric scooter.

Driving outdoors and precautions in bad weather conditions

Although the electric scooter is designed to be used indoors to help with daily living activities, the manufacturer has designed it to perform perfectly outdoors on level surfaces.

**ATTENTION!**

When driving outdoors, avoid uneven terrain, soft surfaces, tall grass, loose gravel, loose sand, fresh and salt water, stream banks, lakes and oceans. If you are unsure about a surface, avoid it.

Exposure to adverse weather conditions should always be avoided. If you find yourself in bad weather conditions while using your electric scooter, immediately go to the nearest shelter. Dry the electric scooter completely before use, before charging or before storing.

Physical limitations

Use awareness, caution, prudence and common sense when operating the scooter. Always be aware of your own limitations and rules of use when operating an electric scooter.

**WARNING!**

It is forbidden to use the scooter while under the influence of alcohol or psychotropic drugs. Never smoke while sitting in the scooter. Damaged or worn upholstery must be replaced as it increases the risk of fire.

**ATTENTION!**

When operating an electric scooter, please be aware of precautions, warnings and safety issues if you are taking medications prescribed by a doctor or available without a prescription.

**WARNING!**

Drive the electric scooter in a straight line up or down a slope, never reverse. Irregular forward and backward movements with changes in slope can increase the risk of scooter rollover.

Information on electromagnetic compatibility (EMC)

Despite complying with applicable EMC directives and standards, an electrically powered wheelchair may interfere with the operation of other electrical devices, such as mobile phones or alarm systems. In addition, this may affect the performance of the wheelchair itself. If you notice this behavior, it is a recommended idea to switch off the third-party device or power wheelchair and move it out of interference range. Remember that other devices emitting electromagnetic fields can also affect the operation of the wheelchair.

WARNING!

Do not use hand-held radio transmitters such as a CB-radio or cell phone near a powered-up scooter.

WARNING!

Pay attention and stay away from nearby transmitters, such as radio or TV stations.

Product information

The Clever Trip scooter is easy to handle and is primarily designed for outdoor use. It is foldable, making it very compact for storage and transport. The illustration below shows all the parts relevant to the user/driver. The clutch lever that disconnects the drive is located at the rear of the wheelchair near the anti-tip wheel.

Illustration 1. Clever Trip scooter component list.

- | | |
|-------------------------|-----------------------------------|
| 1. Control panel | 7. Disengaging drive clutch lever |
| 2. Mirrors | 8. Seat and backrest |
| 3. Hand-grip Delta | 9. Footplate |
| 4. Basket | 10. Batteries (2 boxes) |
| 5. Steering column | 11. Anti-tip wheels |
| 6. Front and rear wheel | |

Illustration 2. Control panel elements.

- | | |
|-----------------------------|--|
| 1. Battery charge indicator | 6. Power switch (key for power on) |
| 2. Emergency lights | 7. Left direction indicator |
| 3. Horn | 8. Right direction indicator |
| 4. Position light | 9. Driving direction lever (forward/reverse) |
| 5. Speed adjustment dial | |

Clutch lever for disengaging the drive

ATTENTION!

It is forbidden to regularly move the scooter by pushing it over long distances - this can damage the internal drive components.

If you ever need to push the scooter for a short distance, there is an option to switch the clutch lever disengaging the drive..

1. The drive disconnection lever is located at the rear of the wheelchair, on the right side near the anti-tip wheels.
2. Push down on the clutch lever to disengage the drive and electromagnetic brake. You can now push the scooter.
3. To engage the drive and brake system, lift the clutch lever upwards. Push mode disengaged.

Illustration 3. Location of the drive disconnect lever.

WARNING!

Remove the key from the power switch before disconnecting or engaging the drive clutch on the scooter. Never sit in a scooter that is on an incline while switching the drive clutch lever. If the scooter has the drive disconnected then the brakes are also not active.

- Disconnect the motor drive only on a level ground.
- Make sure the key is removed from the power switch.
- Take a sit next to the scooter when engaging or disengaging the drive clutch lever.
- Never sit in the scooter while engaging or disengaging the drive.
- When you have finished pushing, engage the drive to lock the brakes.

Battery charge indicator

When the key is turned, power is switched on and the charge indicator shows the power level of the batteries. For more information about the battery charge indicator, look at chapter "Batteries and Charging."

Drive control lever

The lever allows you to drive forwards or reverse up to the maximum speed set by the speed control knob.

- Grab the right handle with your right hand and the left handle with your left hand.
- Using your hand, pull the right side of the lever to disengage the scooter's brakes and move forward.
- Fully release the lever and allow the scooter to fully brake before pulling the left side of the lever allowing the scooter to move backwards.
- When the lever is fully released, it automatically returns to the center "stop" position, the brakes are automatically applied.

Power switch

- Turn the key to the "ON" (power on) position to start the scooter (Illustration 4.).
- Turn the key to the "OFF" position (power off) (Illustration 4.).

When you have finished riding / using the product, remove the key to prevent unauthorized use of the scooter.

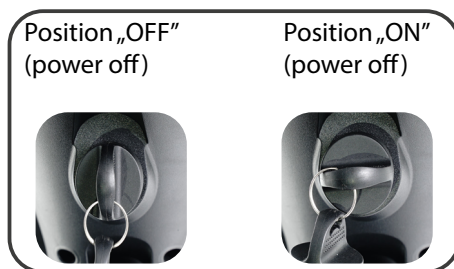
Illustration 4. Key position.

Speed control dial

The dial allows you to set the maximum speed limit.

- Turtle symbol represents the slowest setting.
- The rabbit symbol represents achieving maximum speed.

Horn

This button activates the warning signal. The scooter must be powered on for the horn to be active. Do not hesitate to use the horn if it can prevent an accident or injury.

Directions lights

To signal the direction of travel, activate the indicator by pressing once the button for the particular direction you intend to turn. To deactivate the indicator, press again the button for the direction that is being signaled.

Operate

Before you start using

- Have you fully charged the batteries? The scooter requires two long, full charging cycles (14 hours). See section "Batteries and charging".
- Is the manual lever of the drive disengagement clutch in the upper position (drive engaged).
- Never leave the lever in the lower position (disengaged drive) if you are not going to push the scooter manually.

Getting on the scooter

WARNING!

Never attempt to get on or off the scooter without first turning off the power (key in the "OFF" position). This prevents the scooter from being moved by accidentally switching the drive control lever.

1. Make sure the key is out of the power switch.
2. Take your seat next to the scooter.
3. Unlock the seat rotation lever and rotate the seat so that it faces you.
4. Make sure the seat is locked in this position.
5. Assume a safe and comfortable position on the seat.
6. Unlock the seat rotation lever and position the seat facing forward.
7. Make sure the seat is locked in this position.
8. Make sure your legs are securely located on the footplate.

Pre-ride inspection and adjustment

- Is the seat at the right height? See the section "Regulation and adjustment of scooter components".
- Is the seat securely locked?
- Is the steering column angle properly adjusted and locked? See the section "Regulating and adjusting scooter components".
- Is the key inserted into the power switch?
- Is the horn work correctly?
- Are there no people, animals or other obstacles along the route?
- Have you planned your route to avoid going through difficult terrain and, if possible, as few inclines as possible?

Operating a scooter

WARNING!

The following operations may have a negative effect on the steering and stability of the scooter:

- Keeping and walking an animal attached to a leash.
- Carrying passengers (including animals).
- Hanging an object from the steering column.
- Towing or pushing by another vehicle.

WARNING!

At all times when using your scooter, keep both hands on the handlebars and your feet on the footplate. This position provides the best control of the vehicle.

- Set the speed control dial to the desired speed.
- Pull the desired side of the drive control lever.
- The automatic electromechanical brake will disengage and the scooter will smoothly accelerate to the set speed.
- Pull the left handlebar grip to turn left.
- Pull the right handlebar grip to turn right.
- Move the steering column to the center position to go straight ahead.
- The electromagnetic brakes engage automatically when the lever is released.

ATTENTION!

The scooter, with the same speed control dial setting, moves slower in reverse than in forward gear.

Getting off the scooter

WARNING!

Never attempt to get on or off the scooter without first turning the key to the power off position. This prevents the scooter from being moved by accidentally switching the drive control lever

1. Stop the scooter completely.
2. Turn off the power by turning the key to the “OFF” position.
3. Pull the seat rotation lever and rotate the seat so that it faces the side of the scooter.
4. Make sure the seat is locked in this position.
5. Carefully and safely get off the seat and take a seat next to the scooter.
6. To make it easier to get on the scooter, you can leave the seat facing to the side of the scooter.

Power off function after a certain time

To keep the life of the batteries, the scooter is equipped with an energy-saving function to automatically turn off the power after a certain period of time. If you accidentally leave the key in the power switch to power “ON” position, but do not use it within a 15-minute interval, the scooter controller will turn off the power automatically.

If the timed shutdown function has been activated, you will need to do the following to return to normal use:

1. Turn to position “OFF” the key and pull out of the power switch.
2. Insert and turn the key to the “ON” position to restart the power to the scooter.

Regulation and adjustment of scooter components

Delta steering column and rear view mirrors adjustment

Illustration 5. Mirror adjustment.

It is possible to adjust the rear-view mirrors:

1. Slide the cap upward **3**.
2. Loosen the mirror nut.
3. Grasp the mirror arm gently **2**.
4. Rotate the tube **2** and set the mirror **1** in the desired position.
5. Screw the nut to lock the position of the mirror.

Steering column angle adjustment

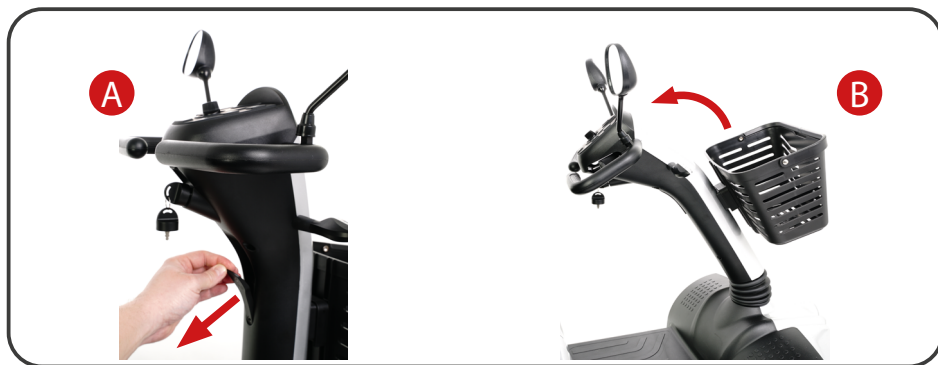
Illustration 6. Steering column tilt adjustment.

1. To adjust the tilt angle of the steering column, first pull and hold the lock shown in illustration 6 **A**.
2. Then set a comfortable steering wheel tilt angle and release the lock lever (Illustration 6 **B**).

ATTENTION!

For transport, the steering column must be lowered completely. Remove the seat and battery pack first. Remove the seat and batteries by pulling it upwards away from the scooter.

**WARNING!**

Remove the key from the power switch before adjusting the steering column or seat. It is prohibited to adjust the steering column or seat if the scooter is in motion.

Seat rotation adjustment

Seat rotation lever **2** located on the right side of the seat column, secures the seat in several positions. See illustration 7.

1. Pull the swivel lever **2** towards the front edge of the seat to unlock the seat.
2. Turn the seat to the desired position.
3. Release the lever to lock the seat in place.

Seat adjustment components

1. Armrest adjustment knobs
2. Seat swivel lever

Illustration 7. Seat adjustment.

Armrest spacing adjustment

The spacing of the armrests can be adjusted inward or outward.

1. Loosen the armrest adjustment knobs **2** - Illustration 7.
2. Move the armrests to the desired width.
3. Tighten the armrest adjustment knobs.

**ATTENTION!**

Raise the armrests to make it easier to get on and off the scooter.

Horizontal seat adjustment

To reduce or increase the distance of the seat from the steering column, adjust the horizontal position of the seat (Illustration 8.).

1. Pull up the adjustment lever **1** to unlock the seat.
2. Move the seat to the desired position.
3. Release the lever to lock the seat in place.

Illustration 8. Horizontal seat adjustment.

Seat height adjustment

The seat can be set at different heights (8 steps). See Illustration 9.

1. Remove the seat and battery pack from the scooter.
2. Use the following spanners: 17mm flat and 8mm Allen spanner to unscrew and remove the locking pin **2** from the lower part of the seat post **3**.
3. Raise or lower the upper seat post **1** to the desired height.
4. Keep the upper post at a height where the adjustment holes align with the lower post.
5. Fully insert the locking pin and tighten the nut securely.
6. Fit the seat and battery pack.

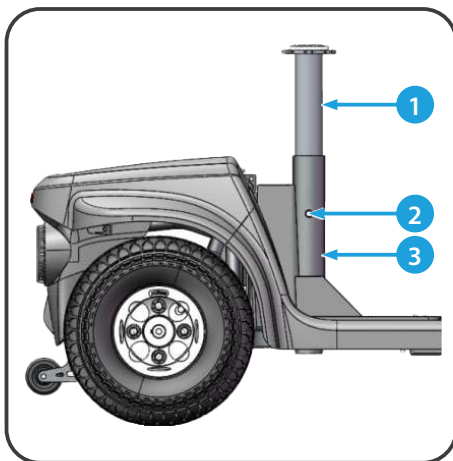
Illustration 9. Seat height adjustment.

Disassembly and assembly the scooter

Disassembly

You can disassemble the scooter to several components: ① seat, ② rear frame, ③ front frame, ④ basket, ⑤ battery pack. Disassembly and assembly does not require any tools. Always carry out disassembly on a level, dry surface in a room big enough to move freely, at a distance of 1.5m in each direction from the scooter.

Disassembly procedures

WARNING!

Do not lift weights beyond your physical capabilities. When disassembling and assembling the scooter, if necessary, ask for assistance.

Identification key

1. Seat
2. Rear frame
3. Battery pack
4. Front frame
5. Basket

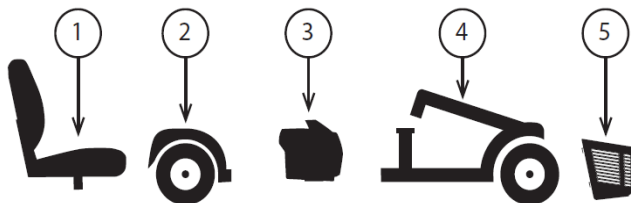
Illustration 10. Disassembled scooter.

1. Remove the seat by lifting it completely above the scooter. If you encounter resistance when removing the seat, lift the rotation adjustment lever and rotate the seat left and right while lifting up.
2. Remove the battery pack by lifting it off the scooter.

Disassembly the frame

ATTENTION!

Before attempting to disassemble, bend the rear of the scooter over the anti-tip wheels, as shown in illustration 11.

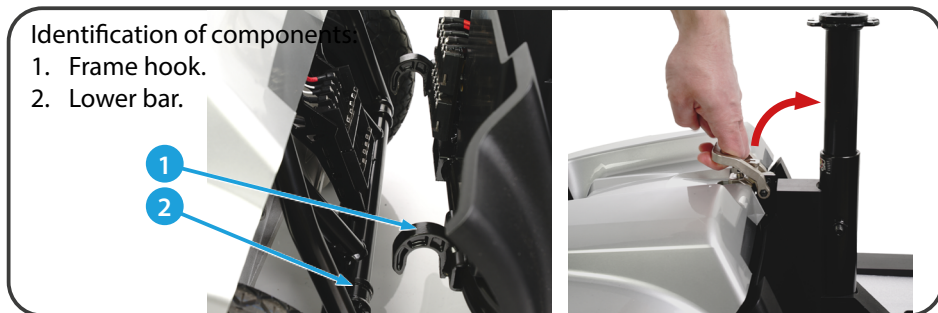
Illustration 11. Disassembling the frame.

1. After removing the seat and battery pack (see disassembly procedure in previous section) Illustration 10.
2. Lift the frame release lever and move half the frame forward until the hooks **1** will not rest on the lower bar of the rear frame **2**.
3. Slowly separate the two parts of the scooter frame.

Assembly

1. Use the front frame release lever to raise and align the front frame hooks to the rear frame bar. See illustration 11.
2. When the front frame hooks are over the rear frame bar, lower the front frame while tilting the rear frame. This will automatically activate the locking mechanism - a click will be heard.
3. Raise the steering column as indicated in illustration 6.
4. Install the battery pack.
5. Install the seat and rotate until it locks in place.

WARNING!

Be especially careful when unfolding and folding the wheelchair, lack of attention may cause body parts to be jammed by moving and folding parts!

Basic troubleshooting

WARNING!

If the battery symbol is flashing on the battery level indicator, the scooter has detected a fault. If this happens, follow the procedures below:

1. Release the drive control lever, then switch the scooter off and on again. If this does not clear the fault, proceed below to step 2.
2. Switch off the scooter and fully charge the battery. If this does not clear the fault, go below to step 3.
3. Count the number of flashes. The battery symbol will flash several times before an extended pause. Count the number of flashes between breaks and then pass this on to your dealer. The following fault codes are listed below:

Number of flashes	Fault description	Meaning and procedure
1	Battery low	Batteries are discharged • Recharge the batteries
2	Low battery fault	Batteries are discharged. Recharge the batteries. Check the batteries and their associated wires and connections.
3	High battery fault	The battery voltage is too high. This could be the result of overcharging and/or moving down the slope. When moving down the slope, reduce speed to minimize regenerative charging.
4	Current Limit Time-out or Controller too hot	Motor exceeds its maximum rated current for too long. • The scooter may have stopped. Turn the power off, wait a few minutes, then turn the power back on. • The motor may be defective. Check the motor and the cables and their connections.
5	Failure of the parking brake	The parking brake release switch is active or the parking brake is faulty. • Check the parking brake cables and their connections. • Ensure any associated switches are in their correct positions.

6	Drive lock	<p>The stop function is active or a charger locked or OONAP conditions have occurred.</p> <ul style="list-style-type: none"> • Release stop condition (e.g. seat raised) • Disconnect the battery charger. • Ensure that the drive control lever is in the neutral position when starting. • The drive control lever may need to be calibrated.
7	Failure of the speed potentiometer	<p>The drive control lever, speed potentiometer, SRW or wires and their connections may have failed.</p> <ul style="list-style-type: none"> • Check the drive control lever, speed potentiometer and wires and their connections.
8	Motor voltage fault	<p>Motor or wiring and connections have failed.</p> <ul style="list-style-type: none"> • Check the motor and wiring and its connections.
9	other error	<p>The controller has internal failure.</p> <ul style="list-style-type: none"> • Check all connections and wiring.

ATTENTION!

If questions or concerns arise about the performance of the product, please contact your distributor.

Batteries and charging

Before charging the batteries, read the instructions and prepare the charger. The battery status indicator on the steering wheel console indicates the approximate power of the batteries by means of a colour code. From right to left, where green indicates fully charged, yellow indicates that some power has been used and red indicates an immediate need to charge the batteries. To determine this exactly, check the battery charge indicator while driving on dry, level ground at top speed.

First charge

The scooter requires two long, full charge cycles (14 hours), sealed and maintenance-free batteries.

Batteries are charged by the charging system outside the vehicle. Before first use, first charge the scooter batteries for 8 to 14 hours. Keep the batteries fully charged so that the scooter can run smoothly.

Connecting the charger

The scooter can be charged through the charging socket located on the battery pack or on the steering column. Connect the charger to the scooter or to the battery pack and then plug into an electrical outlet. When the batteries are fully charged, disconnect the charger from the scooter.

Illustration 12. Connecting the battery charger.

Operating temperatures

The scooter is designed for use, ideally in temperatures between -10°C ~ 50°C (-14°F to 122°F). In extremely cold conditions, the batteries may freeze, depending on charge, use and composition. In extremely high temperatures, the scooter may run at a lower speed. This is due to a built-in mechanism to protect against damage to the gearbox and other electronic components when operating in hot conditions.

Battery operation and charging

If you use your electric scooter on a daily basis, charge the batteries as soon as possible after you have finished using it for the day. The scooter will be ready every morning. When using the scooter on a daily basis, we recommend charging for 8 to 14 hours. We also recommend that you continue to charge for 4 hours after the charger has indicated the end of the cycle. If you use your scooter once a week or less frequently, charge the batteries at least once a week for 24 hours.

Battery charging process

The LED light on the charger indicates the different charging status.

Red light - means that, it is in the process of charging.

Green light - means that, bulk charging (95% charged) is complete and the charger continues to cycle charging.

ATTENTION!

Keep the batteries charged and avoid discharging them completely. To keep the batteries in good condition and to extend their life, we recommend charging a minimum for 48 hours, once a month.

ATTENTION!

There is a charger in the scooter which blocks its functions. The scooter will not start and the battery charge indicator will not be active when charging the battery pack fitted to the scooter.

WARNING!

Removal of the ground pins may cause an electrical hazard. If necessary, when accessing a 2-pin socket, use a suitable, approved 3-pin adapter.

WARNING!

Never use an extension cord to plug in the charger. Plug the charger directly into a properly installed standard electrical outlet.

WARNING!

It is prohibited, while charging the batteries, for children to play unsupervised near the scooter. We recommend that you do not charge the scooter batteries if the scooter is occupied by user.

WARNING!

Explosive gases may be generated when charging the batteries. Keep the scooter and charger away from sources that can cause ignition such as fire, sparks and ensure proper ventilation when charging the batteries.

WARNING!

Charge the scooter batteries with the supplied external charger. Do not use any other type of charger.

WARNING!

Before each use, check the charger, cables and connectors for damage. If you find any damage, contact your supplier.

WARNING!

Do not open the battery charger case. If the charger does not work correctly, contact your authorized supplier.

Warning!

Be aware that during charging, the battery charger case may be hot. Avoid contact with skin and other surfaces that may be damaged by heat.

**WARNING!**

If the charger has not been tested and approved for outdoor use then it should not be exposed to extreme weather conditions. If the battery charger is exposed to bad or extreme weather conditions, allow the charger to adjust to differences in environmental conditions before using it indoors.

Reporting technical defects and repairing the device

If a defect or damage to the product is discovered during use or maintenance, it must be reported immediately to the distributor or manufacturer.

**WARNING!**

Under no circumstances should an unqualified person repair or alter the product or its design.

The only person allowed to make any repairs or adjustments is an employee authorised by the manufacturer.

Care and maintenance

Your scooter requires minimal care and maintenance. If you are not sure of your care and maintenance capabilities listed below, you can have the inspection and service carried out by an authorized supplier. The components listed below require periodic inspection and maintenance.

Tires

Regularly check your electric scooter's tires for signs of damage or wear.

Wheel Replacement

The scooter is equipped with polyurethane (PU) tires. Replace the entire wheel if the tire has become worn or damaged. For information on replacing wheels, contact your authorized electric scooter supplier.

**WARNING!**

The wheels on the scooter must be serviced/replaced by a qualified technician. During this procedure, make sure that, the key is removed from the power switch and the scooter does not have the clutch lever switched disengaging the drive.

External surfaces

Bumpers, tires and rims can be occasionally cleaned with rubber and vinyl cleaner.

Cleaning

Use a damp cloth and a mild, non-abrasive cleaner to clean the plastic and metal parts of the electric scooter. Avoid using products that can scratch the scooter's surfaces.

Disinfection

If necessary, use approved disinfectants. Before using, make sure the disinfectant is safe for the scooter's surface. Only tested and approved disinfectants may be used for disinfection. Information on recognized and approved disinfectants and disinfection methods is available from your national health authority.

WARNING!

Follow the safety instructions provided by the manufacturer of cleaning and disinfecting products. Not complying may result in skin irritation or early damage to the upholstery or electric scooter.

Battery terminal connections

Make sure the connections remain tight and non-corroded.

Electric wiring harnesses

- Regularly check wire connections.
- Regularly check the insulation of the wires, including those from the charger, for wear or damage.
- Have an authorized supplier repair or replace any damaged connectors, connections or insulation before using your electric scooter again.

Daily checks

- With the power off, check the operation of the drive control lever. Make sure that the lever is not bent or damaged and that when released, it returns to the neutral position. Self-repair is prohibited. In case of problems, contact an authorized supplier.
- Visually inspect the steering column wiring. Make sure it is not burnt, cut or wires sticking out. Contact your authorized supplier if there are problems.
- Check for worn (flat) spots on the tires. Worn tires can affect stability.
- Check armrests for loose, strained or damaged parts. If there are problems, contact your authorized supplier.
- Check brakes. Checks should be made on a level surface without obstacles within 3 feet (1 meter) of the scooter.

Weekly checks

- Check the controller and charger connections for corrosion. If necessary, contact an authorized supplier.
- Check the pressure if there is a pneumatic tire on the equipment. If the tire does not hold pressure, contact an authorized supplier to replace the inner tube.

Monthly checks

- Check that the anti-tip wheels do not rub against the ground when operating the electric scooter.
- Check the anti-tip wheels for wear and tear. Replace if necessary.
- Check for tire wear. Contact an authorized supplier for repairs.
- Keep the scooter clean and away from foreign objects such as mud, sand, hair, food, drinks, etc.

Yearly checks

- Take your electric scooter to an authorized supplier for an yearly inspection, especially if you use your scooter daily. This will ensure the correct operation of the scooter and help avoid future breakdowns.

ABS plastic covers

If your electric scooter has a shiny chassis surface then it has been treated with a clear fixative coating. You can apply a small coat of car wax to help restore the glossy surface. If your scooter's chassis has a matte finish, use only products designed for matte finish surfaces. Do not use wax or other products designed for glossy paintwork.

Axle bearings and motor/transaxle

All these components are pre-lubricated, sealed and do not require further lubrication.

Console, Charger, and Rear Electronics

Keep these areas free of moisture. Allow these areas to dry thoroughly if they have been exposed to moisture before operating your electric scooter again.

Nylon Lock Nut Replacement

During periodic maintenance, assembly or disassembly, each securing nylon nut must be replaced with a new one. Nylon nuts must not be reused as this may damage the nylon threads and reduce the effectiveness of the fit. Replacement nylon nuts are available from local hardware stores or an authorized supplier.

Storage of the scooter

If you plan not to use the scooter for a period of time then follow these rules:

- Fully charge batteries before storage.
- Remove the battery pack from the electric scooter.
- Store the scooter in a warm, dry room.
- Avoid storing in a place where the scooter will be exposed to temperature changes.
- Recommended storage temperatures: from +5°C to +40°C (from 41°F to 104°F).
- Recommended ambient humidity for storage: 30% to 70%.

Batteries that are regularly completely discharged, not regularly charged, stored at inappropriate temperatures, or stored without being charged may be permanently damaged resulting in unreliable operation and shortened battery life. To ensure proper operation, it is recommended that batteries be charged regularly, throughout the period of prolonged storage of the scooter.

SUGGESTIONS!

For prolonged storage, you may wish to place several boards under the frame of your mobility scooter to raise it off of the ground. This takes the weight off the tires and reduces the possibility of flat spots developing on the areas of the tires contacting the ground.

Re-use (reusable product)

The product is suitable for reuse. The number of times it can be reused depends on how often and in which way the product is used. Before reuse, thoroughly clean and disinfect the product and check its safety features. If necessary, take repair action. If necessary, take repair action.

Product life cycle

The expected life of a medical device is 5 years, as long as it is used as intended and all maintenance and servicing is performed. The lifespan of a medical device also depends on the frequency of use, the environment in which it is used and its care. By using spare parts, the life of the product can be extended. Spare parts are generally still available for 5 years after the model is discontinued. This expected lifetime of a medical device does not imply an additional guarantee.

Replacement parts

The availability of replacement components and/or spare parts is dependent on the capabilities of the supplier, not the manufacturer. For more information on replacement components and/or spare parts, please contact your authorized supplier.

Disposal of an electric scooter

The electric scooter must be disposed of in accordance with applicable local and national regulations. For information on the proper disposal of packaging, metal frame parts, plastic parts, electronics, batteries, neoprene, silicone and polyurethane materials, please contact your local waste disposal company or authorized supplier.

Warranty

Details of the warranty can be found in the attached “warranty card” document.

The warranty does not cover components that may need to be replaced due to normal wear and tear during use or that have been damaged by misuse or accident, for which neither the manufacturer nor the supplier is responsible. This guarantee does not cover labour or claim calls.

ATTENTION!

The warranty does not cover deterioration of battery performance caused by being left in a fully discharged state, in cold conditions for an extended period of time, or by excessive use.

Technical parameters

Product name	CLEVER TRIP
Type	Scooter, rear wheel drive, Class A
Maximum user weight	150 kg
Maximum speed (driving forward)	10 km/h
Minimum braking distance at max. speed	2100 mm
Energy range	37 km
Maximum obstacle height	50 mm
Maximum safe inclination	9° (15,84%)
Minimum turning radius	1400 mm
Minimum width when reversing	1800 mm
Dimensions without seat (L x W)	1270 x 590 mm
Total mass with batteries	89,95 kg
Battery weight	24,3 kg
Maximum weight of single part (front frame)	26,21 kg
Effective seat depth	455 mm
Effective seat width	450 mm
Seat surface height at front edge	120 mm
Footplate height	155 mm
Seat surface angle	4°
Motor	320 W, electromagnetic brake
Batteries	45 Ah / 24 V Gel Batteries
Controller	DR90
Protection class	IPX5
Battery charging	Wyjście DC 24 V , 4 Amp
Protection class of battery charger	IPX1
Insulation class of battery charger	Class 2
Maximum motor sound level	68 dB
EMC compatible according to	ISO 7176-21
Ignitability of upholstery according to	EN 1021-2

Diameter rear wheels (number)	10" (3.00-4)
Diameter front wheels (number)	10" (3.00-4)
Suspension	front and rear
Control (wig-wag)	Lever to start/stop movement
Operating console	Rotatable switch for speed control
Storage and use temperature	from +5°C to +40°C
Humidity to store and use	30% - 70%
<p>We reserve the right to introduce technical changes. The tolerance in the measurements is ± 15 mm / 1.5 kg / 1.5°. The theoretical turning radius decreases if the scooter is often used on inclines, uneven surfaces or for driving up curbs. The maximum distance covered has been tested under ideal conditions in accordance with ISO7176-4.</p> <p>Speed and range depends on the user's weight, type of terrain, charge and battery condition. Information is current at the time of publication.</p>	

Product safety symbols

The symbols used below on the electric scooter are used to identify warnings, mandatory actions and prohibited actions. Be sure to read everything and understand their meaning.

It does not comply with the requirements of the ISO 7176-19 standard for transporting people in a wheelchair located in a motor vehicle. It is prohibited to sit in a scooter while traveling in a motor vehicle.

Before use, charge the battery to the maximum level. Remove the key from a scooter left unattended.

Symbols

	Medical device
	Date of manufacture
	Manufacturer
	Conformité Européene (compliance with the European Community)
	Catalog number (to re-order)
	Serial number
	Lot code
	Unique Device Identifier
	Follow the instructions for use

	Refer to the instructions for use
	Indicates the maximum weight capacity. Do not exceed the specified weight capacity of the product
	Warning
	Indicates the danger of electric shock
	EMI-RFI - This product has been tested and passed at an immunity level of 20 V/m
	Indicates the risk of explosion
	Corrosive chemicals contained in batteries
	Indicates a mandatory action that should be performed as specified in order to avoid a hazardous condition/situation. Failure to perform mandatory actions can lead to personal injury and/or equipment damage
	Protect from humidity
	Possibility of jamming fingers
	Pinch/Crush danger points
	Permitted temperature
	For indoor/outdoor use
	Contains Lead

	Maximum user weight
	Maximum safe inclination
	Indicates attachment points on the scooter
	The product is designed to be transported by car
	The product is not intended to be used as a seat for transport in a motor vehicle
	Do not stand on the footrest
	Use only AGM or Gel-Cell batteries to reduce the risk of leakage or explosive conditions
	Wear eye protection
	N = Neutral (Brakes disengaged; lever disengaging the drive turned on)
	D = Drive (Brakes engaged; unit in Drive Mode)
	Indicates a prohibited action that should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage
	Do not use batteries with different capacity (Ah). Do not mix old with new batteries. Always replace both batteries at the same time
	Do not mix AGM and Gel-Cell batteries. Always use batteries of the same type and chemistry
	Keep tools and other metal objects away from battery terminals. Contact with tools can cause electric shock

	It is forbidden for children to play, unsupervised, while charging the scooter's batteries
	Removal of the grounding pin may cause an electrical hazard. If necessary, properly install an approved 3-pin adapter in an electrical outlet with access to a 2-pin plug
	Do not connect an extension cord to the AC/DC converter or the battery charger
	Indicates flammable material. Do not expose to heat sources such as open flame or sparks. Do not transport batteries with flammable or combustible items
	Avoid exposure to rain, snow, ice, salt, or standing water whenever possible. Maintain and store in a clean and dry condition
	Do not remove anti-tip wheels
	Keep your hands away from the tires when driving. Be aware that loose fitting clothing can become caught in the drive tires
	Do not operate the drive disengagement clutch on an incline
	Do not operate the drive disengagement clutch on a decline
	Contact your local recycling center or authorized provider for information on proper recycling of product components

SELECTIVE COLLECTION OF ELECTRICAL AND ELECTRONIC EQUIPMENT.

This symbol on the products or accompanying documents indicates that used electrical or electronic products and batteries must not be disposed of in municipal waste. For proper disposal, restoration or recycling, such products should be returned to collection points for this type of waste. Alternatively, in some countries of the European Union or other European countries, you can return your products to a local seller at the time of purchase of a similar new product. By correctly disposing of this product, you will help preserve valuable natural resources and support the prevention of potential negative impacts on the environment and human health, which could result from incorrect waste disposal. More information is available from the communal offices or waste collection sites. In case of incorrect disposal of this product, penalties may be imposed in accordance with local regulations. If you want to dispose of an electrical or electronic device, get the information you need from your dealer or supplier.

Disposal in countries outside the European Union: This symbol is valid in the European Union. If you want to dispose of this device, please obtain the necessary information on the correct disposal at your local authorities or from your dealer.

Contact details

Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

www.mdh.pl

www.viteacare.com

We reserve the right to make technical changes due to technical progress.