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## Dane kontaktowe

NINGBO BAICHEN MEDICAL DEVICES CO., LTD.  
Adres: Room 2101, Diqiu Building, 666 Taikang  
Middle Road, Ningbo, Zhejiang, CN

SUNGO Europe B. V.  
Adres: Fascinatio Boulevard 522, Unit 1.7,  
2909VA Capelle aan den IJssel, The Netherlands

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**English version** on next page

**Users with visual impairments can find the PDF files together with further information on our website:**

**< [www.viteacare.com](http://www.viteacare.com) >.**

** Contact your specialist dealer when required.**

**Alternatively users with visual impairments can have the documentation read out by a helper.**

# Table of contents

<b>Introduction .....</b>	<b>35</b>
Indications .....	36
Contraindications .....	36
<b>Safety of use .....</b>	<b>36</b>
Information on electromagnetic compatibility (EMC) .....	39
Safety control .....	39
<b>Product description .....</b>	<b>40</b>
Medical device characteristics .....	40
Wheelchair description .....	40
<b>Technical specifications .....</b>	<b>42</b>
<b>Use of the wheelchair .....</b>	<b>43</b>
Preparing the wheelchair .....	43
Before use .....	43
Unfolding the wheelchair .....	44
Folding the wheelchair .....	44
Assembly of elements .....	44
Drive disengagement clutch .....	46
Control Panel operation .....	47
Joystick - control handle .....	47
<b>OPTION - Remote control .....</b>	<b>48</b>
Remote control pairing .....	48
<b>Safety .....</b>	<b>50</b>
Safety belt .....	50
Anti-tip wheels .....	51
Transporting the wheelchair .....	51
<b>Use .....</b>	<b>51</b>
Getting in the wheelchair .....	51
Wheelchair driving training .....	51
Driving on uneven terrain .....	52
Getting out of the wheelchair .....	52
<b>Batteries .....</b>	<b>52</b>
Charging process .....	53
Batteries - Storage .....	54
<b>Reporting technical defects and repairing the device .....</b>	<b>54</b>
Modification .....	54
<b>Care and maintenance .....</b>	<b>54</b>
Cleaning .....	54
Disinfection .....	55
Disposal of the electric wheelchair .....	55
Maintenance plan .....	56
<b>Symbols .....</b>	<b>58</b>
<b>Contact Information .....</b>	<b>60</b>

## Introduction

BC-EA8000 is a wheelchair created in accordance with the latest technologies. Its design allows for maximum adaptation to the user's needs. It combines a robust and durable construction, modern electronic control solutions and safety systems. In order to maintain safety and maintain the full efficiency of the product as long as possible, it should be regularly inspected at the service point (at least once a year). In case of any doubts or damage, please contact the distributor. Make sure that you have filled the attached warranty form and that you have submitted it to the distributor, as this will enable us to ensure the best possible service.

# ATTENTION

Always observe the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Please keep these instructions for future use.

### WARNING!

Improper use can lead to death, serious injury, dangerous conditions or product malfunction.

### ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

### SUGGESTIONS!

Follow the instructions below to keep the product in good working order.

If the device is used by another person, a copy of these instructions for use must be provided.

These instructions for use apply to the design and specification of the product at the time of publication. Due to design changes, some illustrations and photos in the manual may not correspond to the product purchased. We reserve the right to make design changes.

The manufacturer disclaims any liability for personal injury or property damage that may result from improper or unsafe use of this product.

## Indications

An electric wheelchair is a device that improves the quality of life, designed to provide mobility.

## Contraindications

Contraindications to the sitting position.

### Suggestion!

To maintain the life of the battery, during transport, they will be disconnected. Remember to connect the red cable with the input marked with the "+" symbol before using the wheelchair for the first time.

### Suggestion!

Avoid completely discharging the battery. Charge the battery after each use of the wheelchair whatever the distance travelled or the time of use. The battery should be charged at least once every 2 weeks.

## Safety of use

### WARNING!

Every "serious incident" that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State where the user resides.

### ATTENTION!

The product is intended for disabled people with musculoskeletal conditions to increase their mobility. If you are not sure whether your health condition allows you to use the device, consult a health care professional.

**WARNING!**

Do not modify the product in any way; this will void the warranty.

**WARNING!**

Exceeding the maximum user weight will void the warranty. The manufacturer is not responsible for injury and damage if the maximum user weight is exceeded.

**WARNING!**

Please read these instructions for use before using the wheelchair and keep them for future reference.

- a. Read and observe the information contained in the instructions for use. It is forbidden to use this product and the available optional equipment without first carefully reading and understanding these instructions. If the warnings, cautions or instructions are incomprehensible, contact the supplier, otherwise personal injury or an accident may occur. Never attempt to use the wheelchair beyond its limitations described in this instructions for use.
- b. Never turn off the control panel while driving, as this may cause the wheelchair to come to a sudden stop, which may result in injury.
- c. Consult a doctor if the medications taken do not affect the ability to drive motor vehicles.
- d. To operate the electric wheelchair safely, you must drive on flat surfaces where all four wheels are in contact with the ground.
- e. If the battery level indicator drops to the red level, charge the wheelchair as soon as possible. Completely discharging the batteries can damage them or shorten their lifetime.
- f. Keep metal objects away from battery connections. Danger of electric shock.
- g. When driving on curves, always slow down and maintain a stable centre of gravity. When taking sharp turns, always reduce the speed.
- h. Take extra caution when reversing. For safety, the reverse speed should be limited to the minimum speed.
- i. Children are not allowed to play near the wheelchair or operate it.
- j. Some parts of the electric wheelchair are sensitive to temperature changes.
- k. Remember that the temperature of the components and frame parts may increase when exposed to the sun or another heat source, and at low temperatures, there is a risk that the metallic parts of the wheelchair will be very cold. This may cause burns or frostbite.
- l. Store the wheelchair in clean and dry conditions.
- m. It is forbidden to drive at night without proper lighting.
- n. Do not remove anti-tip wheels or drive wheels.
- o. Keep hands and feet away from moving parts when driving. Ensure that loose clothing does not become trapped in the drive wheels.

- p. Do not connect the battery charger to the extension cord.
- q. Disassembly of the controller or engine by persons other than the authorized service is prohibited and invalidates all applicable warranty conditions.
- r. It is forbidden to move the wheelchair on motorways. When moving on public roads, follow all current traffic rules for the country.
- s. It is forbidden to use the electric wheelchair under the influence of alcohol, drugs or other substances affecting the ability to drive the vehicle.
- t. Take caution when driving in busy areas and shopping centres.
- u. Never use the wheelchair as a chair for transport in motor vehicles.
- v. Do not try to lift the wheelchair by holding onto parts such as the seat, armrest or parts of the housing. Lift by grasping the frame elements.
- w. When driving, it is forbidden to use a mobile phone or other personal communication devices (e.g. CB-radio).
- x. Do not use the wheelchair outside during rain or snowfall, high (above 30° C) and low (below 0° C) temperatures.
- y. Do not drive the wheelchair onto unprotected wharves, river banks or lake shores.
- z. The wheelchair is not suitable for use in difficult terrain (unpaved roads).
- aa. Do not stop, turn or reverse suddenly to avoid falls and injuries.
- ab. When driving, keep your forearms on the armrests and feet on the footrest platform.
- ac. It is forbidden to stand while driving the wheelchair. Do not lean to the side or lift objects from the ground.
- ad. Do not drive onto raised objects (steps, kerbs) higher than 50 mm.
- ae. The wheelchair is not suitable for moving up or down the stairs. Always use ramps or elevators.
- af. The wheelchair is not designed to negotiate ground gaps greater than 100 mm in size.

**WARNING!**

Drive the electric wheelchair in a straight line up or down the slope, never reverse. Irregular forward and backward movements with slope changes can increase the risk of the wheelchair tipping over.

The maximum user weight is 150 kg

## Information on electromagnetic compatibility (EMC)

Despite complying with applicable EMC directives and standards, an electrically powered wheelchair may interfere with the operation of other electrical devices, such as mobile phones or alarm systems. In addition, this may affect the performance of the wheelchair itself. If you notice this behavior, it is a recommended idea to switch off the third-party device or power wheelchair and move it out of interference range. Remember that other devices emitting electromagnetic fields can also affect the operation of the wheelchair.

## Safety control

### ATTENTION!

The following points should be checked before use:

- Electrical connections and wiring harnesses (visible damage must be reported to the distributor).
- Batteries – Fully charged and not corroded.
- Brakes - check that the handbrake is working.
- Folding lock mechanism - make sure wheelchair is fully unfolded.
- Condition of the tyres – make sure that they are not damaged and excessively worn - if there is a problem, contact the distributor.

## Product description

### Medical device characteristics

The electric wheelchair is a lightweight, robust vehicle with motors powered by lithium or gel batteries, enabling people with limited mobility to move independently. Equipped with a joystick for control, adjustable seat and footrests(or footrest platform), it provides comfort and safety. The range on a single charge is approximately 25 km and the speed is up to 6 km/h. The design allows for easy folding and transport, and built-in safety systems guarantee stability while riding. It is an independent and convenient solution to support everyday life.

### Wheelchair description

Illustration 1. Front of the wheelchair.

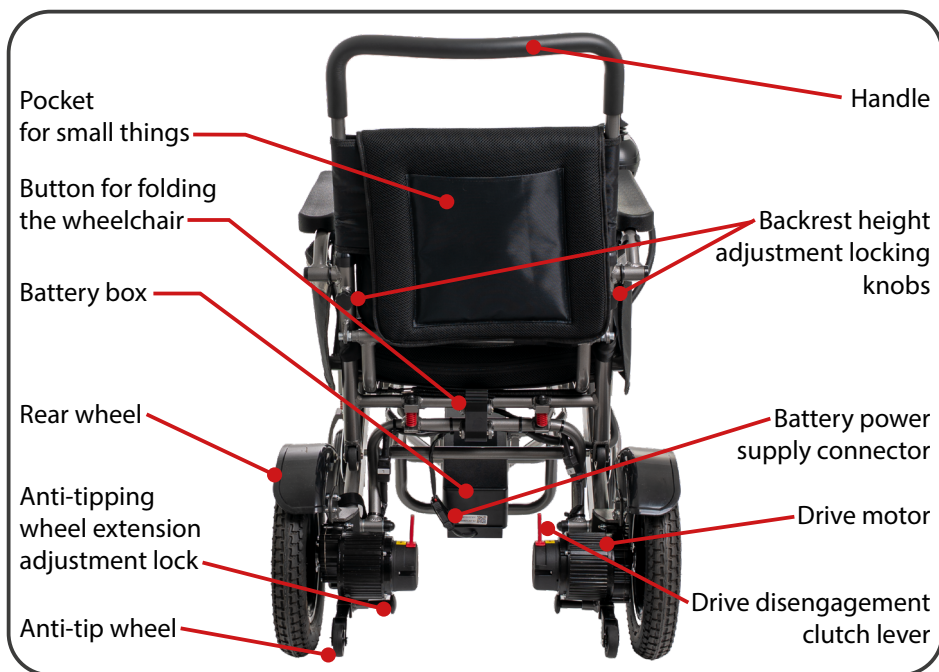
Illustration 2. The back of the wheelchair.

Illustration 3. Control panel (joystick)

## Technical specifications

Model	BC-EA8000
Seat width	420 mm
Maximum user weight	150 kg
Maximum load capacity	
Motors	2 x 300W
Battery	24V/20Ah (Lithium)
Charger (output power)	24V 3A
Maximum speed forward	6 km/h
Maximum speed reverse	3,5 km/h
Energy range	22-25 km $\pm$ 10%
Seat surface height from the floor	510 mm
Seat Depth	420 mm
Backrest height	510 mm
Total length of the wheelchair with/without footrest	1030/940 mm
Total wheelchair height	950 mm
Total width of the wheelchair	630 mm
Dimensions after folding (length x height x width)	630x340x840 mm
Armrest height from the ground	720 mm
Seat cushion thickness	60 mm
Front wheels	8" Solid(4,5x19)
Rear wheels	12" Pneumatic(5x32)
Turning radius	900 mm
Footrests	Folding platform
Weight of wheelchair with/without batteries	30,7 kg / 25,7 kg
Safety	Adjustable safety belt, anti-tip wheels
Maximum safe inclination	13° (~23,1%)
Maximum obstacle to overcome safely	50 mm
Other functions	Frame folds into a so-called "sandwich", reclining armrests.
Minimum braking distance at maximum speed	600 mm
Frame material	Aluminum
Description of the environment in which the product will be used	  
Storage and usage temperature	From +5°C to +40°C
Storage and usage humidity	From 30% to 80%

Speed, max distance of the vehicle depends on the user weight, battery load, battery capacity and the terrain on which the vehicle is used. The information is current at the time of publication, the manufacturer reserves the right to make changes to the specifications.

# Use of the wheelchair

## Preparing the wheelchair

The wheelchair is delivered in a cardboard package, and the fragile parts are additionally protected against damage during transport. Unpack it carefully to avoid any damage.

### Before use

- Check all electrical connections, make sure that they are stable and not corroded.
- Check that the batteries are fully charged.
- Check that the wheelchair is engaged in drive mode (clutch levers engaging the drive in the upper position).
- Always plan your route in such a way as to avoid dangerous (marshy, difficult areas) and obstacles that you will not be able to overcome with a wheelchair.
- Whether the speed is adapted to the user's capabilities and skills.
- Does the horn work.
- Fasten seat belt.

#### ATTENTION!

- When transferring to the wheelchair, do not rest your body weight on the armrests as this can lead to the wheelchair tipping over and causing injury.
- Do not stand on the footrests because this could tip the wheelchair over.
- Remember to check whether the wheelchair power is switched off because unintentionally touching the joystick can cause the wheelchair to move, fall and, consequently cause injury.
- During the transfer, try to be as close to the wheelchair as possible to avoid slipping and falling.

#### WARNING!

The following actions can influence the steering and stability of the wheelchair negatively:

- Holding and walking an animal strapped on a leash.
- Carrying passengers (including pets).
- Hanging the item on the controller.
- Towing or pushing by another vehicle.

## Unfolding the wheelchair

- First of all, sufficient space should be prepared to move safely around the wheelchair.

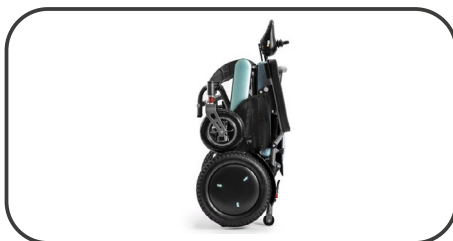
Illustration 4. Folded wheelchair.

- Then, gripping the handle and the front part of the frame under the seat, tilt the frame of the pushchair.
- Check that the lock at the back, at the bottom of the backrest, has locked the folding of the wheelchair - close the lock if necessary.

Illustration 5. Unfolding the frame.

## Folding the wheelchair

- To fold the wheelchair, press the locking button on the back of the wheelchair below the backrest.
- At the same time, while pressing the button, push the handle towards the front wheels of the pushchair.

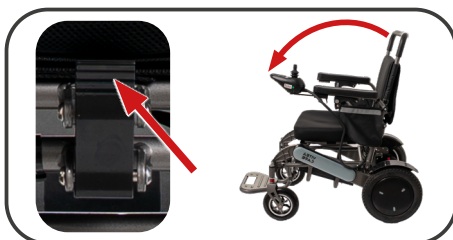
Illustration 7. Folding the frame.

## Assembly of elements

Assembly of the main parts should be carried out in accordance with the instructions given below:

- Install the controller on the user's right armrest.

Illustration 6. Controller installation.

- Then connect the battery plug to the wheelchair socket as shown in the illustration opposite.

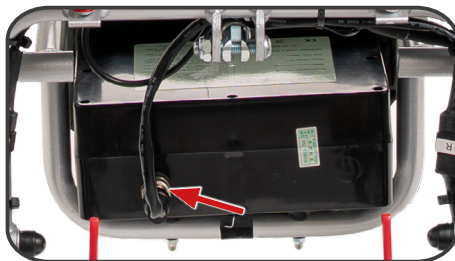
Illustration 8. Connecting the battery power connector.

It is possible to charge the battery outside the wheelchair:

- To remove the battery pack, press the locking button at the bottom of the battery pack.
- To install the battery pack, slide it into the guides at the bottom of the frame until the lock clicks audibly.

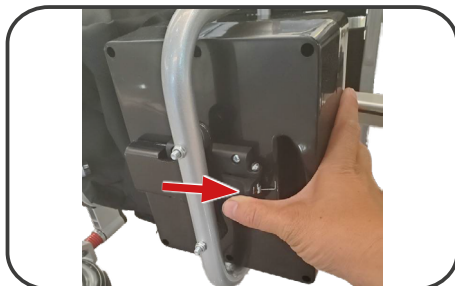
Illustration 9. Battery installation and removal.

### Tilting armrests

- To tilt the armrest, press the grey locking button shown in the illustration opposite.

Illustration 10. Tilting armrest.

### WARNING!

Take special care when tilting the armrests so that no parts of your wardrobe or body are in danger of being crushed. Take special care.

## Drive disengagement clutch

There are two clutch levers in the wheelchair to enable or disengage the electric drive and manually push the wheelchair.

- To disconnect the drive for manual pushing, lower the lever as far down as possible - position **B** in illustration 21.
- Restarting the drive (travel mode) should be done by lifting the lever up - position **A**.

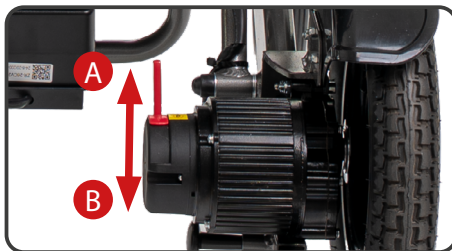
Illustration 11. Disconnect the motor drive.

## Control Panel operation

The electronic control panel is an element that allows you to steer the power wheelchair. It powers all operating systems. It allows control and steering of the wheelchair's movements, monitors the battery charge level and the status of the entire electrical system. Control panel is located on one of the armrests and is connected to the battery and motor.

- To start the electrical system of the wheelchair, press the power button (Illustration 3.).
- To change the speed of the wheelchair, press the left button to reduce the speed or the right button to increase the maximum speed (Illustration 3.). A single LED lighting up indicates the lowest speed. The highest speed is indicated by the lighting of all LEDs.
- In dangerous situations, an warning signal can be used by pressing the horn button (Illustration 3.).

## Joystick - control handle

Thanks to the joystick, you can control the direction and speed at which the wheelchair moves. When you move the joystick from the neutral position (middle position), the electromagnetic brake of the wheelchair will be released allowing it to move. The greater the swing from the neutral position, the faster the wheelchair will move. When the joystick returns to the neutral position, the brake system is activated, which slows down until the wheelchair comes to a complete stop. Moving the joystick backwards from the neutral position causes the wheelchair to move backwards. Swinging the joystick to the side causes the wheels to turn left or right.

### WARNING!

The wheelchair is equipped with electromagnetic brakes. When moving, be careful because the immediate return of the joystick to the neutral position stops the wheelchair at the shortest possible distance.

Installation and adjustment of the control panel (joystick):

- Unscrew the screw locking the control panel handle (Illustration 12.).
- Set the panel in the correct position.
- Then lock by tightening the screw.

Changing side of the control panel:

- Contact the distributor.

Illustration 12. Control panel adjustment screw.

## OPTION - Remote control

① Control handle (joystick). Controls the movement of the wheelchair by swinging it in a given direction.

② Switching on / off the power and activity of the remote control. Pressing and holding the button for 5 second turns on the power of the remote control. Short press the button again turns off the power.

③ Increase speed button. Press to increase the speed of the wheelchair.

④ Decrease speed button. Press to reduce the speed of the wheelchair.

⑤ Horn button. Press the button, in dangerous situations, to activate the audible warning.

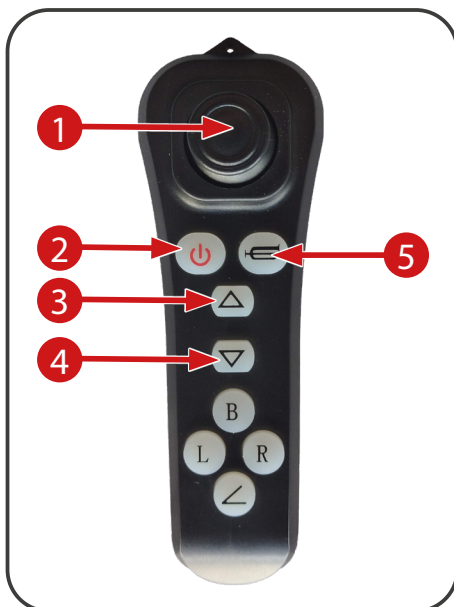
Illustration 13. Remote control description.

## Remote control pairing

1. To pair the remote control with the wheelchair controller, first turn on the power to the controller in the wheelchair.

Illustration 14. Remote control pairing.

2. then power up the remote control.

Illustration 15. Remote control pairing.

3. Then press the speed reduction button on the controller until one centre LED on the speed indicator lights up.

Illustration 16. Remote control pairing.

4. Then activate the pairing process by pressing the sound signal button (horn).

Illustration 17. Remote control pairing.

5. The final step is to press and hold the down arrow on the remote control. Pairing on the remote control will be signalled by the continuous illumination of the power switch LED on the remote control.

The pairing process will be completed when the power is turned off at the wheelchair controller (Joystick).

Illustration 18. Remote control pairing.

### Use of the remote control

#### Preparation before connection:

- Make sure that the wireless remote control is turned on (the signal lamp lights up when the device is turned on).
- Turn off all other controllers and turn on only the one to be connected. It is necessary to prevent interference of "Bluetooth" signals that can affect the efficiency of the connection.
- Turn on the power to the controller in the wheelchair to be connected and make sure that the wireless remote control is in the active state.

**WARNING!**

Do not switch off the electric drive if there is no person to guide the wheelchair.

Do not switch off the electric drive when on an incline as the electric wheelchair could roll away and cause injury.

**Remember that when the electric drive is switched off, the electromagnetic brake system is also switched off.**

Always stand behind the wheelchair when the electric drive is switched off. It is forbidden to do this while sitting in a wheelchair. After moving the wheelchair to the desired place, remember to switch the electric drive back on.

## Safety

### Safety belt

**WARNING!**

Remember to fasten seat belt before each ride.

Care should be taken to ensure that no objects are jammed under the belt. This way you will avoid painful pressure points.

Safety seat belt is not part of the fastening system of an electric wheelchair and/or its user when transported in a motor vehicle.

The hip seat belt is used to stabilize the sitting position and prevent the user from falling out of the electric wheelchair forward.

To install the hip seat belt, pull both ends forward and close the latch so that it locks audibly. To open the hip seat belt, it is necessary to press the buckle buttons and open the belt ends.

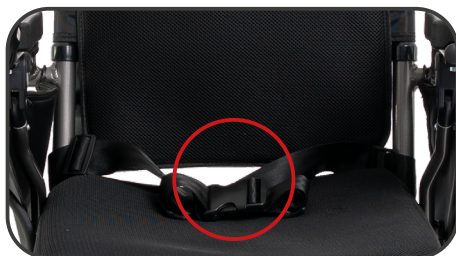
Illustration 19. Properly fastened safety seat belt.

**ATTENTION!**

The length of the hip belt can be adjusted, making sure not to tighten it too much.

## Anti-tip wheels

- To set the extension of the anti-tipping wheels, pull the lock **A**
- Then slide out the wheel mounting rail and adjust the holes **B** so that the lock can be reinserted.

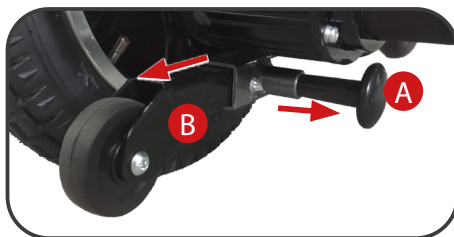
Illustration 20. Anti-tip wheels.

## Transporting the wheelchair

The wheelchair should only be transported by vehicles adapted for this purpose. The wheels should be blocked. It is forbidden to be in a wheelchair during transport, this may cause a fall and serious injuries. Always make sure that the wheelchair is properly fixed in the transport vehicle, otherwise it may cause injuries to people moving in this vehicle as well as serious damage to the transport vehicle itself.

When driving up the ramp into the transport vehicle, protection should be provided at the rear, to protect the wheelchair from tipping over.

All additional elements of the wheelchair during transport should be removed from it or attached to the wheelchair so that they do not cause damage to people or transport equipment during sudden braking.

## Use

### Getting in the wheelchair

#### WARNING!

Entering or exiting the electric wheelchair can only be performed with the wheelchair switched off and the drive disengagement clutch lever set in the driving mode (lever in the upper position).

Unintentional movement of the joystick (driving and steering lever) can lead to uncontrolled movement of the electric wheelchair! – Danger of accident!

## Wheelchair driving training

1. Handling the wheelchair and performing all manoeuvres must be well mastered in order to move safely. First, you need to master moving forward. Ensure that the minimum speed is set.
2. When you have mastered moving straight forward, try to drive by circling the shape of the letter S with a wheelchair.
3. Then you can start backwards training. Regardless of the programmed forward speed, the wheelchair will always move backwards more slowly.

## Driving on uneven terrain

Navigating rough terrain poses more risks. It is important to follow the safety rules. Failure to do so, may result in serious injuries.

- Do not climb hills above 13° of inclination. Note that uneven or wet surface will further reduce grip and weaken brake performance when ascending and descending a hill.
- Do not go backwards from the hills.
- Do not move on uneven, unpaved surfaces (sand, gravel, grass).
- When descending a hill, move at a minimum speed.
- Do not get off the wheelchair or sit on the wheelchair standing on an inclined surface (always place the wheelchair on a level, horizontal surface first).
- When overcoming hills, always approach in a straight line in accordance with the slope of the terrain.

## Getting out of the wheelchair

### WARNING!

Never get in or out of the wheelchair without first turning off the power. This prevents the wheelchair from moving by accidentally switching the joystick of the controller.

1. Stop the wheelchair completely.
2. Turn off the power supply using the control panel (Illustration 3.).
3. Tilt the armrest on the side that will allow the wheelchair to be abandoned safely.
4. Carefully and safely dismount from the seat and take a safe position next to it.
5. To make it easier to get in the wheelchair, you can leave the armrest tilted.

## Batteries

The wheelchair is equipped with long-lasting batteries. Batteries are secured and maintenance-free.

- Perform a full charge cycle of at least 10 hours before using the wheelchair for the first time.
- In order to maintain the best performance of the battery, 5 cycles of full charging should be performed.


### ATTENTION!

Always charge batteries until the green LED on the charger appears, which is a full cycle. If they are not charged enough, their life will be shortened.

## Charging process

1. Park the wheelchair as close to the grounded socket as possible.
2. Switch off the wheelchair.
3. Make sure that the clutch is in the right position – drive mode.
4. Connect the charger to the charging socket in the wheelchair.
5. Connect the charger to a grounded socket, remember to check that the socket has the correct voltage of 230V.
6. Switch on the charger.
7. The red LED light on the charger indicates an active power supply.
8. The yellow LED light on the charger indicates an active charging process.
9. The green LED light on the charger indicates that the charging process is complete.
10. The optimal charging time is 8-10 hours.
11. When charging is complete, disconnect the charger from the wall socket first, and then remove the plug from the charging socket located in wheelchair.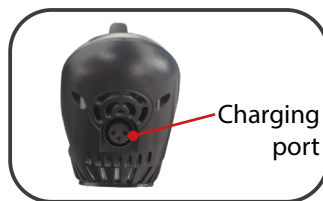

### WARNING!

Do not leave the charger connected to the charging port if the charger is not actively charging the wheelchair.

The charger is for indoor use only.

- Do not expose it to rain or water.
- Do not cover the charging terminal of the battery.
- Always keep the battery dry, never use it high full power and never charge it for more than 10 hours.

## Batteries - Storage

- If the wheelchair is to be unused for a long time, remove the batteries and store them in a dry and cool place.
- Avoid completely discharging the batteries.
- If you use the wheelchair every day, charge the batteries every day.
- If you use the wheelchair once a week, charge the batteries once a week.
- Store the batteries fully charged.
- If the wheelchair is not used regularly, charge it at least once every two weeks in a full charge cycle to maintain battery life.
- Never replace the batteries with batteries other than those supplied with the wheelchair.
- If the batteries are damaged or need to be replaced, contact the distributor.
- Do not dispose of batteries in the waste bin. Dispose of in accordance with local waste management regulations.
- To prevent short-circuiting and damaging the battery, never touch their terminals with metal objects.

## Reporting technical defects and repairing the device

If a defect or damage to the product is discovered during use or maintenance, it must be reported immediately to the distributor or manufacturer.

### WARNING!

Under no circumstances should an unqualified person repair or alter the product or its design.

The only person allowed to make any repairs or adjustments is an employee authorised by the manufacturer.

## Modification

Do not modify, remove, disable or add any parts, components or functions of the electric wheelchair that are not supplied by the manufacturer.

## Care and maintenance

### ATTENTION!

Insufficient or abandoned care and maintenance of the electric wheelchair leads to limitation of the manufacturer's liability.

## Cleaning

In order to ensure long-term and efficient operation of the wheelchair, it should be subjected to basic maintenance and technical condition inspections. Inspections should take place once a year. If you have any doubts or damage, contact the distributor, which has all the competences to solve the problem.

**WARNING!**

Observe the safety instructions given by the manufacturer of cleaning and disinfecting products. Failure to do so may result in skin irritation or premature damage to the upholstery of the electric wheelchair.

**It is not permitted to use:**

- solvents,
- toilet cleaners,
- agents with chlorine content,
- sharp brushes and hard objects,
- corrosive cleaners.

*Do not clean the wheelchair with pressure or steam units!*

**Disinfection**

If the product is used by many people (e.g. in a social care home), you should use commercially available disinfectants. Before proceeding with disinfection, clean the surface. Only biocidal products authorised in Poland by the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products or by another competent authority of a given EU Member State may be used for disinfection.

**WARNING!**

Disinfectants may damage the outer surface, reducing the long-term functionality of the components. Always observe the instructions concerning the use of the products specified by the manufacturer.

**Disposal of the electric wheelchair**

The electric wheelchair must be disposed of in accordance with current local and national regulations. For proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, contact your local waste disposal company or authorised distributor.

## Maintenance plan

When	Scope of control
<b>Before the ride</b>	<p>In general, check for impeccable performance. After turning off the power, check the joystick. Ensure that it is not bent or damaged and that it returns to the neutral position when released. Make sure that the rubber cover around the base of the joystick is not damaged. Check the cover visually. Check the controller cable visually. Ensure that it is not ragged, cut or exposed wires. Check that there are no flat places on solid tires that may have a negative impact on the stability of the wheelchair.</p> <p>In case of problems and questions, contact the distributor.</p>
<b>Weekly</b>	<p>Make sure that all parts of the control panel (joystick) are well attached to the handle.</p> <p>Check the tightening of the screws and tighten with moderate force if necessary.</p> <p>Check the correct tyre pressure. If the tire does not hold air, contact the distributor.</p> <p>Check the operation of the electromagnetic brake. This test must be carried out on a level surface with at least a meter of free space around the electric wheelchair. Try to push the wheelchair with the motor clutches engaged into driving mode. In case of doubt, always contact the distributor.</p>
<b>Monthly</b> Checking the brakes	<p>Turn on the controller and reduce the speed level of the electric wheelchair. After one second, check the battery level indications - make sure that the LED light remains on. Slowly push the joystick forward until you hear the electromagnetic brakes click - release the joystick immediately. The operation of each electromagnetic brake must be audible within a few seconds of the movement of the joystick. Repeat this test three times by pushing the joystick backwards, then left and then right.</p>

<p><b>Monthly</b> (depending on the length of the traveled sections)</p>	<p><b>Inspection of tire profile</b>  <b>Check drive tyres for wear:</b>  Rear wheels - minimum tread depth = 1 mm.  Replacement of tires should be commissioned to an authorized service.</p> <p><b>Front wheels:</b>  Check front, self-aligning wheels for wear. If necessary, contact your distributor.</p> <p><b>Anti-tipping wheels:</b>  Check that the anti-tipping wheels do not rub against the ground when using the electric wheelchair.  If necessary, adjust them.  Check that the anti-tipping wheels are not excessively worn. In case of doubt, always contact the distributor.  Keep the electric wheelchair clean and protected from foreign bodies such as mud, dirt, hair, food, beverages, etc.</p>
<p><b>Every 6 months</b> (depending on the frequency of use)</p>	<p><b>Check</b></p> <ul style="list-style-type: none"> <li>- Recommended inspection of the motor for carbon brush wear.</li> <li>- Cleanliness</li> <li>- Overall condition</li> </ul>
<p>Manufacturer's recommendations  <b>Every 12 months</b> (depending on the frequency of use)</p>	<p><b>Maintenance work - contact with distributor</b></p> <ul style="list-style-type: none"> <li>- Vehicle</li> <li>- Charger</li> </ul>

## Symbols

The symbols used below are used to identify warnings, mandatory actions and prohibited actions. You should read everything and understand their meaning.

	Medical device
	Date of manufacture
	Manufacturer
	Authorized representative in the European Union
	Importer
	Distributor
	Conformité Européene (compliance with the European Union)
	Catalogue number
	Serial number
	LOT number
	Unique Device Identifier
	Model number
	Consult instructions for use
	Follow the instructions for use

	Warning
	Protect from moisture
	Possibility of jamming fingers
	Permitted temperature
	For indoor/outdoor use
	Maximum user weight
	Total permissible mass
	Maximum safe inclination
	The product is designed to be transported by car
	The product is not intended to be used as a seat for transport in a motor vehicle

**SELECTIVE COLLECTION OF ELECTRICAL AND ELECTRONIC EQUIPMENT.** This symbol on the products or accompanying documents indicates that used electrical or electronic products and batteries must not be disposed of in municipal waste. For proper disposal, restoration or recycling, such products should be handed over in collective places for this type of waste. Alternatively, in some countries of the European Union or other European countries, you can return your products to a local seller at the time of purchase of a similar new product. By correctly disposing of this product, you will help preserve valuable natural resources and support the prevention of potential negative impacts on the environment and human health, which could result from incorrect waste disposal. More information is available from the communal offices or waste collection sites. In case of incorrect disposal of this product, penalties may be imposed in accordance with local regulations. If you want to dispose of this device, please obtain the necessary information on the correct disposal at your local authorities or from your dealer.

**Liquidation in countries outside the European Union:** This symbol is valid in the European Union. If you want to dispose of this device, please obtain the necessary information on the correct disposal at your local authorities or from your dealer.

## Contact Information

NINGBO BAICHEN MEDICAL DEVICES CO., LTD.  
Address: Room 2101, Diqu Building, 666 Taikang  
Middle Road, Ningbo, Zhejiang, CN

SUNGO Europe B. V.  
Address: Fascinatio Boulevard 522, Unit 1.7,  
2909VA Capelle aan den IJssel, The Netherlands

### Importer

mdh Sp. z o.o.  
Address: ul. Maratońska 104, 94-007 Łódź, Polska  
tel. +48 42 674 83 84, fax. +48 42 636 52 21  
[www.mdh.pl](http://www.mdh.pl)  
[www.viteacare.com](http://www.viteacare.com)

We reserve the right to make technical changes due to technical progress.