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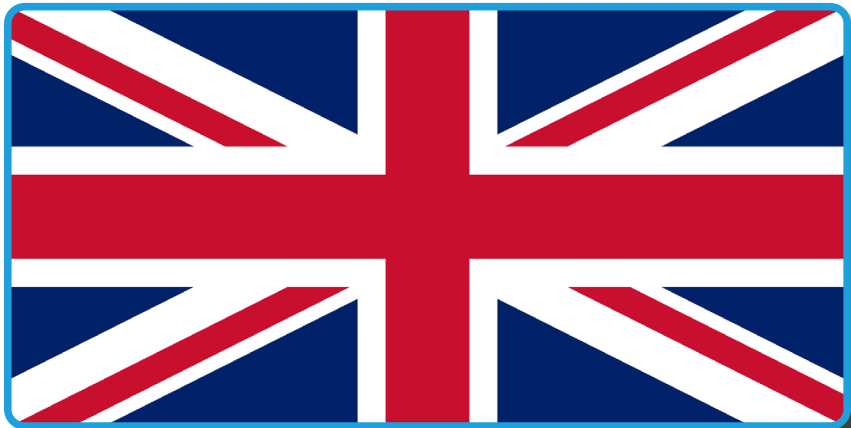
English version on next page

Users with visual impairments can find the PDF files together with further information on our website:

<www.viteacare.com>.

👉 Contact your specialist dealer when required.

Alternatively users with visual impairments can have the documentation read out by a helper.

Table of contents

Introduction from mdh Sp. z o.o.	39
Indications	40
Contraindications	40
Safety instructions	40
General safety rules	42
Modifications	43
Security check	43
Capacity	43
Transport the wheelchair	43
Information on electromagnetic interference - EMI	44
Practical information concerning the usage	45
Before usage	45
Safety belt	45
Driving outdoors and precautions in bad weather conditions	46
Physical limitations	46
Information on electromagnetic interference - EMI	47
Preparing to drive	48
Wheelchair operating training	48
Moving the wheelchair outside	48
Moving on rough terrain	49
Preparing the wheelchair	49
Before the first use of the wheelchair	49
Motor clutches	50
Manual handling of the wheelchair	51
Getting off the wheelchair	51
Wheelchair characteristics	51
Specifications	52
Description of wheelchair	53
Controller present in wheelchairs until August 2022	53
Controller appearing in wheelchairs from August 2022	55
Control panel - operation	56
Armrest	58
Getting off the wheelchair	58
Battery and charging	59
Batteries	59

Fuse.....	61
Error codes and troubleshooting	62
Care and maintenance	63
Disinfection.....	66
Disposal of electric wheelchair	66
Symbols	66
Contact details	68

Introduction from mdh Sp. z o.o.

Limber is a wheelchair manufactured according to the latest technology. Construction of this wheelchair enables to ensure its maximal adjustment to the requirements of the user. The wheelchair combines solid and durable construction, latest solutions as far as electronic control is concerned, and appropriate safety systems. In order to follow safety principles and to maintain the capability of being fully operational, remember to regularly control your wheelchair in the service center (once a year). In case of any doubts or any damages, please contact your service center, as this is the entity having all competences required to solve your problem. Make sure that you have filled the attached warranty form and that you have submitted it to the distributor, as this will enable us to ensure the best possible service.

ATTENTION

Always follow the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Keep this manual for future reference.

WARNING!

Improper use can lead to death, serious injury, unsafe conditions or product malfunction.

ATTENTION!

Improper use may lead to personal injury and/or damage to the wheelchair.

SUGGESTION!

Follow the instructions below to keep your wheelchair in good working condition.

If the electric wheelchair is used by another person, you must provide them with copies of these instructions for use.

These instructions for use relate to the design and specifications of the product at the time of publication. Due to design changes, some illustrations and photos in the manual may not correspond to the wheelchair you have purchased. We reserve the right to make design changes.

The manufacturer disclaims all liability for personal injury or property damage that may result from improper or unsafe use of this product.

SUGGESTION!

Batteries will be disconnected during transport in order to ensure their long lifetime. Remember to connect the red cord with input marked with „+“ symbol before using the wheelchair for the first time.

Indications

The product is intended for disabled people with musculoskeletal conditions to increase their mobility.

Contraindications

Contraindications to sitting position.

Safety instructions

- Read and follow the information in the instructions for use. It is forbidden to use this product and the available optional equipment without first carefully reading and understanding this manual. If you do not understand the warnings, cautions or instructions, contact your supplier, otherwise personal injury or accident may result. Never attempt to use the electric wheelchair beyond its limitations as described in this manual.
- Never deactivate the control panel when driving, as this will make the wheelchair stop suddenly, and this can cause an injury.
- Consult your doctor and ask if medicines you are taking don't pose an influence on your ability to operate mechanical vehicles.
- To operate an electric wheelchair safely, it should be ridden on flat surfaces where all four wheels have sufficient contact with the ground.
- If the battery level indicator drops to red, recharge the wheelchair as soon as possible. Completely discharging the batteries may damage them or shorten their life.

- Keep metal objects away from the battery connections. Risk of electric shock.
- When cornering, always reduce speed and keep the center of gravity stable. Always reduce speed when negotiating sharp bends.
- Ensure that there are no obstacles or objects behind the wheelchair when reversing. For safety, the reversing speed should be limited to the minimum speed.
- It is prohibited for children to play near or operate the electric wheelchair.
- Please note that the temperature of the components and frame parts may increase when exposed to the sun or other heat source, while at low temperatures there is a danger of very high cooling of the metal parts of the wheelchair. This can cause burns or frostbite on parts of the body.
- Store in clean and dry conditions.
- It is forbidden to drive at night without proper lighting.
- Do not remove the anti-tip wheels or transport wheels.
- Keep hands and feet away from moving parts when driving. Take care that loose clothing does not get caught in the drive wheels.
- Do not connect the battery charger to an extension cord.
- Disassembly of the controller, motor or charger by anyone other than an authorized service is prohibited and will invalidate any applicable warranty conditions.
- It is prohibited to drive the wheelchair on public roads and motorways.
- It is forbidden to use an electric wheelchair under the influence of alcohol, medication or any other substance affecting the ability to drive.
- Be careful when driving in busy areas and shopping centers.
- Under no circumstances should the wheelchair be used as a seat when transported by motor vehicles.
- Do not attempt to lift the wheelchair by gripping anything other than the frame (for example, the seat, armrest or chassis components).
- It is forbidden to use a cell phone or other personal communication devices (e.g. CB-radio) while driving.
- Do not use the wheelchair outside when it's raining or snowing, when the temperature is too high (it exceeds 30°C) or too low (below 0°C).
- Do not drive the wheelchair on quaysides, on river banks of lake shores, which have not been secured.
- The wheelchair is not adjusted to be used off-road (dirt roads).
- Avoid sudden stops, turns and making U-turns, as they can lead to fall and injury.
- When operating the wheelchair, keep your forearms on arm-rests, and keep your feet on foot-rests.
- It is forbidden to stand up when operating the wheelchair. Do not lean out to the sides, do not lift any objects from the ground.
- Do not approach any elevations (steps, curbs) with height exceeding 50 mm.
- The wheelchair is not adjusted to move on stairs. Always use wheelchair ramps or lifts.
- The wheelchair is not adjusted to overcome any cracks in the ground that are wider than 100 mm.

WARNING!

Any “serious incident” related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

General safety rules

Read the instructions for use carefully with understanding before using the electric wheelchair.

Users may encounter difficulties with maneuvering when passing through narrow doorways, ascending and descending ramps, driving around corners and navigating uneven terrain. Reduce speed, take your time and maneuver the electric wheelchair carefully.

WARNING!

Do not sit in the vehicle when is moving.

WARNING!

The product is intended for use by people with disabilities and movement disorders. If you are not sure whether your health condition allows you to use the product, consult a healthcare professional.

WARNING!

It is forbidden to use the wheelchair while under the influence of alcohol or psychotropic drugs. Never smoke while sitting in the wheelchair. Damaged or worn upholstery must be replaced as it increases the risk of fire.

ATTENTION!

When operating an electric wheelchair, please be aware of precautions, warnings and safety issues if you are taking medications prescribed by a doctor or available without a prescription.

Modifications

Do not modify, remove, disable or add any parts, components or features to your electric wheelchair. The manufacturer offers a wide range of electric wheelchair accessories to meet your needs.

WARNING!

Do not modify the electric wheelchair in any way; this will void the warranty.

Security check

ATTENTION - Check the following before use:

- Electrical connections and wiring harnesses - protect and remove corrosion.
- Batteries - fully charged and not corroded.
- Brakes - check that the electromagnetic brake is functioning.
- Upright mechanism - Ensure that the wheelchair is fully extended.
- Condition of the tires- make sure they are not damaged or excessively worn - if there is a problem, contact your dealer.

Capacity

Hero stand up electric wheelchair is designed and built to transport one person with a maximum weight of 120 kg. The wheelchair is not designed to carry goods or objects other than those previously described.

Warning!

Exceeding the maximum user weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

Transport the wheelchair

When transporting the electric wheelchair by vehicle, store it safely in the back of a van, truck or in the boot of a car. Moving parts should be removed or properly secured for transport.

WARNING!

Drive the electric wheelchair forward down or up the slope, never reverse. Irregular movements or making forward and backward movements may increase the probability of tipping over. Changes in slope can reduce stability. Under no circumstances should the electric wheelchair be moved up a slope greater than the maximum stability angle of this electric wheelchair.

Information on electromagnetic interference - EMI

Studies of electromagnetic interference (EMI) have shown that it adversely affects the operation and control of electrically powered mobile devices.

EMI can come from a variety of sources, such as cell phones, two-way radios, radio stations, TV stations, amateur radio (HAM) transmitters, wireless computer links, microwave signals, paging transmitters and transmitters used by emergency vehicles. EMI waves can cause unintended movement of the wheelchair or damage to the controller. Every electrically powered mobile device has a certain immunity to electromagnetic interference. The higher the immunity level, the greater the protection. The intensity of interference can be measured in volts per meter (V/m). Current technology allows for an immunity level of 20 V/m, which provides protection against the most common sources of electromagnetic interference. The wheelchair, without any modifications, has an immunity level of 20 V/m. As you get closer to the source, electromagnetic interference becomes more intense. Special attention should be paid to electromagnetic interference from hand-held radio devices. It is possible to unintentionally bring high levels of EMI close to the wheelchair control system, which can affect movement and braking. To prevent possible interference with the wheelchair control system, follow the warnings below.

WARNING!

Do not use hand-held radio transmitters such as a CB-radio or cell phone near a powered-up wheelchair.

WARNING!

Pay attention and stay away from nearby transmitters, such as radio or TV stations.

Warning!

Drive the electric wheelchair in a straight line up or down the slope, never reverse. Irregular forward and backward movements with changes in slope can increase the risk of the wheelchair rollover.

Practical information concerning the usage

Before usage

- Are the batteries fully charged? See section „Battery and charging“.
- Is the motor clutch lever in the drive mode (rear position). Never leave the lever in the sideways position if you do not intend to push the wheelchair manually.

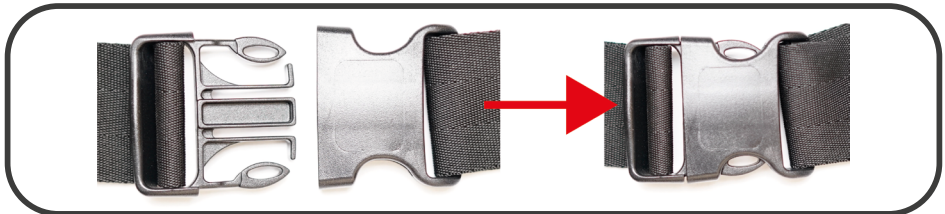
WARNING!
The following operations may adversely affect the steering and stability of the wheelchair:

- Keeping and walking an animal attached to a leash.
- Carrying passengers (including animals).
- Hanging an object on the controller.
- Towing or pushing by another vehicle.

Safety belt

Safety belts will protect the user when driving the wheelchair. Always remember to fasten the safety belts before starting the wheelchair.

Driving outdoors and precautions in bad weather conditions

Although the wheelchair is designed to be used indoors to help with daily living activities, the manufacturer has designed it to perform perfectly outdoors on level surfaces.

ATTENTION!

When driving outdoors, avoid uneven terrain, soft surfaces, tall grass, loose gravel, loose sand, fresh and salt water, stream banks, lakes and oceans. If you are unsure about a surface, avoid it.

Exposure to adverse weather conditions should always be avoided. If you find yourself in bad weather conditions while using your electric scooter, immediately go to the nearest shelter. Dry the electric scooter completely before use, before charging or before storing.

Physical limitations

Use awareness, caution, prudence and common sense when operating the wheelchair. Always be aware of your own limitations and rules of use when operating an electric wheelchair.

WARNING!

The product is intended for use by people with disabilities and movement disorders. If you are not sure whether your health condition allows you to use the product, consult a healthcare professional.

WARNING!

It is forbidden to use the wheelchair while under the influence of alcohol or psychotropic drugs. Never smoke while sitting in the wheelchair. Damaged or worn upholstery must be replaced as it increases the risk of fire.

ATTENTION!

When operating an electric wheelchair, please be aware of precautions, warnings and safety issues if you are taking medications prescribed by a doctor or available without a prescription.

WARNING!

Drive the electric wheelchair forward down or up the slope, never reverse. Irregular movements or making forward and backward movements may increase the probability of tipping over. Changes in slope can reduce stability. Under no circumstances should the electric wheelchair be moved up a slope greater than the maximum stability angle of this electric wheelchair.

Information on electromagnetic interference - EMI

Studies of electromagnetic interference (EMI) have shown that it adversely affects the operation and control of electrically powered mobile devices.

EMI can come from a variety of sources, such as cell phones, two-way radios, radio stations, TV stations, amateur radio (HAM) transmitters, wireless computer links, microwave signals, paging transmitters and transmitters used by emergency vehicles. EMI waves can cause unintended movement of the wheelchair or damage to the controller. Every electrically powered mobile device has a certain immunity to electromagnetic interference. The higher the immunity level, the greater the protection. The intensity of interference can be measured in volts per meter (V/m). Current technology allows for an immunity level of 20 V/m, which provides protection against the most common sources of electromagnetic interference. The wheelchair, without any modifications, has an immunity level of 20 V/m. As you get closer to the source, electromagnetic interference becomes more intense. Special attention should be paid to electromagnetic interference from hand-held radio devices. It is possible to unintentionally bring high levels of EMI close to the wheelchair control system, which can affect movement and braking. To prevent possible interference with the wheelchair control system, follow the warnings below.

WARNING!

Do not use hand-held radio transmitters such as a CB-radio or cell phone near a powered-up wheelchair.

Warning!

Drive the electric wheelchair in a straight line up or down the slope, never reverse. Irregular forward and backward movements with changes in slope can increase the risk of the wheelchair rollover.

WARNING!

Pay attention and stay away from nearby transmitters, such as radio or TV stations.

WARNING!

The following operations may adversely affect the steering and stability of the wheelchair:

- Keeping and walking an animal attached to a leash.
- Carrying passengers (including animals).
- Hanging an object on the controller.
- Towing or pushing by another vehicle.

Preparing to drive

1. Before you start using the wheelchair freely, you should find an open space where you can practice all the maneuvers and become well acquainted with its guidance system.
2. Before you get on, make sure the wheelchair is turned off and the speed is programmed according to your skills.
3. When using it for the first time, the minimum speed should be set to safely practice driving the wheelchair and familiarize yourself with its capabilities.

Wheelchair operating training

1. It is necessary to master well the use of the wheelchair and the execution of all maneuvers to move safely. First, master the forward movement. Make sure that the minimum speed is set.
2. Once you have mastered moving in a straight forward direction, you should try to drive by circling the wheelchair in an S shape.
3. After that, you can start practicing driving backwards. Regardless of the programmed forward speed, the wheelchair will always move slower backward.

Moving the wheelchair outside

- The wheelchair is not designed to move in the street traffic
- Do not use the wheelchair during rain or snow, when the temperature is higher than 30° C or lower than 0°C.
- Do not use the wheelchair on soft, uneven, unprotected surfaces such as grass, gravel, sand or decks, beside a river, lake.
- Do not use the wheelchairs for off-road driving.
- If possible, do not use the wheelchair outdoor at night.
- Make sure that there are no obstacles behind you when you drive backwards. We also recommend setting the speed at the slowest position.
- Do not stop suddenly, always try to make turns gently.
- Always keep your arms on the armrests and feet on the footrest while driving.
- Do not try to climb curbs higher than 7cm.
- Do not try to drive over a hole/gap wider than 10 cm.

Moving on rough terrain

Moving on an undulating terrain creates more risk. It is important to follow the safety principles stated below. Failure to comply with these principles may lead to serious injuries.

- It is not recommended to move the wheelchair on slopes with inclination exceeding 10°. It is essential to remember that wet surface will additionally reduce the grip of the wheelchair and will impair the functioning of brakes when ascending and descending a slope.
- When in vertical position, it is not allowed to ascend on a slope with inclination greater than 3°. The speed is automatically reduced, according to the inclination angle and the selected position. In extreme conditions, the system shall make it impossible to move any further.
- It is not recommended to move backwards when descending slopes.
- It is not recommended to move on uneven, dirt roads (sand, gravel, grass).
- When descending a slope, it is recommended to move with minimum speed.
- Do not get off the wheelchair and do not get on a standing wheelchair on an inclined surface (always place the wheelchair in an even, flat surface first).
- When conquering a slope, always ascend following a straight line, according to the inclination of the slope.

WARNING!

The wheelchair is not designed to transport a patient up stairs, as a stroller or by pulling a patient on it. The wheelchair's structure and connecting parts are not designed for such high overloads if it is used in this way. This risks weakening or damaging the entire support structure of the wheelchair and a possible accident!

Preparing the wheelchair

The wheelchair is delivered in a cardboard package, and its delicate parts are additionally secured against damage during transport. Unpack the wheelchair very carefully, to avoid any damage.

Before the first use of the wheelchair

- Check if batteries are fully charged.
- Check if the drive mode is turned on (clutch lever).
- Always plan your route ensuring that you will omit dangerous areas (boggy, difficult areas) and obstacles, which you won't be able to overcome in a wheelchair.
- Check that the seat is adjusted properly.
- Check if the speed is adjusted to your skills and capacity.
- Check if the horn is functioning properly.

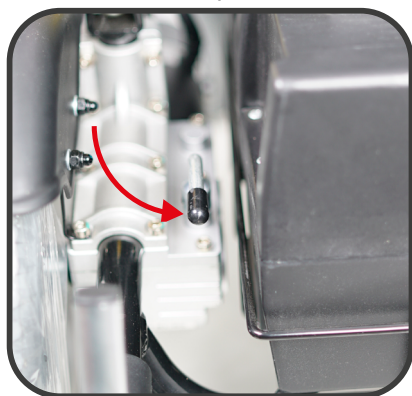
ATTENTION!

- When sitting on the wheelchair do not lean your body weight on arm-rests, as this may lead to personal injuries and the wheelchair may fall.
- Do not stand on foot-rests as the wheelchair may fall (except for the upright position).
- Remember to check whether the wheelchair is deactivated, since inexperienced touching the control shifter when changing seats may make the wheelchair move, and this can lead to fall and personal injury.
- During transfer try to stay as close to the wheelchair as possible, to avoid slip and fall.


Motor clutches

The wheelchair has two clutch levers that allow the electric drive to be disconnected and the wheelchair to be pushed manually.

- The clutch lever must be pulled for the electric drive to be engaged - the end of the lever faces the rear of the wheelchair.
- In order for the electric drive to be disengaged, push the clutch lever so that the end of the lever points to the side of the wheelchair.

Electric drive **connected**

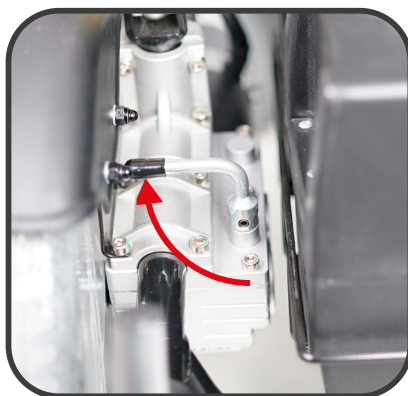Electric drive **disconnected**

WARNING!

Do not deactivate the electric drive, if there is no one who could push your wheelchair.

Do not deactivate the electric drive when the wheelchair is on an inclined surface, as it could roll down and this could lead to serious injuries.

Remember that when the electric drive is deactivated, the electromagnetic braking system is also deactivated.

Always stand behind the wheelchair when you are deactivating the electric drive. Do not deactivate the electric drive when sitting on the wheelchair. When you place your wheelchair in the desired location, remember to activate the electric drive again.

Manual handling of the wheelchair

1. Manual driving of the wheelchair is possible after turning off the electric drive by pushing the clutches to the side.
2. Driving is possible after the electric drive is switched on again by pulling back the clutches.

Getting off the wheelchair

WARNING!
Never attempt to get on or off the wheelchair without first removing the key from the power switch. This prevents the product, from being moved by accidentally switching the drive control lever.

1. Stop the wheelchair completely.
2. Turn off the power o control panel
3. Carefully and safely get off the seat and take place next to the wheelchair.

Wheelchair characteristics

Specifications


Model	W1018 LIMBER	
Length	1080 mm	
Height	1020 mm	
Width overall	600 mm	650 mm
Seat width	400 mm	450 mm
Seat height	540 mm	
Seat depth	450 mm	
Weight	42 kg (without batteries) / 64 kg (with batteries)	
Maximum speed	do 9 km/h	
Maximum safe inclination	5% (2,86°)	
Motor	2x320 W (24V)	
Energy range*	do 25 km	
Maximum user weight	120 kg	
Battery type	2 x 45 Ah	
Charger	24V/5A	
Tyres	Front: 9"(229 mm) / Rear: 14"(356 mm)	
Pressure in tyres	36 Psi (2,5 BAR)	
Brakes	Electromagnetic brakes	
Clearance	80 mm	
Suspension	2 independent dampers	

ATTENTION!

*The range of the wheelchair is given after testing under laboratory conditions (constant speed, no load, the wheelchair moves on rollers). The range of the wheelchair depends on the road surface and the conditions in which it will move.

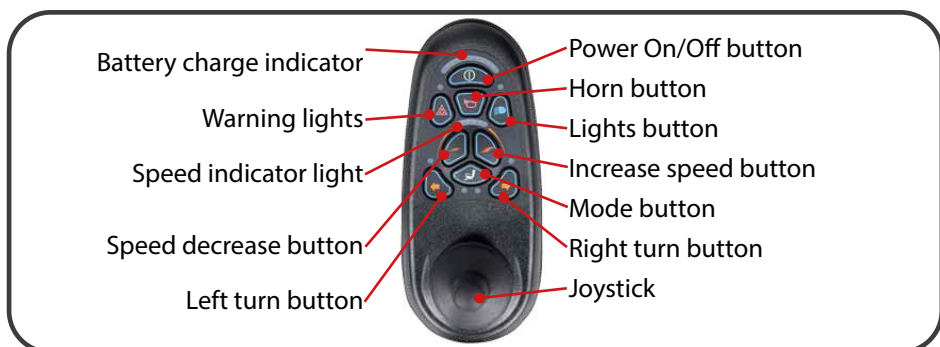
Description of wheelchair


Controller present in wheelchairs until August 2022.


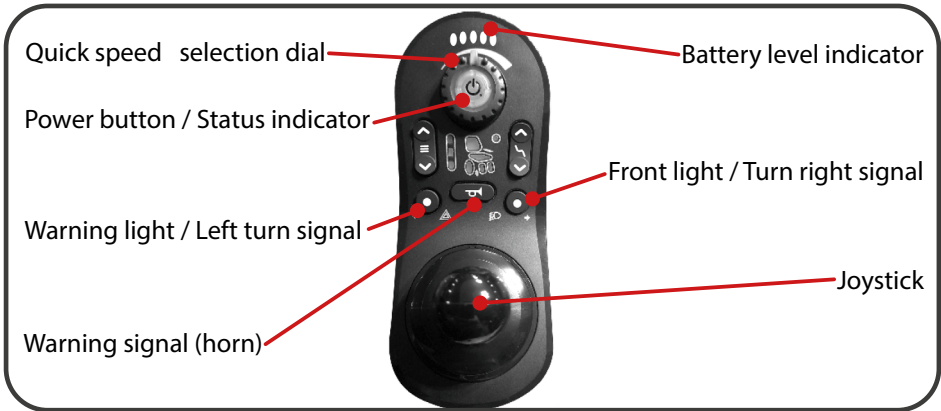
WARNING!
The user should be aware that the surface of the remote control may get heat up when it is exposed to sunlight for an extended period of time.

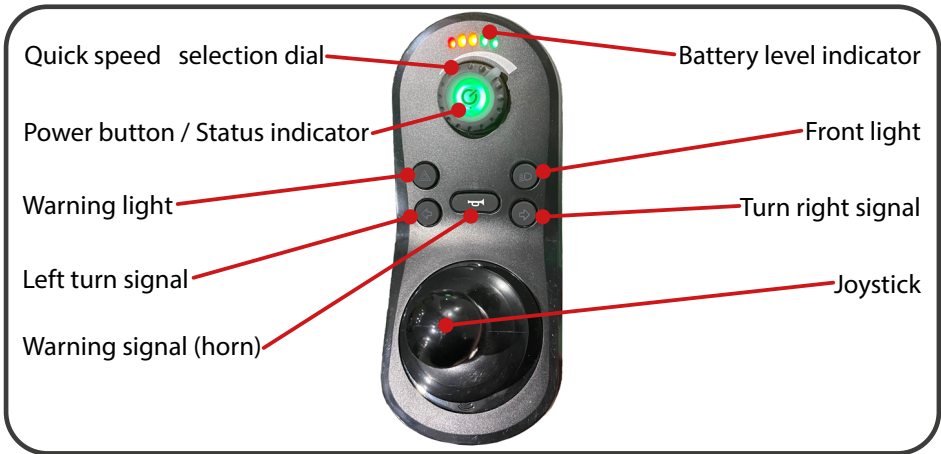
Controller present in wheelchairs from August 2022 to June 2024.

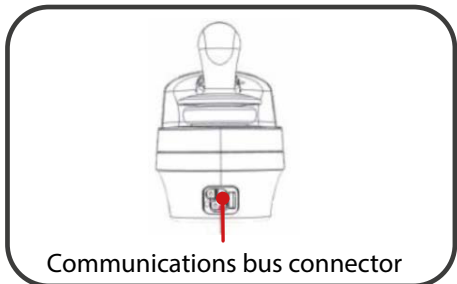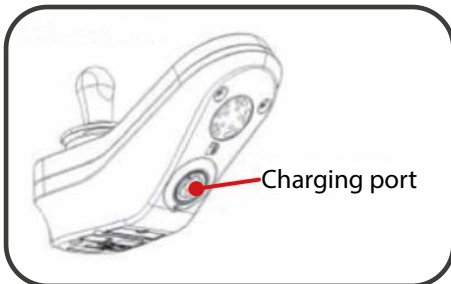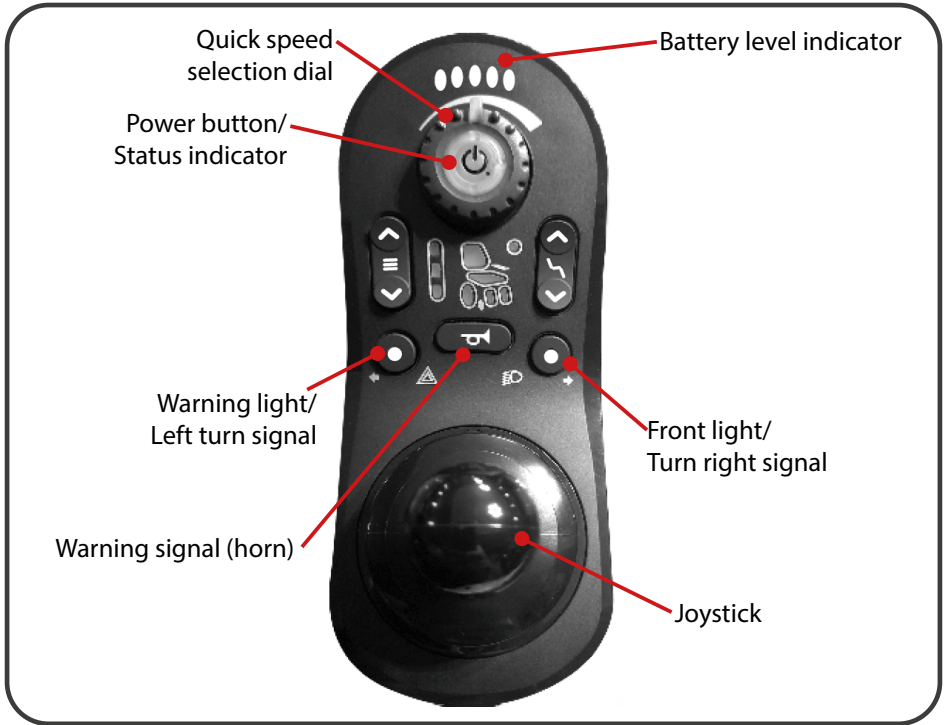
Controller present in wheelchairs from June 2024.

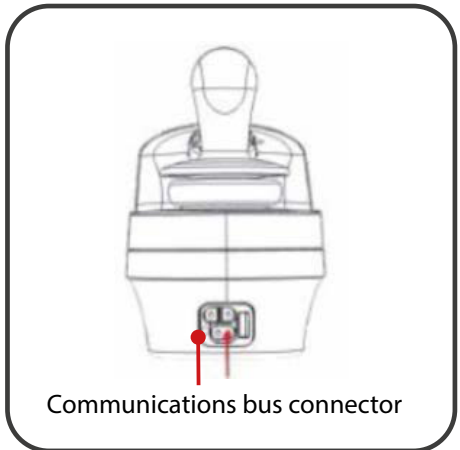
Controller appearing in wheelchairs from August 2022.

Control panel - operation

Adjustment of wheelchair function operation

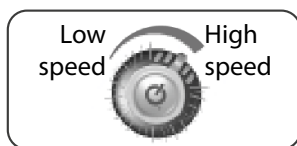Quick speed selection dial

To set the lowest speed, turn the dial as far to the left as possible.

To set the highest speed, turn the dial as far to the right as possible.

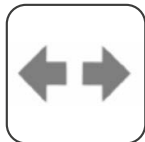
Indicator lamps and displayed information

Emergency indicator light:

To turn on the emergency light, press the button once. To turn off the emergency light, press the left or right light button once.

Turn signal lights:

To turn on left (right) signal lights, please press left (or right) lighting button for a while (3 seconds) and release. Left (or right) side LED will light up.

To turn off signal lights, please press left or right lighting button once.

Front lights:

To turn on the headlight, press the button on the controller once. To turn off the headlight, please press once, the left or right front light or turn signal button.

Battery level indicator

1. The battery level indicator is placed on the top of the control panel. This illuminated display indicates the status of power, battery and electrical system.
2. Constant green light: battery is charged, power and electrical system works properly.
3. Constant red and yellow lights: if possible, charge the battery, power and electrical system OK work properly.
4. Only constant red light or slowly blinking red light – you should charge the batteries as soon as possible, the electric systems and electronic system of control panel are efficient.
5. If the lights blink, the electronic system or electric system does not work properly.
6. If the warning lights blink continuously up and down, the joystick was not in a neutral position when the power wheelchair was switched off. Turn off the power, release the joystick and again turn on the power.

Joystick

With the joystick you can control the direction and speed at which the wheelchair moves. When you move the joystick from the neutral position (middle setting) the wheelchair's electromagnetic brake will be released allowing the wheelchair to move. The more you swing the handle from the neutral position, the faster the wheelchair will move. When the handle is returned to the neutral position, the brake system is activated, slowing down until the wheelchair comes to a complete stop.

Drive mode

The joystick allows you to control and steer the wheelchair's movements. The movement of the joystick forward causes the wheelchair to move forward. Moving the joystick backward causes the wheelchair to move backward, while moving the joystick sideways causes the wheelchair to turn accordingly. To stop the wheelchair release the control handle to the neutral (center) position.

Charger socket

Charger socket allows to charge the wheelchair by connecting it to power outlet of max 12 A.

Warning signal (horn)

Press the horn button to activate the horn.

Front lights

It is used to turn on the wheelchair lighting. When turned on, the LED on the control panel lights up to indicate the function.

Left turn button

It is used to signal a left turn. When you use this button, the indicator light on the control panel comes on. When you want to turn off the turn signal, press the button again.

Turn right signal

It is used to signal a right turn. When you use this button, the indicator light on the control panel illuminates. When you want to switch off the indicator, press the button again.

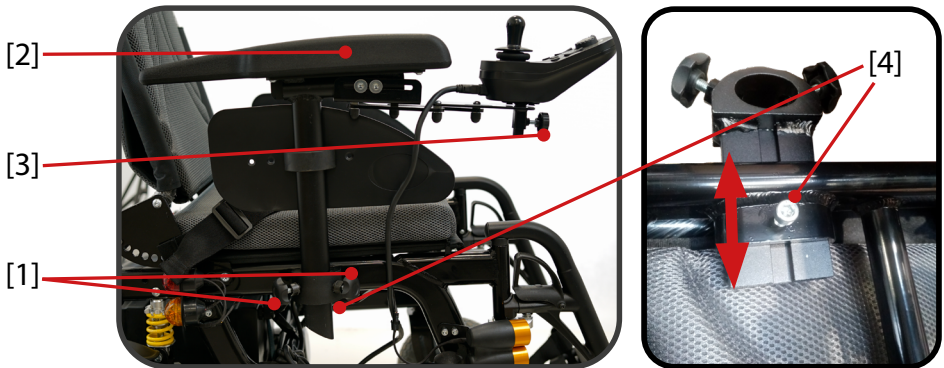
Warning light

Activates the warning lights.

Armrest

Adjusting armrest height:

1. Loosen the screws [1] attaching the armrest [2], indicated in the figure below.
2. Choose required armrest position.
3. Tighten the screws [1].
4. To change the width of the side panels, loosen screw [4], adjust the extension and then tighten the screw firmly.

Adjusting the arrangement of the control panel

To adjust the control panel setting, loosen the fixing screw [3] indicated in the figure above. Adjust the panel so that it is in the most comfortable position, and then tighten the screw [3]. The control panel can be mounted on any armrest (on the right or left side).

Getting off the wheelchair

WARNING!

Never attempt to get on or off the wheelchair without first removing the key from the power switch. This prevents the product, from being moved by accidentally switching the drive control lever.

1. Stop the wheelchair completely.
2. Turn off the power o control panel
3. Carefully and safely get off the seat and take place next to the wheelchair.

Battery and charging

Batteries

- Use a standard 3-pin grounded plug and a grounded contact.
- Do not use an extension cord or adapters to avoid the risk of fire, electric shock and injury.
- Do not use the same socket for other activities while charging the battery.
- Use only the charger that is factory designed for this wheelchair to avoid damage.
- Do not leave the charger in the socket if the batteries are not being charged.
- Keep the charger and batteries away from water and moisture, do not operate it with wet hands.
- Charge the wheelchair in a well-ventilated room.
- Do not cover the charger fan to allow heat dissipation.
- Leave the device at rest until the batteries are fully charged.
- Do not sit on the wheelchair while the batteries are charging.
- Do not stop charging the batteries until the charge indicator shows that the batteries are fully charged.
- Do not place the charger directly on the battery.
- Check that the parameters of the charger correspond to the parameters of the batteries and the mains voltage. Connecting a battery with different parameters can lead to battery damage and serious injury to the user.
- Especially during charging, avoid exposing the wheelchair and charger to high temperatures and smoke to avoid ignition and explosion.
- To avoid electric shock, it is forbidden to touch the charger or battery pack with wet hands or other objects, and to avoid exposure to moisture.
- Check that the charger switch is set to the "O" position before connecting the charger to the mains and the batteries connected to it.
- Do not expose the charger to rain of water and moisture.
- The charging process should take place at an ambient temperature of 0°C to 40°C, under other conditions charging may fail or lead to damage to the batteries.
- If the charging process indicator does not light up, disconnect the cables for 10 minutes and try to connect them again after this time. If this does not help, contact service.
- Contact service if the charge indicator remains yellow after charging for more than 12 hours.

Charging process

1. When charging the new batteries for the first time, the charging should last till all the green control indicators flash, no longer than 24h.
2. Park the wheelchair as close to the grounded socket as possible.
3. Turn off the wheelchair.
4. Make sure that the electric power supply of the wheelchair is turned on (the clutch is in proper position - drive mode).
5. Plug the charger into the charging socket placed on the control panel.
6. Connect the charger to grounded socket, remember to check whether the socket has proper voltage - 115V/230V.
7. Turn on the charger.
8. If the LEDs flash irregularly, turn off the charger. Check its connection and turn it on again.
9. In case when the charging procedure will proceed without complications, diodes marking the battery charge level will light up (as described above).
10. When charging is finished, disconnect the charger from the socket, and then take the plug out of the charging socket.

Plug the charger into the socket on the control panel.

Battery removal

In order to remove the batteries, pull out the battery box.

Batteries - storage

- In case when the wheelchair will remain unused for a longer period of time, take the batteries out and store them in a dry and cold place.
- Avoid complete failure of the battery.
- If you are using the wheelchair every day, charge batteries every day.
- If you are using the wheelchair once a week, charge batteries once a week.
- It is not allowed to partially charge batteries (for example connect batteries for a shorter time than a full charging cycle), as this may damage the battery or limit its durability.
- Only fully charged batteries can be stored.
- If you are not using your wheelchair on regular basis, in order to maintain durability of batteries, charge them at least once every two weeks, observing a full charging process.
- Always replace the batteries with the ones delivered with your wheelchair.
- In case when the battery is damaged or has to be changed, report to your distributor. Do not throw the battery into the recycle bin, instead, handle it to the distributor.
- In order to prevent short circuit and battery damage never touch battery contacts with metal objects.

Fuse

The batteries have a fuse that causes the power to be cut off in the event of a short circuit, damage to the electrical system or overheating of the engine. In the situations mentioned above, the fuse will jump cutting off the power supply. If the motor has overheated, wait until it returns to normal usable temperature, press the fuse and restart the wheelchair. If there is a problem restarting the wheelchair, contact your service center.

ATTENTION!

If the battery has been exposed to very low temperatures (frozen), do not charge it.

If the battery has been frozen, keep it at room temperature for a few days to warm it up and then charge it.

SUGGESTION!

If you are not sure when and how to replace the battery, contact your service center or distributor.

Error codes and troubleshooting

ATTENTION!

If there is an error with the system when it is powered up, then the status indicator will flash red; the number of flashes will indicate the type of error.

Status indicator

The table below describes the error indication, and a few possible actions that can be taken to rectify the problem. The actions listed are not in any particular order and are suggestions only; the intention is that one of the suggestions may help you clear the problem. If in doubt, consult your authorized supplier.

Number of flashes	Error description	Possible action
1	Remote / joystick error	Check cables and connectors. Replace Remote module.
2	Network or configuration error	Check cables and connectors. Check Bluetooth pairing. Reconfigure the system. Recharge the battery. Check charger. Replace modules. Contact supplier.
3	Left motor error	Check cables and connectors. Replace Power Module. Check and/or replace left motor.
4	Right motor error	Check cables and connectors. Replace Power Module. Check and/or replace right motor
5	Left park brake error	Check cables and connectors. Check left park brake is released
6	Right park brake error	Check cables and connectors. Check right park brake is released.
7	Module error (other than Remote module)	Check cables and connectors. Check modules. Replace LiNX Access Key. Replace Power Module. Recharge battery.

WARNING!
If the wheelchair stops, reverse or remove the obstacles. If the wheelchair has been moved while the power is off, turn the power on and off.

Care and maintenance

Insufficient or abandoned care and maintenance of an electric wheelchair lead to a limitation of the manufacturer's liability.

In order to ensure long-lasting and efficient activity of the wheelchair, it is essential to perform fundamental maintenance activities and reviews conducted by the service department. Reviews should be performed once a year. In case of any doubts or damages, contact the service department, as the service centre has all competencies required to solve your problem.

Maintenance plan

When	What	Attention
Before driving	Generally check the correct operation.	Perform the inspection alone or with a support person.
	Magnetic brake control. Set the driving mode switch lever on both sides to driving mode.	Perform the inspection by yourself or with a support person. If the wheelchair can be pushed, immediately contact the service center for brake repair. - Risk of accident!
In especially before driving after dusk	Lighting. Lighting system and reflector, check for proper operation.	Perform the inspection alone or with a support person.

<p>Every 2 weeks (depending on the length of the traveled distances)</p>	<p>Checking tire pressure. Tire pressure: 2,5 bar = 36 psi</p>	<p>Perform the inspection alone or with a support person. Use a manometer.</p>
	<p>Set screw. Check the tightness of the screws and nuts.</p>	<p>Perform the inspection alone or with a person assistance. Tighten loose set screws firmly. If necessary, contact service.</p>
<p>Every 6-8 weeks (depending on the length of the traveled distances)</p>	<p>Bolt fastening. Check nuts and bolts for proper fastening.</p>	<p>Perform these activities by yourself or with the help of another person. Tighten loose wheel nut bolts firmly and re-tighten after 10 hours or 50 km. If necessary, contact service.</p>
<p>Every 2 months (depending on the length of the traveled distances)</p>	<p>Tire profile control Minimum tread depth of tread = 1mm</p>	<p>Perform the inspection by yourself or with the help of another person. In the case of a worn profile or damaged tires, have the repair done by a specialized service.</p>
<p>Every 6 months (depending on the length of the traveled distances)</p>	<p>Check - Cleanliness - Overall condition</p>	<p>Perform the inspection by yourself or with the help of another person.</p>
<p>Manufacturer's recommendations: Every 12 months (depending on frequency of use)</p>	<p>Inspection work - Vehicle - Loader</p>	<p>Performs specialized service.</p>

Yearly inspection

The inspection should take place at a service center. The purpose of the inspection is to confirm that the wheelchair works correctly, all its systems (electric, electronic, driving) are fully operational. It also helps prevent possible complications resulting from the failure of any of the aforementioned systems.

The review should include:

1. Checking the tires.
2. Checking the batteries and their connections.
3. Checking the operation of the control panel (electronic system).
4. Checking the frame.
5. Checking the condition of the upholstery.

Usage Notes

1. Protect the control panel from strong shocks and impacts.
2. Protect the wheelchair from prolonged exposure to high and low temperatures and moisture.
3. Keep the control panel and the steering handle clean.
4. Protect the wheelchair from direct contact with water.
5. Take care to keep the frame and upholstery clean. If dirty, wipe them with a soft, dry cloth. Do not clean with any abrasive materials. Do not apply any cleaning agents directly to the control panel, battery or electrical parts.
6. Oil the wheel axles once every three months.
7. Use only the batteries supplied with the wheelchair. If it becomes necessary to replace them, report it to the distributor.
8. Charge the batteries regularly. Always make sure the charger is properly connected to the outlet. Do not disconnect the charger by pulling the cord.
9. Check the operation of the control handle when the control panel is off. Check if the control handle is damaged, whether it returns to the neutral position. Watch the rubber that the control handle is covered with, make sure it is not damaged. If any damage is noticed, do not repair it yourself, but contact a service center.
10. Check all cables. If you notice any damage (abrasion, tear), contact the service center.
11. Make sure the control panel is fixed properly.
12. Store the wheelchair in a dry place. Long exposure to moisture can lead to corrosion of frame components and damage to the electronic system.

WARNING!

When you find any problem with the motor, charger, control panel or damage to the frame of the forklift, contact the service center immediately. If there are any problems and malfunctions with the forklift, contact the service center to help determine the next procedure.

Disinfection

If the product is used by several people (e.g. in a nursing home), commercially available disinfectants should be used. Before disinfecting, clean the upholstery and handles. Only tested and approved disinfectants may be used for disinfection. Information on recognized and approved disinfectants and disinfection methods is available from your national health authority.

WARNING!

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.

Disposal of electric wheelchair

The electric wheelchair must be disposed of in accordance with applicable local and national regulations. For information on the proper disposal of packaging, metal frame components, plastic components, electronics, batteries, neoprene, silicone and polyurethane materials, please contact your local waste disposal company or authorized dealer.

Symbols

The symbols used below on the electric wheelchair are used to identify warnings, mandatory actions and prohibited actions. Be sure to read everything and understand their meaning.

	Medical device
	Date of manufacture
	Manufacturer

	Distributor
	Conformité Européene (compliance with the European Community)
	Catalog number (to re-order)
	Serial number
	LOT NUMBER
	Unique Device Identifier
	Follow the instructions for use
	Refer to the instructions for use
	Warning
	Protect from humidity
	Possibility of jamming fingers
	Permitted temperature
	For indoor/outdoor use
	Maximum user weight

	Maximum safe inclination
	The product is designed to be transported by car
	The product is not intended to be used as a seat for transport in a motor vehicle
	Indicates attachment points on the scooter
	<p>DISPOSAL OF WASTE ELECTRICAL AND ELECTRONIC EQUIPMENT</p> <p>This symbol on the products or accompanying documents means that used electrical or electronic products should not be disposed of with the normal household waste. For proper disposal, refurbishment or recycling, please take these products to the collection points for this type of waste.</p> <p>Alternatively, in some EU countries or other European countries, you may return your product to your local retailer when purchasing a similar new product. By disposing of this product correctly, you will help to conserve valuable natural resources and support the prevention of potential negative effects on the environment and human health, which could result from incorrect disposal. For further information, please contact your local municipality or collection point waste collection points. Penalties may be imposed in accordance with local regulations if this product is disposed of in an inappropriate manner. For parties within the European Union, If you wish to discard electrical or electronic equipment, please obtain the necessary information from your retailer or supplier.</p> <p>Elimination in countries outside the European Union. This symbol is valid in the European Union. If you wish to decommission this appliance, please obtain the correct decommissioning information from your local authorities or from your retailer.</p>

Contact details

Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Polska

tel. +48 42 674 83 84, fax. +48 42 636 52 21

www.mdh.pl

www.viteacare.com

We reserve the right to make technical changes due to technical progress.