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Dane kontaktowe

NINGBO BAICHEN MEDICAL DEVICES CO., LTD.
Adres: Room 2101, Diqiu Building, 666 Taikang
Middle Road, Ningbo, Zhejiang, CN

SUNGO Europe B. V.
Adres: Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel, The Netherlands

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English version on next page

Users with visual impairments can find the PDF files together with further information on our website:

< www.viteacare.com >.

☞ Contact your specialist dealer when required.

Alternatively users with visual impairments can have the documentation read out by a helper.

Table of contents

Introduction	35
Indications.....	36
Contraindications	36
Safety of use	36
Information on electromagnetic compatibility (EMC)	39
Safety inspection	39
Product description	40
Medical device characteristics	40
List of items included in the package.....	40
Wheelchair description.....	40
Technical specifications	42
Use of the wheelchair.....	43
Preparing the wheelchair.....	43
Before use.....	43
Unfolding and folding the wheelchair.....	44
Drive disengagement clutch	46
Control panel operation.....	47
Joystick - control lever.....	48
Safety	49
Seat belt	49
Transport of the wheelchair.....	50
Use	50
Getting into the wheelchair	50
Wheelchair driving training	50
Driving on uneven terrain	51
Getting out of the wheelchair - end of use	51
Batteries	51
Charging process	52
Batteries - Storage	53
Diagnosing faults and troubleshooting.....	53
Reporting technical defects and repairing the device.....	54
Modification.....	54
Care and maintenance	54
Cleaning	54
Disinfection.....	55
Re-use (reusable product)	55
Product life cycle.....	55
Disposal of the electric wheelchair	55
Maintenance plan.....	56
Symbols.....	58
Contact Information.....	60

Introduction

BC-EC8003 is a wheelchair created in accordance with the latest technologies. Its design allows for maximum adaptation to the user's needs. It combines a robust and durable construction, modern electronic control solutions and safety systems. In order to maintain safety and maintain the full efficiency of the product as long as possible, it should be regularly inspected (at least once a year). In case of any doubts or damage, please contact the distributor. Make sure that you have filled the attached warranty form and that you have submitted it to the distributor, as this will enable us to ensure the best possible service.

ATTENTION

Always observe the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Please keep these instructions for future use.

WARNING!

Improper use can lead to death, serious injury, dangerous conditions or product malfunction.

ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

SUGGESTIONS!

Follow the instructions below to keep the product in good working order.

If the device is used by another person, a copy of these instructions for use must be provided.

These instructions for use apply to the design and specification of the product at the time of publication. Due to design changes, some illustrations and photos in the manual may not correspond to the product purchased. We reserve the right to make design changes.

The manufacturer disclaims any liability for personal injury or property damage that may result from improper or unsafe use of this product.

Indications

An electric wheelchair is a device that improves the quality of life, designed to provide mobility.

Contraindications

Contraindications to the sitting position.

Suggestion!

To maintain the life of the battery, during transport, they will be disconnected. Remember to connect the red cable with the input marked with the "+" symbol before using the wheelchair for the first time.

Suggestion!

Avoid completely discharging the battery. Charge the battery after each use of the wheelchair whatever the distance travelled or the time of use. The battery should be charged at least once every 2 weeks.

Safety of use

WARNING!

Every "serious incident" that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State where the user resides.

ATTENTION!

The product is intended for disabled people with musculoskeletal conditions to increase their mobility. If you are not sure whether your health condition allows you to use the device, consult a health care professional.

WARNING!

Do not modify the product in any way; this will void the warranty.

WARNING!

Exceeding the maximum user weight will void the warranty. The manufacturer is not responsible for injury and damage if the maximum user weight is exceeded.

WARNING!

Please read these instructions for use before using the wheelchair and keep them for future reference.

1. Read and observe the information contained in the instructions for use. It is forbidden to use this product and the available optional equipment without first carefully reading and understanding these instructions. If the warnings, cautions or instructions are incomprehensible, contact the supplier, otherwise personal injury or an accident may occur. Never attempt to use the wheelchair beyond its limitations described in this instructions for use.
2. Never turn off the control panel while driving, as this may cause the wheelchair to come to a sudden stop, which may result in injury.
3. Consult a doctor if the medications taken do not affect the ability to drive motor vehicles.
4. To operate the electric wheelchair safely, you must drive on flat surfaces where all four wheels are in contact with the ground.
5. If the battery level indicator drops to the red level, charge the wheelchair as soon as possible. Completely discharging the batteries can damage them or shorten their lifetime.
6. Keep metal objects away from battery connections. Danger of electric shock.
7. When driving on curves, always slow down and maintain a stable centre of gravity. When taking sharp turns, always reduce the speed.
8. Take extra caution when reversing. For safety, the reverse speed should be limited to the minimum speed.
9. Children are not allowed to play near the wheelchair or operate it.
10. Some parts of the electric wheelchair are sensitive to temperature changes.
11. Remember that the temperature of the components and frame parts may increase when exposed to the sun or another heat source, and at low temperatures, there is a risk that the metallic parts of the wheelchair will be very cold. This may cause burns or frostbite.
12. Store the wheelchair in clean and dry conditions.
13. It is forbidden to drive at night without proper lighting.
14. Do not remove anti-tip wheels or drive wheels.

15. Keep hands and feet away from moving parts when driving. Ensure that loose clothing does not become trapped in the drive wheels.
16. Do not connect the battery charger to the extension cord.
17. Disassembly of the controller or engine by persons other than the authorized service is prohibited and invalidates all applicable warranty conditions.
18. It is forbidden to move the wheelchair on motorways. When moving on public roads, follow all current traffic rules for the country.
19. It is forbidden to use the electric wheelchair under the influence of alcohol, drugs or other substances affecting the ability to drive the vehicle.
20. Take caution when driving in busy areas and shopping centers.
21. Never use the wheelchair as a chair for transport in motor vehicles.
22. Do not try to lift the wheelchair by holding onto parts such as the seat, armrest or parts of the housing. Lift by grasping the frame elements.
23. When driving, it is forbidden to use a mobile phone or other personal communication devices (e.g. CB-radio).
24. Do not use the wheelchair outside during rain or snowfall, high (above 30° C) and low (below 0° C) temperatures.
25. Do not drive the wheelchair onto unprotected wharves, river banks or lake shores.
26. The wheelchair is not suitable for use in difficult terrain (unpaved roads).
27. Do not stop, turn or reverse suddenly to avoid falls and injuries.
28. When driving, keep your forearms on the armrests and feet on the footrest platform.
29. It is forbidden to stand while driving the wheelchair. Do not lean to the side or lift objects from the ground.
30. Do not drive onto raised objects (steps, kerbs) higher than 50 mm.
31. The wheelchair is not suitable for moving up or down the stairs. Always use ramps or elevators.
32. The wheelchair is not designed to negotiate ground gaps greater than 100 mm in size.
33. The product contains lithium, lead and components containing lead.

WARNING!

Drive the electric wheelchair in a straight line up or down the slope, never reverse. Irregular forward and backward movements with slope changes can increase the risk of the wheelchair tipping over.

The maximum user mass is 130 kg.

Information on electromagnetic compatibility (EMC)

To ensure proper use of the wheelchair without electromagnetic interference, it is recommended to use only dedicated cables and accessories specified by the manufacturer. Potential electromagnetic interference from neighboring devices should not significantly affect the operation of the wheelchair during its expected service life, provided that it is used as intended and in accordance with the instructions for use.

Use of accessories and cables: Only use accessories, transformers and cables supplied by the wheelchair manufacturer. The use of other components may lead to electromagnetic interference or reduced electromagnetic immunity and cause the wheelchair to malfunction.

Avoiding close proximity to other devices: If it is necessary to use the device in close proximity to other equipment, monitor all devices to ensure they are functioning properly. Malfunction may result from close proximity to other devices.

Portable communication devices: Communication devices (e.g. mobile phones, radios) operating at high frequencies and their accessories (antenna cables, external antennas) should not be used within 30 cm of electrical components and cables in the described wheelchair. Failure to comply with this recommendation may limit the functionality of the device.

Electromagnetic environment: The wheelchair is designed to operate in a specific electromagnetic environment. The user should ensure that the wheelchair is used in appropriate conditions.

WARNING!

Do not use handheld radio transmitters such as CB radios or mobile phones in close proximity to the controller or the wheelchair battery.

Safety inspection

ATTENTION!

Before use, check the following components:

- Electrical connections and wiring harnesses (report any visible damage to your distributor).
- Batteries - fully charged and free from corrosion.
- Brakes - check the correct operation of the electromagnetic brake.
- Folding lock mechanism - ensure that the wheelchair is fully unfolded.
- Tyre condition - ensure that they are not damaged or excessively worn - if there is a problem, contact your distributor.

Product description

Medical device characteristics

The electric wheelchair is a lightweight, robust vehicle with motors powered by lithium or gel batteries, enabling people with limited mobility to move independently. Equipped with a joystick for control, adjustable seat and footrests(or footrest platform), it provides comfort and safety. The range on a single charge is approximately 10 km and the speed is up to 6 km/h. The design allows for easy folding and transport, and built-in safety systems guarantee stability while riding. It is an independent and convenient solution to support everyday life.

List of items included in the package

Model	BC-EC8003
Wheelchair frame	1 szt.
Charger	1 szt.
Instructions for use	1 szt.
Footrests	2 szt.
Control panel (joystick)	1 szt.

Wheelchair description

Illustration 1. Front of the wheelchair.

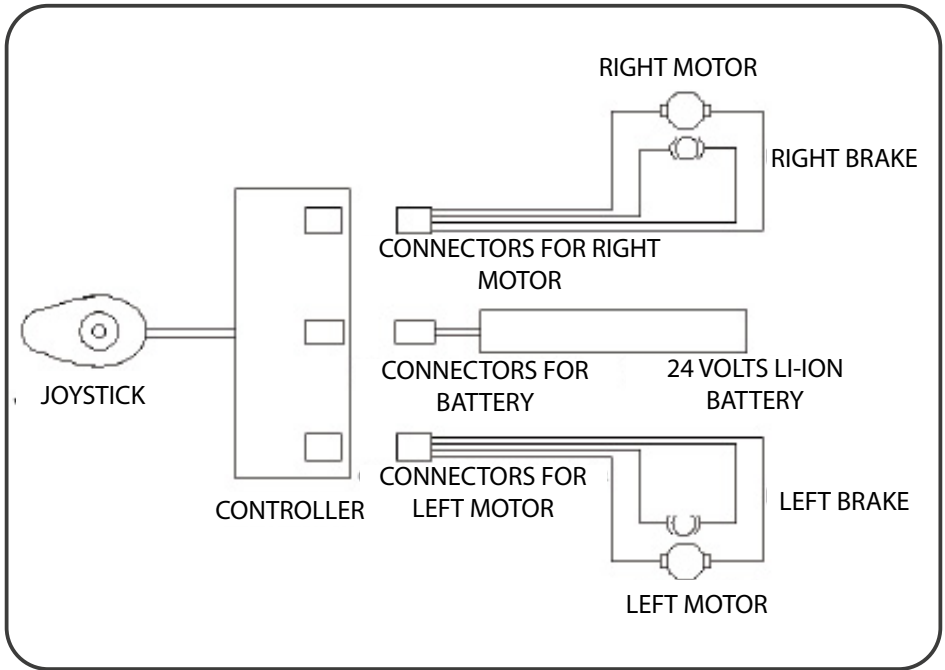
Illustration 2. Electric Diagram.

Technical specifications

Model	BC-EC8003
Seat width	405 mm
Maximum user weight	130 kg
Maximum load capacity	
Motors	2 x 180 W (brushless)
Battery	10,4 Ah / 24V (Li-Ion)
Charger (output power)	24V (3A)
Maximum speed forward	6 km/h
Energy range	do 10 km
Seat surface height from the floor	470 mm
Seat width between the sides	490 mm
Seat Depth	430 mm
Backrest height	400 mm
Total length of the wheelchair with/without footrest	970 / 790 mm
Total wheelchair height	910 mm
Total width of the wheelchair	590 mm
Dimensions after folding (length x height x width)	775 x 310 x 690 mm
Armrest height from the ground	700 mm
Seat cushion thickness	35 mm
Front wheels (tyre type)	6,5" - 160x35 mm (solid)
Rear wheels (tyre type)	11" - 270x35 mm (solid)
Suspension	-
Turning radius	900 mm
Footrests (type)	2 pcs. (folding footrests)
Weight of wheelchair with/without batteries	12,88 kg / 10,7 kg
Safety	Safety belt, anti-tip wheels
Maximum safe inclination	6°
Other functions	Manual folding frame (cruciform mechanism), reclining armrests and folding footrests, folding backrest
Minimum braking distance at maximum speed	1000 mm
Frame material	Carbon
Description of the environment in which the product will be used	  
Storage and usage temperature	From -25°C to +50°C
Storage and usage humidity	From 30% to 80%

Speed, max distance of the vehicle depends on the user weight, battery load, battery capacity and the terrain on which the vehicle is used. The information is current at the time of publication, the manufacturer reserves the right to make changes to the specifications.

Use of the wheelchair

Preparing the wheelchair

The wheelchair is delivered in a cardboard package, and the fragile parts are additionally protected against damage during transport. Unpack it carefully to avoid any damage.

Before use

- Check all electrical connections, make sure that they are stable and not corroded.
- Check that the batteries are fully charged.
- Check that the wheelchair is engaged in drive mode (clutch levers engaging the drive in the upper position).
- Always plan your route in such a way as to avoid dangerous (muddy, difficult areas) and obstacles that you will not be able to overcome with a wheelchair.
- Whether the speed is adapted to the user's capabilities and skills.
- Does the horn work.
- Fasten seat belt.

ATTENTION!

- When transferring to the wheelchair, do not rest your body weight on the armrests as this can lead to the wheelchair tipping over and causing injury.
- Do not stand on the footrests because this could tip the wheelchair over.
- Remember to check whether the wheelchair power is switched off because unintentionally touching the joystick can cause the wheelchair to move, fall and, consequently cause injury.
- During the transfer, try to be as close to the wheelchair as possible to avoid slipping and falling.

WARNING!

The following actions can influence the steering and stability of the wheelchair negatively:

- Holding and walking an animal strapped on a leash.
- Carrying passengers (including pets).
- Hanging the item on the controller.
- Towing or pushing by another vehicle.

Unfolding and folding the wheelchair

First of all, you should prepare enough space to move safely around the wheelchair.

The frame can be unfolded by pressing the seat frame down from both sides.

Illustration 3. Folded wheelchair.

Lift the folding backrest upwards until it locks - an audible click is heard - illustration opposite.

Illustration 4. Backrest unfolding.

Then attach the backrest cover to the frame using Velcro tape and also attach the seat cushion with Velcro.

Illustration 5. Backrest strap and seat cushions.

Tilt the footrests towards the front of the wheelchair and then unfold the footrest platforms - as shown in the illustration opposite.

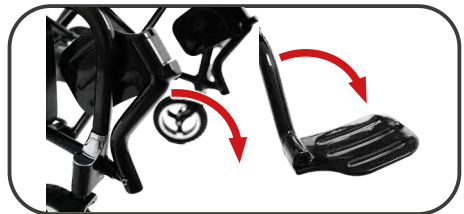
Illustration 6. Folding out the footrests.

Install the controller in the wheelchair armrest as shown.

Tighten the locking screw located under the armrest.

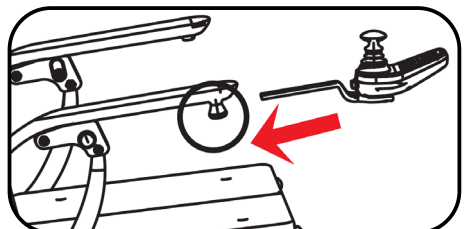
Illustration 7. Control panel (joystick).

Then mount the battery on the side of the frame - depending on which side the battery connector is located (right or left side).

Illustration 8. Battery mounting.

To fold the wheelchair, first tilt the foot-rest platforms as shown in the illustration opposite.

Illustration 9. Folding frame and foot-rests.

Then lift upwards on both sides of the seat frame or use the two straps under the seat cushion.

Illustration 10. Folding the frame.

It is possible to charge the battery outside the wheelchair:

To remove the battery pack, press the lock button on the front of its housing while lifting it upwards.

The charging port is located under the hinged cover on the battery housing (illustration opposite).

To install the battery pack, slide it from the top into the guides at the bottom of the battery holder until the lock clicks audibly.

Illustration 11. Disconnecting the battery and charging outside the wheelchair.

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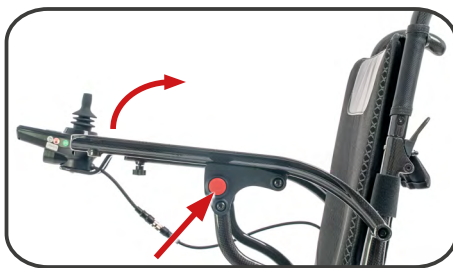
Illustration 12. Tilting armrests.

WARNING!

Attention should be taken when tilting the armrests so that no parts of your wardrobe or body are in danger of being pinched. Take special care.

Drive disengagement clutch

There are two clutch levers in the wheelchair to enable or disengage the electric drive and manually push the wheelchair.

- To disconnect the drive for manual pushing, lower the lever as far down as possible - position **B** in illustration 21.
- Restarting the drive (travel mode) should be done by lifting the lever up - position **A**.

Illustration 13. Motor drive disconnection.

Control panel operation

The electronic control panel is a component that allows you to control the electric wheelchair. It powers all operating systems. It allows you to control and steer the wheelchair, monitors the battery charge level and the condition of the entire electrical system. The control panel is located on one of the armrests and is connected to the battery and motor.

Power button applies power to the control system for the electronic equipment, which then supplies power to the wheelchair motor. Do not use the power button to stop the wheelchair unless there is an emergency. Doing so may shorten the life of the wheelchair's drive components.

Display provides information about the most important functions of the wheelchair. The battery icon indicates the battery charge status. Icon with numbers indicates the selected maximum speed (range 1-5) and shows the current speed while driving the wheelchair.

Speed increase button allows you to select a higher speed at which the wheelchair will move.

Speed reduction button will lower the maximum speed of the wheelchair.

Sound signal (horn) is mainly used to emit an audible signal that warns pedestrians, cyclists or other road users of the presence of the wheelchair, increasing safety in urban traffic and crowded areas.

Mute button is used to mute the voice prompts provided when operating the control panel.

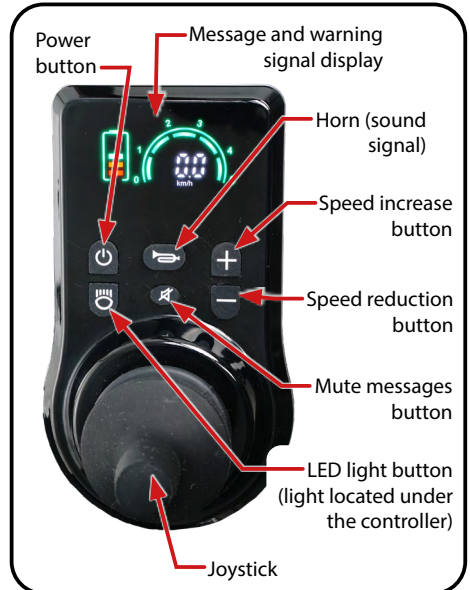
Illustration 14. Control panel (joystick).

The reverse sound can be turned off by pressing the speed reduction and horn buttons simultaneously for 3 seconds. If you want to turn it on, press the acceleration and horn buttons simultaneously for 3 seconds, and the sound will be turned on.

Joystick - control lever

The joystick is mainly used to control the movement of the wheelchair, including its speed and direction – forwards, backwards, left or right, etc. The further you push the joystick from its central position, the faster the wheelchair moves. Whenever you release the joystick, it will automatically return to the centre position and the brake will automatically activate to stop the wheelchair.

WARNING!

The wheelchair is equipped with electromagnetic brakes. Be careful when moving, as immediately returning the joystick to the neutral position will stop the wheelchair in the shortest possible distance.

WARNING!

Do not switch off the electric drive if there is no one available to guide the wheelchair. Do not switch off the electric drive when the wheelchair is on a slope, as it could roll down, posing a risk of injury.

Please note that when the electric drive is switched off, the electromagnetic braking system is also switched off.

Always stay outside the wheelchair when switching off the electric drive. Do not do this while sitting in the wheelchair. After moving the wheelchair to the desired location, remember to switch the electric drive back on.

Safety Seat belt

WARNING!

Remember to fasten your seat belt before starting to drive.

Make sure that no objects are trapped under the belt. This will prevent painful pressure points.

Later installation of the hip safety belt can only be carried out at a specialist service centre!

The safety belt is not part of the securing system for an electric wheelchair and/or its user during transport in a motor vehicle.

The lap belt is used to stabilise the sitting position and prevent the user from falling forward out of the electric wheelchair.

To fasten the lap belt, pull both ends forward and close the buckle until it locks into position with an audible click. To open the lap belt, press the buckle buttons and pull the belt ends apart.

Illustration 15. Fasten the safety belt.

ATTENTION!

The length of the safety belt can be adjusted, but remember not to tighten it too much.

Transport of the wheelchair

WARNING!

It is forbidden to remain in a wheelchair during transport, as this may result in a fall and serious injury.

The wheelchair should only be transported using means designed for this purpose. The wheels should be locked. Always ensure that the wheelchair is properly secured inside the means of transport, otherwise it may cause injury to people travelling in the vehicle and serious damage to the vehicle itself.

When driving up the ramp into the transport vehicle, rear support should be provided to prevent the wheelchair from tipping over.

All additional wheelchair components should be removed or secured to the wheelchair during transport so that they do not cause injury to people or damage to the transport equipment in the event of sudden braking.

Use

Getting into the wheelchair

WARNING!

You may only get in or out of the electric wheelchair when it is switched off and the drive disengagement lever is in drive mode (lever in the upper position).

Accidental movement of the joystick (drive and steering lever) can cause the electric wheelchair to move uncontrolled! – Risk of accident!

Wheelchair driving training

1. You must learn how to use the wheelchair properly and perform all manoeuvres in order to move around safely. First, learn how to move forward. Make sure that the minimum speed is set.
2. Once you have mastered moving straight ahead, try to drive the wheelchair in an S-shape.
3. Then you can start practicing driving backwards. Regardless of the programmed forward speed, the wheelchair will always move slower backwards.

Driving on uneven terrain

Navigating rough terrain poses more risks. It is important to follow the safety rules. Failure to do so, may result in serious injuries.

- Do not climb hills above 13° of inclination. Note that uneven or wet surface will further reduce grip and weaken brake performance when ascending and descending a hill.
- Do not go backwards from the hills.
- Do not move on uneven, unpaved surfaces (sand, gravel, grass).
- When descending a hill, move at a minimum speed.
- Do not get off the wheelchair or sit on the wheelchair standing on an inclined surface (always place the wheelchair on a level, horizontal surface first).
- When overcoming hills, always approach in a straight line in accordance with the slope of the terrain.

Getting out of the wheelchair - end of use

WARNING!

Never get in or out of the wheelchair without first turning off the power. This prevents the wheelchair from moving by accidentally switching the joystick of the controller.

1. Stop the wheelchair completely.
2. Turn off the power supply using the control panel.
3. Tilt the armrest on the side that will allow the wheelchair to be abandoned safely.
4. Carefully and safely dismount from the seat and take a safe position next to it.
5. To make it easier to get in the wheelchair, you can leave the armrest tilted.

Batteries

The wheelchair is equipped with long-lasting batteries. The batteries are protected but require periodic technical and performance inspections.

- Perform a full charge cycle of at least 6 hours before using the wheelchair for the first time.
- Only use the charger supplied by the manufacturer (24V/3A) to charge the batteries.

ATTENTION!

Always charge batteries for 4 to 6 hours or, if available, until the green LED on the charger lights up. If they are undercharged, their service life will be shortened.

Charging process

How to charge batteries:

1. Charging by connecting the charger to the socket located at the bottom of the control unit.
2. Connecting directly to the battery box.

Illustration 16. Connecting the charger to the controller.

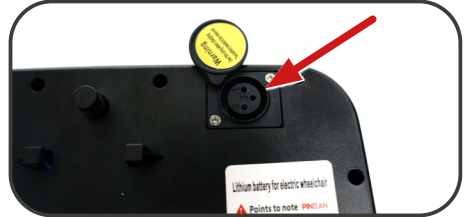
Illustration 17. Direct connection to the battery.

1. Park the wheelchair as close as possible to a grounded socket.
2. Switch off the wheelchair.
3. Ensure that the drive disengagement clutch is in the correct position – drive mode.
4. Connect the charger to the wheelchair's charging socket.
5. Connect the charger to a grounded electrical socket, remembering to check that the socket has the correct voltage of 230V.
6. Switch on the charger.
7. A red LED light on the charger indicates that the power supply is active.
8. A yellow LED light on the charger indicates that the charging process is active.
9. The green LED light on the charger indicates that the charging process is complete.
10. The optimal charging time is 4-6 hours.
11. When charging is complete, first disconnect the charger from the power outlet, then remove the plug from the charging socket in the wheelchair.

WARNING!

During charging, to avoid the risk of electric shock, the device must only be connected to a power supply with protective grounding.

WARNING!

- Do not leave the charger connected to the charging port if the charger is not charging the wheelchair battery.
- The charger is intended for indoor use only.
- Do not expose it to rain or water jets.
- Do not cover the battery charging terminal.
- Always keep the battery dry, never use it at high power, and never charge it for more than 6 hours.

Batteries - Storage

- If the wheelchair is to be unused for a long time, remove the batteries and store them in a dry and cool place.
- Avoid completely discharging the batteries.
- If you use the wheelchair every day, charge the batteries every day.
- If you use the wheelchair once a week, charge the batteries once a week.
- Batteries should be stored at 80% charge.
- If the wheelchair is not used regularly, charge it at least once every two weeks in a full charge cycle to maintain battery life.
- Never replace the batteries with batteries other than those supplied with the wheelchair.
- If the batteries are damaged or need to be replaced, contact the distributor.
- Do not dispose of batteries in the waste bin. Dispose of in accordance with local waste management regulations.
- To prevent short-circuiting and damaging the battery, never touch their terminals with metal objects.

Diagnosing faults and troubleshooting

This product has an automatic fault warning function, which is very convenient. In the event of a malfunction, the LEDs on the controller will start flashing and an alarm signal will be activated by a specific number of sound signals. If, after checking the fault according to the table below, the problem persists, contact your distributor immediately.

Number of alarm signals	Scope of control	Description of the fault solution
2+1	Communication failure	Check the connection inside the joystick.
2+2	Right engine failure	Check the right motor or the wiring connection.
2+3	Right brake fault or poor cable connection	Check that the brake lever is in place and that the brake is not damaged.
2+4	Left engine failure	Check the left motor or the wiring connection.
2+5	Left brake fault or poor cable connection	Check that the brake lever is in place and that the brake is not damaged.
2+6	Overcurrent	Check that the motor is not blocked or short-circuited, and that the joystick is not damaged.

2+7	Joystick lever failure	Check that the joystick lever is working properly or that the internal connection is not loose.
2+8	Joystick failure	Contact your distributor
2+9	Battery failure	Charge the battery, and if it still does not work, replace it with a new one with the same specifications.

Reporting technical defects and repairing the device

If a defect or damage to the product is discovered during use or maintenance, it must be reported immediately to the distributor or manufacturer.

WARNING!

Under no circumstances should an unqualified person repair or alter the product or its design.

The only person allowed to make any repairs or adjustments is an employee authorised by the manufacturer.

Modification

Do not modify, remove, disable or add any parts, components or functions of the electric wheelchair that are not supplied by the manufacturer.

Care and maintenance

ATTENTION!

Insufficient or abandoned care and maintenance of the electric wheelchair leads to limitation of the manufacturer's liability.

Cleaning

In order to ensure long-term and efficient operation of the wheelchair, it should be subjected to basic maintenance and technical condition inspections. Inspections should take place once a year. If you have any doubts or damage, contact the distributor, which has all the competences to solve the problem.

WARNING!

Observe the safety instructions given by the manufacturer of cleaning and disinfecting products. Failure to do so may result in skin irritation or premature damage to the upholstery of the electric wheelchair.

It is not permitted to use:

- solvents,
- toilet cleaners,
- agents with chlorine content,
- sharp brushes and hard objects,
- corrosive cleaners.

Do not clean the wheelchair with pressure or steam units!

Disinfection

If the product is used by many people (e.g. in a social care home), you should use commercially available disinfectants. Before proceeding with disinfection, clean the surface. Only biocidal products authorised in Poland by the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products or by another competent authority of a given EU Member State may be used for disinfection.

WARNING!

Disinfectants may damage the outer surface, reducing the long-term functionality of the components. Always observe the instructions concerning the use of the products specified by the manufacturer.

Re-use (reusable product)

The product is suitable for reuse. The number of times it can be reused depends on how often and in which way the product is used. Before reuse, thoroughly clean and disinfect the product and check its safety features. If necessary, take repair action. If necessary, take repair action.

Product life cycle

The expected life of a medical device is 5 years, as long as it is used as intended and all maintenance and servicing is performed. The lifespan of a medical device also depends on the frequency of use, the environment in which it is used and its care. By using spare parts, the life of the product can be extended. Spare parts are generally still available for 5 years after the model is discontinued. This expected lifetime of a medical device does not imply an additional guarantee.

Disposal of the electric wheelchair

The electric wheelchair must be disposed of in accordance with current local and national regulations. For proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, contact your local waste disposal company or authorised distributor.

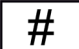
Maintenance plan

When	Scope of control
<p>Before the ride</p>	<p>In general, check for impeccable performance. After turning off the power, check the joystick. Ensure that it is not bent or damaged and that it returns to the neutral position when released. Make sure that the rubber cover around the base of the joystick is not damaged. Check the cover visually. Check the controller cable visually. Ensure that it is not ragged, cut or exposed wires. Check that there are no flat places on solid tires that may have a negative impact on the stability of the wheelchair.</p> <p>In case of problems and questions, contact the distributor.</p>
<p>Weekly</p>	<p>Make sure that all parts of the control panel (joystick) are well attached to the handle.</p> <p>Check the tightening of the screws and tighten with moderate force if necessary.</p> <p>Check the correct tyre pressure (if present). If the tire does not hold air, contact the distributor.</p> <p>Check the operation of the electromagnetic brake. This test must be carried out on a level surface with at least a meter of free space around the electric wheelchair. Try to push the wheelchair with the motor clutches engaged into driving mode. In case of doubt, always contact the distributor.</p>
<p>Monthly Checking the brakes</p>	<p>Turn on the controller and reduce the speed level of the electric wheelchair. After one second, check the battery level indications - make sure that the LED light remains on. Slowly push the joystick forward until you hear the electromagnetic brakes click - release the joystick immediately. The operation of each electromagnetic brake must be audible within a few seconds of the movement of the joystick. Repeat this test three times by pushing the joystick backwards, then left and then right.</p>

<p>Monthly (depending on the length of the traveled sections)</p>	<p>Inspection of tire profile Check drive tyres for wear: Rear wheels - minimum tread depth = 1 mm. Replacement of tires should be commissioned to an authorized service.</p> <p>Front wheels: Check front, self-aligning wheels for wear. If necessary, contact your distributor.</p> <p>Anti-tipping wheels: Check that the anti-tipping wheels do not rub against the ground when using the electric wheelchair. If necessary, adjust them. Check that the anti-tipping wheels are not excessively worn. In case of doubt, always contact the distributor. Keep the electric wheelchair clean and protected from foreign bodies such as mud, dirt, hair, food, beverages, etc.</p>
<p>Every 6 months (depending on the frequency of use)</p>	<p>Check</p> <ul style="list-style-type: none"> - Cleanliness - Overall condition
<p>Manufacturer's recommendations Every 12 months (depending on the frequency of use)</p>	<p>Maintenance work - contact with distributor</p> <ul style="list-style-type: none"> - Vehicle - Charger

Symbols

The symbols used below are used to identify warnings, mandatory actions and prohibited actions. You should read everything and understand their meaning.

	Medical device
	Date of manufacture
	Manufacturer
	Authorized representative in the European Union
	Importer
	Distributor
	Conformité Européene (compliance with the European Union)
	Catalogue number
	Serial number
	LOT number
	Unique Device Identifier
	Model number
	Consult instructions for use
	Follow the instructions for use
	Warning
	Protect from moisture
	Possibility of jamming fingers
	Permitted temperature

	For indoor/outdoor use
	Maximum user weight
	Total permissible mass
	Maximum safe inclination
	The product is designed to be transported by car
	The product is not intended to be used as a seat for transport in a motor vehicle
	<p>SELECTIVE COLLECTION OF ELECTRICAL AND ELECTRONIC EQUIPMENT. This symbol on the products or accompanying documents indicates that used electrical or electronic products and batteries must not be disposed of in municipal waste. For proper disposal, restoration or recycling, such products should be handed over in collective places for this type of waste. Alternatively, in some countries of the European Union or other European countries, you can return your products to a local seller at the time of purchase of a similar new product. By correctly disposing of this product, you will help preserve valuable natural resources and support the prevention of potential negative impacts on the environment and human health, which could result from incorrect waste disposal. More information is available from the communal offices or waste collection sites. In case of incorrect disposal of this product, penalties may be imposed in accordance with local regulations. If you want to dispose of this device, please obtain the necessary information on the correct disposal at your local authorities or from your dealer.</p> <p>Liquidation in countries outside the European Union: This symbol is valid in the European Union. If you want to dispose of this device, please obtain the necessary information on the correct disposal at your local authorities or from your dealer.</p>

Contact Information

NINGBO BAICHEN MEDICAL DEVICES CO., LTD.
Address: Room 2101, Diqu Building, 666 Taikang
Middle Road, Ningbo, Zhejiang, CN

SUNGO Europe B. V.
Address: Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel, The Netherlands

Importer

mdh Sp. z o.o.
Address: ul. Maratońska 104, 94-007 Łódź, Polska
tel. +48 42 674 83 84, fax. +48 42 636 52 21
www.mdh.pl
www.viteacare.com

We reserve the right to make technical changes due to technical progress.