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Instructions for use
Cameleon Stab
Special wheelchair stabilizing back and head
VCWK7AC
Edition EJ 05.2022

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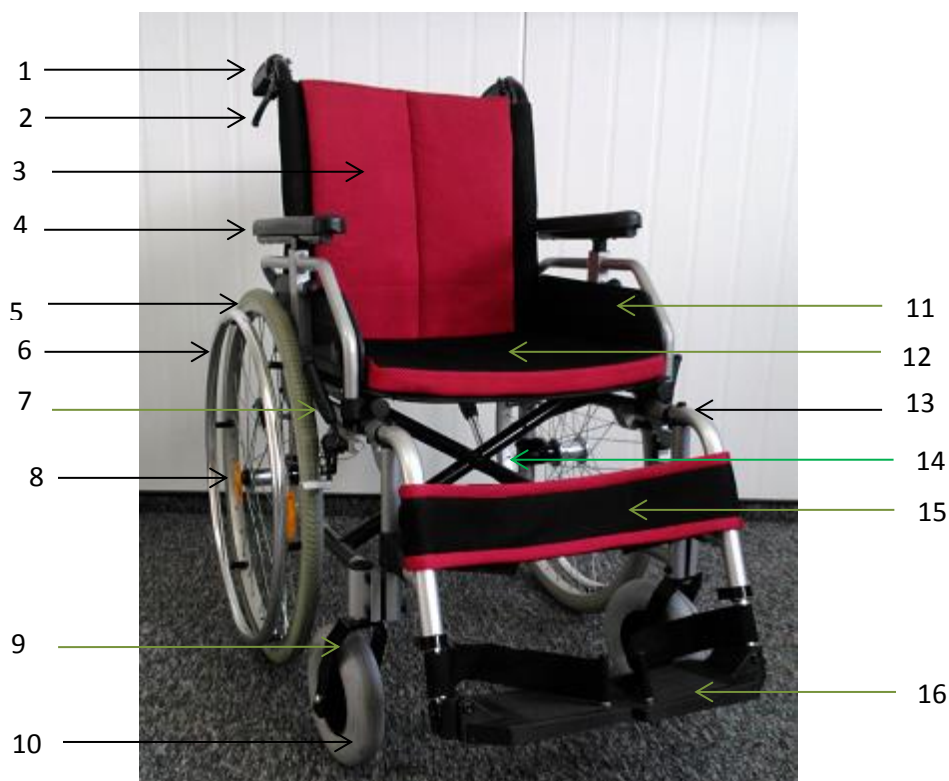
PLEASE READ CAREFULLY BEFORE USING
WHOLE OF INSTRUCTIONS

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Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual was made in order to provide the necessary information on the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to distinguish that to date we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual.

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1. Handle holder
2. Brakes for caretaker
3. Backrest
4. Armrest
5. Rear wheels
6. Metal handrim
7. The parking brake lever
8. Quick connect handle
9. The front wheel fork
10. The front wheel

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11. The side plate of the armrest
12. Seat cushion
13. Footrest frame
14. Frame of cross construction
15. Stabilizing belt
16. Footrest platform

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The wheelchair is used in order to commute by people with disabilities e.g. after the spinal cord damages, cerebral palsy or various lower limb injuries, neuromuscular diseases, post-traumatic cranial cerebral or for geriatric patients.

It is also intended for people who want to actively and comfortably participate in everyday life.

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Contraindications to a sitting position. Conditions that require stabilization of the upper spine and head. Patients with mental disorders should not be left unattended in the wheelchair.

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UNFOLDING THE WHEELCHAIR.

- The wheelchair must be placed with rear wheels in front to You
- Unfold the handle holder towards the outside (Illustration 7.).
- Then place wheelchair with front wheels in front of you
- Press seat frames, until complete unfolding. (Illustration 8., 9.)
- On the seat should put the seat cushion. Then should attach the backrest upholstery.

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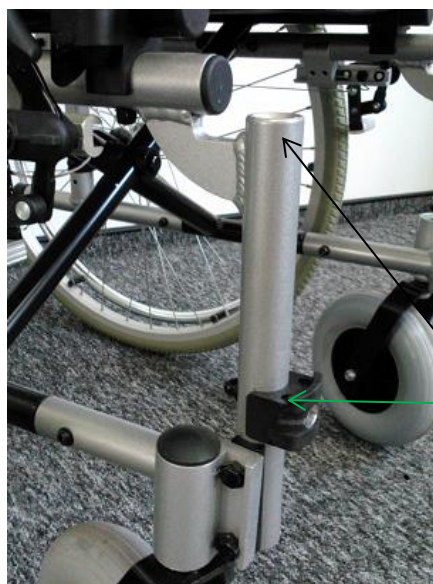
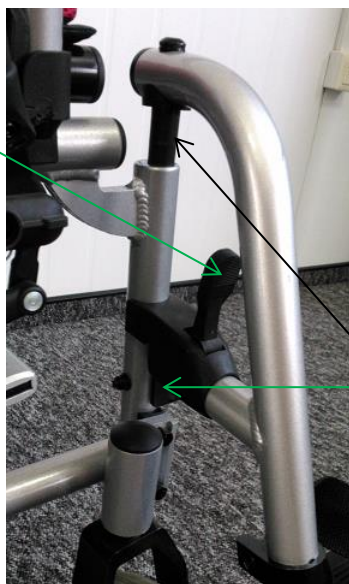
to avoid injury to the hands or damage the wheelchair.

- f) Using public roads, follow accurate traffic regulations.
- g) Like other vehicles, do not drive the wheelchair under the influence of alcohol, or drugs. This also applies to indoor riding.
- h) The wheelchair is not intended for patient transporting on stairs, just like on stretcher or by pulling or lowering the patient. The construction of the wheelchair and its joining elements are not designed for such a large overload if it is used in this way. This can cause weakness or damage of the entire structure the wheelchair and possible accidents!
- i) The wheelchair is designed to use indoors and outdoors only on hard and even surfaces.
- j) Using the wheelchair on an unstable surface (covered with ice, wet, greasy, sandy) is unacceptable- it causes risk of falling,
- k) You can only use the wheelchair on surfaces where all four wheels are touching the ground.
- l) To overcome architectural barriers (steep or long climbs, thresholds or curbs, etc.) help of caretaker is needed.
- m) If the user does not have sufficient experience in the wheelchair use, should ask caretaker for help with overcoming obstacles.
- n) Do not leave the wheelchair with the user close to open sources of fire.
- o) Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of wheelchair. It may cause burns or frostbite parts of the body.
- p) In order to transfer wheelchair, grab the frame. Never transfer the wheelchair by grabbing the armrests or footrest.

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finger entrapment:

- when folding the wheelchair
- in the area moving parts.
- in the area of brakes.
-

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SITTING ON THE WHEELCHAIR:

- Place the wheelchair in a position that will make easier transporting patient (by the bed, chair, close to lifting distance)
- Block both brakes of wheelchair, fold or remove the footrests.
- Pull the brake lever up and fold it. (Illustration 15., 16., 17.)
- Lift the front of the armrest up by pressing the armrest lever. (Illustration 18.)
- The armrest can be dismantled. To do this, after raising the front part of the armrest, turn the armrest block in to the middle part of the wheelchair. (Illustration 20; 21.)
- Place the patient in the wheelchair.
- Mount the armrest, proceeding in the reverse order when dismantling.
- Unfold the brake lever.
- Install and open the footrests and put on it patient's feet.
- Place the patient wheelchair so that the patient is in upright sitting position (knees should be bent at an angle of $\sim 90^\circ$).
-

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PUSHING THE WHEELCHAIR BY CARETAKER

- Before use, make sure that no part of the patient's body or his / hers clothing is not exposed to hook with the moving parts of the wheelchair.
- If patient can not sit on one's own, wheelchair should be equipped with stabilizing belts (belts, which are in standard equipment to protect the patient from sliding off the seat).
- Wheelchair should be pushed with both hands.

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- While moving on slopes, use caution and reduce your speed.
- Do not move on too high slopes. The maximum allowable slope angle up and down is 5%.

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- Consider the possibility of assistance from the caretaker. If the person on the wheelchair did not have enough force to control the wheelchair, the brakes must be on.
- To prevent tipping over while moving along the sloping surface, You should be slightly lean forward to move your centre of gravity forward.

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GOING DOWN OF THE THRESHOLD OR CURBS

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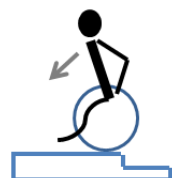
- Ahead without the help of a caretaker
 - Experienced wheelchair user can drive on his own from the threshold or curbs.
 - With a low curb, you can drive by moving forward. Make sure that the footrests platform does not touch the ground.
- Please keep balance on the rear wheels to reduce the pressure on the front wheels.
- Then cross the curb.

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
- Ahead with the help of a caretaker
 - Higher curbs can be overcome with caretaker help.
 - Caretaker should slightly tilt the wheelchair back.
 - Cross the curb moving on the rear wheels.
- Then the wheelchair must be paced on all four wheels.

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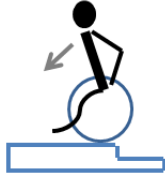- Backwards without the help of a caregiver
 - Experienced wheelchair user can cross on his own the threshold or curbs.
 - Turn the wheelchair in such a way that both rear wheels were turned toward the curb
 - Lean forward to move the centre of gravity in forward.
- Come closer with the wheelchair to the edge of the curb.
- Then, by using the handrim in a controlled manner, user must slide the wheelchair from the curb.

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MOVING ON THE THRESHOLD OR CURBS

- Ahead with the help of a caregiver

- Experienced wheelchair user can drive on his own from the threshold or curbs.

• Caregiver should slightly tilt the wheelchair back.

- Wheelchair user should keep balance on the rear wheels to reduce pressure on the front wheels.
- Go with front wheels on the curb and then put the rear wheels over the curb.

- Higher curbs can be crossed going backwards with the help of a caregiver

• Turn the wheelchair in such a way that both rear wheels are turned toward the curb.

• Wheelchair user should keep balance on the rear wheels to reduce the pressure on the front wheels.

- Caregiver should pull the wheelchair over a curb.
- Wheelchair user should return to its normal position on the wheelchair.

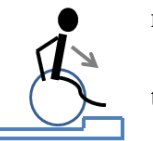
- Ahead without the help of a caregiver.

• Experienced wheelchair user can go down on his own from the threshold or curb.

• Place the front wheels to the edge of threshold or curb.

- It is important to make sure that the footrests do not touch the threshold or curb

• Now, can go with the front wheels on the curb.

• Then, may enter the rear wheels on to the curb.

TRANSPORT IN THE CAR

- In order to make the wheelchair for easy to transport remove the seat cushion.
- Then should fold footrests platforms, or remove footrests.
- In order to do that should unlock the footrest lock, rotate the footrests on the outside of wheelchair and remove the footrests.

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- Two pairs of wheels are mounted on movable axes; you can change the wheel localization front axles and rear axles.
- Changing the wheels height can be made only in situations when the patient is not sitting in a wheelchair.
- Press the black button to remove the wheels and remove the wheel (Illustration 24.)

- In order to shift center of gravity hold the 10mm nut with wrench, then unscrew two screws located from the inner side hub fixing by using 5mm allen wrench. (Illustration 25.)
- Then turn the hub fixing as shown in Illustration 27.
- Hold the nuts and tighten the previously removed screws.

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POMMEL ADJUSTMENT OF THE WHEEL FOR

- In order to adjust fork pommel should holding the nut with wrench 10mm, unscrew two mounting screw of fork pommel by using wrench 10 mm mounting. (Illustration 26.)
- Select the appropriate arrangement of shims.

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(Illustration 27.)

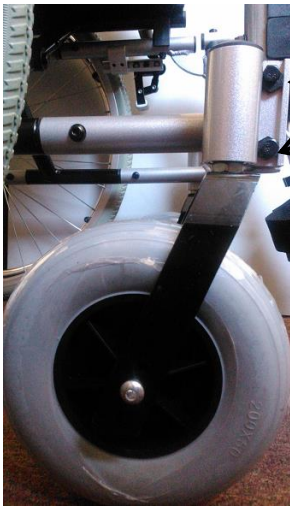
- Hold the nuts and tighten the previously removed screws.

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
All of the following adjustment should be made by qualified personnel:

- Pommel adjustment of front wheel fork
- Shift center of gravity
- Wheelchair height

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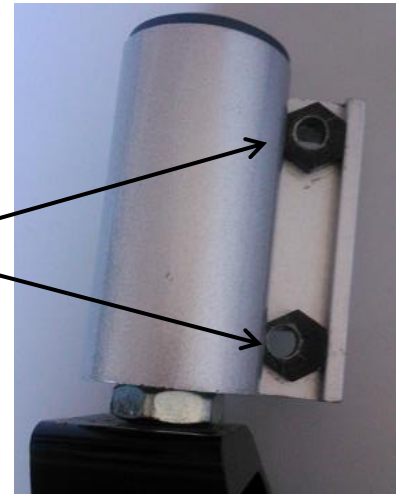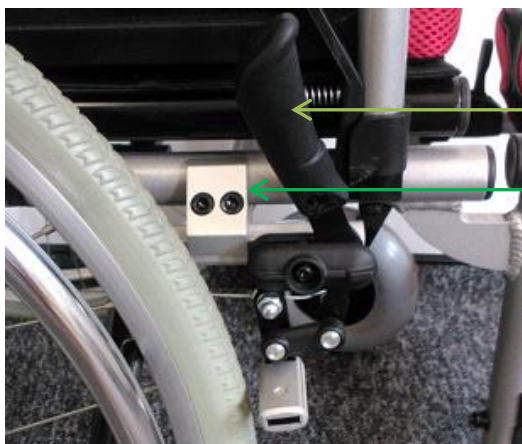
- Illustration 28. shows the brake mechanism.
- To activate the brake, pull the brake lever towards the front wheel to the point where the brake is tightly touching to the rear wheel
- Make sure that the brake lever does not automatically switch off when trying to move the wheelchair.

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- To disable the brake, pull the brake lever toward the rear wheel.
- To reposition the brake mechanism, loosen two screws by using Allen wrench 5 mm. (Illustration 28.)
- Move the entire mechanism in the direction of the rear wheel and tighten the screws.
- When you turn on the brake, the wheel should not rotate (test the correctness of adjustment carried out in a wheelchair with a patient).

FOOTREST

FOOTREST HEIGHT ADJUSTMENT

- Footrest could be adjustable in length.
- To set the desired length of the footrest, should loosen the footrest height adjustment screw by using allen wrench 5 mm. (Illustration 29.)
- Depending on the needs push in or pull out the footrest platform located in the frame footrest.
- After setting the proper length of the footrest, tighten the adjustment screw.

FOOTREST ANGLE ADJUSTMENT

- To set desired footrest platform position should loosen the footrest plate adjustment screw by using allen wrench 5 mm. (Illustration 29.)
- Set the desired position of the footrest platforms.
- After choosing the footrest platform correct position, tighten the footrest adjustment screw.

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ANTI-FLIP WHEELS

- The wheelchair is equipped with anti flip wheels, which play an important role when a person sitting in a wheelchair has a problem with the stabilization of the body.
- Before using the wheelchair, set the anti-flip wheels.
- To do this, press the anti-flip wheel lock located in the lower part of the anti-flip wheel handle
- Turn anti-flip wheel towards the ground. (Illustration 30., 31.)
- This procedure must be performed on both anti-flip wheels.

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PUMPING TYRE

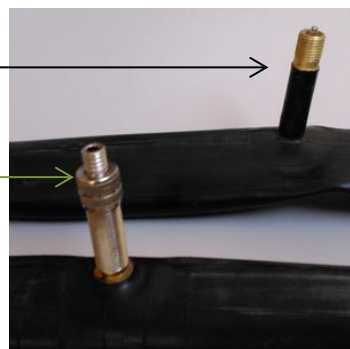
(It applies to wheelchair fitted with wheels pumped)

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valve which is located on the tire. This may be one of the two valves shown in Illustration 32 and 33.

- Prest valve also has an additional small top, which must be loosened before pumping.
- After identifying the valve, prepare the pump designed to valve type.
- Most of the new pumps are equipped to service both Prest valve and Schrader. Many of them let to reverse the tip. Smaller hole is Prest valve, Schrader valve greater. If the pump does not support Prest, you will need to buy yourself the "adapter Prest", which is available at any bike store.
- At this point, if the wheel is provided with a valve Prest, it will be necessary to unscrew the top located on the vent. It is a small top, which can be unscrewed with fingers. When doing this, the top should move upwards. Then valve is considered open.
- Then remove the plastic top from the valve and set it aside in a safe place. It is important to not lose it, because the valves will get dirt, leading to difficulties in pumping and rapid air loss.
- Place the nozzle of the pump to the valve and check if it stick properly. If it is needed, push the pump to the valve to make sure that the air does not escape to the sides.
- If the pump has a lever with clamp at the nozzle, tighten it. If the nozzle of the pump and valve have a threaded tip, screw them together. As a result, air will be streamed directly to the inner tube and the nozzle can not be detached from the valve.
- At the beginning of pumping the pressure in tyres should be checked at some points. Check tire pressure with your fingers at the end of the tread. If it is possible, make only a millimeter

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indentation. It means that the tire has enough air. If necessary continue pumping.

- After pumping, remove or unscrew the nozzle from the valve outlet while compressing it's ends by thumb. It is recommended to do this quick enough in order to not loose too much air.
- For Prest valve, tighten previously loosened additional small top.
- Screw the plastic top on the valve.

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CHANGE OF TYRES (It applies to wheelchar fitted with wheels pumped)

Before beginning the change of tyres it is necessary to get acquainted with the following notes.

First, it is necessary to release air from an inner tube and then by pressing a tyre in the direction of the centre of a rim to loosen a tyre-rim contact. Using a special spoon it is necessary to pull a tyre border out of the rim what will give the possibility to shift the whole wall of the tyre outside. After this operation pulling the inner tube out is very natural.

Prior to the installation of a new inner tube attention should be devoted to ensuring that:

- a bottom band is not damaged and has not foreign bodies,
- external walls of the rim are smooth, without foreign insertions and other damages.

The bottom band and rim should be precisely cleaned if need be.

To install the tyres it is necessary to put a hole of the bottom band over a valve, and then put together to a seating in the rim. Keeping such a position, put the bottom band on the whole perimeter of the rim, paying attention to ensuring that the band is not twisted or damaged and it covers spoke tighteners thoroughly.

Pulling the tyre wall away from the rim, insert the valve to the seating in the rim and slightly pump the inner tube that it achieves a round shape and then to put the inner tube in the rim.

To be sure, it necessary to check if the inner tube is not twisted (in case of deviation or deformation, pump air additionally). Manually put the tyre wall onto the rim, starting from an opposite side of the valve and ending by it.

It is necessary to check on both sides of the wheel if the inner tube is not pressed by the tyre and if the tyre regularly adheres to the rim. The valve should be slightly

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pressed inside and then repressed outside that it sets well in the seating.

To check that the tyre installation is correct it is necessary to pump such air that it is possible to press a thumb to the bottom of the rim. Then, check if the tyre adheres evenly to the rim on both sides and if it is centrally arranged. If such defects occur it is necessary to release air from the wheel and correct the setting of the tyres. When an opinion is positive it is necessary to pump the wheel to the maximum pressure value and put a cap onto the valve end.

PRESSURE IN WHEELS AND BRAKING POWER

- Pressure in wheels should be select to the user weight and then with assistance caregiver check the operation of the parking brakes (wheelchair with user facing toward the fall, can not roll on a plane with a slope of 15%.)
- After all the adjustments, be sure to thoroughly check all connections.

COMMENTS

- Observe proper tire inflation level with the information provided by the manufacturer on the sides of the tire.
- Use a pump with a pressure gauge or manual pump cycling.

EVERYDAY USE AND MAINTANCE

- Clean the upholstery and frame with a moist cloth, without the addition of detergents and other similar chemicals.
- Systematically remove the dirt from the movable elements of the wheelchair (screws, resistance mechanisms).
- Keep clean all wheelchair elements for to save functional parameters of the product.
- Storing in rooms of very high humidity (baths, laundries etc.) or in the open air, in the combination with the lack of hygiene will cause the fast loss of functional and aesthetic parameters.
- Do not expose the wheelchair on to extreme weather conditions (rain, snow very strong sun), because certain parts can change colour.

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You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,

Do not clean the wheelchair with pressure or steam aggregates!

TECHNICAL REVIEW

The review should be carried out regularly, it should be preceded by thorough cleaning the wheelchair (so as to reveal any hidden damage).

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Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.

Before each use	Control of wheels, brakes,	The brakes can worn out, check the condition of the brakes before each use. You should also check the condition of wheels and clean them from dust and sand.
Every two weeks	Screw and nut control regarding their correct fixing.	Make sure if all the screws and top are fixed tightly.
Every two months	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.
Once a month	wheel axles	Wheel axles must be conserved, seting between the axle and the hub a few drops of lubricant preservative. In addition, make test each wheel. Raise the wheelchair so that the tested wheel was located above the ground, the wheel should rotate freely but without backlash on axle.
Once a month	State connections, threaded, soldered, the system of quick release rear wheels	Pay attention to the cracks on frame material in connections threaded, soldered, the system of quick release rear wheels.
Once a month	Condition of the tires	Make sure that the tire tread is not worn and that the tires have no visible signs of damage.

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Repair

- In case of reparation, please contact the producer.

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Customer service

- If you have any questions or need help, contact the producer.

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